

Fitting all the benefits of the renowned Spex lateral supports into a small package suited perfectly for kids!

Introducing the Paediatric Lateral Support from Spex!



Its slim componentry allows for easier placement of the lateral support because positioning lateral supports can be tricky on paediatric wheelchairs where there is limited space for additional accessories and attachments.




## Renowned Axial Technology

Featuring the popular Axial technology that is known and loved from Spex, this allows you to achieve the correct support structure through a unique mechanically rotational system where the lateral pad can be placed in the optimum position for correct alignment of the user's trunk.

Wheelchair transfers made easy

The Paediatric Lateral Support has a clever swing-away feature, much like that found on the existing adult lateral supports, which allows unimpeded transfers in and out of the chair.



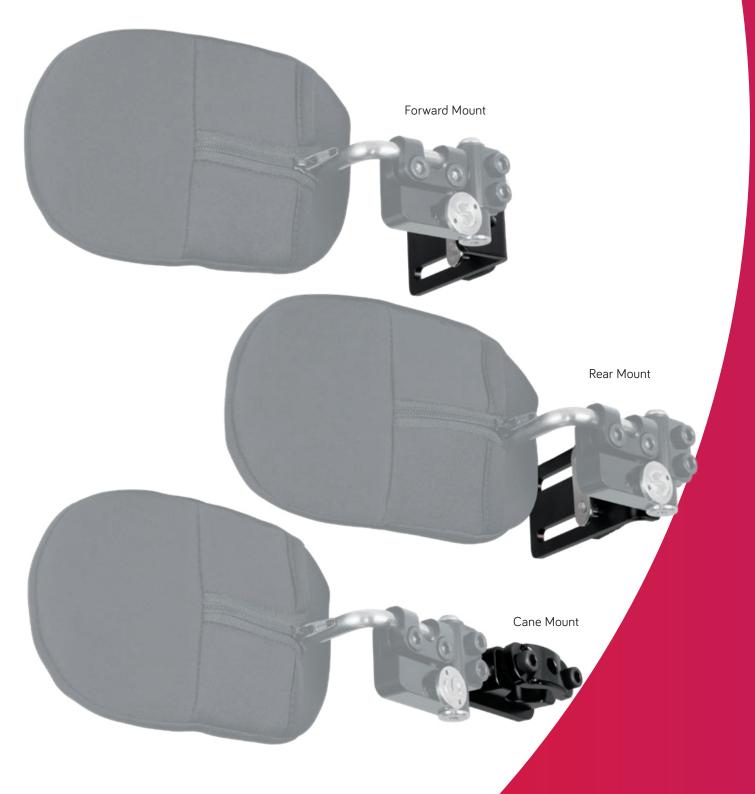

# Versatile mounting options

- Forward Mount: For mounting on thicker back supports. It mounts directly to the back shell, allowing for some width adjustment on the support.
- Standard (Rear Mount): For a snug fit on thinner back supports. It mounts directly to the back shell, allowing for some width adjustment on the support. This is the default Paediatric Lateral mounting option.
- Cane Mount: Mounts directly to the back support canes of the wheelchair for greater flexibility when installing.


### spex

Paediatric Lateral Support Specifications

# Invertible mounting capabilities


The Paediatric Lateral Support has invertible mounting capability depending on where you want the Swing-Away release button to be positioned. These are available in left hand and right hand configurations where the Swing-Away release button is positioned on the lower side of the bracket. However, these can be used on either left hand and right hand side.


### Dimensions

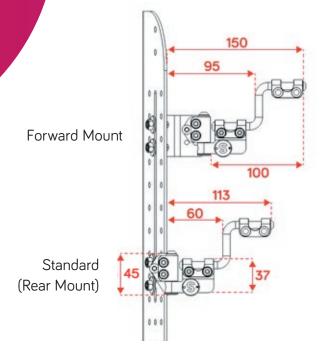

### Axial Adjustment

Range of adjustment available within the Axial lateral hardware

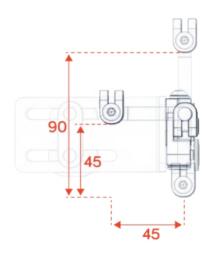

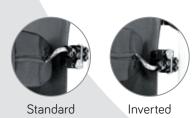

|                          | Left Hand Side  | Right Hand Side | Weight                                      | Weight<br>Limit |
|--------------------------|-----------------|-----------------|---|-----------------|
| STANDARD<br>(REAR MOUNT) | 1266-2623-100-L | 1266-2623-100-R | 0.15kg<br>to<br>0.25kg<br>(bracket<br>only) | 50kg            |
| FORWARD<br>MOUNT         | 1266-2623-200-L | 1266-2623-200-R |   |                 |
| CANE MOUNT               | 1266-2623-300-L | 1266-2623-300-R |   |                 |

## Lateral pad options



#### **Broad Pad**

Long trunk support, effective pressure reduction.

| SIZE   | HEIGHT | DEPTH  | CODE          |
|--------|--------|--------|---------------|
| SIZE 1 | 105 mm | 130 mm | 1266-3031-XXX |
| SIZE 2 | 125 mm | 140 mm | 1266-3032-XXX |
| SIZE 3 | 140 mm | 150 mm | 1266-3033-XXX |
| SIZE 4 | 160 mm | 155 mm | 1266-3034-XXX |
| SIZE 5 | 195 mm | 165 mm | 1266-3035-XXX |
| SIZE 6 | 230 mm | 175 mm | 1266-3036-XXX |

### Pad Cover Options

Standard Cover: Removable and washable. Code: -400





**Comfi-Cover:** Extended padding over mechanism to protect arms from metal hardware. Code: -800



### Narrow Straight pad

Freedom of movement, move forwards and back for functional activities and transfers.

| SIZE   | HEIGHT | DEPTH | CODE          |  |
|--------|--------|-------|---------------|--|
| SIZE 0 | 85mm   | 115mm | 1266-3010-XXX |  |
| SIZE 1 | 95mm   | 145mm | 1266-3011-XXX |  |
| SIZE 2 | 95mm   | 180mm | 1266-3012-XXX |  |
| SIZE 3 | 115mm  | 210mm | 1266-3013-XXX |  |
| SIZE 4 | 115mm  | 240mm | 1266-3014-XXX |  |



### Narrow Curved pad

Anterior support, for increased comfort and spinal extension.

| SIZE   | HEIGHT | DEPTH | CODE          |
|--------|--------|-------|---------------|
| SIZE 1 | 95mm   | 145mm | 1266-3021-XXX |
| SIZE 2 | 95mm   | 180mm | 1266-3022-XXX |
| SIZE 3 | 115mm  | 210mm | 1266-3023-XXX |
| SIZE 4 | 115mm  | 240mm | 1266-3024-XXX |

