

Carrot 3000

SAFETY FEATURES

Our most
comprehensive
special purpose
car seat

Ensure the
HEAD SUPPORT
is adjusted as your
child grows taller

**5 POINT
HARNESS** ensure
this is tightened
sufficiently through
grey strap at the
front of the seat

Large **CHEST
SUPPORT** holds the
body securely and
should be adjusted
for height through
front of seat

Always use **VEHICLE
SEATBELT** to secure
the child



Carrot 3000

INSTALLATION

- The Carrot 3000 should be installed in the back seat of the vehicle
- **The mandatory points of installation:**
 - The vehicle seatbelt must be used every time
 - Ensure the Carrot 3000 back is in contact with the vehicle seat back (no big gap behind headrest)
- As standard, the Carrot 3000 is provided with 2 fixing straps which provide the seat with additional stability during transfers. Their use is optional.
- Additional accessories are available for fixing the Carrot 3000 to the vehicle seat. These are optional and may be prescribed and used as desired

USE

- Place the child in the seat, ensuring the shoulder straps are level with the child's shoulder height. If they are not inline with the child's shoulder height, the seat requires size adjustment.
- Ensure the harness buckle is done up at the front of the child, you will hear an audible click
- The harness is then tightened using the grey strap at the front of the seat to ensure a supportive and snug fit
- Ensure the vehicle seatbelt is used EVERY time. This is to be securely fastened OVER the child and seat.
- The seatbelt must be threaded through the red belt guides provided. From where it originates, the top portion of the vehicle seatbelt must pass through the red guide located at the side of the head support, the lower section must pass through the red guide at the side of the seat base. On the side of the car seat that the vehicle seatbelt clips in, both sections of the seatbelt should pass through the red belt guide, before being clipped into the receiver.
- Familiarise yourself with the weight limit on your child's seat and ensure they have not outgrown this. For the Carrot 3000 this is 49kg
- Periodically check for loose or missing parts.

Contact the car seat specialists for more information:
0800 543 343
solutions@medifab.com

Distributed by **medifab**


The quick reference guide does not replace the requirement to read the user manual and safety information therein.



The car's seatbelt must be worn over top of the child and car seat harness



Ensure back is in contact with the vehicle seat back

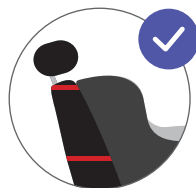

Ensure back straps are firmly fixed around the vehicle's headrest and seat



Ensure shoulder straps are firmly secure



Ensure child has not outgrown the weight capacity limit