

Kidsflex

SAFETY FEATURES

Excellent posture support options for comfort and postural management

Secure seat using the **EXISTING 3 POINT SAFETY BELT** fitted in the rear seat.

Additional belt guide can be added for other side of Car Seat #4170-6960-000

Backrest with **RECLINE FUNCTION**
Increased tilt angle - from 10°-20° can be added #4170-0250-010

5 point harness: comes standard with **ABDOMINAL PADDING**

SLIMLINE seat cushions allow more space and headroom inside the car

SWIVEL BASE



Kidsflex

Excellent posture support options and suitable for installation with most car types. Available with a range of options and adjustments, allowing complete comfort and postural management for the child.

- Ensure the Kidsflex's back is in contact with the vehicle seat back (no big gap behind headrest).
- There is no tether for this seat, but ensure the clamp bow is slotted correctly between the seat and backrest of the car, OR that the ISOFIX/LATCH connectors are correctly in use.
- Familiarise yourself with the weight limit on your child's seat, and ensure they have **not outgrown** this.
- Ensure the car's seatbelt is used **EVERY** time. This is to be securely **fastened OVER the child and seat**. Please note this seat does not have capacity to have a belt threaded through the back as it is a booster seat. You are able to purchase and install a padded sleeve for the seatbelt if you feel the belt is going to irritate your child's neck.
- Please **note the belt guides are to be used to ensure the belt is sitting correctly**, is kept low across the child's hips and is kept off the child's face.
- If this unit is fitted with a swivel base check the **latch is working correctly**, and that the car seat swivels freely.
- Periodically **check for loose or missing parts**.
- Keep child's fingers **clear of moving parts and swivel latches**.



Ensure back is in contact with the vehicle seat back



Ensure shoulder straps are firmly secure



Ensure child has not outgrown the weight capacity limit



The car's seatbelt must be worn over top of the child and car seat harness

Contact the car seat specialists for more information:
0800 543 343
solutions@medifab.com

Distributed by **medifab**[®]