

# Carrot 3000 Swivel Base Installation and User Manual



# 4116-0180-104

**IMPORTANT CONSUMER INFORMATION NOTICE:**

This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end user.

**SUPPLIER:**

This manual must be given to the user of this product.

**USER:**

Before using this product, read entire manual and save for future reference.

**medifab**<sup>®</sup>

User Manual for all Carrot Swivel Base  
Manual: Carrot Swivel- 20180813

Serial Number:

Dear parents

with the ISO-TURN, you have chosen an especially high-quality product. Because of the extremely solid construction, the ISO-TURN can be needed for many years. Your dealer can help you with the assembly.

EC declaration of conformity of medical devices (in accordance with Annexes II, V, VI and VIII of the EC Directive 93/42/EEC concerning medical devices):

Hereby we declare that the ISO-TURN is equivalent to the relevant provisions of the Act and the Medical Device Directive 93/42 EEC. The ISO-TURN with ISOFIX connection is produced under the control of quality management in accordance to DIN EN ISO 9100 and DIN EN 46001.

Distributed in Australia/New Zealand by: Medifab Ltd  
 Head Office: 32 Detroit Drive, Rolleston 7675, New Zealand  
 AU Office: 26 Pardoe Street, East Devonport, TAS 7310, Australia  
 Telephone +64 3 307 9790 Fax +64 3 307 2820  
 Email: solutions@medifab.com      www.medifab.com



## Index

<b>Before using the ISO-TURN</b>	page 3
<b>Safety arrangements</b>	page 4
<b>Use of the ISO-TURN</b>	page 5
➤ 1. Fix the Carrot/Carrot II on the ISO-TURN	page 5
➤ 2. Proper fix on the car seat with the ISOFIX connection	page 5
➤ 3. Back angle adjustment	page 6
➤ 4. Adjustment of the footwell brace to the footwell	page 6
➤ 5. Spinning process	page 7
➤ 6. How to remove the ISO-TURN	page 7
<b>Warranty</b>	page 8


# ISO-TURN with ISOFIX connection

## Instruction manual

### Before using the ISO-TURN

To ensure the safe use of the ISO-TURN please read and use the instruction manual and especially the safety arrangements. Please understand that the instruction manual may be different to the various designs of the product.

Keep the instruction manual well. A warranty card is attached on the last page. Please keep it carefully so you can submit the warranty card in case of a defect. The instruction manual always have to be present also if you commit the product to others or lend it.

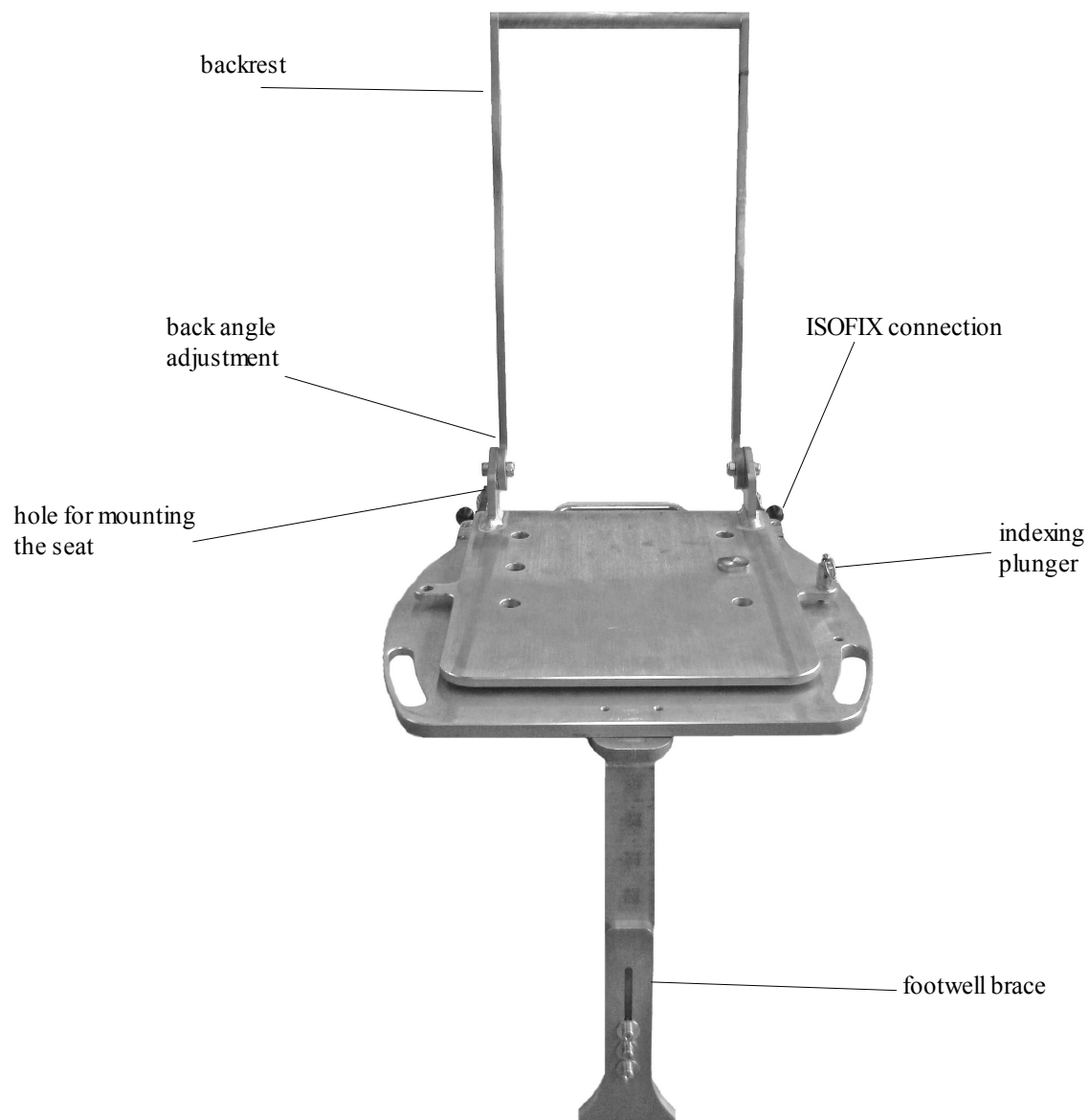

## SAFETY ARRANGEMENTS



Attention (this symbol means that misuse can cause a high risk of death or injury.)

The following car seats are not suitable for the use with the ISO-TURN:

- car seats without ISOFIX connection
- car seats which are fixed sideways or backwards
- car seats which are less than 40cm deep
- car seats with a pronounced bulge on the seat which can lead to an unstable attachment
- car seats with an two-point seat belt

### **What you should be aware of before using the ISO-TURN**

Please transport your child with the five-point seat belt of the Carrot/Carrot II and with the three-point seat belt of the car. A modification of the product is your responsibility. We are not responsible for any damage in case of a incorrect modification. Only the accompanying person should handle the product. Just for the use in vehicles which are equipped with a three-point seat belt. Please pay attention when installing that no part of the ISO-TURN can be wedged by a movable seat or in the door if you use it the normal way.

An insufficiently fixed ISO-TURN can cause a high risk of death or injury. Use only for specified children and only with the above-described car seats. We recommend the space behind the front passenger. Regular checks are necessary if you drive on bad roads as well as before and after a ride. The instruction manual always have to be attached to the product. Do not forget this if you let it to other+persons.

Make sure that the ISOFIX connection of the ISO-TURN is properly locked with the car seat. This applies also if you travel without a child in the car seat or there is no car seat on the turntable. Without the adjustment the turntable could injure inmates in case of an accident.

Do not use a car seat without a ISOFIX connection to guarantee the safe of the patient.

Secure your baggage that you do not get injured in the event of an accident.

Please do not leave your child unattended in the car.

### **When you should not use the ISO-TURN**

Please do not use the ISO-TURN if it not fixed correctly as described in the instruction manual and also if the ISO-TURN is not firmly locked even after fitting. Do not use this product without an ISOFIX connection of the car seat. Do not use other seat belts or forms than those that are described in the instruction manual. To ensure safety please do not use the product any longer if it was involved in an accident. If the ISO-TURN is fixed on the front passenger seat and impede you actuating the gear shift and the handbrake you should not use it. Do not use seat belts that are damaged, e.g. by cigarettes burns.

### **What also should be noticed**

The ISOFIX connections can not be solved easily. But you should still pay attention that nobody handle it improperly. If you have any further questions regarding to the use of this product or the repair work please do not hesitate to contact your dealer.

Some cars have in the foot rest area an integrated storage shelf. If the foot well brace should stand on such a shelf the stability in a crash situation is reduced. Please contact your dealer. We will give him a solution for this problem.

## Use of the ISO-TURN

RULES how to fix:


### **1. Fix the Carrot/Carrot II on the ISO-TURN**

At first please solve and remove the adjustment screws of the Carrot/Carrot II. Now put the car safety seat on to the turntable so that the back of the seat is between the holders of the back angel adjustment. Please mount the Carrot/Carrot II on the ISO-TURN with the bolts that keep the seat and the back of the car safety seat together. The screws should be firmly tighten on both sides of the seat. After that please screw the clamp lever for the back angle adjustment in to the designated holes.



### **2. Proper fix on to the car seat with the ISOFIX connection**

Now place the ISO-TURN with the attached Carrot/Carrot II on to the car seat. Please note that the ISOFIX connection of the ISO-TURN and of the vehicle are the same height. Just after that you can insert the ISO-TURN into the ISOFIX connection of the car seat until it clicks hearable into the right place.

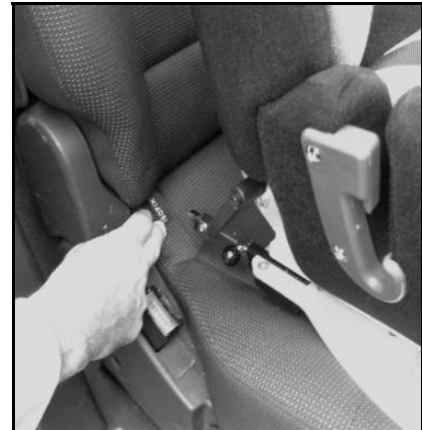


### **3.Back angel adjustment**

The back of the ISO-TURN and of the Carrot/Carrot II can be fitted to the back of the car seat if you solve the clamp lever at the back angle adjustment. Please push the back frame of the ISO-TURN according to the inclination of the car seat. Now fix it again with the previously dissolved clamp lever. Make sure that the clamp lever is firmly applied. After that you can adjust the back of the Carrot/Carrot II to the car seat.




### **4.Adjustment of the footwell brace to the footwell**

First of all loose the screws of the footwell brace on the underneath of the ISO-TURN. Afterwards you can adapt the brace to the depth of the car's seat. Then tighten the screws firmly. Now loosen the three screws on the footwell brace to cease the height. Leave the brace glide to the car floor and then tighten the bolts firmly.

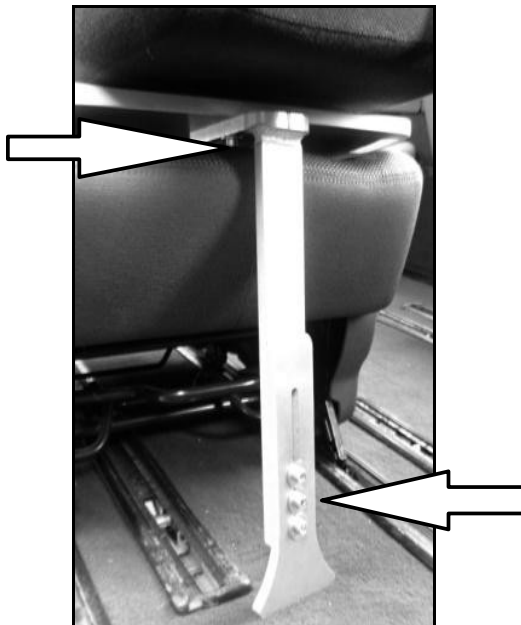


### **5. Spinning process**

In order to turn the ISO-TURN the indexing plunger have to be unlocked. The indexing plunger is located at the top of the rotation plate, drag it upward to unlock. Now you can turn the Carrot/Carrot II with the ISO-TURN in the direction of the car door to exit it. After that you can fix the turntable with the indexing plunger in the desired position. You should make sure that the indexing plunger snaps into the right position.

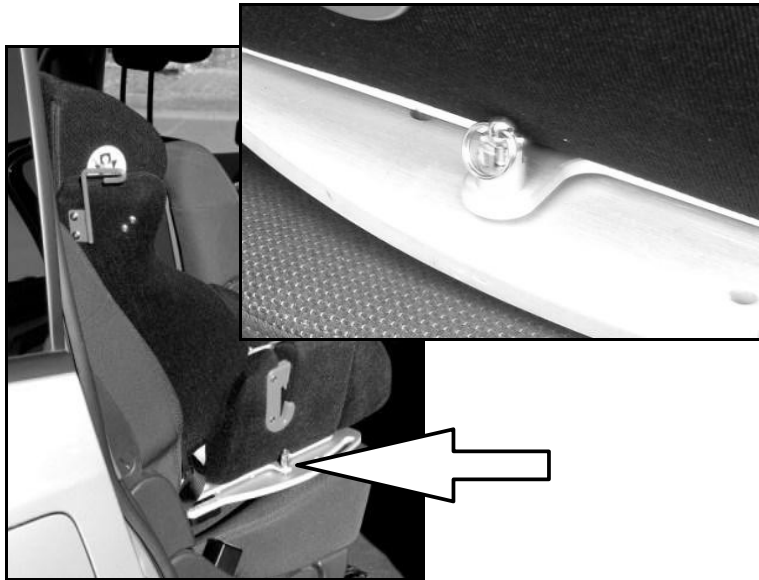

### **6. How to remove the ISO-TURN**

You have to unlock the ISOFIX connection to remove the turntable. Unlock of the ISO-TURN by pushing the black balls in the driving direction on both sides, push the ISO-TURN some millimeters back to loose the ISOFIX hooks out of the mounting link, lift it up and then pull the turntable. Now you can remove the Carrot/Carrot II with the ISO-TURN from the vehicle.

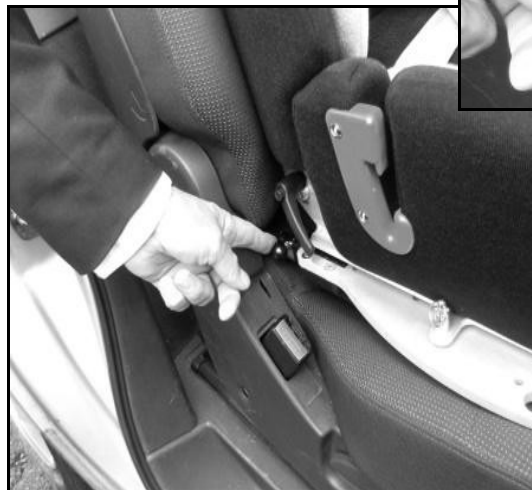

## Garantie

1. During the warranty period the product is guaranteed in case of damage caused by a defect of the materials.
2. The length of warranty is limited to one year after acquiring the product.
3. The warranty do not cover following cases:
  - ✗ incorrect and improper use
  - ✗ unauthorized change of the product
  - ✗ damage caused by accidents
  - ✗ The warranty card is without the name and stamp of the dealer where you bought the product.
  - ✗ If you are not able to provide the warranty card.

**Maintenance service:** If a problem occurs during the warranty period please contact your dealer where you have purchased this product. You have to submit the warranty card.

**CAUTION:** The warranty card will not be replaced. Keep it well.

<b><u>WARRANTY CARD</u></b>			
model	<b><u>ISO-TURN turntable with ISOFIX connection</u></b>		
client	address		
	name	phone	
Date of purchase			
dealer	Medifab Ltd 32 Detroit Drive, Rolleston 7675, New Zealand		

# medifab®

New Zealand Head Office  
32 Detroit Drive,  
Rolleston 7675  
Call 0800 543 343

Australian Head Office  
26 Pardoe Street,  
East Devonport, TAS 7310  
Call 1300 543 343

solutions@medifab.com

**medifab.com**

Your enthusiastic distributor is