

Bingo Mini Canopy Mount Installation



- Suits 10 – 16" chairs.
- 19 – 25mm tube diameter.

Step 1:

For ease of installation, it is recommended a single bolt is connected to the bracket prior to fitting.



Step 2:

Sit the bracket on the canes near the handgrip, fit the clamp to the bracket using the single screw to locate and hold. L-shape of the bracket is to face backward.



Step 3:

Using a 4mm Allen key, fit the second bolt and tighten to lock the bracket to the cane. Repeat for the opposite side.



Step 4:

Clip the Bingo Mini Canopy into the bracket and adjust the height of brackets and angle of the canopy to suit chair.

