

Bingo Mini Canopy

Mount Installation



- Suits 10" – 16" Chairs
- 19mm – 30mm Tube Diameter

Step 1:

For ease of installation, it is recommended a single bolt is connected to the bracket prior to fitting.



Step 2:

Sit the bracket on the canes near the handgrip, fit the clamp to the bracket using the single screw to locate and hold.

L-shape of the bracket is to face backward.



Step 3:

Using a 4mm Alan key fit the second bolt and tighten to lock the bracket to the cane.

Repeat for the opposite side.



Step 4:

Clip the Bingo Mini Canopy into the bracket and adjust the height of brackets and angle of the canopy to suit chair.

