



This installation checklist is made available to facilitate the delivery of this special purpose car seat or restraint only, it does not replace the need to consult the user manual as required.


### 1 - Client and vehicle details

Date of Installation		Client age at time of installation	
Client Name			
Client weight at time of installation	IMPORTANT: Check user weight is within this range prior to commencing installation: 36-75kg		
Main vehicle that client uses for the majority of their travel	Make and model:		
	Year of manufacture:		
Other vehicles used for transport have been checked for compatibility with the Carrot XL		<input type="checkbox"/> Yes	<input type="checkbox"/> No

### 2 - Installation of Carrot XL into vehicle - Mandatory Installation Points

Establish if Carrot XL is to be installed with any of the following optional accessories:

- Horizontal or vertical fixing straps
- ISOFIX lower straps
- Top tether strap

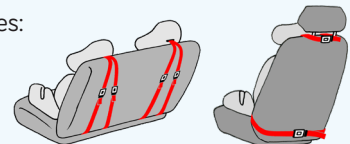

Attach and secure any of these accessories if being used for installation. Adjust and tighten all accessory straps and ensure all strap slack has been removed.


- The above accessories are optional for installation but may provide additional stability during transfers. Any accessory above not required for installation should be removed from the Carrot XL prior to installation and use.

Position of the headrest is adjusted to ensure proper fitment of the inbuilt harness and vehicle seat belt path.

Position of lateral supports are adjusted to suit needs of user (under direction from child's OT/PT)

The vehicle seat belt secures over the user **FIRST** before the Carrot XL inbuilt harness is secured. **This is specific to the Carrot XL only.** The route for the vehicle seat belt is as follows:

- The upper section of the seat belt is fed through the red belt guide located at the side of the head support
- The lower section of the seat belt passes through the red belt guide on the side of the seat base closest to the seat belt origin
- The vehicle seat belt then passes over the top of the user. Both the upper and lower sections of the vehicle seat belt pass through the red seat belt guide located on the opposite side of the seat base and are buckled into the receiver

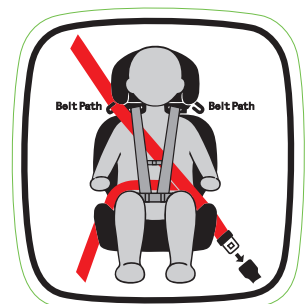

After fitting the vehicle seat belt over the user through the above path, adjust and secure the Carrot XL inbuilt harness to suit the size of the user (ensure the shoulder straps of the inbuilt harness are in line with the top of the shoulders).

Ensure the webbing of the restraint and vehicle seat belt is not twisted or trapped.

Remove any residual slack from the vehicle seat belt

### 3 - Parent/caregiver training on the Carrot XL

Training and education has been given to parent/caregiver on how to:

Remove and reinstall Carrot XL

Operate anti escape release on harness (if using an anti escape harness option)

If anti escape present - Webbing slicer kit has been provided and explained to family

Adjust the restraint

Use and fit all accessories delivered with the Carrot XL

### IMPORTANT

There are differences in the road laws between each state and territory in Australia regarding requirements of transporting children with disabilities and medical conditions. Please refer to the MACA website [www.macahub.org/resources/forms-and-templates](http://www.macahub.org/resources/forms-and-templates) to find out about requirements and processes that apply to the specific state or territory that the Carrot XL will be used in.

### All below parties to complete and sign the below

Completion of sections 1, 2 and 3 of this form

Parent/caregiver has been supplied with Parent Safety Checklist relevant to product

Prescribing therapist to be supplied a copy of completed form and upload to client records stored by the prescribing therapist

AT Product Consultant name and organisation:	Parent/caregiver name:	Prescribing therapist name and organisation:
AT rep signature:	Parent/caregiver signature:	Prescribing therapist signature:
Date:	Date:	Date:

AT rep to retain and store copy of completed form. Parent/caregiver can request copy of completed form.