



This installation checklist is made available to facilitate the delivery of this special purpose car seat or restraint only, it does not replace the need to consult the user manual as required.

1. Client and vehicle details

Date of Installation		Client age at time of installation	
Client Name			
Client weight at time of installation		IMPORTANT: Check user weight is within this range prior to commencing installation: 0-18kg	
Main vehicle that client uses for the majority of their travel	Make and model:		
	Year of manufacture:		
Other vehicles used for transport have been checked for compatibility with the Hero-NXT		☐ Yes	☐ No

2. Installation and fitting of Hero-NXT

It is recommended images are taken of each completed installation step. These images can be stored in both AT supplier and prescribing therapist's client record system for future reference.

Mandatory Installation Points – tick when each step is complete

- ☐ ISOFIX connection made through factory installed ISOFIX points when you can see a green strip displayed on both ISOFIX arms – If no ISOFIX is present, abort delivery of Hero-NXT
- ☐ The Hero-NXT is pushed rewards in the vehicle seat so the seat is as close to the vehicle backrest as possible
- ☐ Foot prop is swung out and is angled away from the seat – (click heard and no red section visible at the top of the foot prop shaft)
- ☐ Length of foot prop is extended and is clicked into place to reach the floor of the footwell, as indicated by thin green strip being visible on foot prop. If the foot prop does not reach the floor of the footwell, cease installation
- ☐ Identify whether restraint to be used in reward or forwards facing position using the below parameters:
 - **Reward facing** – for children 61-105cm tall
 - **Forward facing** – from 15 months and up, 76-105cm tall



- ☐ Shoulder straps are same level as child's shoulder or slightly higher; they must not angle down behind the shoulder or angle upwards inline or above the ear
- ☐ Harness has been secured and tightened after it has been fitted to the user
- ☐ The webbing of the inbuilt harness is not twisted or trapped

3. Parent/caregiver training on the Hero-NXT

Training and education has been given to parent/caregiver on how to:

- ☐ Remove and reinstall the Hero-NXT
- ☐ Adjust the restraint
- ☐ Use and fit all accessories delivered with Hero-NXT
- ☐ Use swivel function within Hero-NXT
- ☐ Operate the tilt mechanism
- ☐ Check whether the restraint is locked in the reward or forwards facing position

IMPORTANT

There are differences in the road laws between each state and territory in Australia regarding requirements of transporting children with disabilities and medical conditions. Please refer to the MACA website macahub.org/resources/forms-and-templates to find out about requirements and processes that apply to the specific state or territory that the Hero-NXT will be used in.

All below parties are required to complete and sign the following:

- ☐ Completion of sections 1, 2 and 3 of this form
- ☐ Parent/caregiver has been supplied with Parent Safety Checklist relevant to product
- ☐ Prescribing therapist to be supplied a copy of completed form and upload to client records stored by the prescribing therapist

AT Product Consultant name and organisation:	Parent/caregiver name:	Prescribing therapist name and organisation:
AT rep signature:	Parent/caregiver signature:	Prescribing therapist signature:
Date:	Date:	Date:

AT rep to retain and store copy of completed form. Parent/caregiver can request copy of completed form.