

Safespace Assembly Instructions

List of Materials:

- 1x PVC Fabric (Anti-toe-stubbing foam)
- 1x Floor Foam
- 4x Horizontal Bars (Short)
- 4x Horizontal Bars (Long)
- 4x Vertical Bars (Bottom bracket attached)
- 4x Top Corner Mounting Brackets (M20 Nuts)
- 4x Corner Foam Pads
- 1x Foam Mattress
- 32x M8 45mm Hex Head Bolts
- 32x M8 Nyloc Nuts
- 32x M8 Washers
- 4x M20 Washers

Tools Required:

- 1x 13mm Spanner
- 1x Power Drill with 13mm Socket Head



Safespace Position:

If possible, move and position the Safespace into the location where it is to be situated in the room i.e. door to door, window to window.



Step 1: Insert the Foam Floor

Roll out the PVC fabric and position it with the floor facing up and mesh side facing down. Fully open the floor zip on the PVC fabric. Place the floor foam on top of the PVC fabric and align the marked * with the * on the inside of the PVC cover. This is to ensure that the foam fits snug into the corners. Insert each corner of the floor foam into the PVC fabric. Once fully fitted, begin to close the zip leaving a 30cm gap. Push the air out of the fabric to make it smooth and even. Then fully zip up and push the zip puller inside the fabric to conceal it. Once the floor is fully zipped up, turn the Safespace over so the mesh side is now facing up.

Step 2: Insert the Horizontal Bars

Identify the 4x horizontal bars (Long) and 4x horizontal bars (Short). The long bars will be for the longer length sides and the shorter bars for the shorter length sides of the Safespace. Please Note: The bars will be the same length if the Safespace is square. Position the top and bottom bars on each side of the Safespace, alongside the pole pockets. The poles will be colour coordinated to indicate which pole pocket they will be placed in. Please Note: If the Safespace is one colour, all poles will be the same colour. The seam on the coloured tape should face downwards (bottom pole) or upwards (top pole). Insert the bars into the pole pockets making sure that the seam isn't visible. Position the holes so that they are facing away from the Safespace.



Step 3: Insert the Top Corner Mounting Brackets


Identify the top corner mounting brackets and align the holes on the horizontal bars with the holes in the bracket, making sure the bracket is facing the correct way. Fit the bracket using an M8 45mm hex head bolt through the holes from the front of the horizontal bar. Attach a M8 washer and M8 nyloc nut to the threaded end of the bolt on the inside of the bar. Repeat for all corners. Use the spanner to help align the bar with the bracket and then tighten all the nuts with a drill nut driver and then finally tighten with a 13mm hand spanner.



Step 4: Insert the Vertical Bars (This step requires 2 people)

Working in a pair, lift the Safespace up so that the threaded bar on the top corner brackets is easily accessible. (As demonstrated in the photo). Once the threaded bar is accessible, place an M20 washer onto the threaded bar so it is in between the vertical bar and the M20 nut (This will be attached to the threaded bar at pre-assembly). Once the M20 washer is on, connect the vertical bar to the threaded bar. Rest these two poles at an angle (shown in photo) so it is resting on the floor mounting brackets and then repeat on the other side of the Safespace. The Safespace is now on all four vertical uprights.

IMPORTANT: READ and FOLLOW these instructions carefully. Review all instructions before beginning assembly. These instructions are provided to avoid problems that may occur from improper assembly or installation.

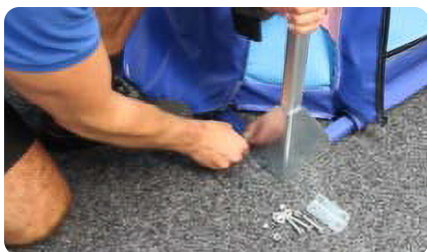

Step 5: Attach the Floor Mounting Brackets

Align the holes on the horizontal bars with the holes in the floor mounting brackets then fit the bracket using an M8 45mm hex head bolt through the holes from the front of the horizontal bar. Then attach an M8 washer and M8 nyloc nut to the threaded end of the bolt on the back of the Safespace bar.
Repeat the same for all four corner brackets.

Step 5.5: Securing to the Floor

Floor Brackets

The Safespace can be screwed to the floor with the four-floor brackets.

Velcro

The Safespace can be secured to the floor using Velcro straps.

Step 7: Tension the PVC

Feel the 'slack' in the PVC at one of the corners.

Lift the top corner mounting bracket up and tighten the nut on the threaded bar downwards to create tension. Tighten the nut to a distance of 80-90mm away from the bracket.

Tighten all the nuts with a power drill and then tighten with a 13mm spanner.

Repeat on all 4 corners.

Step 8: Insert the Corner Foam Pads

Insert the corner foam pads inside the PVC pockets on the inside of the PVC fabric.

Repeat on all 4 corners.



Step 9: Attach the Vertical Bars

Make sure all the zips of the doors and windows of the Safespace are closed.

Then begin to wrap the Velcro straps around the vertical bars, at first with only a small amount of tension. Thread the back of the Velcro around the pole, thread it through the buckle. Then fold it back on itself to hold and tension the straps more firmly.

The door and window zips need to open and close freely with no 'straining'. Check this, and if there is straining loosen the tension off slightly till the zips run freely.

Step 10: Zip up the Corner Pockets

Once you are satisfied with the tension and the positioning of the safespace is correct, zip up all the corner pockets.



Step 11: Mattress - Only applicable if Safespace has a mattress

Place the mattress into the Safespace, the position of the mattress will be shown on the plan.

Your Safespace is now ready to use.



Read User Manual Before Use:

It is important to fully understand the content of the user manual before attempting to use the equipment.