


Spare Parts & Components

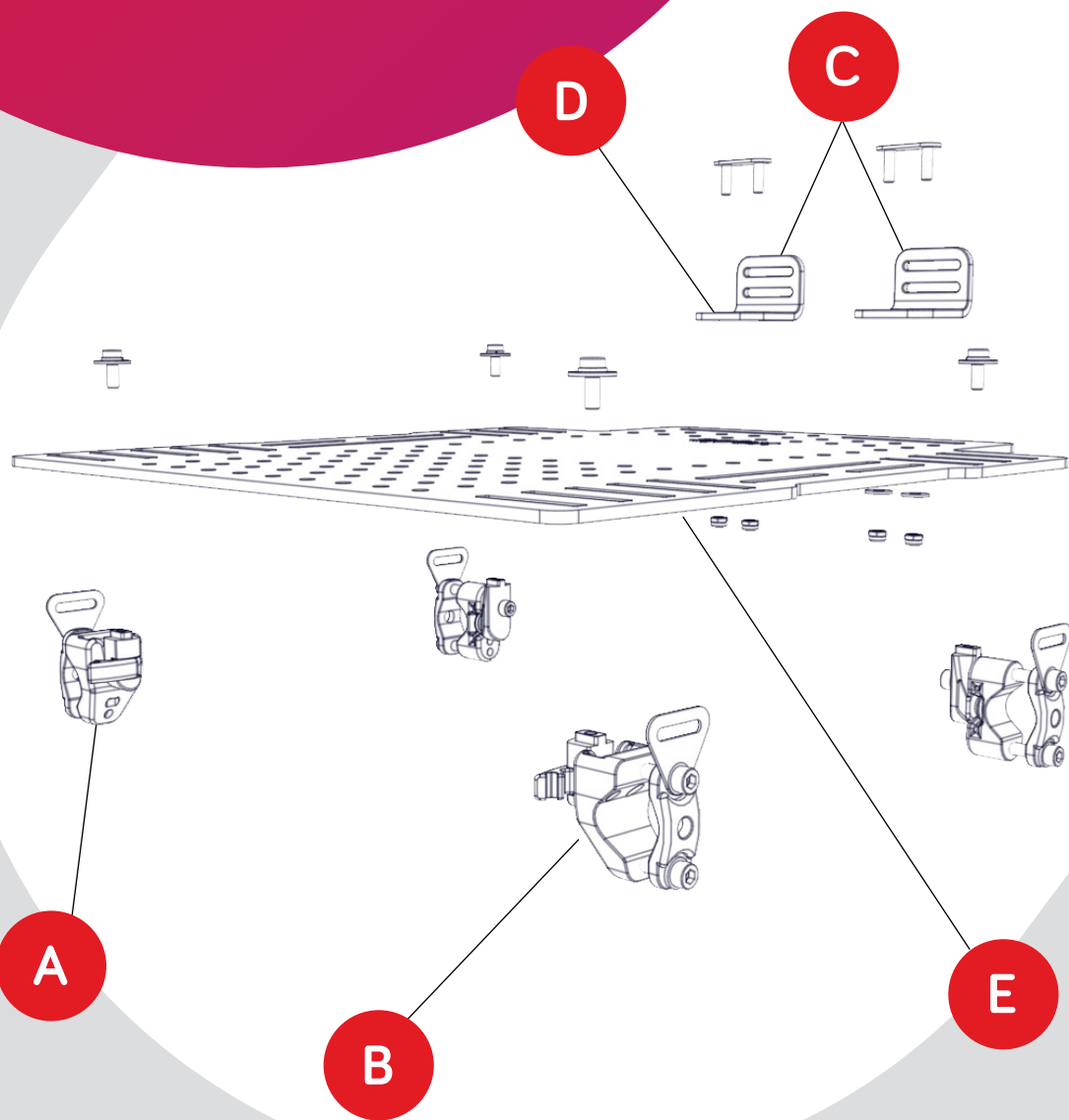

spex

SPARE PARTS

SCRIPTING AND ORDERING GUIDE

CONTENTS

SEAT BASE	3
CUSHION COVERS	4 - 11
CONSTRUCTA SPARE PARTS	11 - 13
CUSHION COVER MODIFICATIONS	14
BACK SUPPORTS	15 - 23
BACK SUPPORT CUSTOMISATIONS	24
BACK SUPPORT HARDWARE	23 - 33
BACK SUPPORT FASTENERS	34
LATERAL TRUNK SUPPORT	35 - 38
MEDIAL THIGH SUPPORT	39
ARM/THIGH SUPPORT	40 - 43
HEAD SUPPORTS	44 - 46
HARNESS & HIP BELTS	47
TRAY TABLES	48
WONDERSEAT BIT BOX	49

CONTACT US

If you need help finding a spare part or have a product enquiry, you can contact us using the following details:



Operations Head Office
1300 543 343
solutions@spexseating.com




Home of Innovation
0800 543 343
solutions@spexseating.com



International
+64 3 307 9790
solutions@spexseating.com

SEAT PAN (DISCONTINUED)


#	CODE	DESCRIPTION	QTY
A	1009-0251-100	Fixing plate	QTY
B	1009-1012-100	Main Plate 10"/11" - 12" WC Width	QTY
B	1009-1212-100	Main Plate 12"/13" - 12" WC Width	QTY
B	1009-1214-100	Main Plate 12"/13" - 14" WC Width	QTY
B	1009-1414-100	Main Plate 14"/15" - 12" WC Width	QTY
B	1009-1416-100	Main Plate 14"/15" - 14" WC Width	QTY
B	1009-1412-100	Main Plate 14"/15" - 16" WC Width	QTY
B	1009-1616-100	Main Plate 16"/17" - 16" WC Width	QTY
B	1009-1618-100	Main Plate 16"/17" - 18" WC Width	QTY
C	1009-0010-100	Seat pan side wings - 10" Depth	QTY
C	1009-0012-100	Seat pan side wings - 12" Depth	QTY
C	1009-0014-100	Seat pan side wings - 14" Depth	QTY
C	1009-0016-100	Seat pan side wings - 16" Depth	QTY
C	1009-0018-100	Seat pan side wings - 18" Depth	QTY
C	1009-0020-100	Seat pan side wings - 20" Depth	QTY
D	1009-0210-100	Seat pan rail clamps	QTY

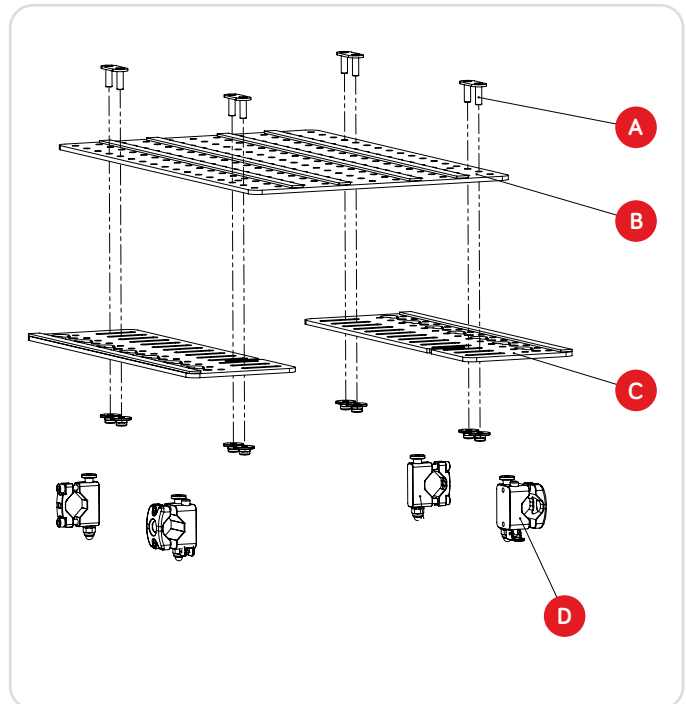

SEAT BASE

#	CODE	DESCRIPTION	QTY
E	1008-1012-100	Spex Seat Base only, 10" x 12"	QTY
E	1008-1014-100	Spex Seat Base only, 10" x 14"	QTY
E	1008-1212-100	Spex Seat Base only, 12" x 12"	QTY
E	1008-1214-100	Spex Seat Base only, 12" x 14"	QTY
E	1008-1216-100	Spex Seat Base only, 12" x 16"	QTY
E	1008-1218-100	Spex Seat Base only, 12" x 18"	QTY
E	1008-1414-100	Spex Seat Base only, 14" x 14"	QTY
E	1008-1416-100	Spex Seat Base only, 14" x 16"	QTY
E	1008-1418-100	Spex Seat Base only, 14" x 18"	QTY
E	1008-1517-100	Spex Seat Base only, 15" x 17"	QTY
E	1008-1520-100	Spex Seat Base only, 15" x 20"	QTY
E	1008-1616-100	Spex Seat Base only, 16" x 16"	QTY
E	1008-1618-100	Spex Seat Base only, 16" x 18"	QTY
E	1008-1620-100	Spex Seat Base only, 16" x 20"	QTY
E	1008-1622-100	Spex Seat Base only, 16" x 22"	QTY
E	1008-1717-100	Spex Seat Base only, 17" x 17"	QTY
E	1008-1720-100	Spex Seat Base only, 17" x 20"	QTY
E	1008-1722-100	Spex Seat Base only, 17" x 22"	QTY
E	1008-1818-100	Spex Seat Base only, 18" x 18"	QTY
E	1008-1820-100	Spex Seat Base only, 18" x 20"	QTY
E	1008-1822-100	Spex Seat Base only, 18" x 22"	QTY
E	1008-2020-100	Spex Seat Base only, 20" x 20"	QTY

#	CODE	DESCRIPTION	QTY
A	1008-0204-100	Fixed Hardware Kit	QTY
B	1008-0206-100	Quick Release Hardware Kit	QTY
C	1008-0250-100	Pair of Cushion Retainers	QTY
D	1008-0251-100	Single Cushion Retainer	QTY

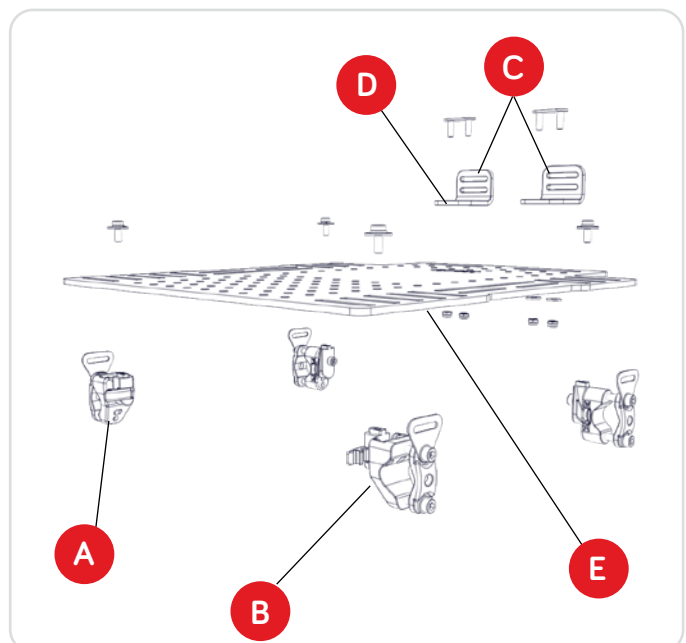

DUALTEC CUSHION COVERS

DUALTEC CUSHION (PICTURED)




A



B

BUILD CODE FOR DUALTEC CUSHION

1 1 X 1 - W W H H - X 0 0

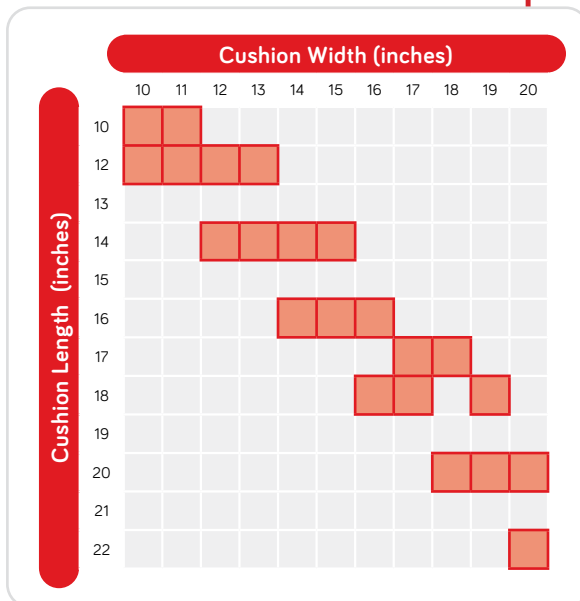

A

OUTER COVER

XX2X-XXXX-6XX

B

SPARE INCONTINENCE COVER

XX3X-XXXX-7XX

CUSTOM SIZING

ADD XXXX IN WWHH ON CODE BUILDER - RESTRICTIONS APPLY

XX
WW = ____

XX
HH = ____



SIZES AVAILABLE FOR DUALTEC CUSHIONS


VIGOUR STANDARD CUSHION COVERS

VIGOUR STANDARD CONTOUR CUSHION (PICTURED)



BUILD CODE FOR VIGOUR STANDARD CUSHION

1 1 X 3 - W W H H - X S C

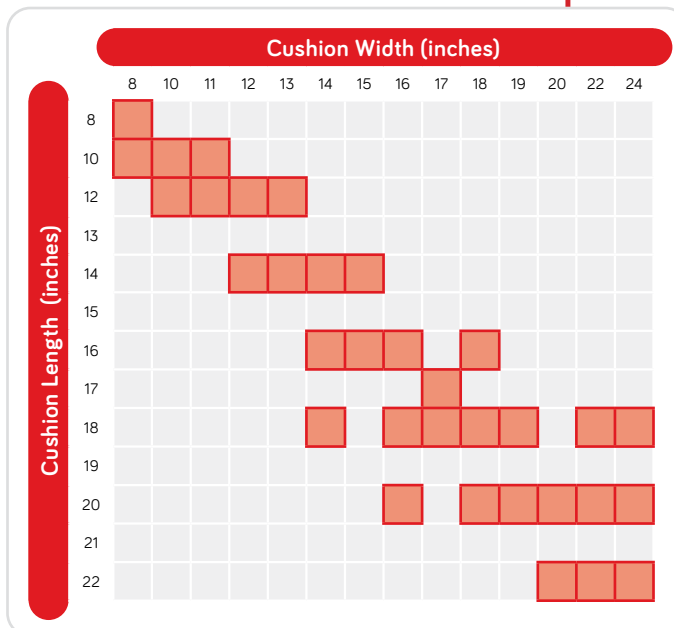

A

OUTER COVER

XX2X-XXXX-6XX

B

SPARE INCONTINENCE COVER

XX3X-XXXX-7XX

CUSTOM SIZING

ADD XXXX IN WWWW ON CODE BUILDER - RESTRICTIONS APPLY

XX

WW = _ _ _ _

XX

HH = _ _ _ _



SIZES AVAILABLE FOR VIGOUR
STANDARD CONTOUR CUSHION


VIGOUR HIGH CUSHION COVERS

VIGOUR HIGH CONTOUR CUSHION (PICTURED)



BUILD CODE FOR VIGOUR HIGH CUSHION

1 1 X 3 - W W H H - X H C

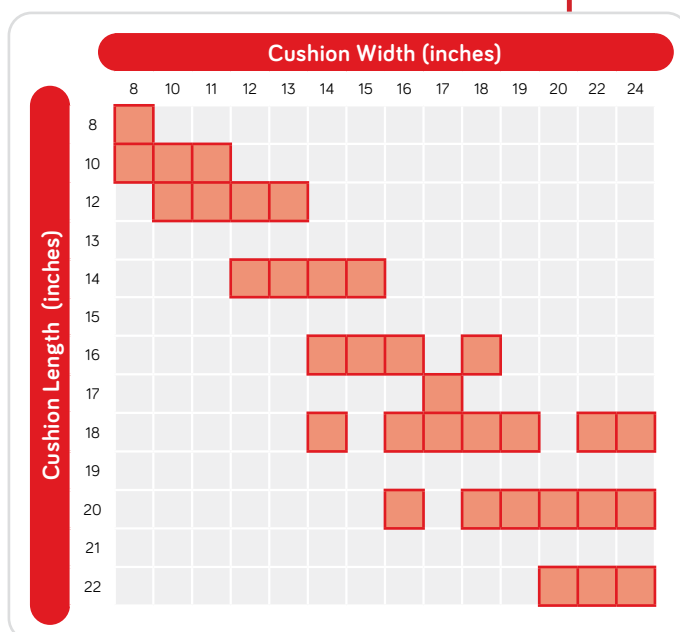

A OUTER COVER
XX2X-XXXX-6XX

B SPARE INCONTINENCE COVER
XX3X-XXXX-7XX

CUSTOM SIZING

ADD XXXX IN WWHH ON CODE BUILDER - RESTRICTIONS APPLY

XX
WW = ____

XX
HH = ____



SIZES AVAILABLE FOR VIGOUR HIGH CONTOUR CUSHION


SPEX STANDARD CONTOUR COVERS

SPEX STANDARD CONTOUR CUSHION (PICTURED)



BUILD CODE FOR SPEX STANDARD CONTOUR CUSHION

1 1 X 5 - W W H H - X S C

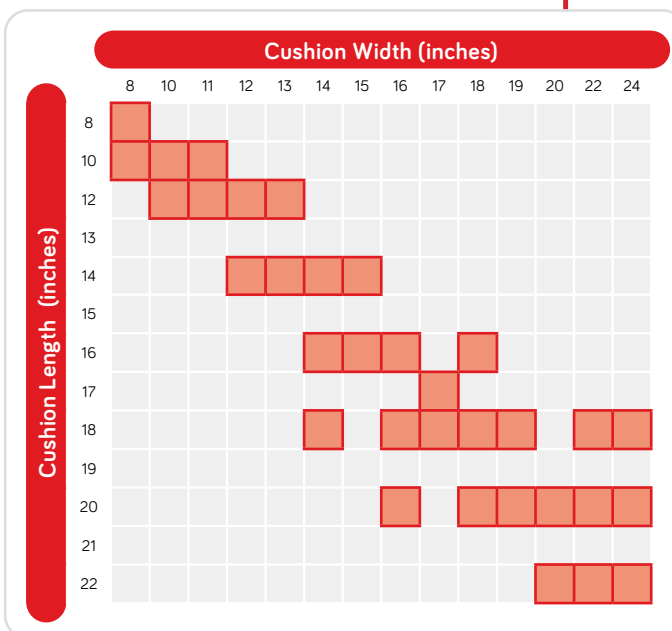

A OUTER COVER
XX2X-XXXX-6XX

B SPARE INCONTINENCE COVER
XX3X-XXXX-7XX

CUSTOM SIZING

ADD XXXX IN WWHH ON CODE BUILDER - RESTRICTIONS APPLY

XX
WW = ____ HH = ____

■ — SIZES AVAILABLE FOR SPEX STANDARD CONTOUR CUSHION

CUSTOMISATIONS & MODIFICATIONS

CUSHION ACCESSORIES

Additional Contouring Kit


CODE

1149-XXXX-060

CUSHION SIZE

XX X XX

CONTOURING KIT

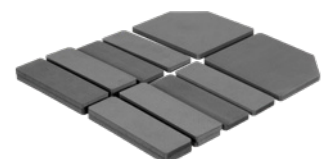

SPEX HIGH CONTOUR COVERS

SPEX HIGH CONTOUR CUSHION (PICTURED)




A



B

BUILD CODE FOR SPEX HIGH CONTOUR CUSHION

1 1 X 5 - W W H H - X H C

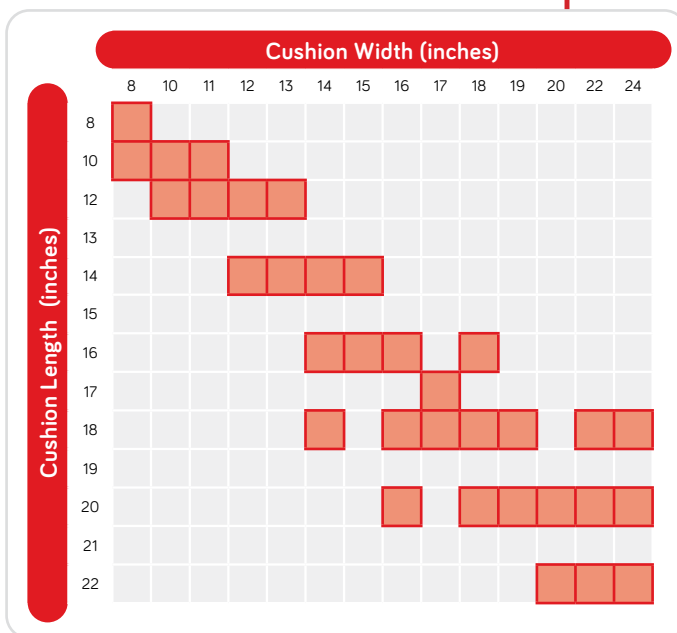

A

OUTER COVER

XX2X-XXXX-6XX

B

SPARE INCONTINENCE COVER

XX3X-XXXX-7XX

CUSTOM SIZING

ADD XXXX IN WWWW ON CODE BUILDER - RESTRICTIONS APPLY

XX
WW = ____

XX
HH = ____



**SIZES AVAILABLE FOR SPEX
HIGH CONTOUR CUSHION**

CUSTOMISATIONS & MODIFICATIONS

CUSHION ACCESSORIES

Additional Contouring Kit


CODE

1149-XXXX-060

CUSHION SIZE

" x "

CONTOURING KIT

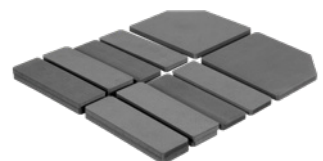

SPEX SUPERHIGH CONTOUR COVERS

SPEX SUPERHIGH CONTOUR CUSHION (PICTURED)




A



B

BUILD CODE FOR SPEX SUPERHIGH CONTOUR CUSHION

1 1 X 5 - W W H H - X S S

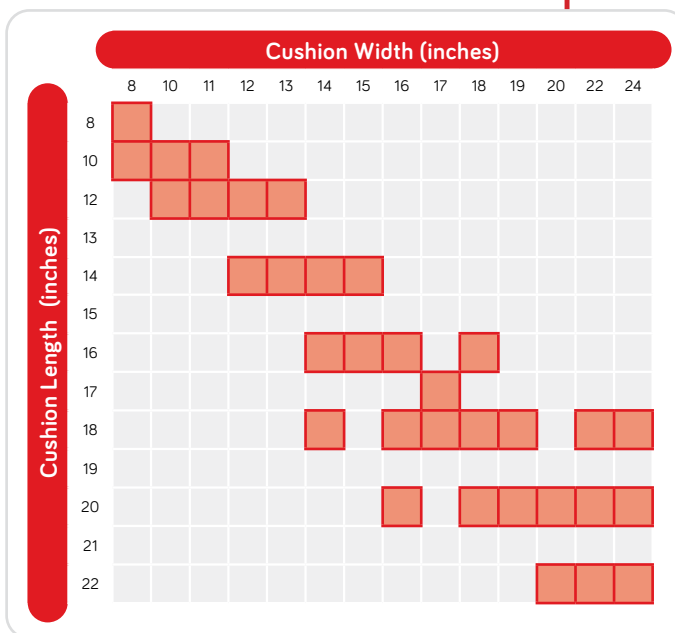

A

OUTER COVER

XX2X-XXXX-6XX

B

SPARE INCONTINENCE COVER

XX3X-XXXX-7XX

CUSTOM SIZING

ADD XXXX IN WWWW ON CODE BUILDER - RESTRICTIONS APPLY

XX
WW = _ _ _ _

XX
HH = _ _ _ _



**SIZES AVAILABLE FOR SPEX
SUPERHIGH CONTOUR
CUSHION**

CUSTOMISATIONS & MODIFICATIONS

CUSHION ACCESSORIES

Additional Contouring Kit


CODE

1149-XXXX-061

CUSHION SIZE

" x "

CONTOURING KIT

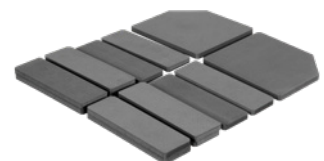

SPEX CONSTRUCTA COVERS

SPEX CONSTRUCTA CUSHION (PICTURED)



A



B



C



D




E



1




2

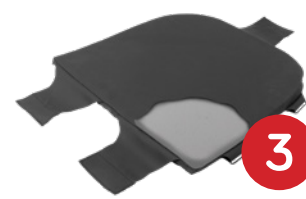

3

BUILD CODE FOR SPEX CONSTRUCTA CUSHION COVER

1 1 0 2 - W W D D - X X X - X X

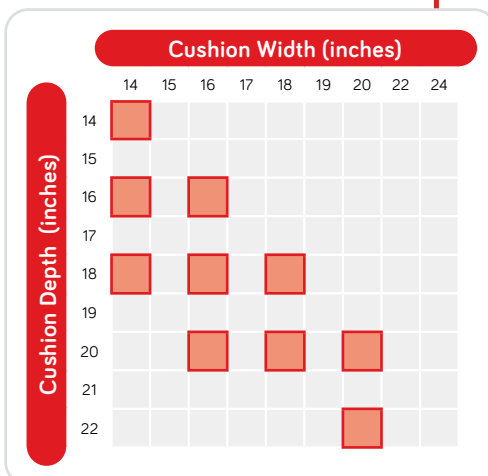

■ — SIZES AVAILABLE FOR
SPEX CONSTRUCTA
CUSHION

STYLE

A CONSTRUCTA SHAPE
XXXX-XXXX-**O**XX

B CONSTRUCTA FLEX
XXXX-XXXX-**F**XX

CONTOUR

C STANDARD OR HIGH
XXXX-XXXX-**X**OC

D SUPERHIGH
XXXX-XXXX-**X**SS

E ANY (INNER)
XXXX-XXXX-**X**00

COVER

1 OUTER COVER
XXXX-XXXX-XXX-**C**X

2 SPARE INNER COVER
XXXX-XXXX-X00-**I**X

3 SPARE OVERLAY
XXXX-XXXX-XXX-**F**I

WONDERSEAT CONSTRUCTA COVERS

WONDERSEAT CONSTRUCTA CUSHION (PICTURED)




A



B

BUILD CODE FOR SPEX CONSTRUCTA CUSHION COVER

1 1 0 2 — W W D D — W S C — X

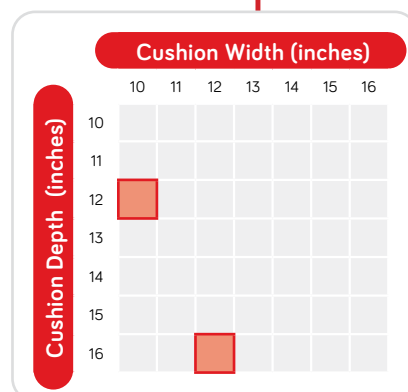

SIZES AVAILABLE FOR WONDERSEAT
CONSTRUCTA CUSHION

COVER

A

OUTER COVER

XXXX-XXXX-XXX-C

B

SPARE INCONTINENCE COVER

XXXX-XXXX-XXX-I

WONDERSEAT CONSTRUCTA SPARE PARTS

SPARE CUSHION FOAM UNIT



Size 0 (10" wide x 12" deep)

1102-1012-WSC-F

Size 1 (12" wide x 16" deep)


1102-1216-WSC-F

SPEX CONSTRUCTA SPARE PARTS

CONSTRUCTA SHAPE BASE

Specify cushion size - Width (WW) x Depth (DD) (inches)
Quantity per cushion specified in red text

Constructa Shape
Positioning Base
1 Set

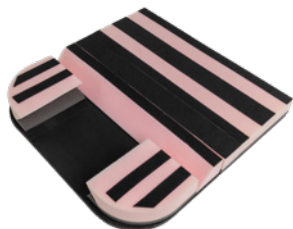

1102- WWDD -213

Strategic Positioning Kit
1 Set




1102- WW 00-000

Constructa Shape Thigh Block
1 Each



1102- WWDD -212

Constructa Shape 5cm Thigh Block
1 Each

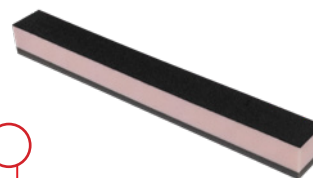

1102- WWDD -222

CONSTRUCTA FLEX BASE


Insert values as relevant - Width (WW) x Depth (DD) (inches)
Quantity per cushion specified in red text

Constructa Flex Rear
Underpanel Pivot
1 Each




1102- WW 00-311

Constructa Flex Thigh Block
2 Each

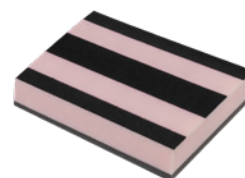


1102- WWDD -312

Constructa Flex Leg Wedge
2 Each

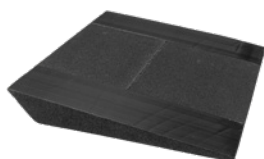


1102- WWDD -321

Constructa Flex 5cm Thigh Block
2 Each

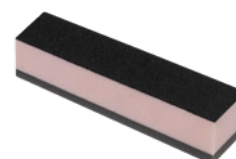

1102- WW 00-322

Constructa Flex Leg with
Positioning Blocks
2 Set

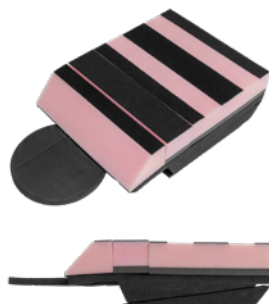

1102- WWDD -323

Pelvic Adjustment Kit
1 Set



1102- WW 00-F00

SPEX CONSTRUCTA SPARE PARTS CONTINUED


POSITIONING BLOCKS

Insert values as relevant - Width (WW) x Depth (DD) (inches)

Quantity per cushion specified in red text

Constructa Trochanter Blocks

1 Pair

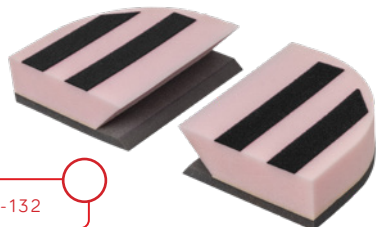

1102-**WW** 00-132

Constructa Ischial Insert

1 Each

IMPORTANT: 18" and 20" cushions use the same size ischial insert, please specify 20 in the code builder.



1102-**WW** 00-142

CONTOURS

Insert values as relevant - Width (WW) x Depth (DD) (inches)

Quantity per cushion specified in red text

Constructa Standard Lateral Thigh Contours

1 Pair



1102-00 **DD** -11SC

Constructa High Lateral Thigh Contours

1 Pair



1102-00 **DD** -11HC

Constructa Superhigh Lateral Thigh Contours

1 Pair



1102-00 **DD** -11SS

Constructa Standard Lateral Pelvic Contours

1 Pair



1102-0000-12SC

Constructa High Lateral Pelvic Contours

1 Pair



1102-0000-12HC


IMPORTANT:

14" and 16" cushions use the same size medial contour, please specify 16 in the code builder.

18" and 20" cushions use the same size medial contour, please specify 20 in the code builder.

Constructa Shape Standard Medial Contour

1 Each

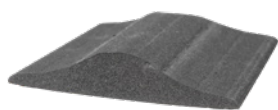

1102-**WW** 00-23SC

Constructa Shape High Medial Contour

1 Each



1102-**WW** 00-23HC

Constructa Shape Superhigh Medial Contour


1 Each



1102-**WW** 00-23SS

Constructa Flex Standard Medial Contours


1 Pair

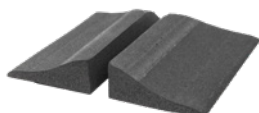

1102-**WW** 00-33SC

Constructa Flex High Medial Contours


1 Pair

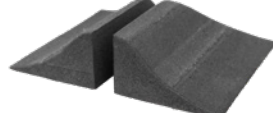

1102-**WW** 00-33HC

Constructa Flex Superhigh Medial Contours

1 Pair

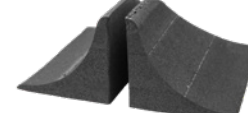

1102-**WW** 00-33SS

Flexy Contour

1 Each



1102-**WW** 00-00F

CUSHION COVER MODIFICATIONS

PRODUCT

OUTER COVER

SPARE INCONTINENCE COVER

DUALTEC	1121-2949-600	QTY	1131-2949-700	QTY
VIGOUR	1123-2949-600	QTY	1133-2949-700	QTY
SPEX	1125-2949-600	QTY	1135-2949-700	QTY
FLEX	1126-2949-600	QTY	1136-2949-700	QTY

CUSTOMISATIONS

Describe existing cushion modifications when above covers are ordered. Original order number for existing custom cushion must be specified when ordering replacement covers


CUSTOMISATION

#

MEASUREMENT

CUSHION WIDTH	A	MM
CUSHION LENGTH	B	MM
BASE HEIGHT	C	MM
CONTOUR WIDTH	D	MM
THIGH LATERALS FRONT LENGTH	E	MM
THIGH LATERALS REAR LENGTH	F	MM
MEDIAL THIGH HEIGHT	G	MM
THIGH LATERALS REAR HEIGHT	H	MM
MEDIAL THIGH FRONT WIDTH	I	MM
MEDIAL THIGH BASE LENGTH	J	MM
FRONT CUSHION HEIGHT	K	MM
REAR CUSHION HEIGHT	L	MM

CUSTOMISATIONS GUIDE

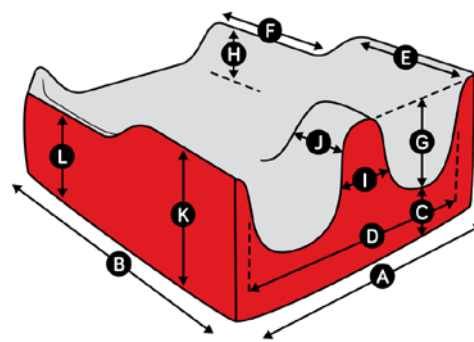

USER CONTACT SURFACE (TOP OF CUSHION)

DUAL FABRIC
(default)

SPEXTEX WATERPROOF
PU (1111-0399-209)

SPEXTEX KNITTED
FABRIC (1111-0399-219)

MERINO
(1111-0399-330)

BORDER FABRIC (SIDE PANELS)

DUAL FABRIC
(1111-0491-286)


SPEXTEX WATERPROOF
PU (1111-0491-209)

SPEXTEX KNITTED
FABRIC (1111-0491-219)


AIR MESH
(default)

CUSTOM FABRIC CHOICES ON COVER


DUAL FABRIC

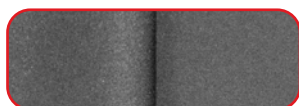

SPEXTEX WATERPROOF PU

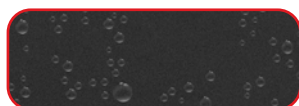

SPEXTEX KNITTED FABRIC

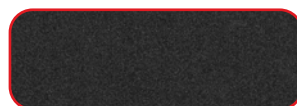

MERINO



COLOURS AVAILABLE

Black
(default)

Chilli
1111-0491-022

Mango
1111-0491-030

Calypso
1111-0491-028

Royal
1111-0491-077

Bumblebee
1111-0491-044


Ocean
1111-0491-060

Leaf
1111-0491-054

Granite
1111-0491-090


VIGOUR

VIGOUR BACK SUPPORT - EXPLODED

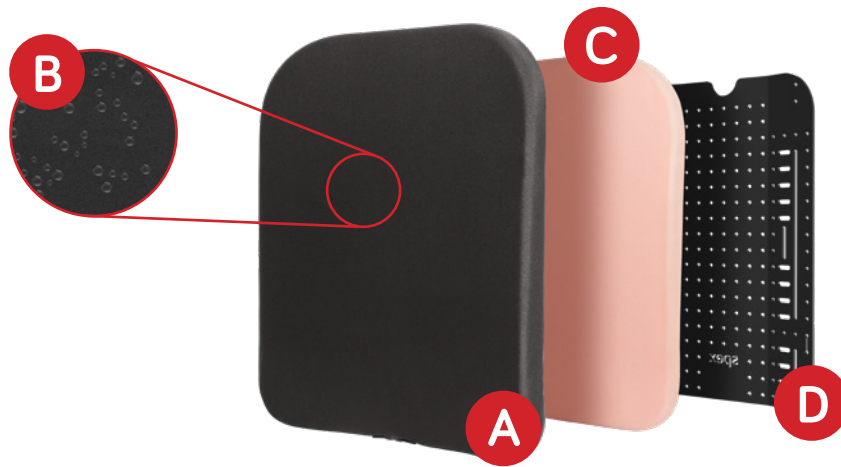

BUILD CODE FOR VIGOUR BACK SUPPORT

1 2 5 2 - W W H H - 0 0 H - X

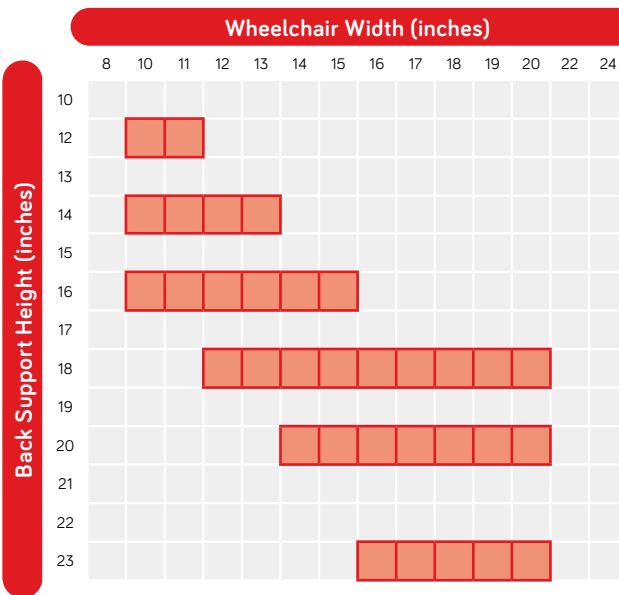

A OUTER COVER
XXXX-XXXX-XXX-C

B INCONTINENCE COVER
XXXX-XXXX-XXX-I

C FOAM INNER
XXXX-XXXX-XXX-F

D SHELL
XXXX-XXXX-XXX-S

— SIZES AVAILABLE FOR VIGOUR BACK SUPPORT

CUSTOMISATIONS & MODIFICATIONS

T-SHAPE MODIFICATIONS

T-shape upgrade - foam

CODE

1209-2561-OTS-F

QTY

T-shape upgrade - cover

1209-2561-OTS-C

QTY

FOAM INNER MODIFICATIONS

Air-Flow Upgrade

CODE

1209-2351-300

QTY

T-SHAPE MODIFICATIONS FOR SHELL

T-shape upgrade - shell: 10 - 15"

1209-2110-OTS-S


QTY

T-shape upgrade - shell: 16"+

1209-2116-OTS-S


QTY

T-SHAPE

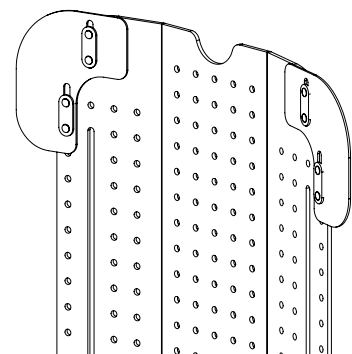


VIGOUR MID

VIGOUR MID BACK SUPPORT - EXPLODED

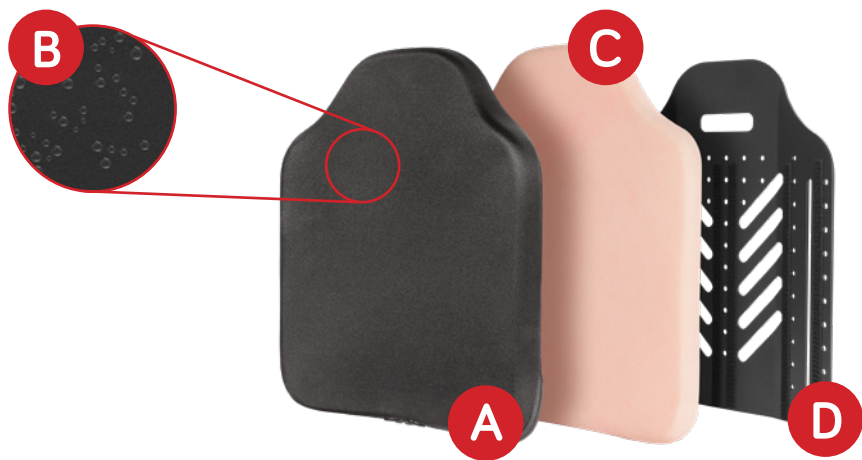

BUILD CODE FOR VIGOUR MID BACK SUPPORT

1 2 5 2 - W W H H - 0 0 M - X

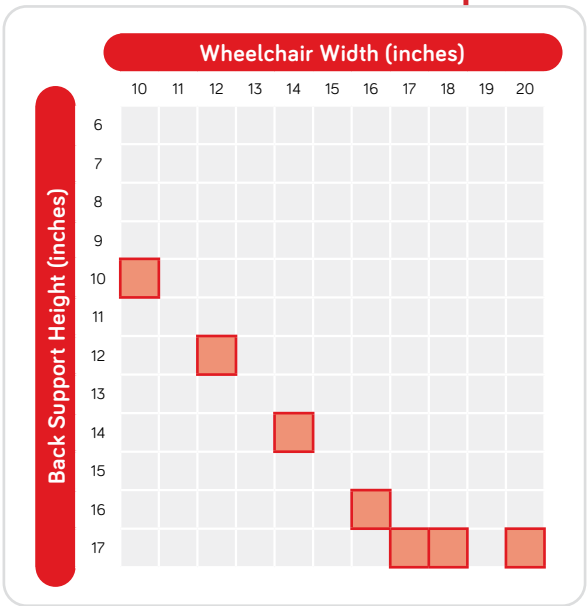


- A** OUTER COVER
XXXX-XXXX-XXX-C
- B** INCONTINENCE COVER
XXXX-XXXX-XXX-I
- C** FOAM INNER
XXXX-XXXX-XXX-F
- D** SHELL
XXXX-XXXX-XXX-S

Available sizes for Vigour Mid Back Support


COMFI

COMFI BACK SUPPORT - EXPLODED

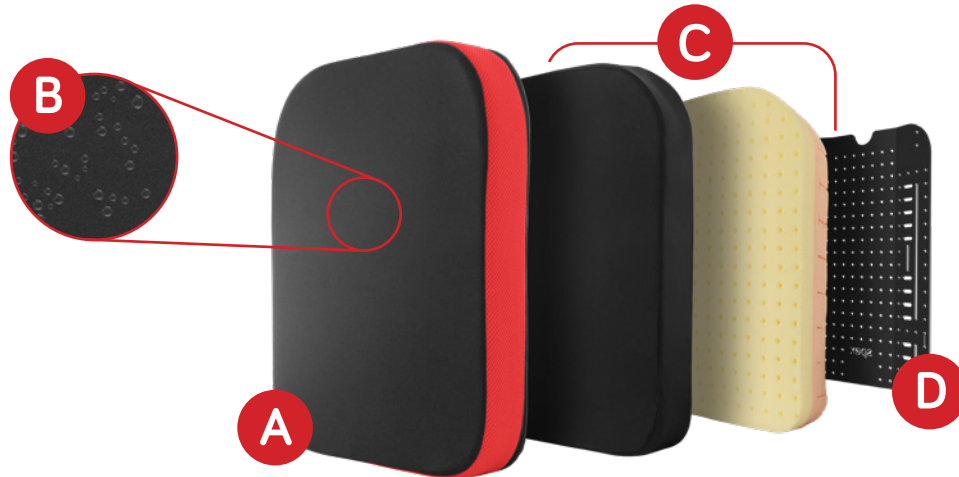

BUILD CODE FOR COMFI BACK SUPPORT

1 2 5 3 - W W H H - 0 0 H - X

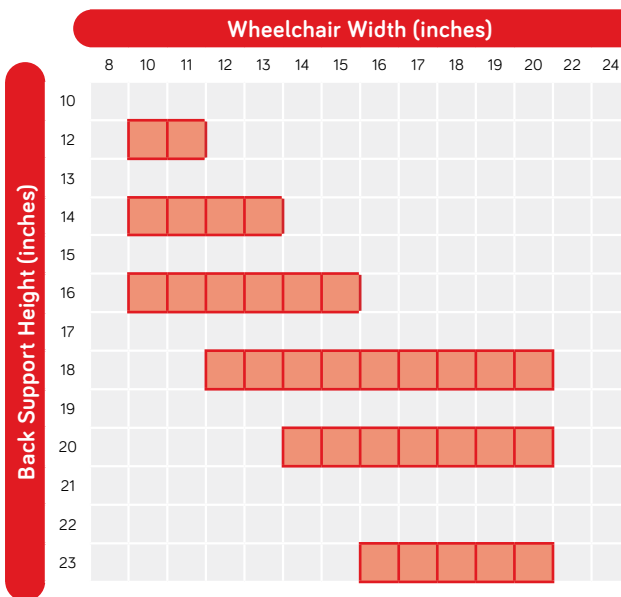

A OUTER COVER
XXXX-XXXX-XXX-C

B INCONTINENCE COVER
XXXX-XXXX-XXX-I

C FOAM INNER
XXXX-XXXX-XXX-F

D SHELL
XXXX-XXXX-XXX-S

— SIZES AVAILABLE FOR COMFI BACK SUPPORT

CUSTOMISATIONS & MODIFICATIONS

SURFACE STYLE MODIFICATIONS

Surface-Style upgrade - foam

CODE

1209-2552-2TS-F

QTY

Surface-Style upgrade - cover

1209-2551-2TS-C

QTY

SURFACE STYLE MODIFICATIONS FOR SHELL

Surface-Style upgrade - shell: 10-15"

1209-2110-2TS-S

QTY

Surface-Style upgrade - shell: 16"+

1209-2116-2TS-S

QTY

Spex Positioning Kit (Pack 10 Cubes)


1209-2712-300

x

FOAM CUBES




SURFACE SHAPE

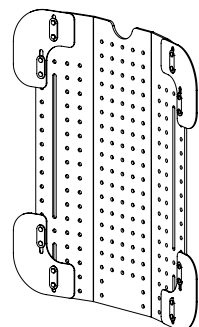


MANTA

MANTA BACK SUPPORT - EXPLODED

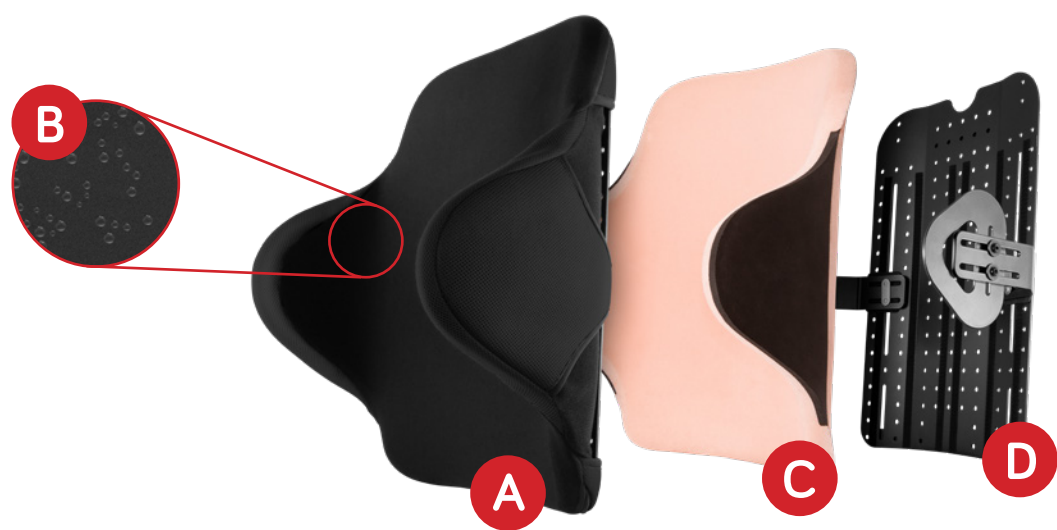

BUILD CODE FOR MANTA BACK SUPPORT

1254 - WWHH - 000 - X

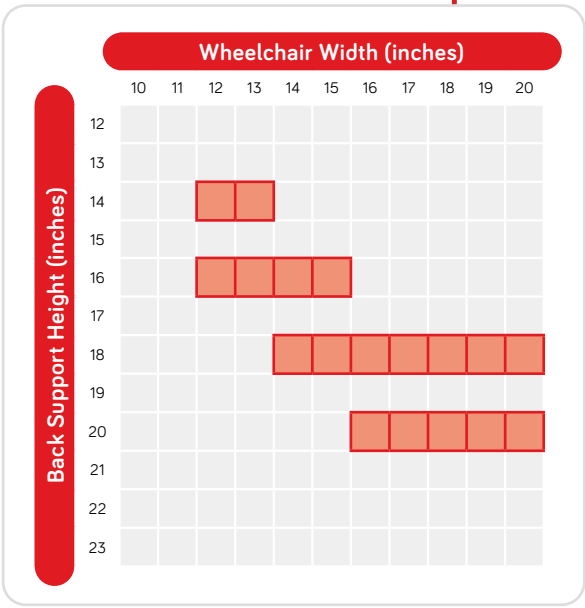

- A

OUTER COVER
XXXX-XXXX-XXX-C
- B

INCONTINENCE COVER
XXXX-XXXX-XXX-I
- C

FOAM INNER
XXXX-XXXX-XXX-F
- D


SHELL
XXXX-XXXX-XXX-S

 — SIZES AVAILABLE FOR MANTA BACK SUPPORT


CUSTOMISATIONS & MODIFICATIONS

WING BRACKET ASSEMBLIES

Size 0 - for 10-12" Back Support	1267-2700-100	EA
Size 1 - for 13-15" Back Support	1267-2710-100	EA
Size 2 - for 16"+ Back Support	1267-2720-100	EA

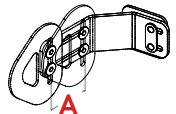

A

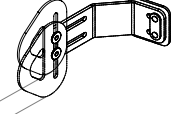


B

A	50mm - 10-13" Width	78mm - 14"+ Width
B	30mm - 10-13" Width	40mm - 14"+ Width


SPEX CLASSIC

SPEX CLASSIC BACK SUPPORT - EXPLODED

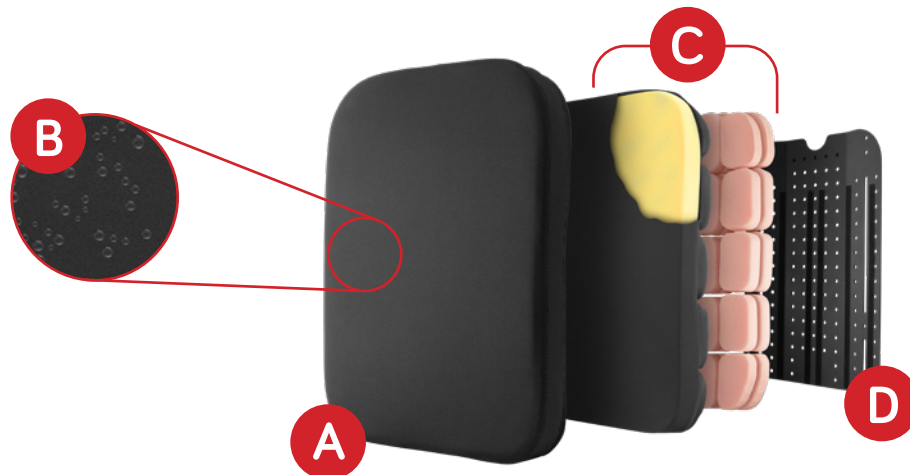

BUILD CODE FOR SPEX CLASSIC BACK SUPPORT

1 2 5 5 - W W H H - 0 S S - X

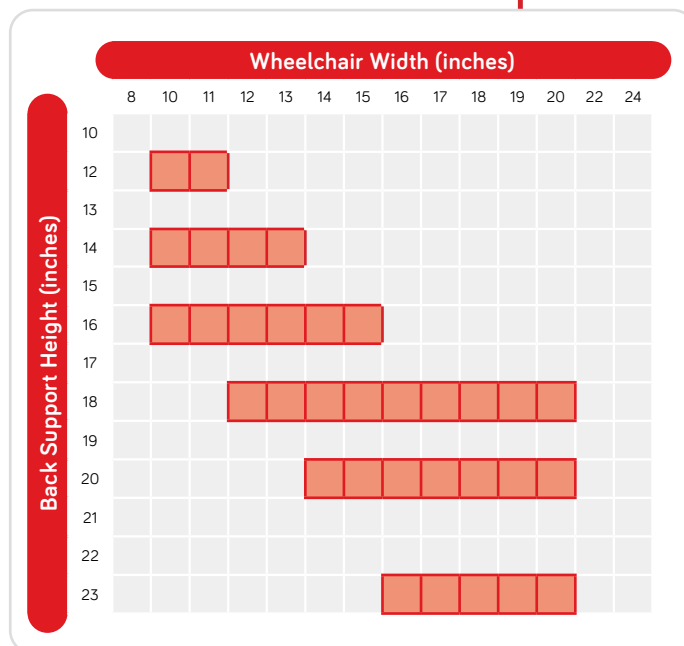

A OUTER COVER
XXXX-XXXX-XXX-C

B INCONTINENCE COVER
XXXX-XXXX-XXX-I

C FOAM INNER
XXXX-XXXX-XXX-F

D SHELL
XXXX-XXXX-XXX-S

— SIZES AVAILABLE FOR SPEX CLASSIC

CUSTOMISATIONS & MODIFICATIONS

SURFACE STYLE MODIFICATIONS

Surface-Style upgrade - foam

1209-2552-2TS-F

QTY

Surface-Style upgrade - cover

1209-2551-2TS-C

QTY

SURFACE STYLE MODIFICATIONS FOR SHELL

Surface-Style upgrade - shell: 10-15"

1209-2110-2TS-S

QTY

Surface-Style upgrade - shell: 16"+

1209-2116-2TS-S

QTY

Spex Positioning Kit (Pack 10 Cubes)

1209-2712-300


QTY

x

FOAM CUBES




SURFACE SHAPE

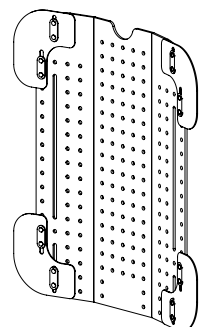


SPEX CLASSIC T-SHAPE

CLASSIC T-SHAPE BACK SUPPORT - EXPLODED

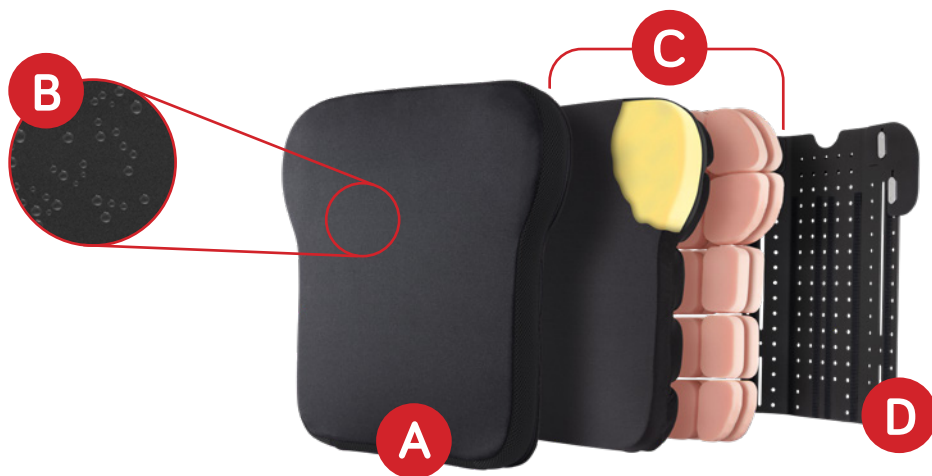

BUILD CODE FOR CLASSIC T-SHAPE BACK SUPPORT

1 2 5 5 - W W H H - O T S - X

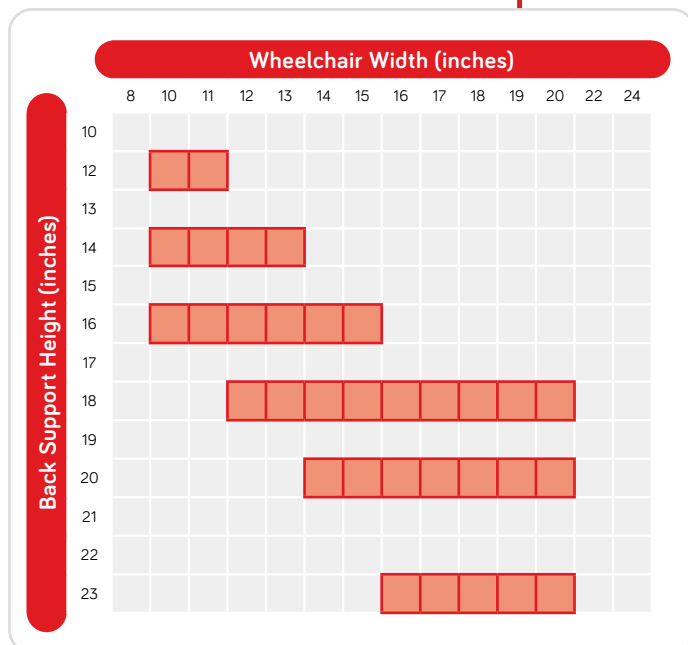**A****OUTER COVER**

XXXX-XXXX-XXX-C

B**INCONTINENCE COVER**

XXXX-XXXX-XXX-I

C**FOAM INNER**

XXXX-XXXX-XXX-F

D**SHELL**

XXXX-XXXX-XXX-S



**SIZES AVAILABLE FOR
CLASSIC T-SHAPE
BACK SUPPORT**

CUSTOMISATIONS & MODIFICATIONS

SURFACE STYLE MODIFICATIONS

Surface-Style upgrade - foam

1209-2552-2TS-F

QTY

Surface-Style upgrade - cover

1209-2551-2TS-C

QTY

SURFACE STYLE MODIFICATIONS FOR SHELL

Surface-Style upgrade - shell: 10-15"

1209-2110-2TS-S

QTY

Surface-Style upgrade - shell: 16"+

1209-2116-2TS-S

QTY

Spex Positioning Kit (Pack 10 Cubes)


1209-2712-300

x

FOAM CUBES




SURFACE SHAPE

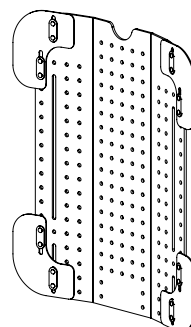


SPEX SUPERSHAPE

SUPERSHAPE BACK SUPPORT - EXPLODED

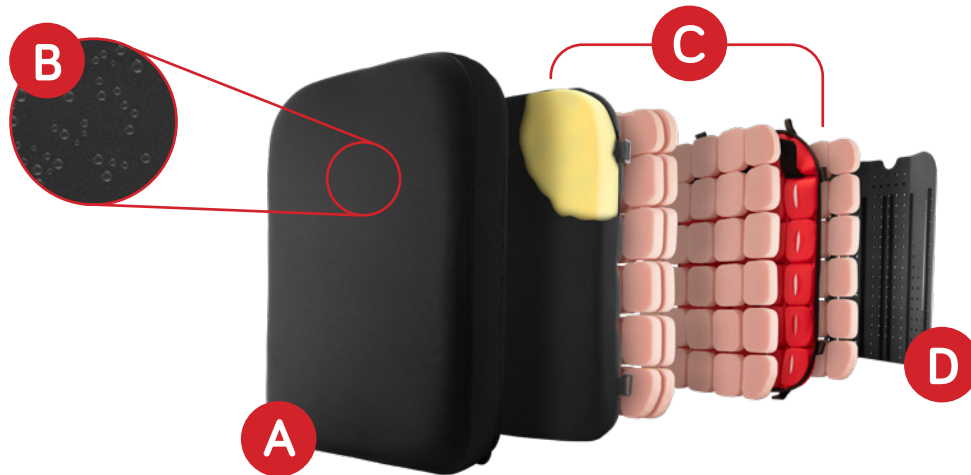

BUILD CODE FOR SUPERSHAPE BACK SUPPORT

1 2 5 6 - W W H H - 0 S S - X

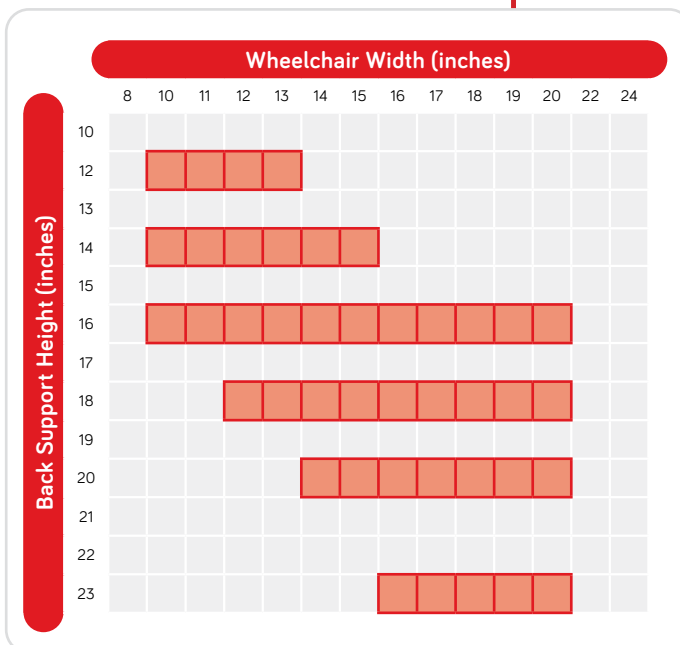

A OUTER COVER
XXXX-XXXX-XXX-C

B INCONTINENCE COVER
XXXX-XXXX-XXX-I

C FOAM INNER
XXXX-XXXX-XXX-F

D SHELL
XXXX-XXXX-XXX-S

— SIZES AVAILABLE FOR SUPERSHAPE BACK SUPPORT

CUSTOMISATIONS & MODIFICATIONS

T-SHAPE MODIFICATIONS

T-shape upgrade - foam

CODE

1209-2561-OTS-F

QTY

T-shape upgrade - cover

1209-2561-OTS-C

QTY

T-SHAPE MODIFICATIONS FOR SHELL

CODE

T-shape upgrade - shell: 10 - 15"

1209-2110-OTS-S

QTY

T-shape upgrade - shell: 16"+

1209-2116-OTS-S

QTY


Supershape Positioning Kit (20 Pink Cubes + 20 CCF Cubes)

1209-2715-300

FOAM CUBES



T-SHAPE

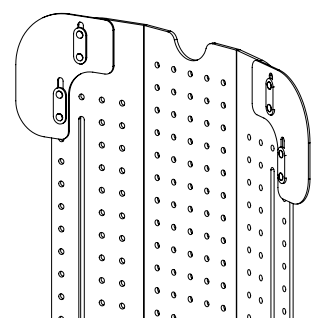


GROWTH-ADJUST

GROWTH-ADJUST BACK SUPPORT - EXPLODED



BUILD CODE FOR GROWTH-ADJUST BACK SUPPORT

1 2 5 2 - W W H H - O G A - X

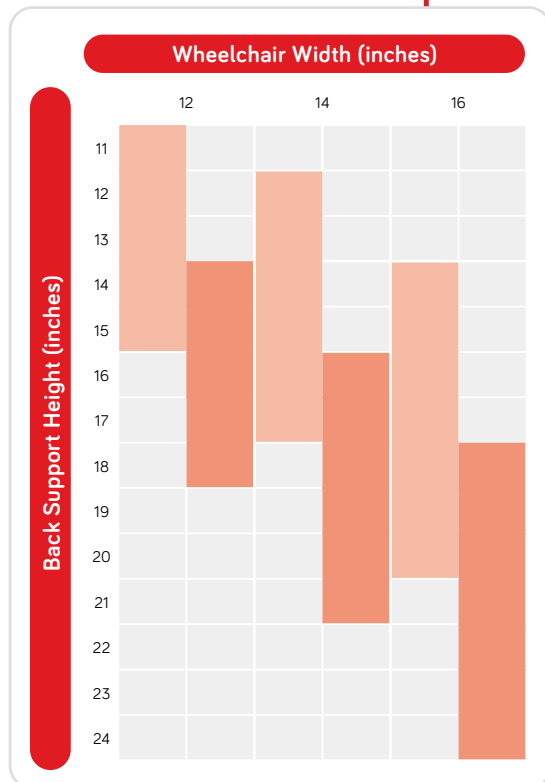

- A** OUTER COVER
XXXX-XXXX-XXX-C
- B** WIPE-DOWN COVER
XXXX-XXXX-XXX-W
- C** FOAM INNER
XXXX-XXXX-XXX-F
- D** SHELL
XXXX-XXXX-XXX-S


- Sizes available for Growth-Adjust Back Support - Growth Range 1
- Sizes available for Growth-Adjust Back Support - Growth Range 2

IMPORTANT: Enter the lowest height in growth range to build code.

e.g. a height range of 11-15" is **1252-WW11-OGA-X**


GROWTH-ADJUST SPEX CLASSIC / T SHAPE

GROWTH-ADJUST SPEX CLASSIC BACK SUPPORT - EXPLODED

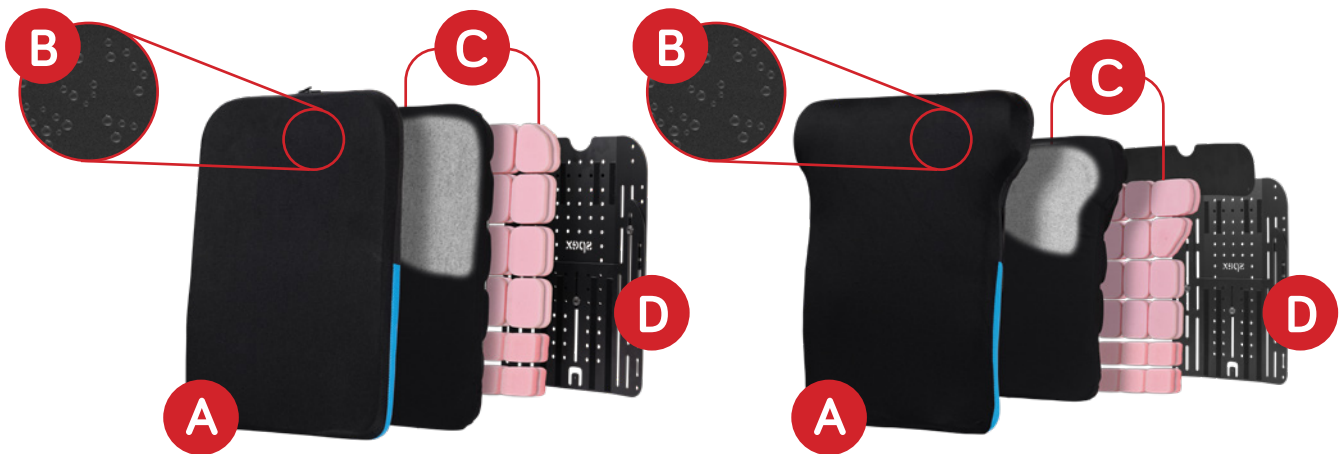

BUILD CODE FOR GROWTH-ADJUST SPEX CLASSIC BACK SUPPORT

1 2 5 5 - W W H H - X G A - X

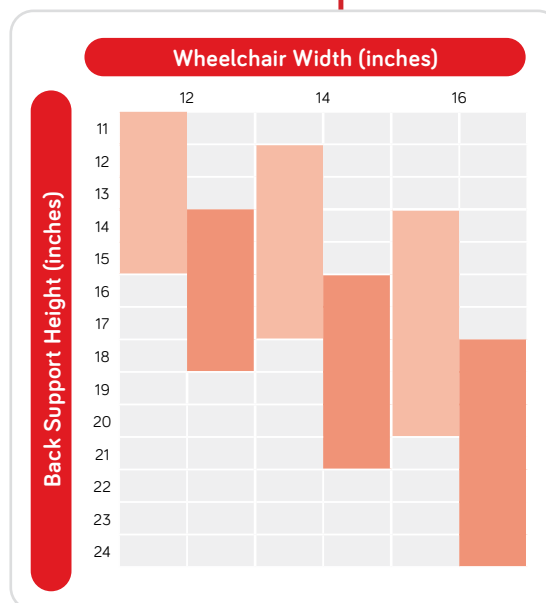

A OUTER COVER
XXXX-XXXX-XXX-C

B WIPE-DOWN COVER
XXXX-XXXX-XXX-W

C FOAM INNER
XXXX-XXXX-XXX-F

D SHELL
XXXX-XXXX-XXX-S

O STANDARD SHAPE
XXXX-XXXX-0XX-X

T T SHAPE
XXXX-XXXX-TXX-X

IMPORTANT: Enter the lowest height in growth range to build code.

e.g. a height range of 12-17" is **1255-WW12-0GA-X**


— Sizes available for Growth-Adjust Back Support - Growth Range 1

— Sizes available for Growth-Adjust Back Support - Growth Range 2

BACK SUPPORT CUSTOMISATIONS

CUSTOM FABRIC CHOICES ON COVER

SPEXTEX WATERPROOF PU

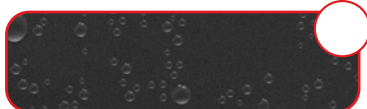

SPEXTEX WATERPROOF PU
(1208-2949-209)

SPEXTEX KNITTED FABRIC



SPEXTEX KNITTED FABRIC
(1208-2949-219)

MERINO



AIR MESH
(default)

COLOURS AVAILABLE

Black
(default)

Chilli
1209-2491-022

Mango
1209-2491-030

Calypso
1209-2491-028

Royal
1209-2491-077

Bumblebee
1209-2491-044

Ocean
1209-2491-060

Leaf
1209-2491-054

Granite
1209-2491-090

CUSTOM OPTIONS

CUSTOMISATION

CUSTOM THICKNESS (UNAVAILABLE FOR COMFI OR AIR-FLOW)

½" EXTRA SOFT PRESSURE-RELIEVING FOAM OVERLAY IN
FRONT BACK SUPPORT SECTION

30MM SOFT PRESSURE-SENSITIVE FOAM IN FRONT SECTION OF
BACK SUPPORT INNER

½" GEL-FOAM IN SPINAL REGION

ENVELOPE COVER ONLY FOR SUPRACOR® INSERT

ENVELOPE COVER WITH SUPRACOR® INSERT

CODE

1208-2942-300

1208-2945-300

1209-2346-300

1208-2947-300

1208-2951-300

1208-2952-300

MEASUREMENTS

_____ mm

CONTOURING WEDGES

CUSTOMISATION

BACK SUPPORT POSTERIOR PELVIC SUPPORT AND LATERAL
STABILITY WEDGES

ASYMMETRIC CONTOUR WEDGE KIT
(CONTAINS WEDGES BELOW)

ASYMMETRIC DEEP CONTOUR WEDGE
(20 HIGH x 20 WIDE x 4 DEEP)

ASYMMETRIC BI-ANGULAR CONTOUR WEDGE
(20 HIGH x 10 WIDE x 4 DEEP)

ASYMMETRIC NARROW CONTOUR WEDGE
(20 HIGH x 10 WIDE x 4 DEEP)

CODE

1209-2713-300

1209-2720-300

1209-2721-300

1209-2723-300


1209-2722-300

MEASUREMENT/IMAGE



QUAD MOUNT HARDWARE

QUAD MOUNT BACK SUPPORT HARDWARE

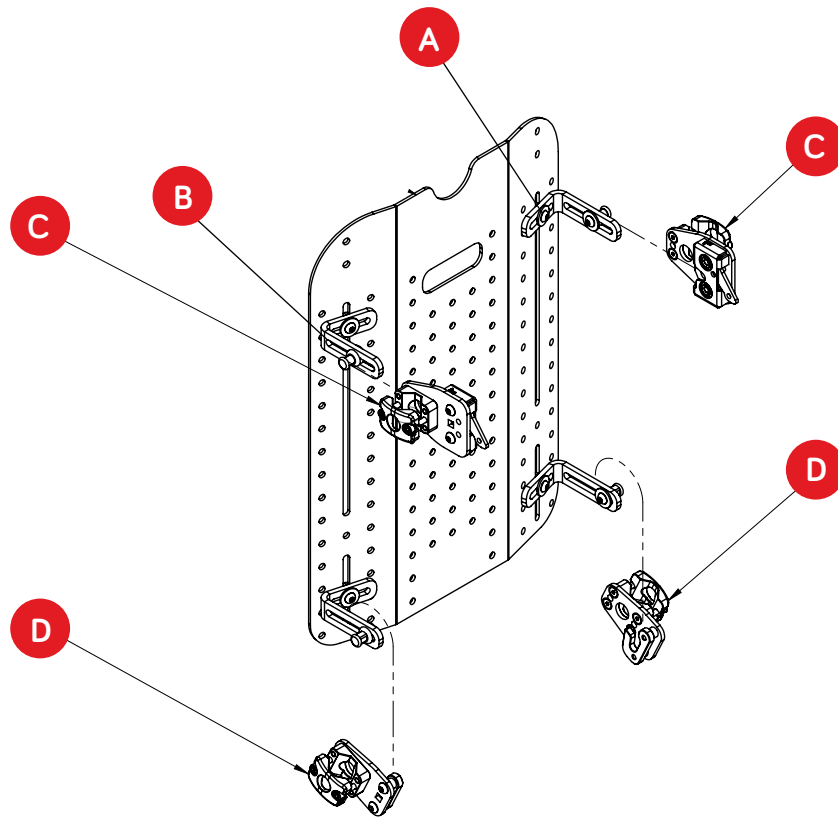


ORDERING CODES FOR QUAD MOUNT

DESCRIPTION	CODE	#	PER	
T-SLOT ASSEMBLY SET	A-00866	A	4	QTY
4 POINT MOUNTING BRACKET		B	4	QTY
QUAD MOUNT LOWER		C	2	QTY
QUAD MOUNT UPPER		D	2	QTY
QUAD MOUNT COMPLETE SET	1259-2430-100	A + B + C + D	SET	QTY
QUAD MOUNT FOR SECOND CHAIR	1259-2420-100	C + D	SET	QTY
QUAD MOUNT BRACKETS ONLY	1259-2408-100	A + B	SET 4	QTY
QUAD MOUNT ADDITIONAL 2 POINTS OF ATTACHMENT (6 TOTAL) FOR WHEN BACK HEIGHT IS OVER 25" OR FOR EXTRA SECURE ATTACHMENT	1259-2440-100	Top 1/2 B + D	SET 2	QTY
QUAD MOUNT EXTENDED UPGRADE: INCREASE BACK ANGLE BY $\approx 10^\circ$	1259-2450-100		1 PR	QTY
QUAD MOUNT EXTENDED UPGRADE: INCREASE SEAT DEPTH	1259-2452-100		2 PRS	QTY

QUAD MOUNT HARDWARE (UPPER)


QUAD MOUNT LEFT UPPER

1

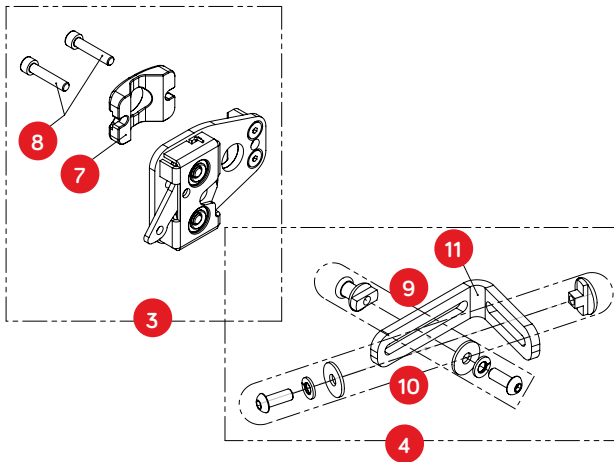

QUAD MOUNT RIGHT UPPER

2

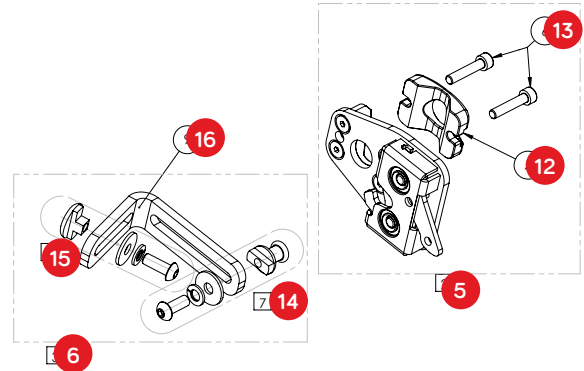

ORDERING CODES FOR QUAD MOUNT

WHOLE ASSEMBLIES

QUAD MOUNT LEFT UPPER
QUAD MOUNT RIGHT UPPER

CODE

-
-

#

1
2

PER

1	QTY
1	QTY

SUB ASSEMBLIES

LEFT QUICK RELEASE
QUAD MOUNT BRACKET
RIGHT QUICK RELEASE
QUAD MOUNT BRACKET

CODE

A-00085
A-00524
A-00084
A-00524

#

3
4
5
6

PER

1	QTY
1	QTY
1	QTY
1	QTY

COMMON PARTS/SETS

30 CLAMP
M5 x 25 CAP SCREW
LOCKING PEG SET
6MM T NUT SET
QUAD MOUNT BRACKET SINGLE
30 CLAMP
M5 x 25 CAP SCREW
LOCKING PEG SET
6MM T NUT SET
QUAD MOUNT BRACKET SINGLE

CODE

P-00292
0052-4CS0-M5X25BK
A-00892
A-00866
P-00388
P-00292
0052-4CS0-M5X25BK
A-00892
A-00866
P-00388

#

7
8
9
10
11
12
13
14
15
16


PER

1	QTY
2	QTY
1	QTY
1	QTY
1	QTY
1	QTY
2	QTY
1	QTY
1	QTY
1	QTY

QUAD MOUNT HARDWARE (LOWER)


QUAD MOUNT LEFT LOWER

1

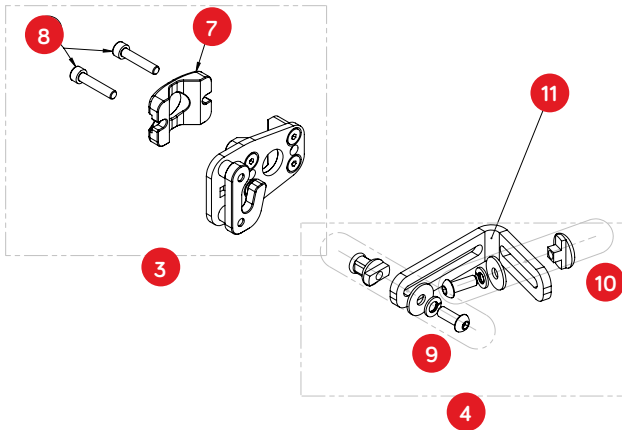

QUAD MOUNT RIGHT LOWER

2

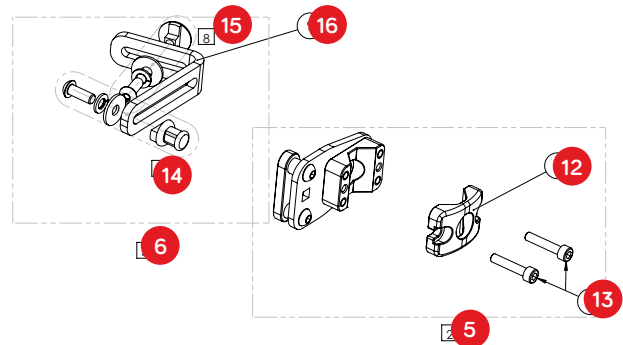

ORDERING CODES FOR QUAD MOUNT

WHOLE ASSEMBLIES

QUAD MOUNT LEFT LOWER
QUAD MOUNT RIGHT LOWER

CODE

-
-

#

1
2

PER

1	QTY
1	QTY

SUB ASSEMBLIES

LEFT QUAD HOOK
QUAD MOUNT LOWER
RIGHT QUAD HOOK
QUAD MOUNT LOWER

CODE

A-00088
A-00525
A-00089
A-00525

#

3
4
5
6

PER

1	QTY
1	QTY
1	QTY
1	QTY

COMMON PARTS/SETS

30 CLAMP
M5 x 25 CAP SCREW
ANTI TILT PEG SET
6MM T NUT SET
QUAD MOUNT BRACKET SINGLE
30 CLAMP
M5 x 25 CAP SCREW
ANTI TILT PEG SET
6MM T NUT SET
QUAD MOUNT BRACKET SINGLE

CODE

P-00292
0052-4CS0-M5X25BK
A-00892
A-00866
P-00388
P-00292
0052-4CS0-M5X25BK
A-00892
A-00866
P-00388

#


7
8
9
10
11
12
13
14
15
16

PER


1	QTY
2	QTY
1	QTY
1	QTY
1	QTY
1	QTY
2	QTY
1	QTY
1	QTY
1	QTY

HEAVY DUTY QUAD MOUNT

HEAVY DUTY QUAD MOUNT OVERVIEW

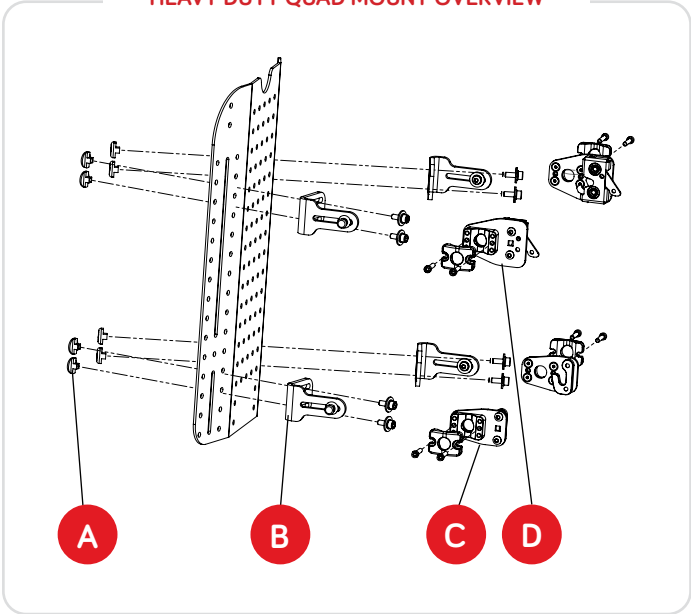

SUB - ASSEMBLIES

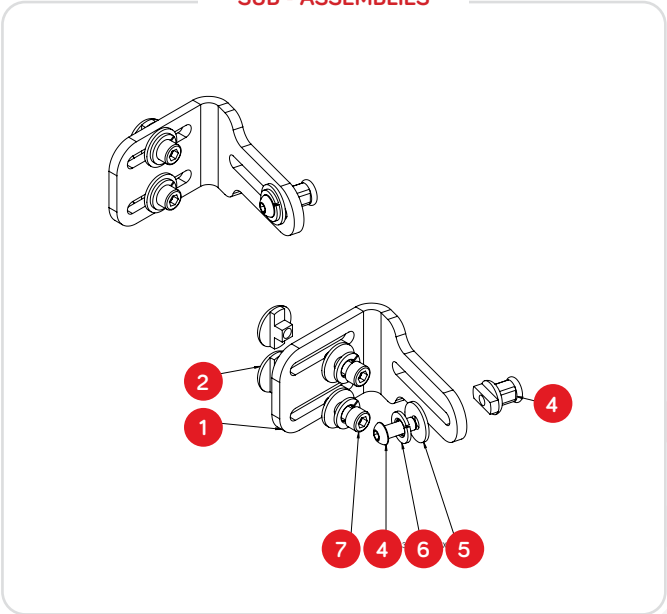

ORDERING CODES FOR QUAD MOUNT

SUB ASSEMBLIES

HEAVY DUTY QUAD MOUNT COMPLETE SET A + B + C + D
HEAVY DUTY QUAD MOUNT BRACKETS ONLY A + B

CODE

1259-2435-100
1259-2438-100

PER

SET	QTY
SET	QTY

COMMON PARTS/SETS

4 POINT BRACKET DOUBLE
SLOT T-NUT
TILT LOCKING PEG
BUTTON CAP SCREW M6 x 16 S/S (BLACK FEROPHOS COAT)
WASHER FLAT M6 x 19 x 2mm S/S BLACK COATED
WASHER SPRING M6 S/S BLACK COATED

CODE

P-02552
P-00391
P-00481
0052-3BCS-M6X16BK
0057-0LFT-M6X19X2BK
0057-0SPR-SSM6BK

#


1
2
3
4
5
6

PER

1	QTY
2	QTY
1	QTY
3	QTY
3	QTY
3	QTY

DUAL MOUNT HARDWARE

DUAL MOUNT BACK SUPPORT HARDWARE

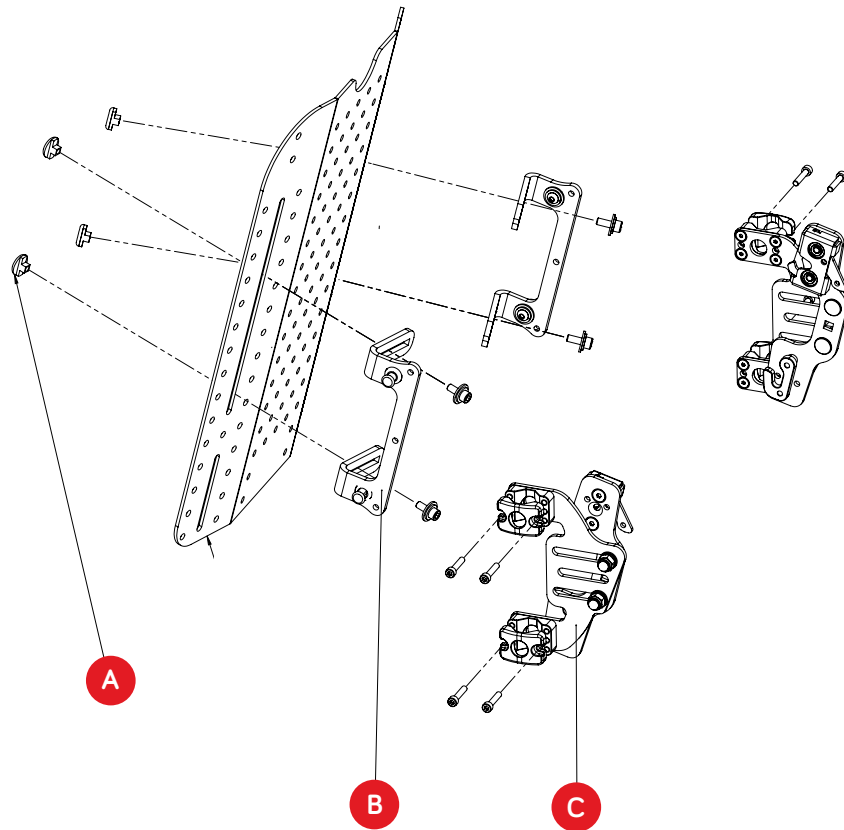


ORDERING CODES FOR QUAD MOUNT

DESCRIPTION	CODE	#	PER	
T-SLOT ASSEMBLY SET	A-00866	A	4	QTY
2 POINT MOUNTING BRACKET		B	2	QTY
DUAL MOUNT HARDWARE		C	2	QTY
DUAL MOUNT COMPLETE SET A + C + D	1259-2230-100		SET	QTY
DUAL MOUNT FOR SECOND CHAIR D	1259-2220-100		SET	QTY

DUAL MOUNT HARDWARE (CONTINUED)


DUAL MOUNT LEFT

1

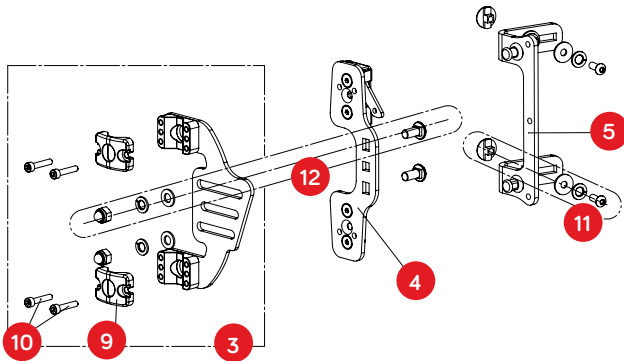

DUAL MOUNT RIGHT

2

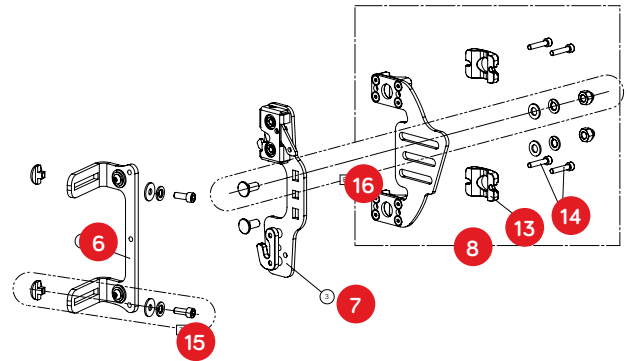

ORDERING CODES FOR DUAL MOUNT

WHOLE ASSEMBLIES

	CODE	#	PER	QTY
DUAL MOUNT LEFT	A-00889	1	1	QTY
DUAL MOUNT RIGHT	A-00890	2	1	QTY

SUB ASSEMBLIES


	CODE	#	PER	QTY
CLAMP BRACKET	A-00115	3	1	QTY
CATCH BRACKET	A-00114	4	1	QTY
PIN BRACKET	A-00117	5	1	QTY
PIN BRACKET	A-00115	6	1	QTY
CATCH BRACKET	A-00113	7	1	QTY
CLAMP BRACKET	A-00112	8	1	QTY

COMMON PARTS/SETS

	CODE	#	PER	QTY
OUTER CLAMP SMALL	P-00292	9	1	QTY
M5 x 25 CAP SCREW	0052-4CS0-M5X25BK	10	2	QTY
6MM T NUT SET	A-00866	11	1	QTY
M8 SQUARE NUT SET	A-00867	12	1	QTY
OUTER CLAMP SMALL	P-00388	13	1	QTY
M5 x 25 CAP SCREW	0052-4CS0-M5X25BK	14	2	QTY
6MM T NUT SET	A-00866	15	1	QTY
M8 SQUARE NUT SET	A-00867	16	1	QTY

SINGLE MOUNT HARDWARE

SINGLE MOUNT BACK SUPPORT HARDWARE

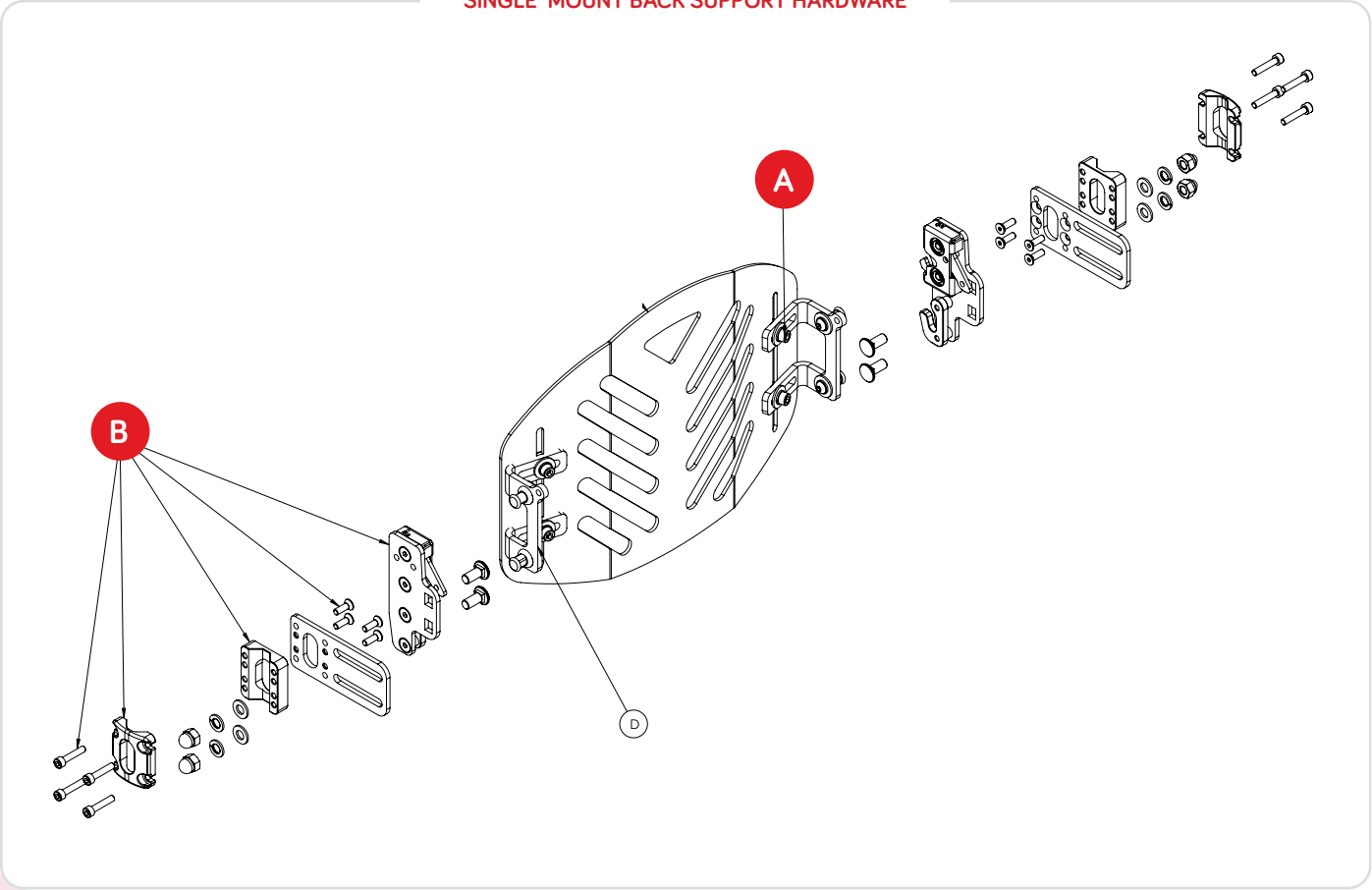


ORDERING CODES FOR SINGLE MOUNT

DESCRIPTION	CODE	#	PER	
T-SLOT ASSEMBLY SET	A-00866	A	4	QTY
SINGLE MOUNT ASSEMBLY		B	2	QTY
SINGLE MOUNT BRACKET		C	2	QTY
SINGLE MOUNT COMPLETE SET A + B + C	1259-2130-100		SET	QTY
SINGLE MOUNT FOR SECOND CHAIR B	1259-2120-100		SET	QTY

SINGLE MOUNT HARDWARE (CONTINUED)


SINGLE MOUNT LEFT

1

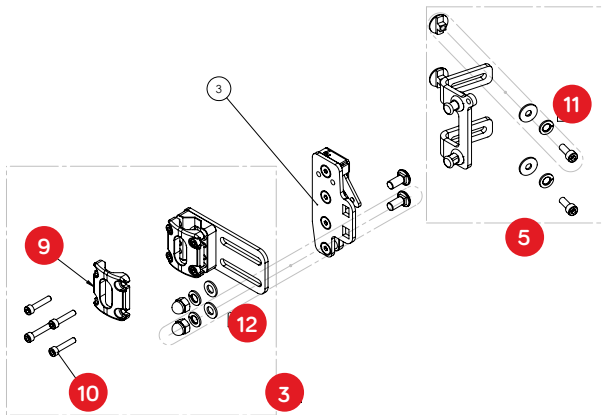

SINGLE MOUNT RIGHT

2

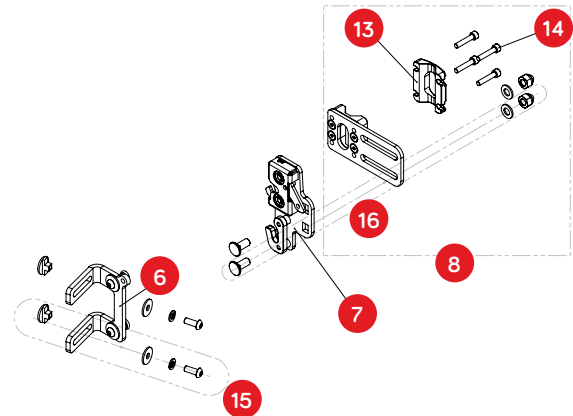

ORDERING CODES FOR DUAL MOUNT

WHOLE ASSEMBLIES

SINGLE MOUNT LEFT
SINGLE MOUNT RIGHT

CODE

-
-

#

1
2

PER

1	QTY
1	QTY

SUB ASSEMBLIES

SINGLE MOUNT ASSEMBLY
BACKREST MOUNT LEFT
SINGLE MOUNT BRACKET LEFT
SINGLE MOUNT BRACKET RIGHT
BACKREST MOUNT RIGHT
SINGLE MOUNT ASSEMBLY

CODE

A-00129
A-00121
A-01032
A-01031
A-00128
A-00129

#

3
4
5
6
7
8

PER

1	QTY
1	QTY
1	QTY
1	QTY
1	QTY
1	QTY

COMMON PARTS/SETS

50MM CLAMP
M5 x 25 CAP SCREW
6MM T NUT SET
M8 SQUARE NUT SET
50MM CLAMP
M5 x 25 CAP SCREW
6MM T NUT SET
M8 SQUARE NUT SET

CODE

P-00469
0052-4CS0-M5X25BK
A-00866
A-00867
P-00469
0052-4CS0-M5X25BK
A-00866
A-00867

#


9
10
11
12
13
14
15
16

PER

1	QTY
4	QTY
1	QTY
1	QTY
1	QTY
4	QTY
1	QTY
1	QTY

DROP MOUNT HARDWARE

DROP MOUNT BACK SUPPORT HARDWARE

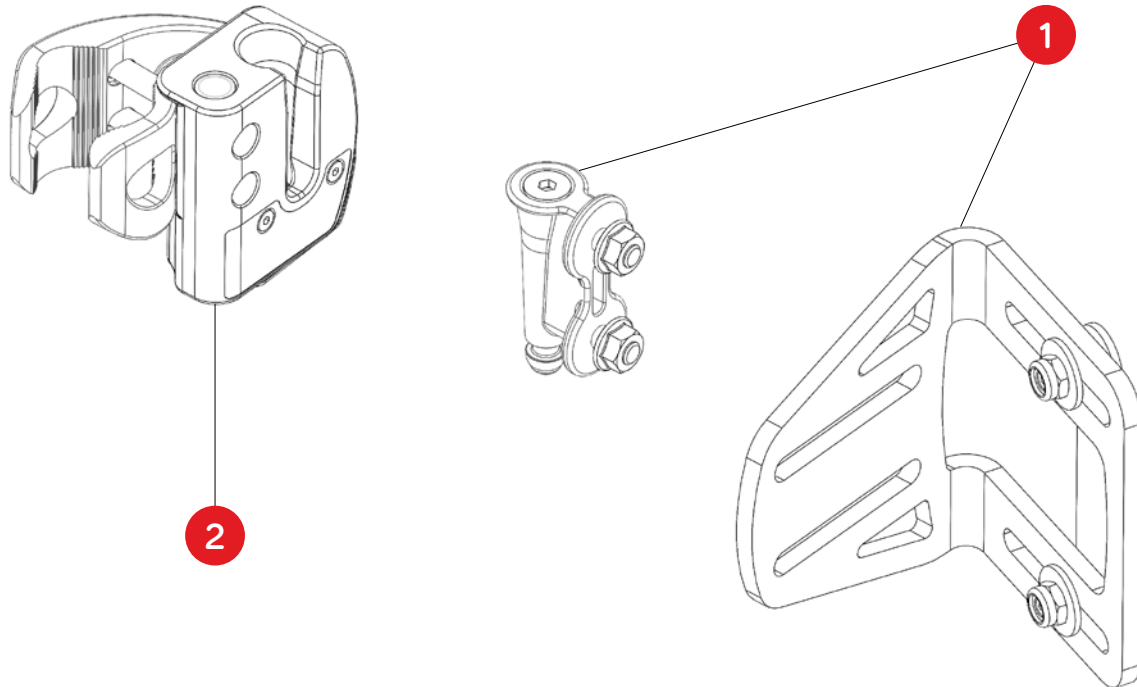

ORDERING CODES FOR DROP MOUNT

WHOLE ASSEMBLIES


	CODE	#	PER	QTY
PAEDIATRIC	1259-1400-00P	/	2	QTY
SMALL	1259-1400-00S	/	2	QTY
MEDIUM	1259-1400-00M	/	2	QTY
LARGE	1259-1400-00L	/	2	QTY

SUB ASSEMBLIES

	CODE	#	PER	QTY
BRACKETS AND PINS - PAEDIATRIC	1259-1410-00P	1	1	QTY
BRACKETS AND PINS - SMALL	1259-1410-00S	1	1	QTY
BRACKETS AND PINS - MEDIUM	1259-1410-00M	1	1	QTY
BRACKETS AND PINS - LARGE	1259-1410-00L	1	1	QTY
Drop Mount Receivers	1259-1410-000	2	1	QTY

BACK SUPPORT FASTENERS

FASTENER KIT

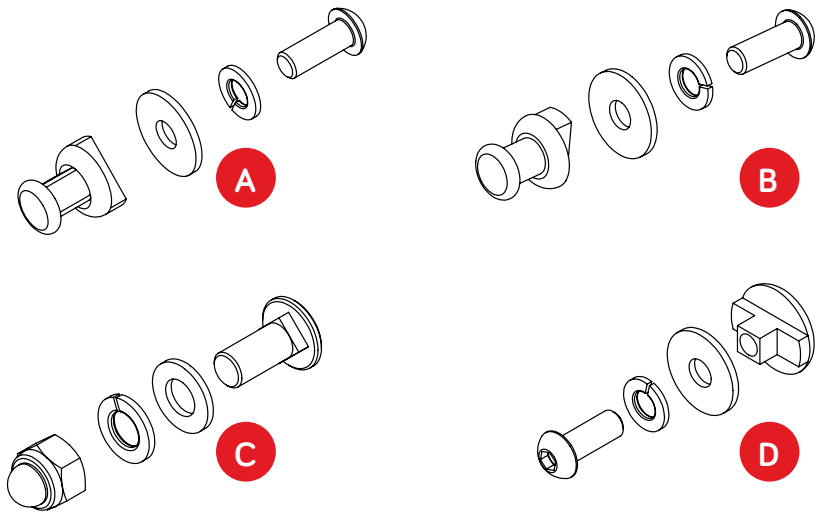


ORDERING CODES FOR FASTENER KIT

DESCRIPTION	CODE	#	PER	
ANTI-TILT LOCKING PEG SET	A-01106	A	EA	QTY
LOCKING PEG SET	A-00892	B	EA	QTY
SQUARE NECK BOLT SET	A-00867	C	EA	QTY
T-SLOT ASSEMBLY SET	A-00866	D	EA	QTY


ORDERING CODES FOR FIXING STUDS

DESCRIPTION	CODE	
16MM FIXING STUD	1258-0030-100	QTY
20MM FIXING STUD	1258-0031-100	QTY
25 MM FIXING STUD	1258-0032-100	QTY

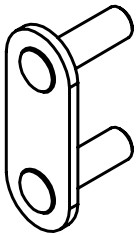

QUAD MOUNT EXTENDED UPGRADES FOR EXISTING HARDWARE

DESCRIPTION	CODE	
QUAD MOUNT EXTENDED UPGRADE FOR EXISTING HARDWARE TO INCREASE BACK ANGLE BY ≈10°	1259-2451-100	QTY
QUAD MOUNT EXTENDED UPGRADE FOR EXISTING HARDWARE TO INCREASE SEAT DEPTH	1259-2453-100	QTY

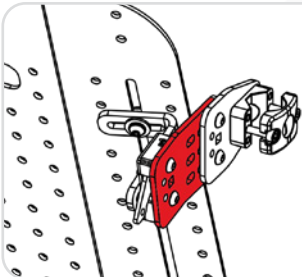

LATERAL TRUNK SUPPORT PADS

PAD WITH STD COVER



NARROW STRAIGHT

PAD WITH COMFY COVER



NARROW CURVED




STD COVER ONLY



COMFY COVER ONLY



BROAD

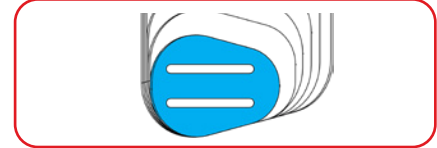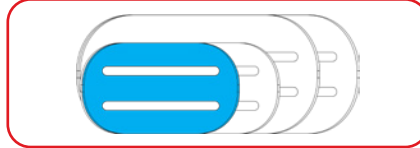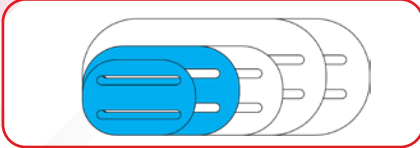

BUILD CODE FOR LATERAL TRUNK SUPPORT PAD & COVER

1 2 6 6 - 3 X X X - X 0 0

NARROW STRAIGHT
XXXX-XX**1**X-XXX

NARROW CURVED
XXXX-XX**2**X-XXX

BROAD
XXXX-XX**3**X-XXX

PAD WITH STD COVER
XXXX-X**0**XX-**4**XX

PAD WITH COMFY COVER
XXXX-X**0**XX-**8**XX

STD COVER ONLY
1267-X6XX-6XX

COMFY COVER ONLY
1267-X6XX-8XX

NARROW STRAIGHT 0
85mm H x 115mm D

0

NARROW STRAIGHT 1
95mm H x 145mm D

1

NARROW STRAIGHT 2
95mm H x 180mm D

2

NARROW STRAIGHT 3
115mm H x 210mm D

3

NARROW STRAIGHT 4
115mm H x 240mm D

4

NARROW CURVED 1
95mm H x 145mm D

1

NARROW CURVED 2
95mm H x 180mm D

2

NARROW CURVED 3
115mm H x 210mm D

3

NARROW CURVED 4
115mm H x 240mm D

4

BROAD 1
105mm H x 130mm D

1

BROAD 2
125mm H x 140mm D

2

BROAD 3
140mm H x 150mm D

3

BROAD 4
160mm H x 155mm D

4

BROAD 5
195mm H x 165mm D

5

BROAD 6
230mm H x 175mm D

6

CUSTOM OPTIONS

DESCRIPTION

CODE

SMALL STRAP - PAEDIATRIC SWING AWAY BRACKET

1267-3851-800

QTY

MEDIUM STRAP - REGULAR SWING-AWAY & AXIAL PAEDIATRIC BRACKET

1267-3852-800

QTY

LARGE STRAP - AXIAL BRACKET

1267-3853-800

QTY

FABRIC CUSTOM OPTIONS

CODE

DUAL LAYER ANTI-SHEAR FABRIC ON INTERNAL FACE

1261-3849-286

QTY

SPEXTEX - WIPE DOWN & WATERPROOF

1261-3849-209

QTY

SPEXTEX - KNITTED SIDE FABRIC FINISH

1261-3849-219

QTY

OTHER CUSTOM OPTIONS

CODE

½" PRESSURE RELIEVING OVERLAY - INTERNAL/TOP SURFACE

1261-3846-400

QTY

½" GEL-FOAM OVERLAY - INTERNAL/TOP SURFACE

1261-3847-400

QTY

NEOPRENE SLEEVE TO PROTECT HARDWARE

1261-3848-400

QTY

OTHER CUSTOMISATIONS

1261-5839-400


LATERAL TRUNK SUPPORT HARDWARE

FIXED BRACKET

DESCRIPTION

CODE

FIXED	1266-2101-100	QTY
PAEDIATRIC FIXED	1266-2111-100	QTY

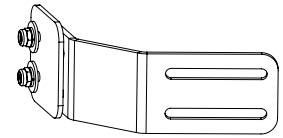


STANDARD SWING-AWAY


COMPLETE LATERAL ASSEMBLIES

CODE

SWING AWAY, STRAIGHT	1266-2201-100	QTY
SWING AWAY, STRAIGHT EXTEND	1266-2202-100	QTY
SWING AWAY, OFFSET 20MM	1266-2203-100	QTY
SWING-AWAY, OFFSET 40MM	1266-2204-100	QTY

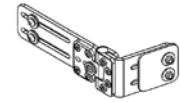

STRAIGHT

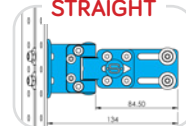


OFFSET



LATERAL BRACKET COMPONENTS

CODE

STRAIGHT BRACKET	1267-1001-100	QTY
PAEDIATRIC STRAIGHT BRACKET	1267-1011-100	QTY
STRAIGHT EXTENDED BRACKET	1267-1002-100	QTY
OFFSET 20MM BRACKET	1267-1003-100	QTY
PAEDIATRIC OFFSET 15MM BRACKET	1267-1013-100	QTY
OFFSET 40MM	1267-1004-100	QTY
BACK MOUNT BRACKET	1267-1100-100	QTY
PAEDIATRIC BACK MOUNT BRACKET	1267-1110-100	QTY

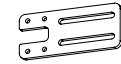


AXIAL

DESCRIPTION

CODE

AXIAL	1266-2301-100	QTY
PAEDIATRIC SWING AWAY AXIAL	1266-2623-400	QTY
PAEDIATRIC AXIAL LATERAL - RIGHT	1266-2613-100-L	QTY
PAEDIATRIC AXIAL LATERAL - LEFT	1266-2613-100-R	QTY

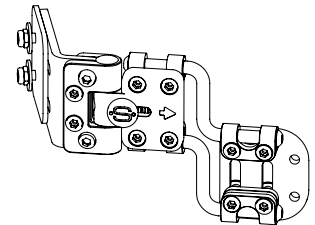


QUICK-RELEASE

DESCRIPTION

CODE

BACK MOUNT FIXED ANGLE - LEFT	1266-2601-100-L	QTY
BACK MOUNT FIXED ANGLE - RIGHT	1266-2601-100-R	QTY
CANE MOUNT FIXED ANGLE - LEFT	1266-2601-200-L	QTY
CANE MOUNT FIXED ANGLE - RIGHT	1266-2601-200-R	QTY
BACK MOUNT, SWING-AWAY, ANGLE ADJUSTMENT - LEFT	1266-2621-100-L	QTY
BACK MOUNT, SWING-AWAY, ANGLE ADJUSTMENT - RIGHT	1266-2621-100-R	QTY
CANE MOUNT, SWING-AWAY, ANGLE ADJUSTMENT - LEFT	1266-2621-200-L	QTY
CANE MOUNT, SWING-AWAY, ANGLE ADJUSTMENT - RIGHT	1266-2621-200-R	QTY

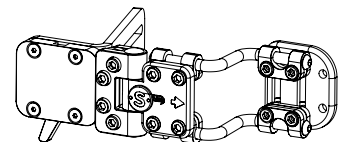

LATERAL TRUNK SUPPORT HARDWARE (CONTINUED)


BRACKET UPGRADE OPTIONS (PAEDIATRIC AND STANDARD BRACKETS/LATERALS ARE NOT COMPATIBLE)

DESCRIPTION	CODE	AVAILABLE FOR:			
		FIXED	S/AWAY	AXIAL	
LEFT CANE MOUNT	1266-2411-100	✗	✓	✓	QTY
RIGHT CANE MOUNT	1266-2412-100	✗	✓	✓	QTY
REINFORCEMENT PLATE FOR FORCEFUL SIDE LEANING OR CLIENTS OVER 100KG	1266-2450-100	✓	✓	✓	QTY
DEEP CONTOUR BRACKET KIT (EXTENDS DEPTH BY 30MM)	1266-2420-100	✗	✓	✓	QTY
DEEP CONTOUR EXTENDED BRACKET (EXTENDS DEPTH UP TO 115MM USED WITH SUPERSHAPE WITH EXTENSIVE CONTOURING)	1266-2430-100	✗	✓	✓	QTY
SWING-AWAY LATERAL EXTENDER BRACKET KIT (EXTENDS DEPTH UP TO 40MM)	1266-2444-100	✗	✓	✓	QTY
EXTENDER KIT FOR PAEDIATRIC SWING-AWAY LATERAL (COMPATIBLE WITH 1266-2623-400 ONLY)	1266-2440-100	✗	✓	✓	QTY


**Cane
Mounts**




**Deep Contour
Extended
Bracket**

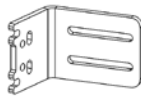


**Extender Kit for
Paediatric Swing-
Away Lateral**

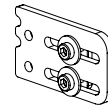

**Reinforcement
Plate**

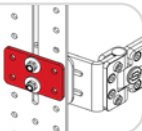

**Paediatric
Lateral
Extender**

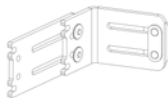

BIANGULAR LATERAL TRUNK SUPPORT

HARDWARE

DESCRIPTION

CODE

AXIAL HARDWARE

1266-2501-100


QTY

HINGE PLATE


1266-2511-100

QTY


Axial Biangular Bracket

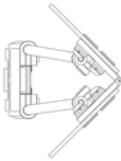

Hinge Plate

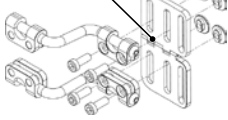

BIANGULAR PAD SIZING OPTIONS

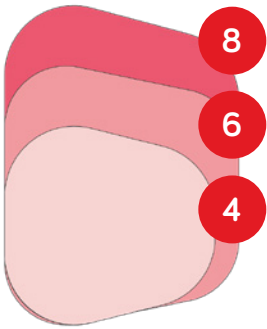

BUILD CODE FOR BIANGULAR LATERAL PAD

1 2 6 6 - 2 5 4 X - X 0 0

STANDARD AXIAL 4
165mm H x 175mm D

4

STANDARD AXIAL 6
215mm H x 195mm D

6

STANDARD AXIAL 8
265mm H x 195mm D

8

AXIAL PRESSURE REDUCTION 4
165mm H x 175mm D

4

AXIAL PRESSURE REDUCTION 6
215mm H x 195mm D

6

AXIAL PRESSURE REDUCTION 8
265mm H x 195mm D

8


PAD WITH STD COVER
XXXX-XXXX-8XX

PAD WITH COMFY COVER
XXXX-XXXX-4XX

MEDIAL THIGH SUPPORT


BRACKET COMPONENTS

DESCRIPTION	#	CODE	
SWING-DOWN, LIFT OFF MECHANISM ONLY	A	1162-1510-100	QTY
STANDARD PAD - SIZE 1	B	1162-1531-400	QTY
STANDARD PAD - SIZE 2	B	1162-1532-400	QTY
STANDARD PAD - SIZE 3	B	1162-1533-400	QTY
STAND PAD - COVER ONLY - SIZE 1	B	1162-1541-600	QTY
STAND PAD - COVER ONLY - SIZE 2	B	1162-1542-600	QTY
STAND PAD - COVER ONLY - SIZE 3	B	1162-1543-600	QTY
AXIAL ADJUST PAD - SIZE 1	C	1162-1631-400	QTY
AXIAL ADJUST PAD - SIZE 2	C	1162-1632-400	QTY
AXIAL ADJUST PAD - SIZE 3	C	1162-1633-400	QTY
AXIAL ADJUST PAD - COVER ONLY - SIZE 1	C	1162-1641-600	QTY
AXIAL ADJUST PAD - COVER ONLY - SIZE 2	C	1162-1642-600	QTY
AXIAL ADJUST PAD - COVER ONLY - SIZE 3	C	1162-1643-600	QTY

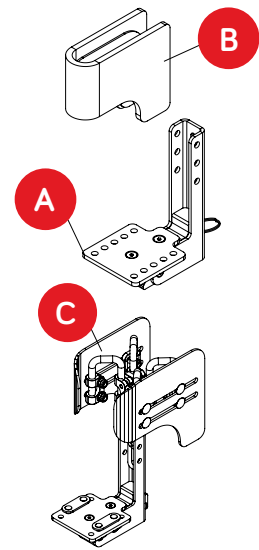

SIZING (mm)

DIMENSIONS	SIZE	A	B	C	D	E	F
STANDARD PAD	1	60	145	95	40	40	65
	2	60	155	100	60	50	65
	3	60	185	105	60	60	75
AXIAL ADJUST PAD	1	70 - 140	145	95	50 - 120	40	65
	2	70 - 140	155	100	60 - 140	50	65
	3	70 - 160	185	105	60 - 140	60	75

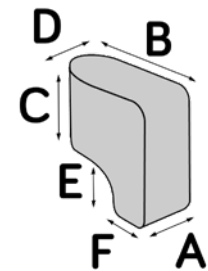

CUSTOM OPTIONS

DESCRIPTION	CODE	PER	
SPEXTEX - WIPE DOWN & WATERPROOF FINISH FOR STANDARD PAD ONLY	1160-1579-209	EA	QTY
SPEXTEX - KNITTED SIDE FABRIC FINISH FOR STANDARD PAD ONLY	1160-1579-219	EA	QTY
GLIDEWEAR ANTI-SHEAR FABRIC	1160-1579-295	EA	QTY
½" PRESSURE RELIEVING OVERLAY	1160-1576-000	EA	QTY
½" GEL-FOAM OVERLAY	1160-1577-000	EA	QTY

DROP DOWN BRACKET

DESCRIPTION	CODE	
SWING-DOWN/LIFT OFF BRACKET	1162-1515-100	QTY



WONDERSEAT MEDIAL THIGH SUPPORT


WONDERSEAT QUICK-RELEASE MEDIAL THIGH SUPPORT

DESCRIPTION	CODE	
WONDERSEAT ADJUSTABLE MEDIAL THIGH SUPPORT	1162-1401-000	QTY
WONDERSEAT ADJUSTABLE MEDIAL THIGH SUPPORT COVER	1162-1401-000-C	QTY




ARM/THIGH SUPPORT HARDWARE


ARM/THIGH COMPONENTS

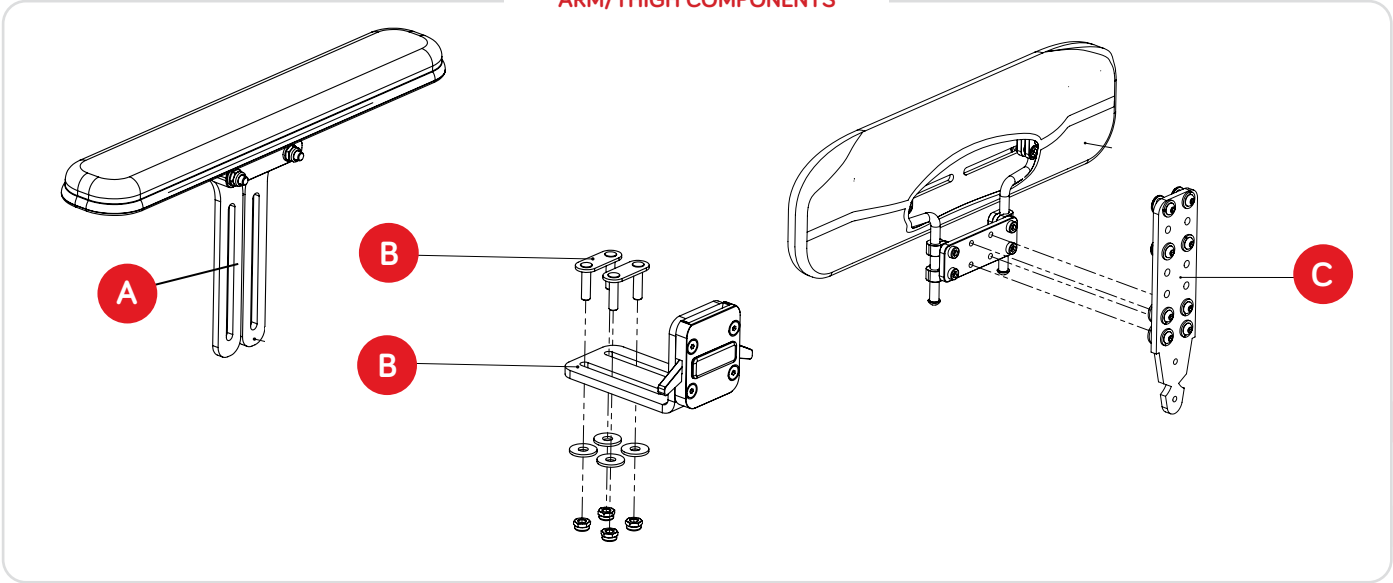


QUICK RELEASE

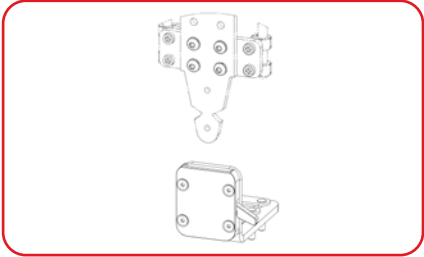

SWING DOWN

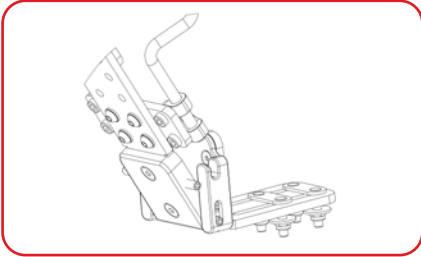


FIXED MOUNT

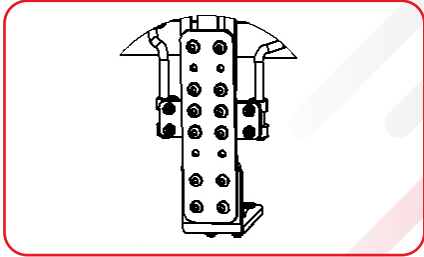


(A) ARM PLATFORMS

DESCRIPTION	CODE	QTY
FIXED ARM PLATFORM	1156-0100-000	QTY
FIXED PAEDIATRIC ARM PLATFORM	1156-0200-000	QTY
ANGLE-ADJUST ARM PLATFORM	1156-0300-000	QTY

FIXED ARM

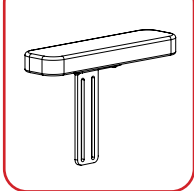


ANGLE ADJUST

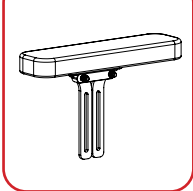


(B) MOUNTS

DESCRIPTION	QUICK RELEASE	SWING DOWN	FIXED MOUNT	QTY
SEAT PAN	1156-0000-110	1156-0000-120	1156-0000-130	QTY
SEAT RAIL	1156-0000-210	1156-0000-220	1156-0000-230	QTY

SEAT PAN

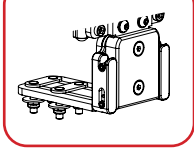


SEAT RAIL

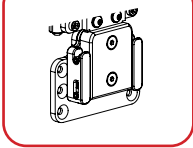


(C) THIGH PLATFORMS

DESCRIPTION	QR & SWING DOWN	FIXED MOUNT	QTY
FIXED THIGH PLATFORM FOR THIGH SUPPORT ONLY	1156-0012-000	1156-0022-000	QTY
FIXED THIGH PLATFORM FOR COMBO ARM/THIGH SUPPORT	1156-0032-000	1156-0042-000	QTY
AXIAL THIGH PLATFORM FOR THIGH SUPPORT ONLY	1156-0011-000	1156-0021-000	QTY
AXIAL THIGH PLATFORM FOR COMBO ARM/THIGH SUPPORT	1156-0031-000	1156-0041-000	QTY

AXIAL THIGH

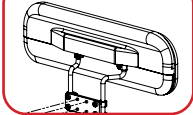

FIXED THIGH

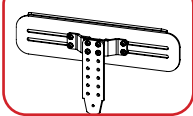

WONDERSEAT COMBO ARM/THIGH SUPPORT HARDWARE

COMBO ARM/THIGH HARDWARE



DESCRIPTION

WONDERSEAT COMPLETE-RELEASE ARM & THIGH SUPPORT HARDWARE, LEFT (LHS)
WONDERSEAT COMPLETE-RELEASE ARM & THIGH SUPPORT HARDWARE, RIGHT (RHS)
WONDERSEAT SIDE-RELEASE ARM & THIGH SUPPORT HARDWARE, LEFT (LHS)
WONDERSEAT SIDE-RELEASE ARM & THIGH SUPPORT HARDWARE, RIGHT (RHS)


CODE

1156-0403-110-L	QTY
1156-0403-110-R	QTY
1156-0403-130-L	QTY
1156-0403-130-R	QTY

Note: These products are designed to work as part of a complete Wonderseat Seating System, and are not compatible with other components of the Spex Seating range

THIGH SUPPORT PAD COVERS

ARM/THIGH COMPONENTS

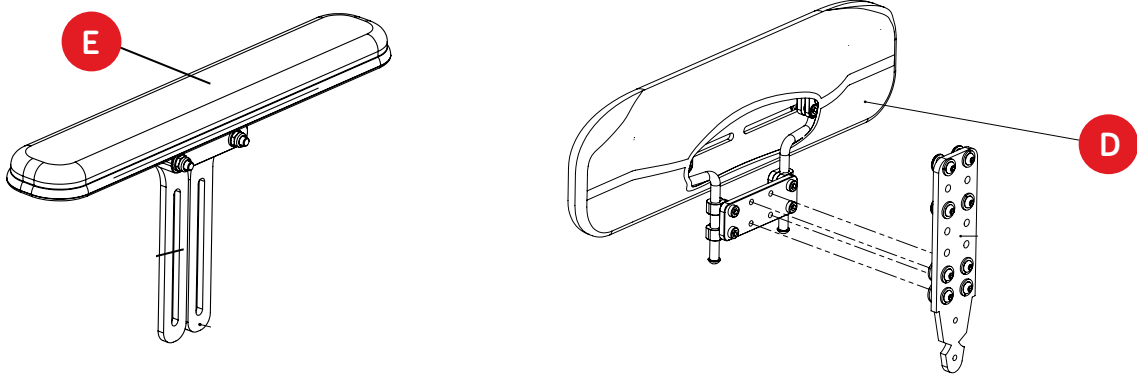

BUILD CODE FOR ARM OR THIGH PAD COVERS

1 1 5 6 - X X X X - X X X

D THIGH SUPPORT PAD COVERS

10" PAD COVER 260mm L x 90mm H	1010-600
12" PAD COVER 310mm L x 90mm H	1012-600
14" PAD COVER 360mm L x 125mm H	1014-600
16" PAD COVER 410mm L x 140mm H	1016-600
18" PAD COVER 450mm L x 140mm H	1018-600
20" PAD COVER 510mm L x 160mm H	1020-600

E ARM SUPPORT PAD COVERS

1 NARROW 40mm W x 250mm H	5861-410
2 NARROW 50mm W x 300mm H	5862-410
3 NARROW 60mm W x 350mm H	5863-410
1 WIDE 70mm W x 250mm H	5871-410
2 WIDE 80mm W x 300mm H	5872-410
3 WIDE 90mm W x 350mm H	5873-410

CUSTOM THIGH SUPPORT PAD COVERS

1156-1029-600 - RESTRICTIONS APPLY

XX

WW = ____

XX

HH = ____

CUSTOM ARM SUPPORT PAD COVERS

1156-5879-410 - RESTRICTIONS APPLY

XX

WW = ____

XX

HH = ____

THIGH SUPPORT FABRIC CUSTOMISATIONS

CUSTOMISATION

SPEXTEX - WIPEDOWN & WATERPROOF FINISH
SPEXTEX - KNITTED SIDE FABRIC FINISH
DUAL LAYER ANTI SHEAR FABRIC - INTERNAL FACE

CODE

1153-1149-209	QTY
1153-1149-219	QTY
1153-1149-286	QTY

WONDERSEAT ARM/THIGH SUPPORT PADS & COVERS

THIGH SUPPORT PADS

DESCRIPTION

CODE

WONDERSEAT THIGH SUPPORT PAD, 5" DEEP

1156-1105-400

QTY

WONDERSEAT THIGH SUPPORT PAD, 8" DEEP

1156-1108-400

QTY

WONDERSEAT SNUG THIGH SUPPORT PAD LEFT (LHS), 8" DEEP

1156-1208-400-L

QTY

WONDERSEAT SNUG THIGH SUPPORT PAD RIGHT (RHS), 8" DEEP

1156-1208-400-R

QTY

STANDARD



SNUG



THIGH SUPPORT PAD COVERS

DESCRIPTION

CODE

WONDERSEAT THIGH SUPPORT PAD COVER, 5" DEEP

1156-1105-400-C

QTY

WONDERSEAT THIGH SUPPORT PAD COVER, 8" DEEP

1156-1108-400-C

QTY

WONDERSEAT SNUG THIGH SUPPORT PAD COVER LEFT (LHS), 8" DEEP

1156-1208-400-L-C

QTY

WONDERSEAT SNUG THIGH SUPPORT PAD COVER RIGHT (RHS), 8" DEEP

1156-1208-400-R-C

QTY

ARM SUPPORT PADS

DESCRIPTION

CODE

WONDERSEAT ARM SUPPORT PAD, SIZE 0 LHS

1156-6410-310-L

QTY

WONDERSEAT ARM SUPPORT PAD, SIZE 0 RHS

1156-6410-310-R

QTY

WONDERSEAT ARM SUPPORT PAD, SIZE 1 LHS

1156-6411-310-L

QTY

WONDERSEAT ARM SUPPORT PAD, SIZE 1 RHS

1156-6411-310-R

QTY

WONDERSEAT ARM & ELBOW SUPPORT PAD, SIZE 0 LHS

1156-6420-310-L

QTY

WONDERSEAT ARM & ELBOW SUPPORT PAD, SIZE 0 RHS

1156-6420-310-R

QTY

WONDERSEAT ARM & ELBOW SUPPORT PAD, SIZE 1 LHS

1156-6421-310-L

QTY

WONDERSEAT ARM & ELBOW SUPPORT PAD, SIZE 1 RHS

1156-6421-310-R

QTY

STANDARD



WITH ELBOW SUPPORT



ARM SUPPORT PAD COVERS

DESCRIPTION

CODE

WONDERSEAT ARM SUPPORT PAD, SIZE 0 LHS

1156-6410-310-L-C

QTY

WONDERSEAT ARM SUPPORT PAD, SIZE 0 RHS

1156-6410-310-R-C

QTY

WONDERSEAT ARM SUPPORT PAD, SIZE 1 LHS

1156-6411-310-L-C

QTY

WONDERSEAT ARM SUPPORT PAD, SIZE 1 RHS

1156-6411-310-R-C

QTY

WONDERSEAT ARM & ELBOW SUPPORT PAD, SIZE 0 LHS

1156-6420-310-L-C

QTY

WONDERSEAT ARM & ELBOW SUPPORT PAD, SIZE 0 RHS

1156-6420-310-R-C

QTY

WONDERSEAT ARM & ELBOW SUPPORT PAD, SIZE 1 LHS

1156-6421-310-L-C

QTY

WONDERSEAT ARM & ELBOW SUPPORT PAD, SIZE 1 RHS

1156-6421-310-R-C

QTY

HEAD SUPPORT PADS



CIRCLE PAD



COMFORT PAD



CONTOUR PAD



STANDARD
LATERAL PAD



EXTENDED
LATERAL PAD



ADJUSTABLE
LATERAL PAD

BUILD CODE FOR HEAD SUPPORT COVER

1 2 7 5 - 0 X X X - X 0 0

PAD & COVER
XXXX-X3XX-3XX

COVER ONLY
XXXX-X6XX-6XX

CUSTOM COMFORT PAD/COVER

1275-4319-300 - PAD & COVER
1275-4619-600 - COVER ONLY

XX
WW = ____

XX
HH = ____

RESTRICTIONS APPLY

CIRCLE 150

175mm W
x 55mm H

-OX11-

CIRCLE 250

220mm W
x 70mm H

-OX12-

COMFORT 250

260mm W
x 90mm H

-OX31-

COMFORT 350

340mm W
x 120mm H

-OX32-

CONTOUR 250

280mm W
x 140mm H

-OX21-

CONTOUR 350

350mm W
x 170mm H

-OX22-

STD LAT PAEDIATRIC

220mm W
x 180mm H

-OX41-

STD LAT STANDARD

250mm W
x 145mm H

-OX43-

EXTD LAT PAEDIATRIC

230mm W
x 160mm H

-OX42-

EXTD LAT STANDARD

260mm W
x 180mm H

-OX44-

ADJUSTABLE LAT 150

175mm W
x 55mm H

-OX61-

ADJUSTABLE LAT 250

220mm W
x 70mm H

-OX62-

SPARE PARTS

DESCRIPTION

CODE

ADJUSTABLE WING BRACKET COMPLETE

1272-4151-100

QTY

STYLO HEAD SUPPORT MOUNT BALL


1274-4179-100

QTY

HEAD SUPPORT PAD MODIFICATIONS


CUSTOM FABRIC CHOICES ON COVER

SPEXTEX WATERPROOF PU

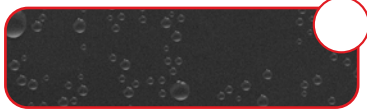

SPEXTEX WATERPROOF PU
(1208-2949-209)

SPEXTEX KNITTED FABRIC

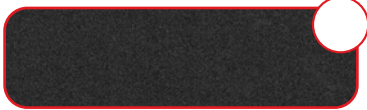

SPEXTEX KNITTED FABRIC
(1208-2949-219)

MERINO



AIR MESH
(default)

COLOURS AVAILABLE

Black
(default)

Chilli
1209-2491-022

Mango
1209-2491-030

Calypso
1209-2491-028

Royal
1209-2491-077

Bumblebee
1209-2491-044

Ocean
1209-2491-060

Leaf
1209-2491-054

Granite
1209-2491-090

CUSTOM OPTIONS

CUSTOMISATION

CUSTOM THICKNESS (SPECIFY)

1/2" EXTRA SOFT PRESSURE-RELIEVING FOAM OVERLAY

MEASUREMENTS

1271-5879-000

1271-0019-000

_____mm

WONDERSEAT HEAD SUPPORT HARDWARE

WONDERSEAT HEAD SUPPORT



LITE



QUICK-RELEASE

DESCRIPTION

WONDERSEAT LITE HEAD SUPPORT HARDWARE (WITH BALL MOUNT)

WONDERSEAT QUICK-RELEASE HEAD SUPPORT HARDWARE (WITH BALL MOUNT)

CODE

1274-0232-100

1274-3232-100


QTY

QTY

Note: These products are designed to work as part of a complete Wonderseat Seating System, and are not compatible with other components of the Spex Seating range

STYLO HARDWARE

STYLO HEAD SUPPORT

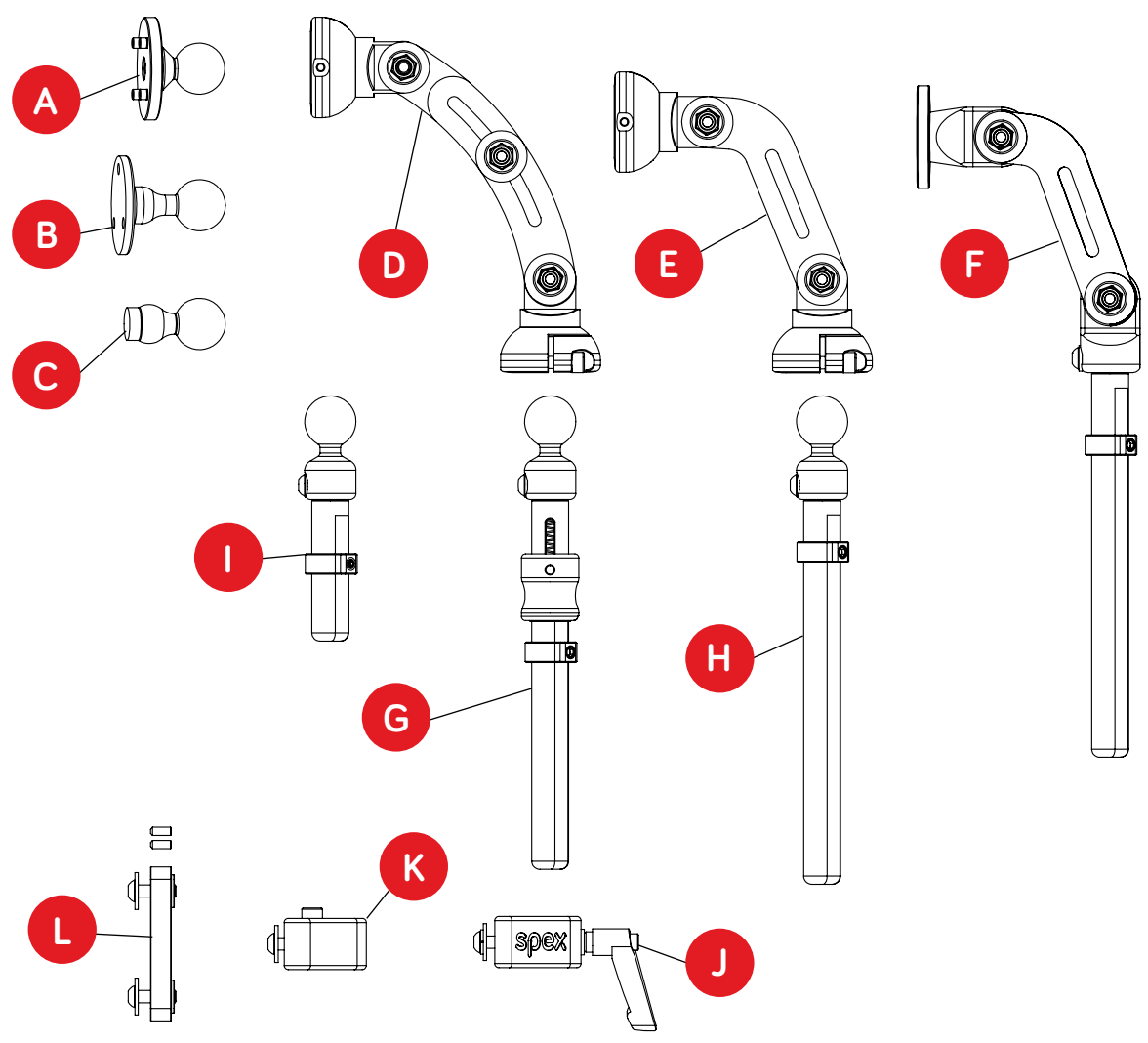

ORDERING CODES FOR STYLO HEAD SUPPORTS

DESCRIPTION	CODE	#	PER	
STYLO HEADREST MOUNT BALL	1274-4179-100	A	EA	QTY
SPEX HEAD SUPPORT BALL JOINT MOUNT 3 HOLE S/S	1272-4172-100	B	EA	QTY
WHITMYER TO SPEX HEAD SUPPORT BALL MOUNT ADAPTOR	1274-4185-100	C	EA	QTY
STYLO 260 ARM	1274-0082-100	D	EA	QTY
STYLO 160 ARM	1274-0052-100	E	EA	QTY
STYLO 130 ARM	1274-1232-100	F	EA	QTY
STYLO SWING-AWAY STEM	1274-2200-100	G	EA	QTY
STYLO FIXED STEM	1274-1200-100	H	EA	QTY
STYLO FIXED STEM - SHORT	1274-1205-100	I	EA	QTY
STYLO BACKREST MOUNT ASSEMBLY	1274-4140-100	J	EA	QTY
STYLO SLIM MOUNT	1274-4140-200	K	EA	QTY
STYLO HEAVY-DUTY COMPATIBILITY KIT	1274-0000-HDK	L	EA	QTY

HARNESSES & HIP BELTS

WONDERSEAT HIP BELTS & HARNESSES

DESCRIPTION

CODE

WONDERSEAT 4-POINT PADDED HIP BELT, SIZE 0	1406-5420-027	QTY
WONDERSEAT PADDED CENTREPOINT SHOULDER HARNESS, SIZE 0*	1409-5660-017	QTY
WONDERSEAT PADDED CENTREPOINT SHOULDER HARNESS, SIZE 1*	1409-5661-017	QTY

*These products are designed to work as part of a complete Wonderseat Seating System, and are not compatible with other components of the Spex Seating range



FOOT FAST MOUNTING KITS

DESCRIPTION

CODE

PER

SPARE MOUNTING KIT FOR FOOT FASTS	1400-6160-000	KIT	QTY
SPARE MOUNTING KIT FOR FOOT FASTS WITH REAR POSTERIOR RETAINER STRAPS	1400-6165-000	KIT	QTY



HIP BELT MOUNTING KITS

DESCRIPTION

CODE

PER

HIP BELT ATTACHMENT HARDWARE 25MM	1400-6160-000	KIT	QTY
HIP BELT ATTACHMENT HARDWARE 38MM	1400-6165-000	KIT	QTY
HIP BELT ATTACHMENT HARDWARE 50MM	1400-6165-000	KIT	QTY

SIZE

MOUNT KIT

XSM	25MM X 2
SML	25 + 38MM
MED	25 + 38MM
LGE	25 + 50MM



HARNESS MOUNTING KITS

DESCRIPTION

CODE

FITS HARNESS SIZE

PER

HARNESS MOUNTING KIT 25MM	1400-6155-000	XSM, SML, MED	KIT	QTY
HARNESS MOUNTING KIT 38MM	1400-6156-000	LGE, XLG	KIT	QTY



HARNESS & HIP BELT MOUNTING HARDWARE

DESCRIPTION

CODE

PER

HARNESS MOUNT 25MM STAINLESS STEEL	1400-6025-022	EA	QTY
HARNESS MOUNT 38MM STAINLESS STEEL	1400-6038-022	EA	QTY
HARNESS MOUNT 50MM STAINLESS STEEL	1400-6050-022	EA	QTY



DESCRIPTION

CODE

PER

TRI-BAR ADJUSTER 25MM PLASTIC	1400-6525-199	EA	QTY
TRI-BAR ADJUSTER 38MM PLASTIC	1400-6538-199	EA	QTY
TRI-BAR ADJUSTER 50MM PLASTIC	1400-6550-199	EA	QTY



DESCRIPTION

CODE

PER

TRI-BAR ADJUSTER 25MM STAINLESS STEEL	1400-6525-022	EA	QTY
TRI-BAR ADJUSTER 38MM STAINLESS STEEL	1400-6538-022	EA	QTY
TRI-BAR ADJUSTER 50MM STAINLESS STEEL	1400-6550-022	EA	QTY



SHOULDER STRAP GUIDES/RETAINER

DESCRIPTION

CODE

SHOULDER STRAP GUIDES AND ATTACHMENT HARDWARE	1400-6692-000	QTY
COMPACT SHOULDER STRAP GUIDES AND ATTACHMENT HARDWARE	1400-6693-000	QTY



TRAY TABLES

WONDERSEAT TRAY TABLE

DESCRIPTION	CODE	WIDTH	DEPTH	
WONDERSEAT TRAY TABLE	1311-7410-013	520MM	420MM	QTY
WONDERSEAT TRAY TABLE PADDING	1311-7410-013-C			QTY

Note: These products are designed to work as part of a complete Wonderseat Seating System, and are not compatible with other components of the Spex Seating range



BLACK ABS TRAY TOP

DESCRIPTION	CODE	SIZE	WIDTH	DEPTH	
BLACK ABS TRAY SMALL	1311-7411-013	SML	450MM	450MM	QTY
BLACK ABS TRAY MEDIUM	1311-7412-013	MED	550MM	550MM	QTY
BLACK ABS TRAY LARGE	1311-7413-013	LGE	650MM	650MM	QTY

CUSTOM OPTION (1311-7419-013) - RESTRICTIONS APPLY

WW = ____

XX

DD = ____



CLEAR POLYCARBONATE TRAY TOP

DESCRIPTION	CODE	SIZE	WIDTH	DEPTH	
CLEAR POLYCARBONATE TRAY SMALL	1311-7401-012	SML	450MM	450MM	QTY
CLEAR POLYCARBONATE TRAY MEDIUM	1311-7402-012	MED	550MM	550MM	QTY
CLEAR POLYCARBONATE TRAY LARGE	1311-7403-012	LGE	650MM	650MM	QTY

CUSTOM OPTION (1311-7409-012) - RESTRICTIONS APPLY

WW = ____

XX

DD = ____



TRAY TOP CUSTOMISATIONS

CUSTOM STOMACH REGION CUT-OUT

1311-7299-000 - RESTRICTIONS APPLY

ELBOW PADS TO REDUCE PRESSURE/SHEAR

1311-7301-400 - RESTRICTIONS APPLY

XX

WW = ____

DD = ____

WONDERSEAT BIT BOX

WONDERSEAT BIT BOX



DESCRIPTION

WONDERSEAT STANDALONE BIT BOX (FOR EXPERTS)

QTY

Note: These products are designed to work as part of a complete Wonderseat Seating System, and are not compatible with other components of the Spex Seating range

spex

Manufactured by Spex Ltd

solutions@spexseating.com

spexseating.com



DISTRIBUTED BY:

US	REP	EC	REP
Medicept 200 Homer Ave, Ashland MA 01721, United States Tel: +1-508-231-8842		BEO MedConsulting Berlin GmbH Helmholtzstr. 2 D-10587 Berlin, Germany Tel: +49-30-318045-30	



Spex Spare Parts Script Form_20250911_EN_v7