


# *Harness Cross Strap*

# *User Manual*

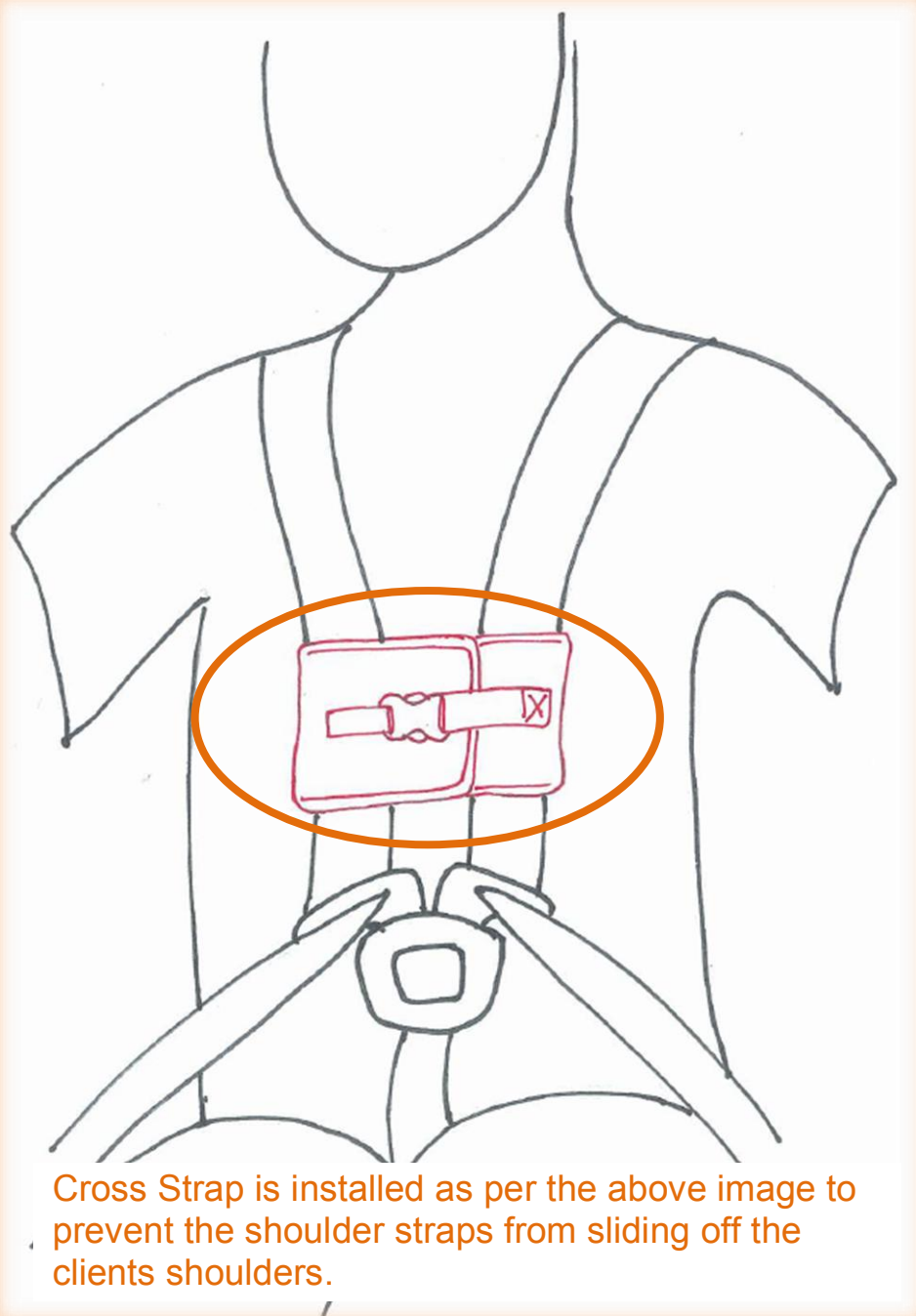

Cross Strap is installed as per the above image to prevent the shoulder straps from sliding off the clients shoulders.

ATTENTION: READ WARNING ON REVERSE PAGE BEFORE USE

**[www.medifab.com](http://www.medifab.com)  
[solutions@medifab.com](mailto:solutions@medifab.com)**

**NZ Call 0800 543 343  
AU Call 1300 543 343**

**7-9 Allens Road, Ashburton  
New Zealand**

## **WARNING: May Present Hazard**

The Harness Cross Strap should only be used if required to prevent shoulder straps falling down, or as a restraint mechanism when a user is escaping from their harness.

Potential risk of injury increases due to the device's proximity to the neck area.

Please ensure that the cross strap is located far enough down to be safe for the user.

Cross Straps must not be used on Harnesses with Chest Pads as the Cross Strap would be too near to the neck.