# User manual

Edition 8, July 2021

## Multi Joystick

Multi Joystick Omni (P002-61) - Multi Joystick R-net (P002-62)

**mo-vis**  
moving forward together

# Contents

<b>Other languages .....</b>	<b>8</b>
<b>About this manual .....</b>	<b>9</b>
User manual .....	9
mo-vis bv .....	9
<b>User manual (English) .....</b>	<b>10</b>
Warning labels .....	10
Support, scrapping & recycling .....	11
Limited liability .....	12
Intended Use .....	12
Features .....	13
Operation .....	16
Parts and accessories .....	16
Qualifications .....	20
First time use .....	20
Cleaning .....	21
Warnings and limitations .....	21
Connection options .....	22
Combinations .....	24
Dimensions .....	24

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EN: You can find this manual in English on our website.

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# About this manual

## User manual

Thank you for choosing a mo-vis product!

This manual contains **useful and important information** about your device.

**Please read it carefully before use and store safely for future reference.**

Our team (or your authorized dealer) will be happy to answer your questions.

## mo-vis bv

Biebuyckstraat 15D . 9850 Deinze . Belgium

<http://www.mo-vis.com> . [contact@mo-vis.com](mailto:contact@mo-vis.com) . +32 9 335 28 60

# User manual (English)

## Warning labels

*Please read this manual, the safety instructions and warning texts carefully, in order to reduce the risks associated to the device. Our products are safe under normal and reasonably foreseeable operating conditions.*

**NOTE:** This symbol indicates general notes and information.

**CAUTION:** This symbol indicates caution for a hazardous situation that, if not avoided, could result in minor or moderate injury.

**WARNING:** This symbol indicates a warning for a hazardous situation that, if not avoided, could result in death or serious injury.

## Support, scrapping & recycling

### *Technical support*

**TROUBLE:** In case of technical problems:

- Contact your dealer
- If your dealer is not available or unknown, please contact mo-vis: [contact@mo-vis.com](mailto:contact@mo-vis.com) or +32 9 335 28 60.

Always state the product code and device serial number when contacting us. This ensures you are provided with the correct information.

### *Spare parts and accessories*

Contact mo-vis or your dealer for more information about spare parts and accessories.

### *Scrapping & recycling*

**CAUTION:** For scrapping, adhere to your local waste legislation. Dispose of obsolete electronic parts responsibly in accordance with local recycling regulations.

## Limited liability

*mo-vis accepts **no liability** for personal injury or damage to property that may arise from the failure of the user or other persons to follow the recommendation, warnings and instructions in this manual.*

**CAUTION:** This product should only be installed by a qualified service engineer.

**NOTE:** All mo-vis manuals can be found at <http://www.mo-vis.com> where they can be consulted in PDF format.

**NOTE:** In case any serious incident occurs in relation to this device, this should be reported immediately to mo-vis and the competent authority of the Member State in which the user is established.

## Intended Use

The Multi Joystick:

- is a joystick module which can be connected directly to the wheelchair electronics to control the wheelchair functions

- is a small proportional joystick, which requires significant less power and movement compared to a normal joystick
- can easily be controlled by any body part, including finger, chin, lip... thanks to its flexibility
- is recommended for people with tetraplegia, neuromuscular diseases (ALS, MS...) and limited muscular force

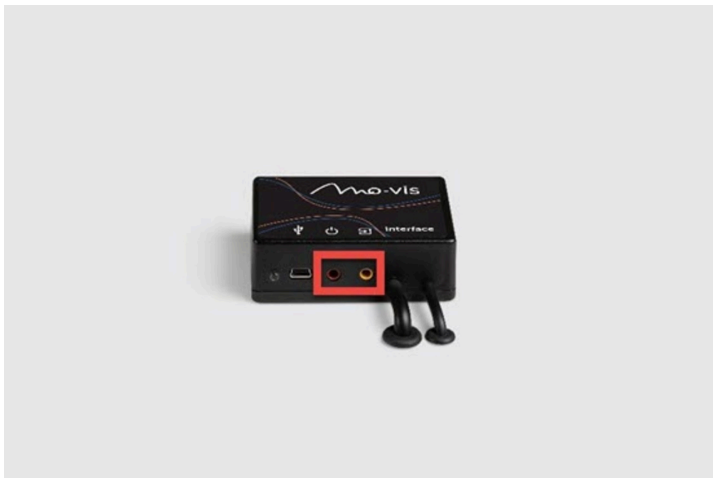
## Features

The Multi Joystick is:

- Fully proportional, with reduced throw (8 mm - 0.31 in) and force (ca. 50 gr - 1.76 oz)
- Built in a compact and discrete housing, with two 3.5 mm jacks for switch input

- has 2 switches that can be linked to the separate interface: (1) on/off power wheelchair - (2) mode of the power wheelchair.

- is fully adjustable to individual possibilities and needs with well-thought-out electronics (mini USB connection).
- is compatible with multiple types of wheelchair electronics.
- has a LED light on the interface which will start to flash green in case of an error or orange in case of active tilt (see *Installation manual*).

## Operation

The movements of the joystick are translated into according movements of the wheelchair, e.g. driving or menu navigation.

Common practice to navigate the wheelchair with the joystick is as follows:

- **Direction:** point the joystick into the direction you want the wheelchair to move. The wheelchair then moves in that direction.
- **Speed:** the further you move the joystick from the default (center) position, the faster the wheelchair moves.
- **Stop:** whenever you release the joystick, the joystick moves back to the default (center) position and the wheelchair stops.

**CAUTION:** Avoid hitting obstacles during driving.

**WARNING:** When the LED light flashes and/or after every incident with the wheelchair or the mo-vis device, contact your dealer immediately to perform a functional test.

## Parts and accessories

### *Package*

The Multi Joystick package consists of:

EXAMPLE	P002-61 - MULTI JOYSTICK OMNI	P002-62 - MULTI JOYSTICK R-NET
	P002-60 - Multi Joystick Sensor	
	M002-05 - Multi Joystick ball	
	M002-18 - Multi Joystick Mounting set	
<i>See Installation Manual</i>	SUB D9 cable and Joystick Omni Interface	R-net cable and Joystick R-net Interface
<i>D-P002-61-70-MX Multi Joystick User Manual</i>		
<i>D-P002-61-70-M1 Multi Joystick Installation Manual</i>		

## *Unit detail*

The connections of the Multi Joystick unit have protective covers to avoid the intrusion of dust and moisture during usage.

**CAUTION:** Before inserting a connector, remove the protective cover.

**CAUTION:** Always put the protective covers in, or keep them, in case the connections are not used.

## Spare parts

Spare parts and accessories must be ordered by the dealer at mo-vis bv.  
Always mention the product code of the desired item.

## Qualifications

**CAUTION:** Only a qualified service engineer may install the device.

**CAUTION:** An incorrect programming of the wheelchair or device electronics, may cause damage to the devices or injury to the user.

## First time use

**CAUTION:** Only a qualified service engineer may install the device.

**CAUTION:** An incorrect programming of the wheelchair or device electronics, may cause damage to the devices or injury to the user.

**CAUTION:** A qualified service engineer should assist during first time use of the device and make sure that all functions are clear to the user, including what to do in case of problems or doubt.

## Cleaning

**WARNING:** As dust and dirt could lead to reduced functionality, all parts of the device should be cleaned on a regular basis (monthly) or whenever needed.

- Gently remove dust and dirt with a damp cloth.
- Use only non-aggressive disinfectant cleaning agents.

**CAUTION:** Do not immerse in water or use excessive amounts of liquid.

**CAUTION:** Do not apply additional lubricants to the moving parts.

The handles of the joystick can be removed to clean them.

- You may shortly immerse the handle in warm water with a gentle cleaning agent. Let the handle dry before putting it back on.
- If the handle is damaged, worn out or remains dirty, please contact your authorized dealer or mo-vis to replace it.

## Warnings and limitations

**NOTE:** This product complies with the limit values for Electromagnetic Compatibility (EMC) with respect to power wheelchairs, as set out in

the harmonized standards for the EU in the Medical Device Regulation, No. 2017/745.

**WARNING:** Do not cover or block the device to avoid uncontrolled behavior of the device or the wheelchair.

## Connection options

The connection interface depends on the joystick version. Please see the *Installation Manual* for more information.

You can connect 1 or 2 switches to the joystick:

- **Red** = power on/off
- **Yellow** = mode

**WARNING:** The on/off switch must be available to the user at all times. This allows to instantly stop the wheelchair in case of problems or an emergency.

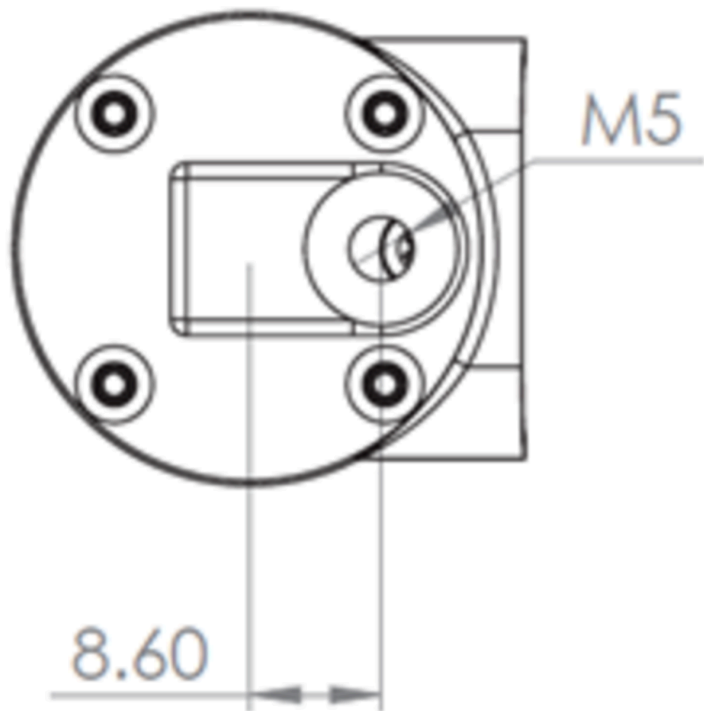
**WARNING:** Always power off the device when the user is no longer in the wheelchair to avoid unauthorized access or accidental movements.

## Combinations


There are different versions of this device available for combination with different wheelchair electronics. Please refer to the *Installation Manual* of this device for more information.

## Dimensions

Bottom view:

Side view:

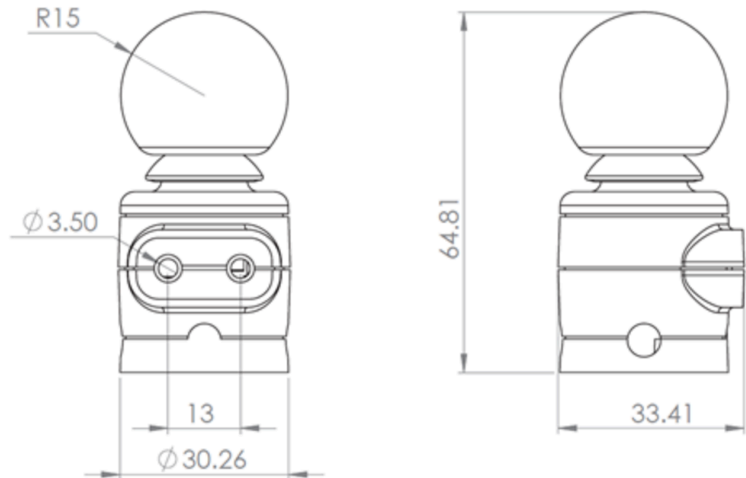PART	DIMENSIONS (METRIC)	DIMENSIONS (IMPERIAL)
Joystick	65 mm x 31 mm x 31 mm (HxWxD)	2.55 in x 1.22 in x 1.22 in
Ball	26 mm x 30 mm x 30	1.02 in x 1.18 in x 1.18

<b>PART</b>	<b>DIMENSIONS (METRIC)</b>	<b>DIMENSIONS (IMPERIAL)</b>
	mm	in

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VOORBEELD	P002-61 - MULTI JOYSTICK OMNI	P002-62 - MULTI JOYSTICK R-NET
 A black, cylindrical joystick sensor with a black cable and a small antenna-like protrusion on top.	P002-60 - Multi Joystick Sensor	
 A small, solid black spherical ball.	M002-05 - Multi Joystick ball	
 A set of mounting hardware including two black screws, a black plastic bracket, and a small black ball.	M002-18 - Multi Joystick Mounting set	
Zie de <i>Installatiehandleiding</i>	SUB D9-kabel en Joystick Omni Interface	R-net-kabel en Joystick R-net Interface

**VOORBEELD****P002-61 - MULTI  
JOYSTICK OMNI****P002-62 - MULTI  
JOYSTICK R-NET**

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