

# Primo Positioning System



**INTRODUCING EARLY  
YEARS POSITIONING  
FOR 0-3 YEAR OLDS**

Distributed by **medifab**<sup>®</sup>

# It's never too soon to start

## Introducing Early Years Positioning

Although Symmetrisleep® have been designing, manufacturing and supplying market leading postural care products for over 20 years, for the first time, with the introduction of Primo, the specific needs of 0-3 year olds are now being catered for.

Early Intervention Teams have been supporting families by teaching to position and handle their young child, in positive stimulation, sensory and play. Primo now offers a solution for day and night time positioning in lying.

## Why supported lying?

In your Early Years learning to lie straight against the effects of gravity can be challenging. If you can be supported in a Symmetrical lying position and learn to lie straight, day time can be so much more fun. Postural Care has become more widely accepted as a route to better positioning, easier manual handling, improved ability to function, therapy and play.

## Successful outcomes

We work closely as a team with Therapists, Families and Infants, successfully providing our Symmetrisleep® systems. With a long history of supplying our simple in bed support systems we are in contact daily with people who are showing improvements in comfort, body shape, range of movement and reduced tone.

## Simple

- Primo is simple and easy to use – unzip the carry bag and you're ready to go. You can use it on any flat surface where there is no risk of falling\* Simply lay your child on the built in Temperature Regulating top sheet and start using.

## Comfortable

- Symmetrisleep® have used the unique Temperature Regulating Technology, CoolOver, and Pressure Relieving materials to ensure the 'Baby Soft' Primo is so comfortable for the young infant.

## Portable

- The Primo is Compact, Light and comes in an 'All in One' Carry bag. You can take it anywhere.

## Hygienic

- The 'Baby Soft' Primo is easy to keep clean. All fabrics are machine washable and can be cool dried. The base unit, cushions and wedges are wipe clean.

## Safe

- Flammability, Breathability, Hardness and Airflow. Complies with Furniture and Furnishings (FIRE SAFETY) regulations 2010. Complies with EN597-2, BS1877-8 1974, BS7177, BS6807 and BS4578.



## Where can I use my Primo?

Symmetrisleep® Primo can be used in an existing cot, crib, carry cot or even on the floor for day time therapy and play\*



\* Please seek guidance on positioning and using your Primo from the Qualified Health Professional who has prescribed your Primo System.

# Primo Starter Pack

## What is in the Bag?

Primo starts with a bag that unfolds to form a base mattress. This can be secured to a cot or bed with the securing straps supplied. Attached to the top face of the mattress is a Velcro sheet and on top of that is your Coolover TR3 sheet.

Various soft supports are supplied in the bag. These can be added or removed when your baby is lying on the system to mould to the desired position offering soft and stable support without being restrictive.

As the supports are held in place by the Velcro they don't drift out of position and you don't need to reposition them until you actually want to, so reducing disturbances.

## Contains:

- Primo Base c/w Airmantle Core & Carry Bag Outer x 1
- Primo Base Velcro Sheet x 1
- Primo Coolover Oversheet x 2
- Primo Head Pillow c/w Coolover Cover x 1
- D Wedge 130 x 120mm c/w Air Permeable Cover x 4
- Primo Roll Cushion Small Crumb Foam c/w Air Permeable Cover x 1
- Security Strap 1500mm (Pair) x 1
- 7001-0100-000



Or select individual components to build your kit

Item	Dimensions	Code	Coolover cover only	Component
Base Unit, wipe-down cover with Airmantle inner, portable carry pouch with zip closure		7001-1000-PMO		
Velcro sheet with dome connectors		7001-2000-PMO		
Coolover oversheet		7001-3000-CVR		
Elliptical head pillow with Coolover cover		7001-4000-CVR	7001-4001-CVR	
Segmented 5 section pillow, with spacer fabric cover	40 x 45 x 0.6 cm	7001-4100-S10	7001-4100-CVR	
Lateral wedge with spacer fabric cover	10 x 13 x 7 cm	7001-4201-S10	7001-4201-CVR	
Small roll cushion, solid foam filling with spacer fabric cover		7001-4203-S10	7001-4203-CVR	
Long roll solid foam filling, with spacer fabric cover.	80 x 0.6 cm	7001-4208-S10	7001-4208-CVR	
Small horseshoe pillow, crumbed foam filling, with spacer fabric cover	37 x 28 cm	7001-4301-S10	7001-4301-CVR	
Small roll cushion, crumbed foam filling with spacer fabric cover		7001-4350-S10	7001-4350-CVR	
Knee cosy with solid foam filling, with spacer fabric cover	30 x 0.6 x 0.8 cm	7001-4405-S10	7001-4405-CVR	
Securing strap	150 cm long	7001-9000-000		
Securing strap	200 cm long	7001-9010-000		

# medifab®

Distributed by Medifab Ltd

32 Detroit Drive  
Rolleston 7675  
New Zealand

+64 3 307 9790

[solutions@medifab.com](mailto:solutions@medifab.com)

[www.medifab.com](http://www.medifab.com)

