

Quickie Q500 Mini Spare Parts



**SPARE
PARTS
GUIDE**

medifab®

MEDIFAB.COM

Version 1, July 2022

Contents

Base	5
Suspension	5
Castors	7
Drive Wheel	8
Motors	9
Shrouds	11
Mudguards	11
Tilt & Lift Cassette	12
Assembly	12
Parts	12
Sedeo Pro Seating	13
Assemblies	13
Short Seat Frame	15
Long Seat Frame	16
Back Frame Universal	17
Tool Adjustable Fixed Back	18
Power Semi-Recline (130 Degrees)	19
Power Recline (150°) with Anti Shear	21
Sedeo Lite Seating	22
Seat	22
Back	22
Sedeo Pro Armrests	24
Single Post	24
Cantilever Style Recline Back	25
2 Pt Armrest Lock	26
Sedeo Sideguards	27
Sedeo Lite Armrests	27
Cantilever	27


Mounting Parts	28
Armrest Pads	29
Standard Sedeo Armrest Pad	29
Comfort Gel Armrest Pad	30
Comfort Foam Armrest Pad	31
Gel Trough Armrest Pad	31
Sedeo Pro Legrests	32
Swingaway Mounts	32
Swing Away 70 Degree Assembly	32
Swing Away 70 Degree Hangers	32
Swingaway Manual ELR Assembly	33
Swingaway Manual ELR	33
Swingaway Power ELR Assembly	33
Swingaway Power ELR Parts	34
Swingaway Hanger Footplates	34
Knee & Calf Pads	35
Power Centre Mount Mechanical Ext	37
Power Centre Mount ELR Power Ext	37
Power Centre Mount Parts	37
Power Centre Mount Wiring	39
Sedeo Lite Legrests	40
Angle Adjustable Centre Mount	40
Fixed Centre Mount Assembly	41
Fixed Centre Mount Parts	42
Fixed Centre Mount Dual Footplate	42
Parts	42
Knee Pads	43
Headrests	44
Joystick	46

CJSM2	46
CJSM1	46
LED	46
Mounts	47
Parts	48
Attendant Joystick	51
Attendant Mounting Systems	51
Accessory Electronics	52
RNET Cables	54
Bus Cables/Harness	54
Battery Harness	56
Wiring	56
Lighting	59

Base


1600-168109

Quickie/Zippie Power, Mini Base, Complete Base, including Suspension, Casters, Seat interface and Tie Downs

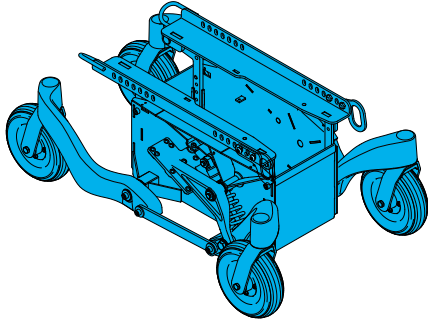




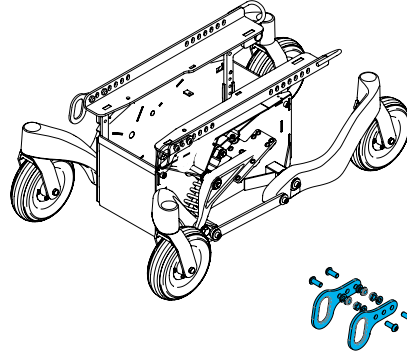

1600-168199

Quickie/Zippie Power, Mini Base, Tie Down Kit, Front Tie downs Only

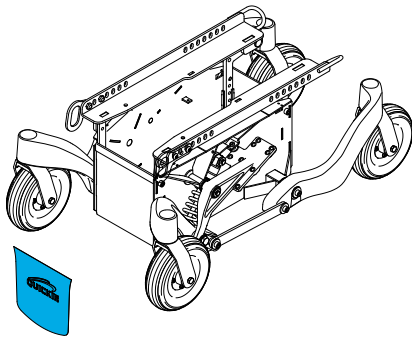

Quickie/Zippie Power, Mini Base, Base
 _____ Carbon Colour Effect




Colours	Product Codes
	1600-254004
	1600-254005
	1600-254006
	1600-254007

Suspension


1600-21501030

Quickie/Zippie Power, Mini Base, Suspension Rubber Bumper Kit

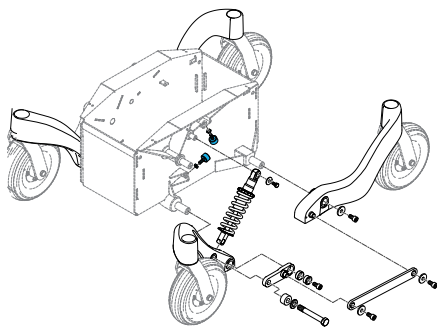


1600-21515000

Quickie/Zippie Power, Mini Base, Suspension Spring 350 LBs

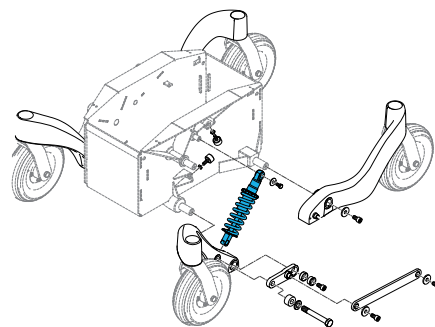


1600-21501046-021

Quickie/Zippie Power, Mini Base, Rear Left
Caster Arm Assembly, Black Textured

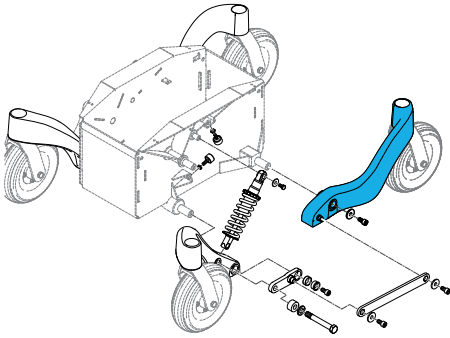


1600-21501046-022

Quickie/Zippie Power, Mini Base, Rear Right
Caster Arm Assembly, Black Textured

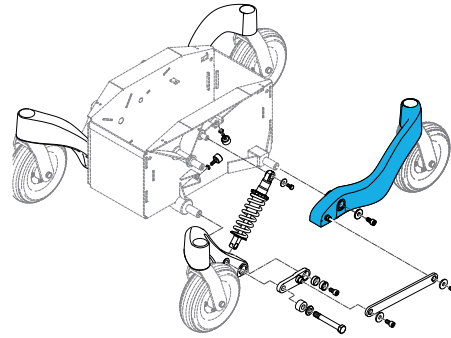


1600-21501042-022

Quickie/Zippie Power, Mini Base, Front Left
Caster Arm Assembly, Black Textured

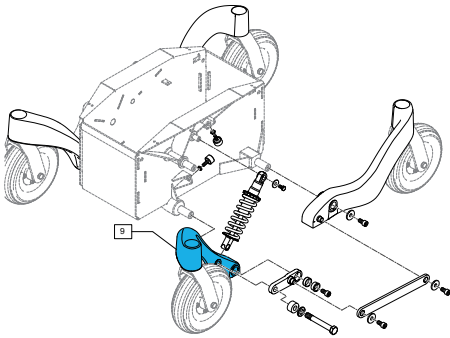


1600-21501042-021

Quickie/Zippie Power, Mini Base, Front Right
Caster Arm Assembly, Black Textured

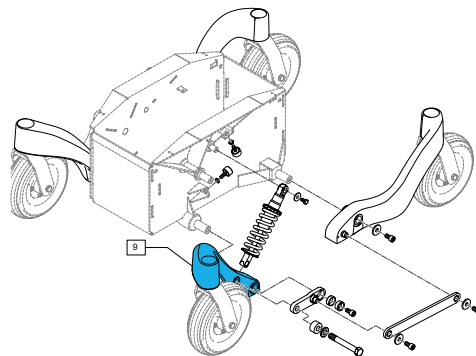


1600-21501034-001

Quickie/Zippie Power, Mini Base, Suspension
Midlink, Connects Front Suspension arm to Left
Suspension Arm

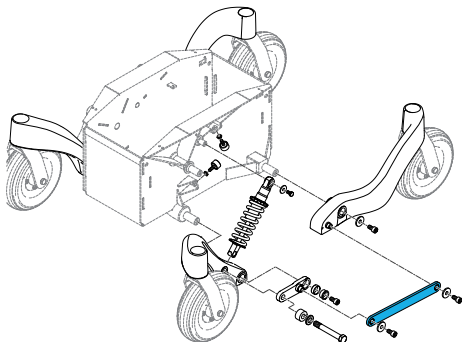


1600-21501036

Quickie/Zippie Power, Mini Base, Caster Bearing
Kit, Includes 2 Bearings & 1 Nut

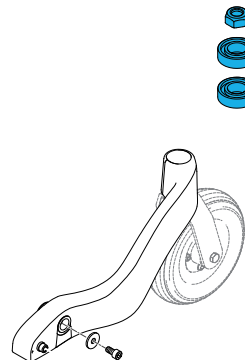


1600-21501037

Quickie/Zippie Power, Mini Base, Front Suspension Stop Bushing Kit

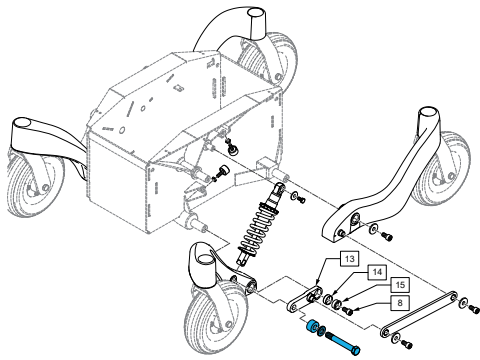


1600-21501038

Quickie/Zippie Power, Mini Base, Screw Suspension

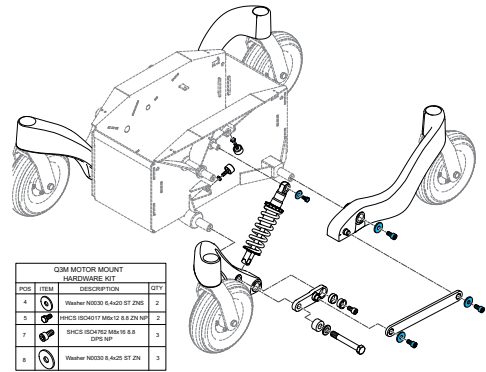

1600-168248


Quickie/Zippie Power, Mini Base, Suspension Hardware Kit

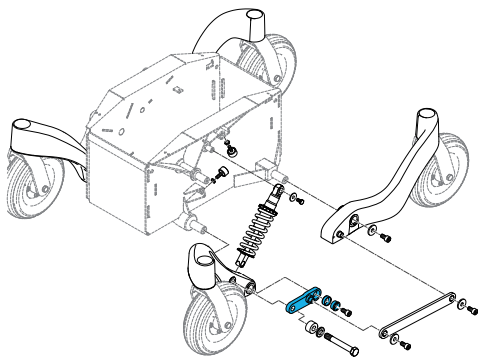

Castors


1600-247565

Quickie Power, Q400M 7" Solid Castor and Fork Assembly ea

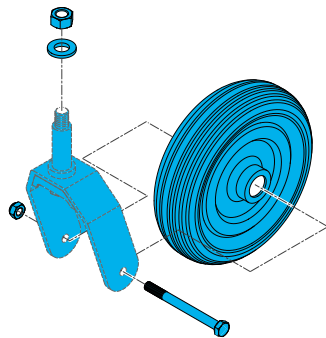

1600-247566


Quickie Power, Q400M 7" Castor with Hardware

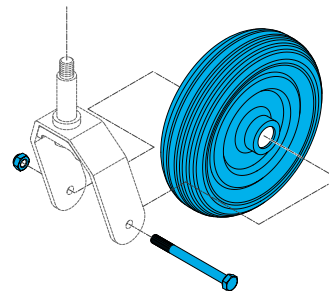

Drive Wheel


1600-243578

Quickie Power, 13" Drive Wheel, 2.5" Peumatic Tyre only with Rounded Tread, Per Each

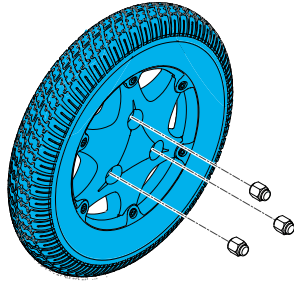


1600-244823

Quickie Power, 13" Pneumatic Drive Wheel with rounded tread, Per Each

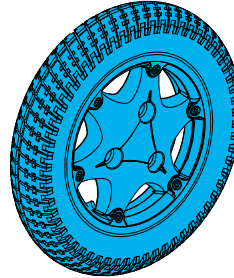


1600-161965

Quickie Power, Lug Nut Set (set of 3)

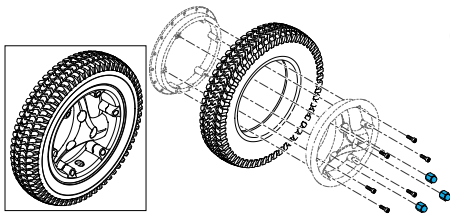


1600-126537

Quickie Power, 13" Knobby Dual Denisty (Solid) Tire Only with Rounded Tread, Per Each

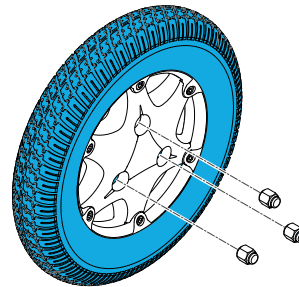


1600-247671

Quickie Power, 13" Drive Wheel, 2.5" Peumatic Tyre only with Rounded Tread, Per Each

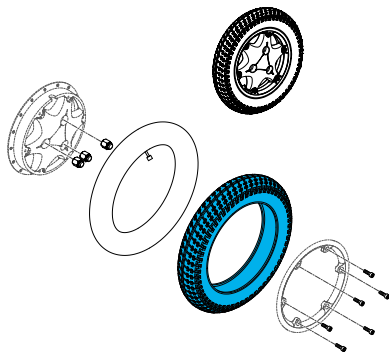


1600-249809

Quickie Power, 13" Drive Wheel Pneumatic Tube only, Per Each

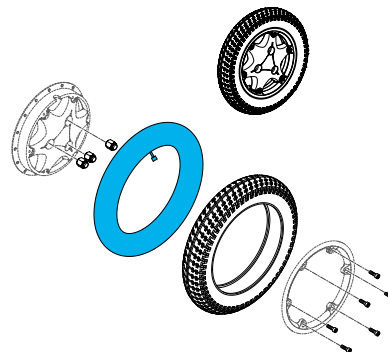

1600-163240


Quickie Power, Drive Wheel Hardware Kit (set of 6)

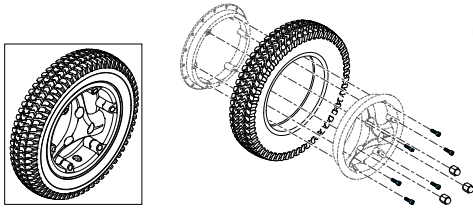

Motors


1600-250287

Quickie Power, 4 Pole 10 kph Motor Right Side

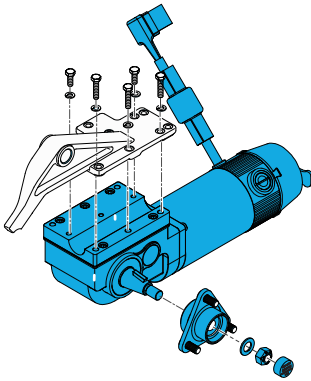


1600-250286

Quickie Power, 4 Pole 10 kph Motor Left Side

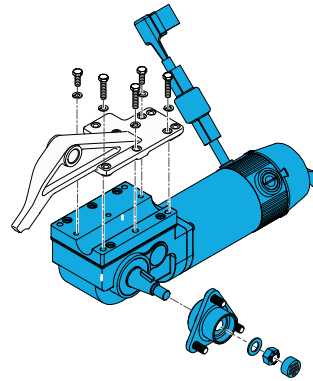


1600-349611

Quickie Power, Hub Nut Vinyl Cover

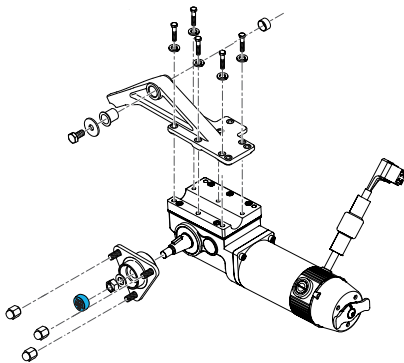


1600-112283

Quickie Power, 4 Pole Motor Brake Cover

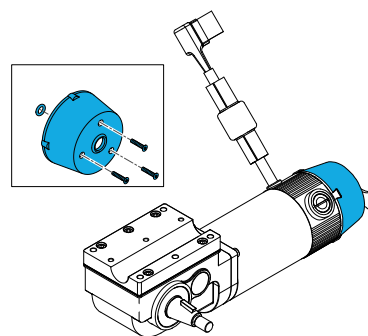


1600-21501055-001

Quickie Power, Mini Base, 4 Pole Motor Mount
Left Side

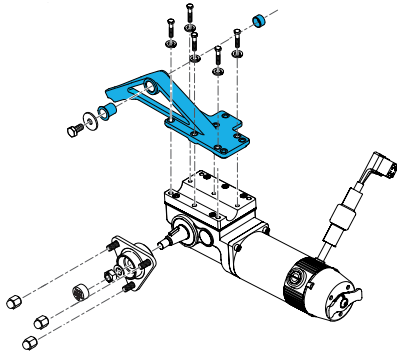


1600-21501055-002

Quickie Power, Mini Base, 4 Pole Motor Mount
Right Side

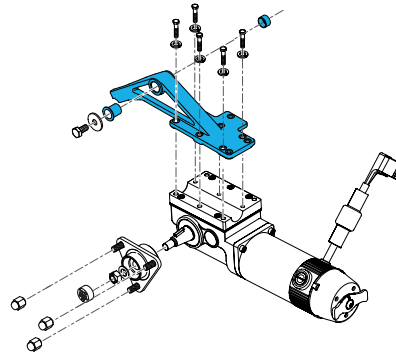


1600-164130

Quickie Power, 4 Pole Motor Mount Hardware

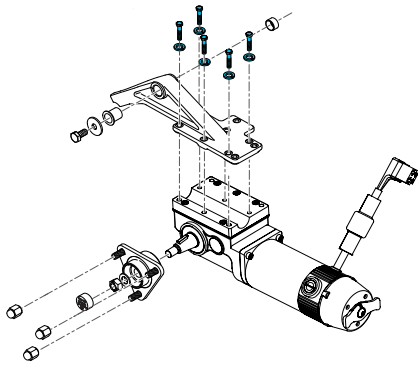


1600-112281

Quickie Power, 4 Pole Motor Brush Kit

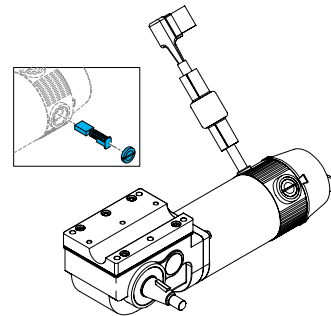


1600-124936

Quickie Power, Clip, Wire Saddle For Motor Cable

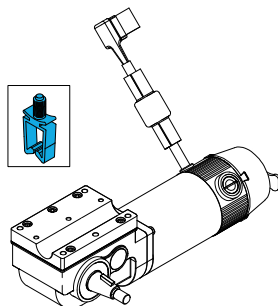


1600-252367

Quickie Power, 4 Pole Motor Brake Lever Set

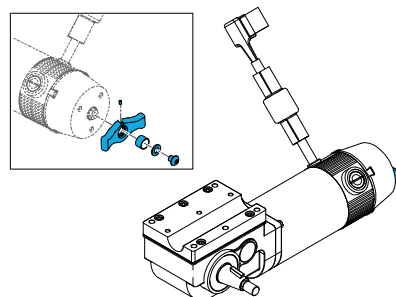


1600-161956
Quickie Power, 4Pole Motor 3 Spoke Hub Kit

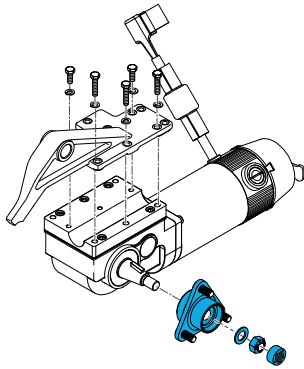







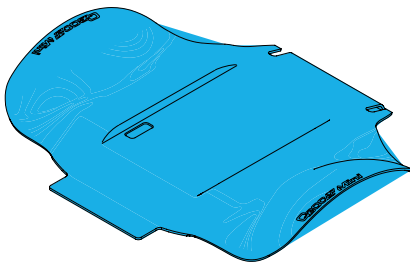


Shrouds

Mudguards







Quickie/Zipie Power, Q500Mini Main Shroud _____



Colours	Product Codes
	1600-168112
	1600-168113
	1600-168114
	1600-168115
	1600-168113
	1600-168195
	1600-168196
	1600-168197
	1600-168198

Quickie Power, Q500Mini Colour Accent Kit _____




Colours	Product Codes
	1600-161948
	1600-161949
	1600-161950
	1600-161951
	1600-161952
	1600-161953

Tilt & Lift Cassette

Assembly

1600-163327


Quickie Power, Sedeo Pro Tilt/Lift Combi Seating
Cassette Complete Assembly with Fasteners

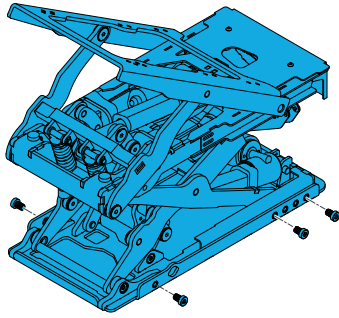

Parts


1600-163476

Quickie Power, Sedeo Pro Tilt Actuator Only
With Fasteners

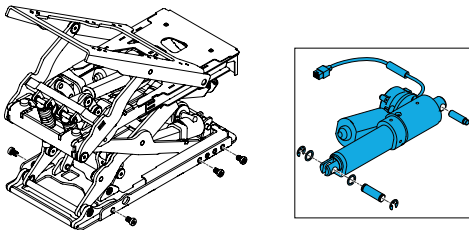


1600-163477

Quickie Power, Sedeo Pro Lift Actuator Only
With Fasteners

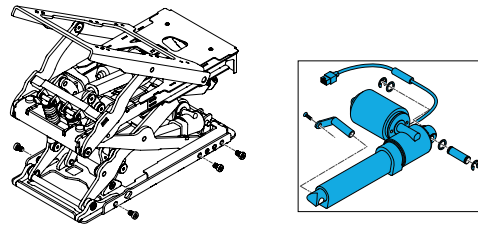


1600-20010139

Quickie Power, Sedeo Pro Seating Cable track
Cover only Incl mounting Fasteners

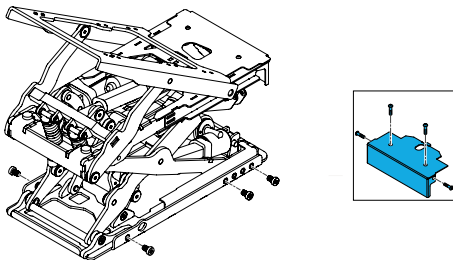


1600-248517

Quickie Power, Sedeo Pro Tilt/Lift Combo Har-
ness Wiring

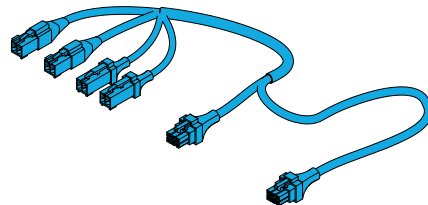


1600-248502

Quickie Power, Sedeo Pro Cassette Mounting Bolts
M8*125*12

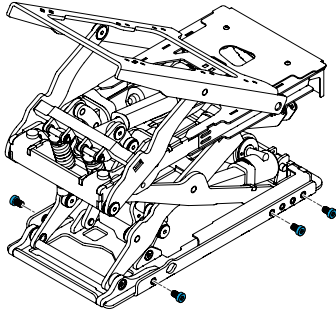

1600-20010137

Quickie Power, Sedeo Pro Cassette Cable Track Incl
mounting Fasteners

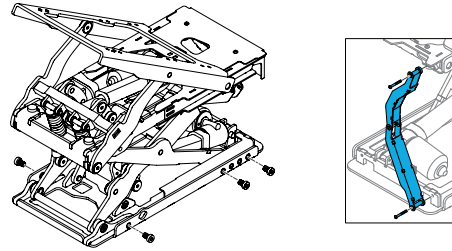


Sedeo Pro Seating

Assemblies


1600-163831

Quickie Power, Sedeo Pro, Tool Adjustable Fixed Back,
Short Seat Complete Assembly

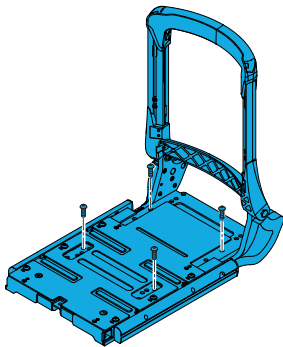


1600-163833

Quickie Power, Sedeo Pro, Tool Adjustable Fixed Back,
Long Seat Complete Assembly

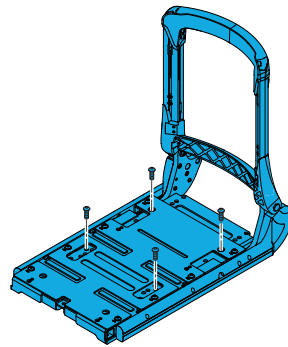


1600-163832

Quickie Power, Sedeo Pro, Tool Adjustable Fixed Back,
Short Seat Complete Assembly With Cantilever
Armrests

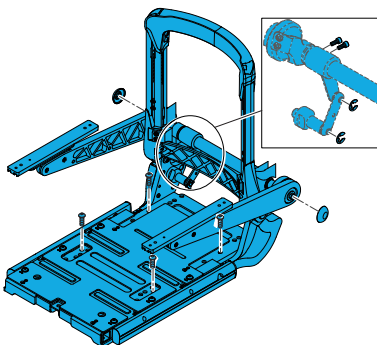


1600-163834

Quickie Power, Sedeo Pro, Tool Adjustable Fixed Back,
Long Seat Complete Assembly With Cantilever
Armrests

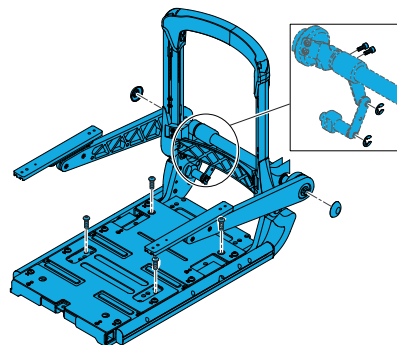


1600-163839

Quickie Power, Sedeo Pro, 130 Degree Power Recline, Short Seat Complete Assembly

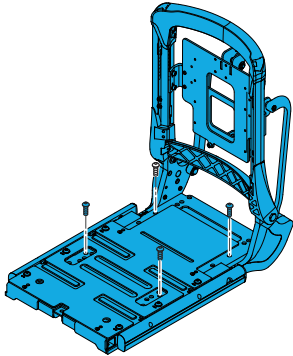


1600-163841

Quickie Power, Sedeo Pro, 130 Degree Power Recline, Long Seat Complete Assembly

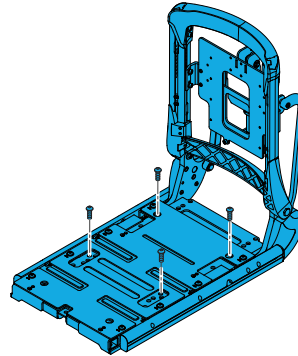


1600-163840

Quickie Power, Sedeo Pro, 130 Degree Power Recline, Short Seat Complete Assembly With Cantilever Armrests

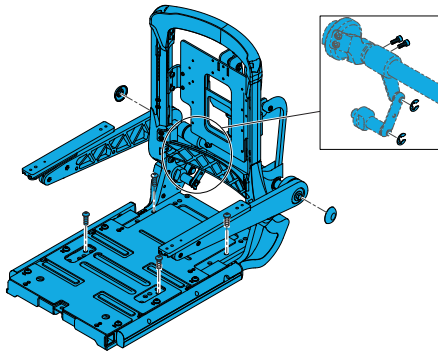


1600-163842

Quickie Power, Sedeo Pro, 130 Degree Power Recline, Long Seat Complete Assembly With Cantilever Armrests

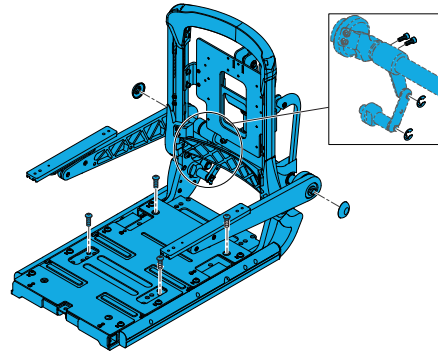


1600-163843

Quickie Power, Sedeo Pro, 150 Anti-Shear Degree Power Recline, Short Seat Complete Assembly

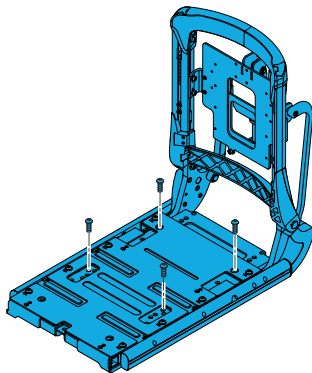


1600-163845

Quickie Power, Sedeo Pro, 150 Anti-Shear Degree Power Recline, Long Seat Complete Assembly

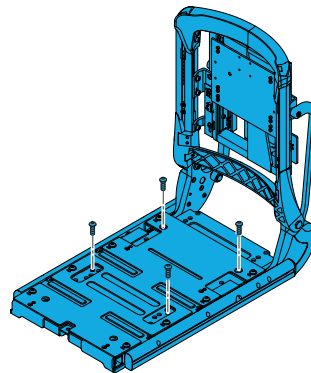


1600-163844

Quickie Power, Sedeo Pro, 150 Anti-Shear Degree
Power Recline, Short Seat Complete Assembly
With Cantilever Armrests

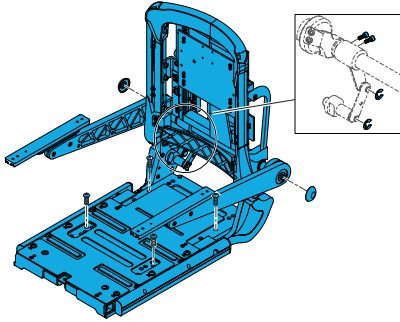

1600-163846


Quickie Power, Sedeo Pro, 150 Anti-Shear Degree
Power Recline, Long Seat Complete Assembly
With Cantilever Armrests

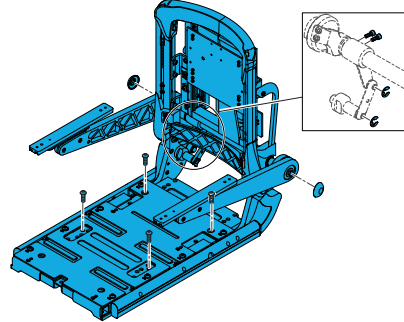

Short Seat Frame


1600-163323

Quickie Power, Sedeo Pro Short Seat Plate Hardware Kit

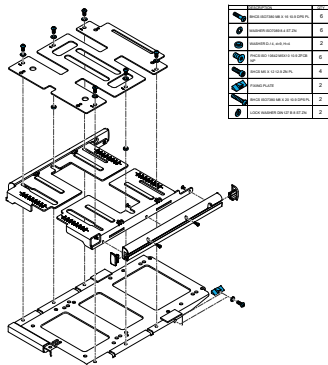


1600-22502019

Quickie Power, Sedeo Pro To Convert Seat Assembly from Long (19-22" Deep) to Short (16-20" Deep)

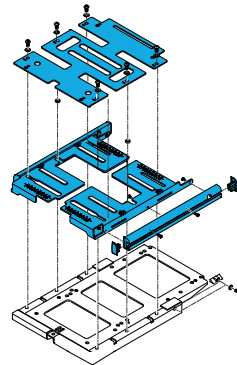


1600-22502000-001

Quickie Power, Sedeo Pro Seat Assembly Complete Assembly Short 16-20" Depth

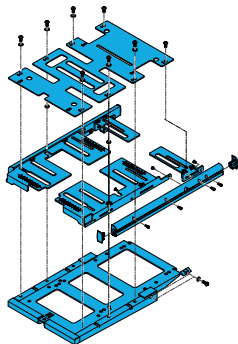

1600-22502018


Quickie Power, Sedeo Seat Rail Caps, Pair

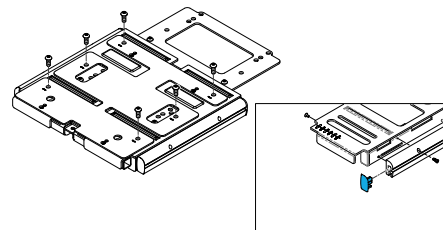

Long Seat Frame


1600-163324

Quickie Power, Sedeo Pro Long Seat Plate Hardware Kit

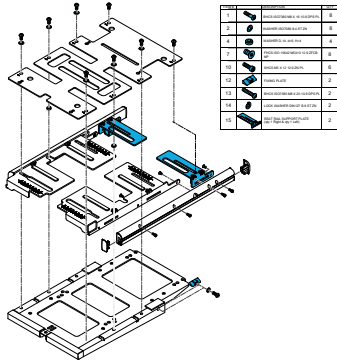


1600-22502015

Quickie Power, Sedeo Pro To Convert Seat Assembly from Short (16-20" Deep) to Long (19-22" Deep)

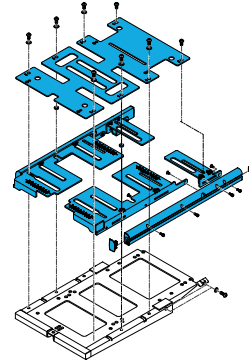


1600-22502000-002

Quickie Power, Sedeo Pro Seat Assembly Complete Assembly Long (19-22" Deep)

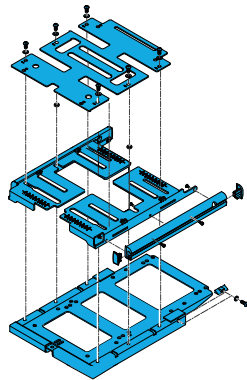


1600-341220

Quickie Power, Sedeo Pro Cable Clip, Used with 2 cables

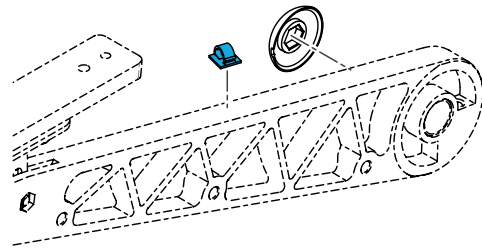


1600-560794

Quickie Power, Sedeo Pro Cable Clip, Used with 1 Cable

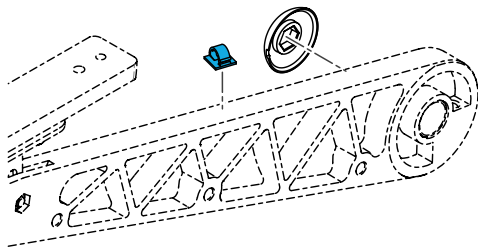

1600-22502018


Quickie Power, Sedeo Seat Rail Caps, Pair

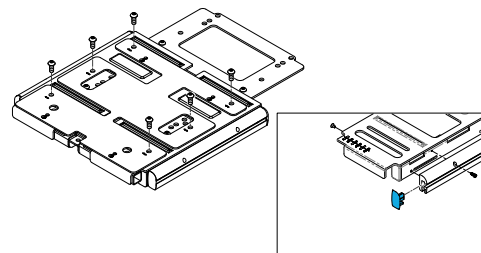

Back Frame Universal


1600-22503057

Quickie Power, Sedeo Pro Fixed Back Cover/Shroud

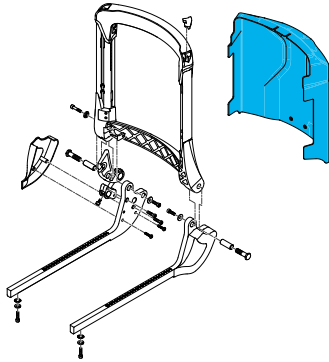


1600-22502024

Quickie Power, Sedeo Pro Shift Piece, Back Frame
Seat Rail Per Each, including Hardware

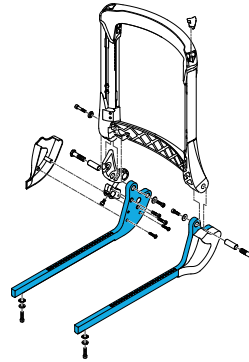


1600-22503020

Quickie Power, Sedeo Pro Back Frame Assembly

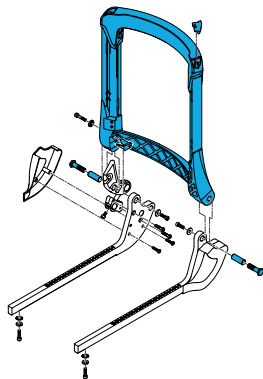


1600-163340

Quickie Power, Sedeo Pro Fixed Back, Shift Piece
Cover Kit

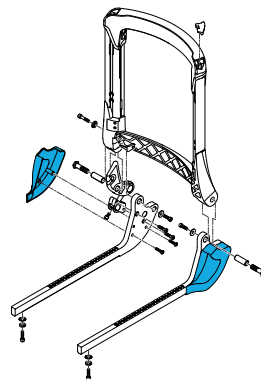


1600-22503116-001

Quickie Power Sedeo Pro Back Cane Cap LHS

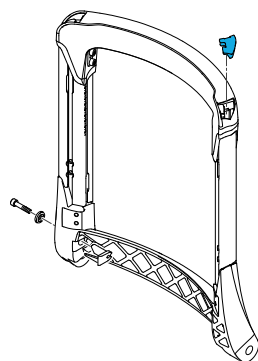


1600-22503116-002

Quickie Power Sedeo Pro Back Cane Cap RHS

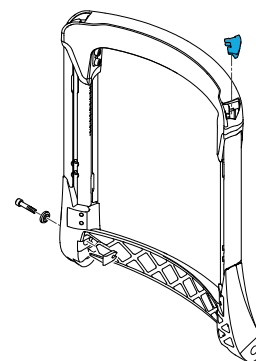


1600-163949

Quickie Power, Sedeo Pro Power Recline Back Quick Release Hardware Kit

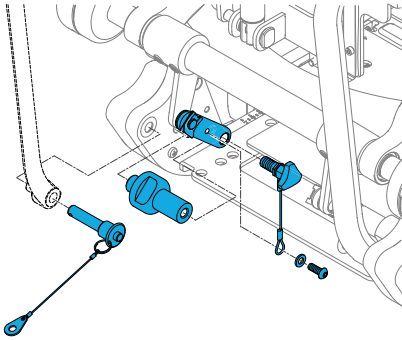

1600-163481


Sedeo Pro Trailing Push Handles, Pair

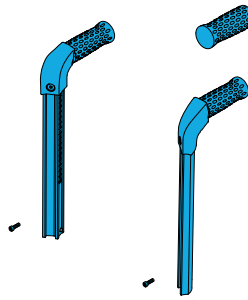

Tool Adjustable Fixed Back


1600-163948

Quickie Power, Sedeo Pro Quick Release Upgrade Assembly

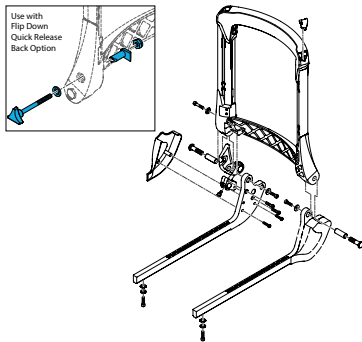


1600-22500101

Quickie Power, Sedeo Pro Fixed Backrest Complete Assembly

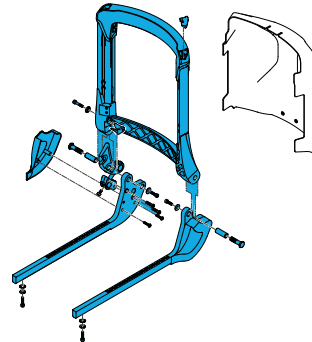


1600-22503210

Quickie Power, Sedeo Pro Fixed Backrest Recline Mechanism

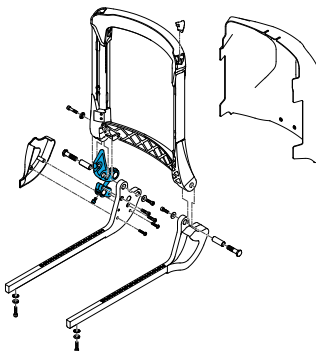


1600-163343

Quickie Power, Sedeo Pro Fixed Back Hinge/Axis Fasteners Kit

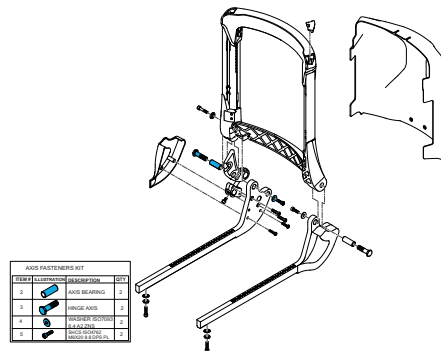

1579-248123


J3/Spex Sedeo Pro Fixed back adaptor 18" w

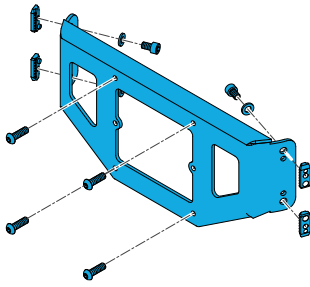

Power Semi-Recline (130 Degrees)


1600-22500102

Quickie Power, Sedeo Pro Power Semi-Recline Back Assembly (does not include Actuator)

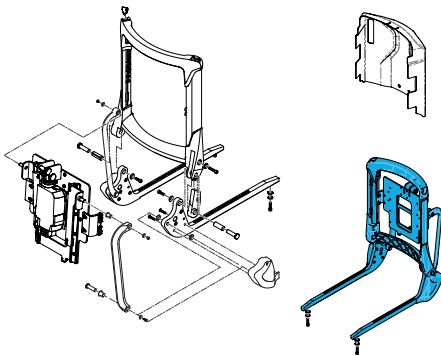


1600-163342

Quickie Power, Sedeo Pro Recline Back Bushing kit

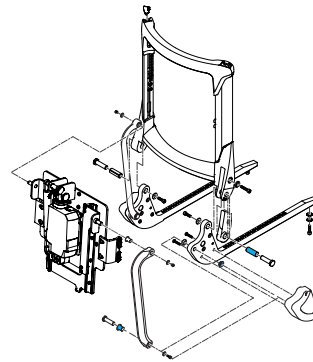


1600-22503300-001

Quickie Power, Sedeo Pro Power Semi-Recline Actuator Assembly

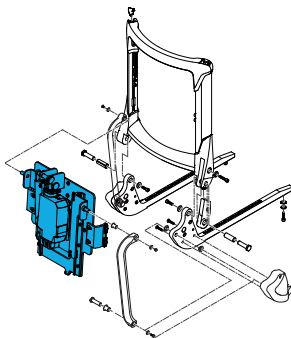


1600-22503056

Quickie Power, Sedeo Pro Power Recline Back Cover/ Shroud

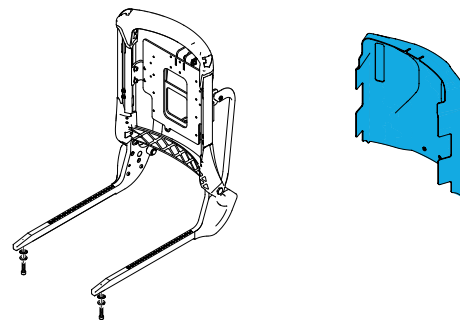


1600-163448

Quickie Power, Sedeo Pro Power Semi-Recline
Actuator Only

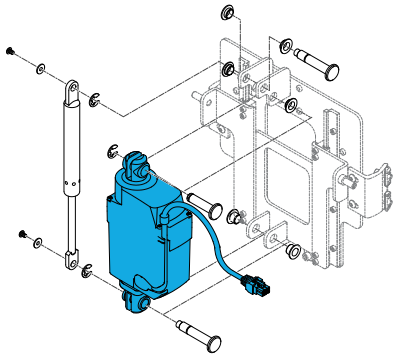


1600-163568

Quickie Power, Sedeo Pro Power Semi-Recline
Gas Strut Only

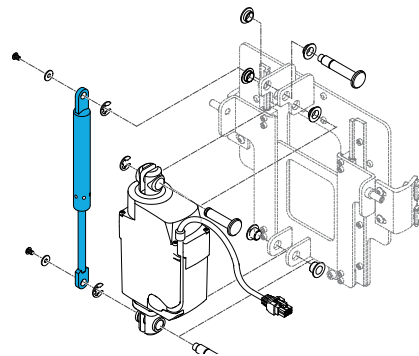


1600-163569

Quickie Power, Sedeo Pro Power Semi-Recline
Actuator Fasteners/Hardware Kit

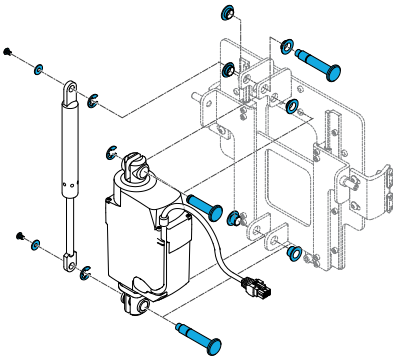


1600-22503100-002

Quickie Power, Sedeo Pro Power Recline Linkage
Assembly Right side

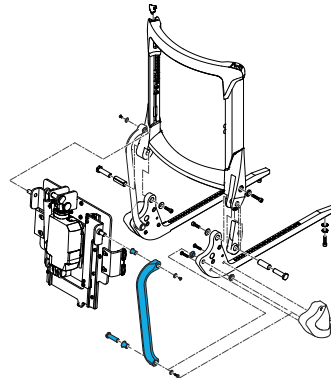


1600-22503100-001

Quickie Power, Sedeo Pro Power Recline Linkage
Assembly Left Side

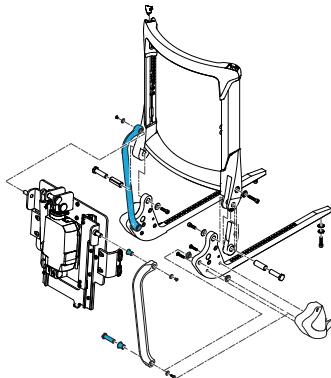

1600-248128


J3 Back Hardware Power Pro 20" (for 16" to 20"
width)

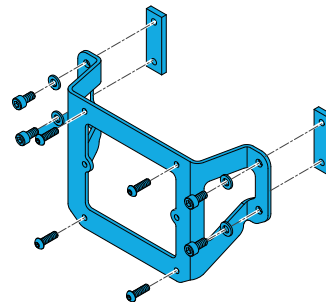

Power Recline (150°) with Anti Shear

1600-253371


Quickie Power, Sedeo Pro Power Shear-Recline Backrest Kit, Without Actuator, After S/N Q7MP-086838 / Q5MP-095960

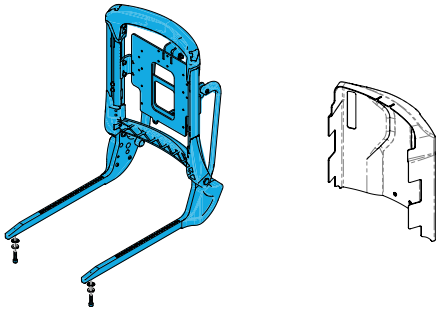

1600-163478


Quickie Power, Sedeo Pro Power Shear-Recline Actuator Only

BACKREST WITH SHEAR RECLINE ACTUATOR (150 DEGREE)
AFTER Q7MP-086838 / Q5MP-095960 without Actuator

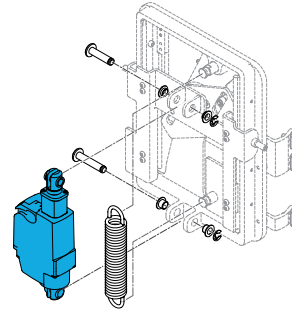


1600-163567

Quickie Power, Sedeo Pro Power Shear-Recline Backrest Actuator Gas Pull Spring

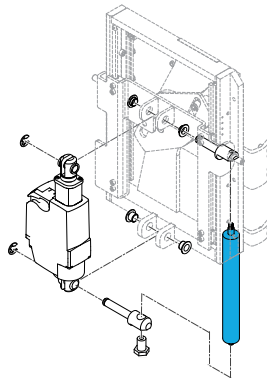


1600-22503000-001

Quickie Power, Sedeo Pro Power Shear-Recline Actuator Assembly

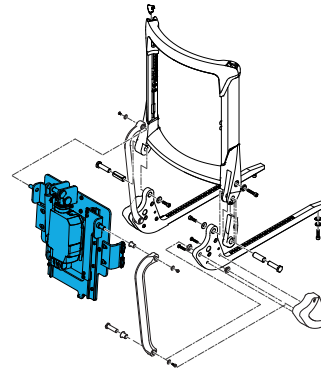


1600-22503056

Quickie Power, Sedeo Pro Power Recline Back Cover/ Shroud

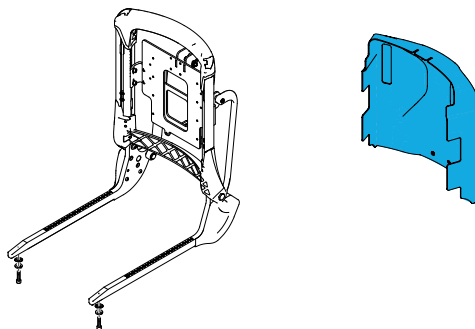


1600-22503100-002

Quickie Power, Sedeo Pro Power Recline Linkage Assembly Right side

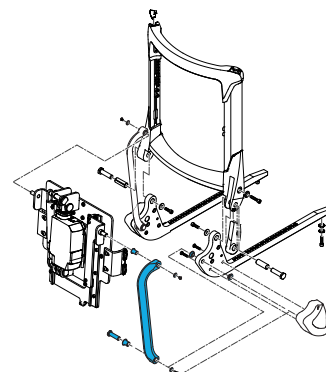


1600-22503100-001

Quickie Power, Sedeo Pro Power Recline Linkage Assembly
Left Side

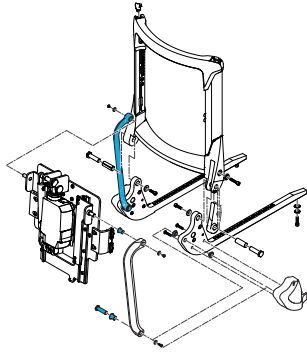

1600-248128

J3 Back Hardware Power Pro 20" (for 16" to 20" width)

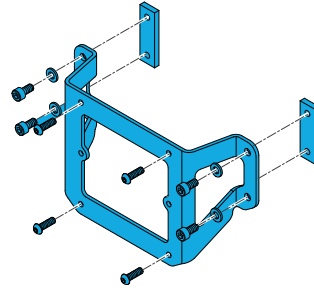


Sedeo Lite Seating

Seat


1600-22502018

Quickie Power, Sedeo Seat Rail Caps, Pair

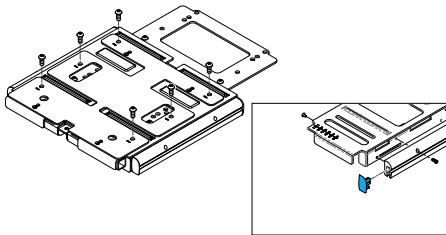

1600-22802000


Quickie Power, Sedeo Lite Seat Plate Assembly

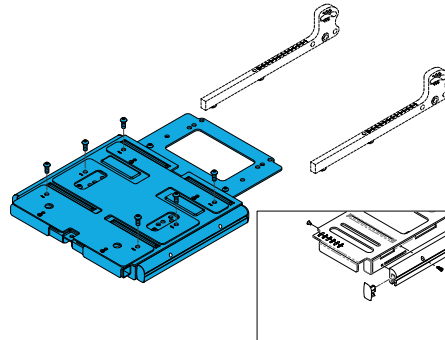

Back


1600-168121

Quickie Power, Sedeo Lite Fixed Backrest Quick
Release Lever Kit

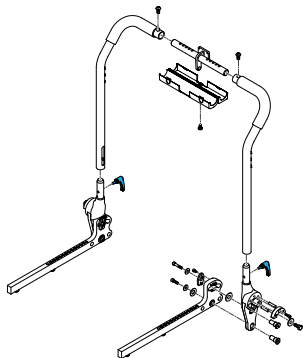


1600-168125

Quickie Power, Sedeo Lite Backrest Receiver Cover,
for use with Cantilever Arm

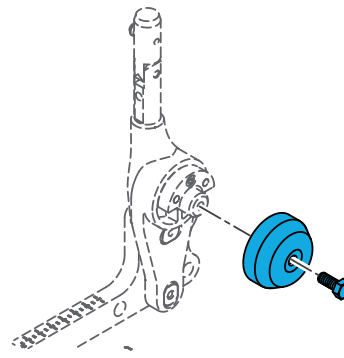


1600-22802001-001

Quickie Power, Sedeo Lite Backrest Frame Inter-
face Non Folding

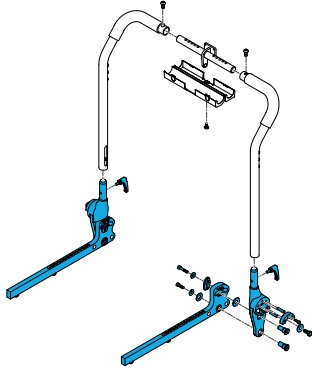


1600-22802001-011

Quickie Power, Sedeo Lite Backrest Frame Inter-
face with Folding Bracket

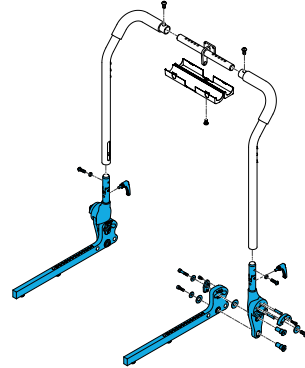


1600-22803010-011

Quickie Power, Sedeo Lite Back Frame Assembly
Standard Height, 15-19" Seat Width

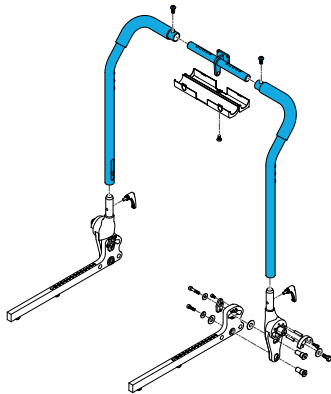


1600-22803010-012

Quickie Power, Sedeo Lite Back Frame Assembly
Standard Height, 18-20" Seat Width

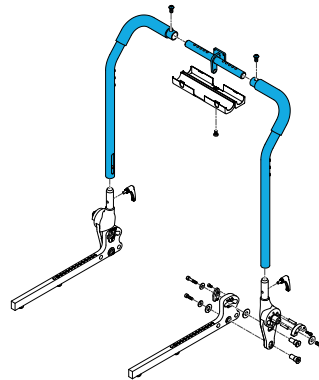


1600-168118

Quickie Power, Sedeo Lite Back Canes Center Ad-
justment Bracket Narrow, 15-19" Wide

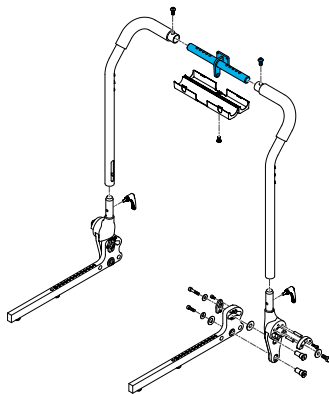


1600-168119

Quickie Power, Sedeo Lite Back Canes Center Ad-
justment Bracket Wide, 18-20" Wide

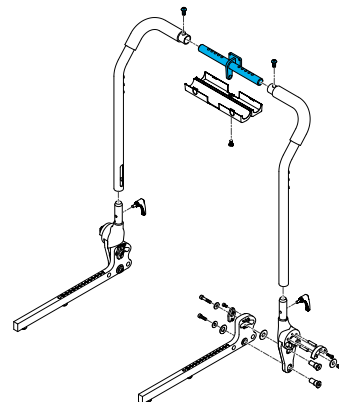

1600-168120

Quickie Power, Sedeo LiteBack Headrest Mounting
Bracket Cover

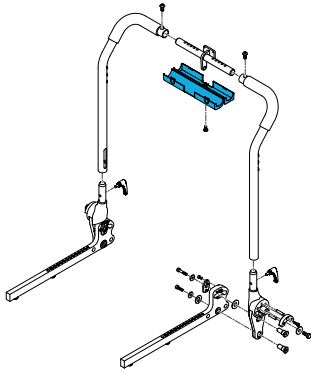


Sedeo Pro Armrests

Single Post


1600-163362

Sedeo Pro Single Post Height Adjustable Armrest
Right Side

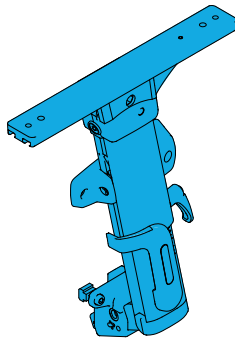


1600-163363

Sedeo Pro Single Post Height Adjustable Armrest
Left Side

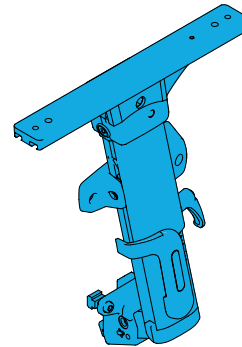


1600-163366

Sedeo Pro Flip Back Single Post Height Adjustable
Armrest Right Side

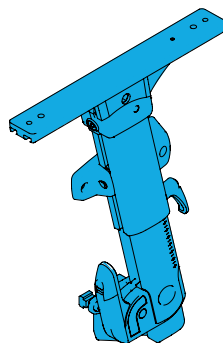


1600-163367

Sedeo Pro Flip Back Single Post Height Adjustable
Armrest Left Side

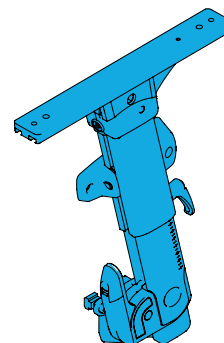


1600-163364
Quickie Power, Sedeo Pro Single Post Arm Rest,
Height Adjustment Handle Kit

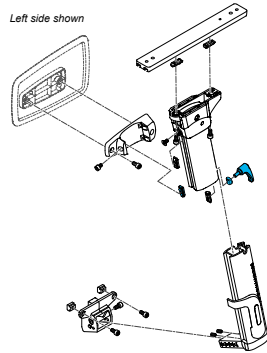


Cantilever Style Recline Back


1600-55-22504020-002
Quickie Power, Sedeo Pro Reclining Flip Back Arm-
Rest Assembly Right Side

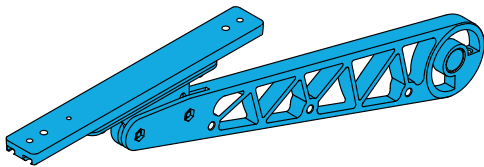


1600-55-22504020-001
Quickie Power, Sedeo Pro Reclining Flip Back Arm-
Rest Assembly Left Side

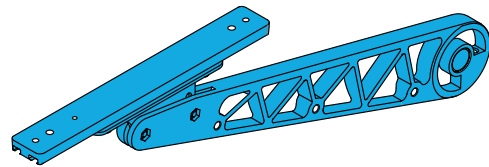

1600-55-22504222-001
Armrest Pivot Cover Black

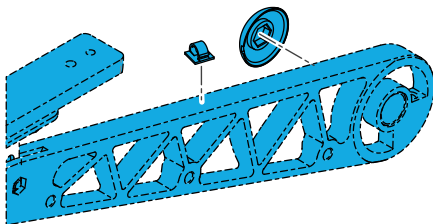


1600-22504110-001
Quickie Power, Sedeo Reclining Back Armrest
Receiver Assembly Left Side

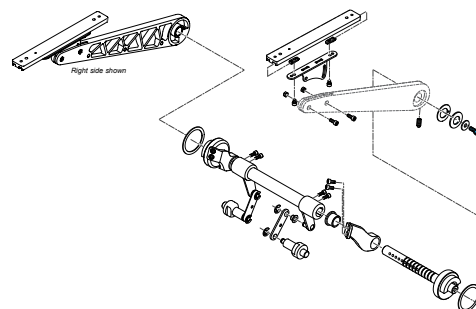


1600-22504110-002

Quickie Power, Sedeo Reclining Back Armrest Receiver
Assembly Right Side

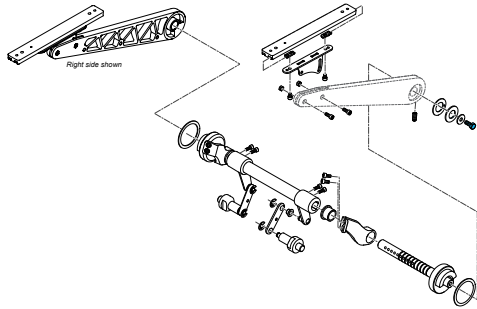


1600-168374

Quickie Power, Sedeo Pro Reclining Flip Back Armrest
Assembly

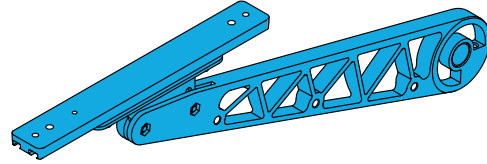

1600-164254


Quickie Power, Sedeo Pro Reclining Back, Cantilever
Arm Hardware Kit

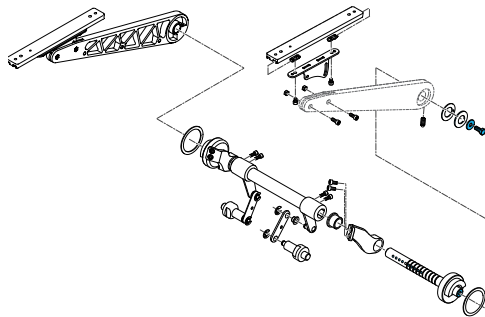

2 Pt Armrest Lock


1600-247172

Quickie Power 2nd Attachment point for Cantilever
Armrest LHS

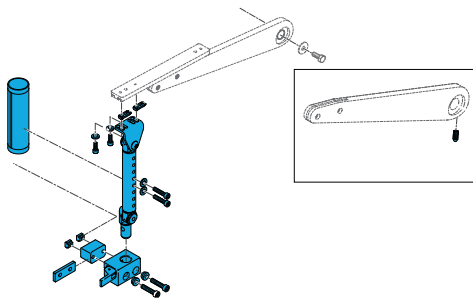

1600-247167


Quickie Power 2nd Attachment point for Cantilever
Armrest RHS

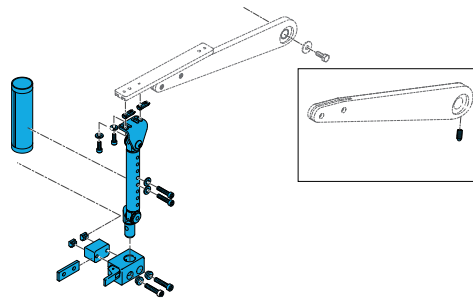

Sedeo Sideguards


1600-163441

Quickie Power, Sedeo Side Guard Plate Fasteners
for One Side

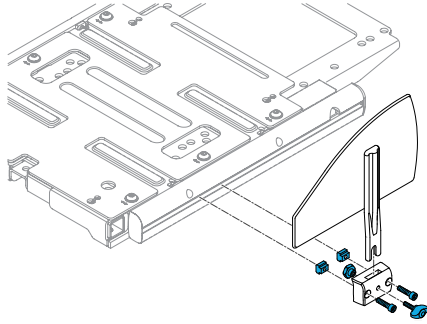


1600-22804030-001

Quickie Power, Sedeo Side Guard Receiver and
Assembly Left Side

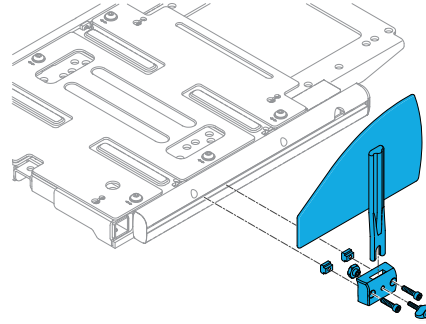

1600-22804030-002

Quickie Power, Sedeo Side Guard Receiver and
Assembly Right Side

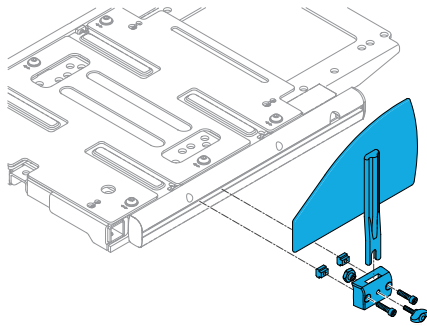


Sedeo Lite Armrests

Cantilever


1600-22804010-001

Quickie Power, Sedeo Lite Cantilever Armrest
Assembly Left Side

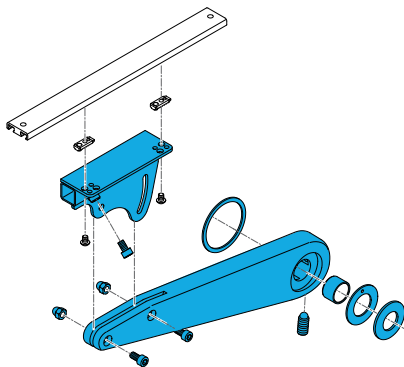


1600-22804010-002

Quickie Power, Sedeo Lite Cantilever Armrest
Assembly Right Side

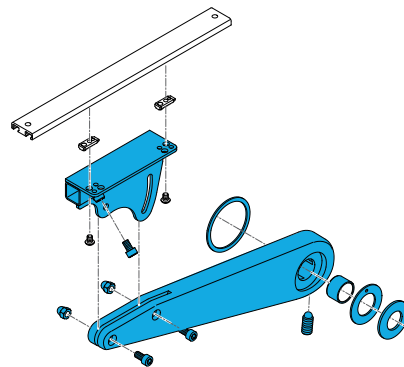


1600-22804061

Quickie Power, Sedeo Lite Cantilever Armrest
Rail for Sedeo Lite Armpad

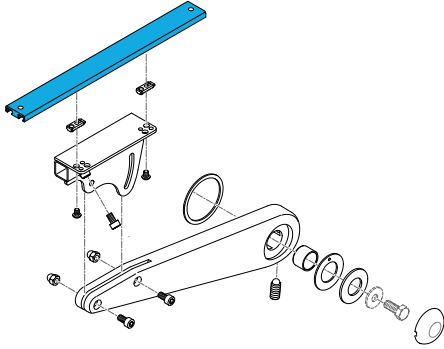

1600-164257


Quickie Power, Sedeo Lite Cantilever Armrest
Hardware Kit for Both Sides

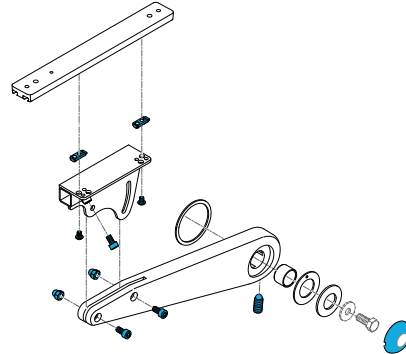

Mounting Parts


1600-248518

Quickie Power Sedeo Pro Armrest Adaptor Plate

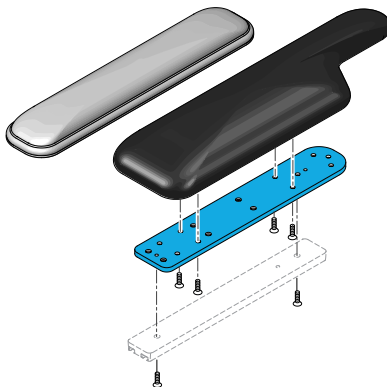


1600-103499-006

M5X16 8.8 DPS NP

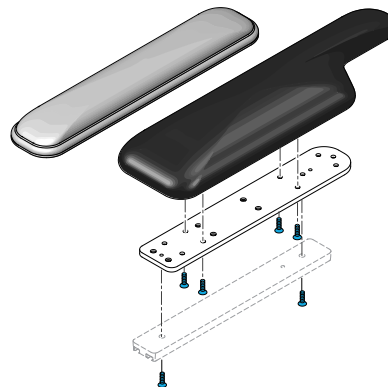


1600-341220

Quickie Power, Sedeo Pro Cable Clip, Used
with 2 cables

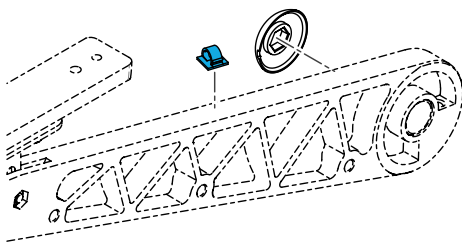


1600-560794

Quickie Power, Sedeo Pro Cable Clip, Used
with 1 Cable

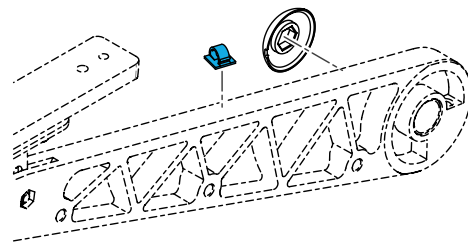

1600-22504042

Quickie Power Armpad Profile for Gel, Comfort
and Standard

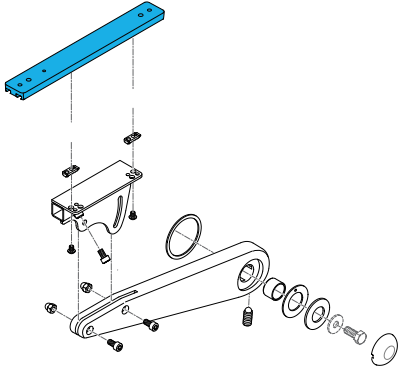


Armrest Pads

Standard Sedeo Armrest Pad


1600-163356

Quickie Power, Sedeo Pro Standard Armrest Pad
30 LH

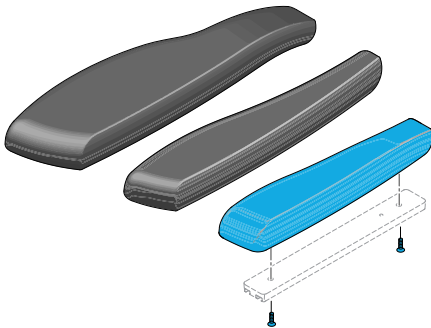


1600-163357

Quickie Power, Sedeo Pro Standard Armrest Pad
30 RH

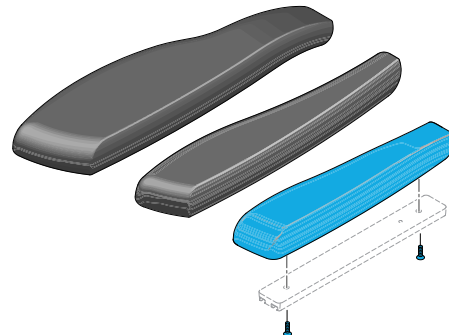


1600-163359

Quickie Power, Sedeo Pro Arm Pad 400 RHS

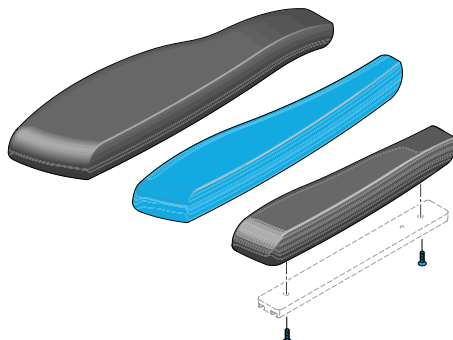


1600-163358

Quickie Power, Sedeo Pro Arm Pad 400 LHS

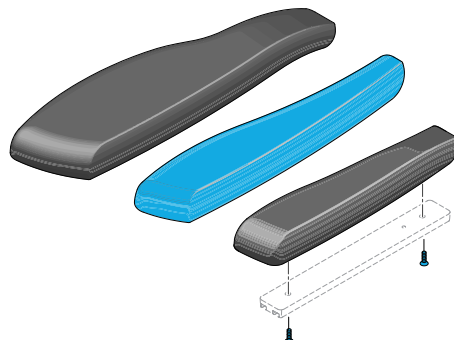


1600-163361

Quickie Power, Sedeo Large Standard Armrest Pad
45 RH

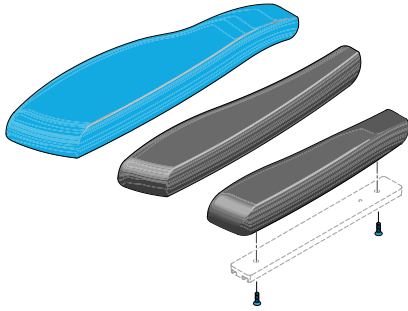

1600-163360


Quickie Power, Sedeo Large Standard Armrest Pad
45 LH

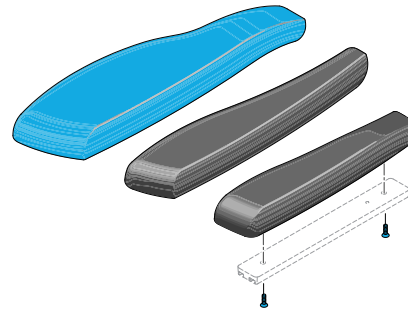

Comfort Gel Armrest Pad


1600-163345

Quickie Power, Sedeo Pro Comfort 2" Gel Pad RH

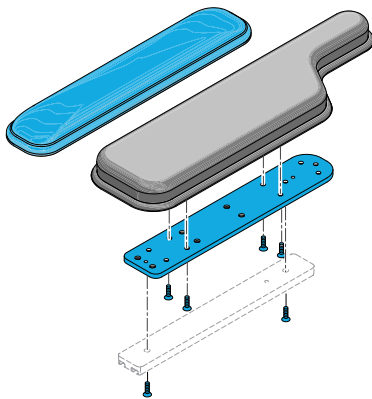


1600-163344

Quickie Power, Sedeo Pro Comfort 2" Gel Pad LH

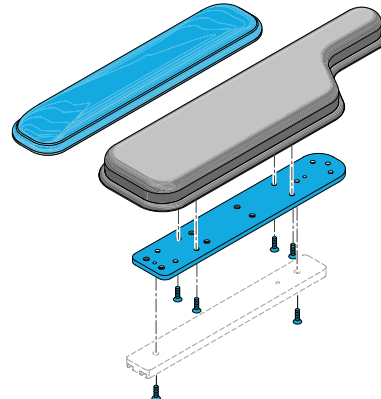


1600-163347

Quickie Power, Sedeo Pro Comfort 4" Gel Pad RH

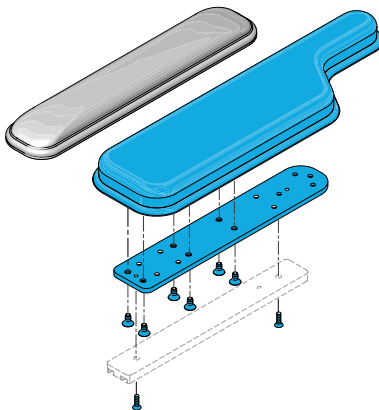

1600-163346


Quickie Power, Sedeo Pro Comfort 4" Gel Pad LH

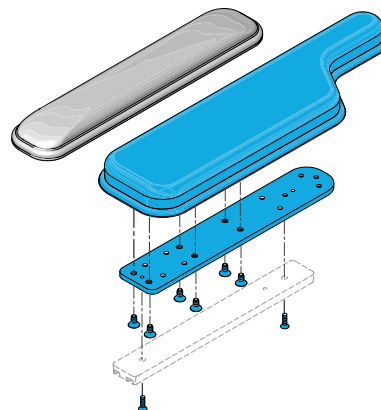

Comfort Foam Armrest Pad


1600-163352

Quickie Power, Sedeo Pro Comfort 2" Wide Foam Arm Pad, Left Side

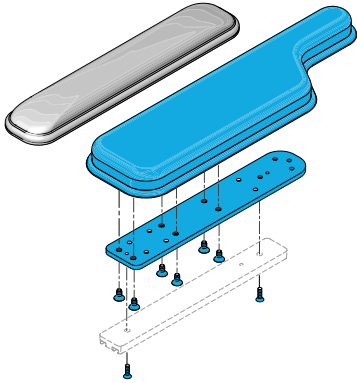


1600-163353

Quickie Power, Sedeo Pro Comfort 2" Wide Foam Arm Pad, Right Side

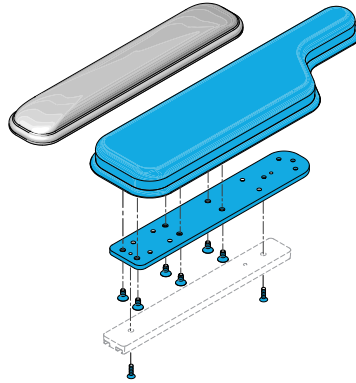


1600-163354

Quickie Power, Sedeo Pro Comfort 4" Wide Foam Arm Pad, Left Side

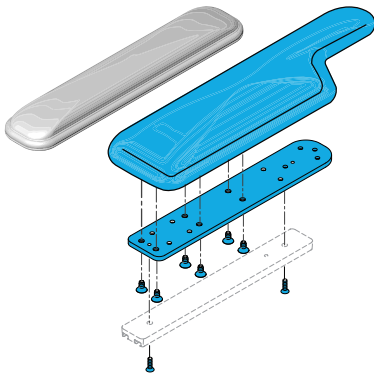


1600-163355

Quickie Power, Sedeo Pro Comfort 4" Wide Foam Arm Pad, Right Side

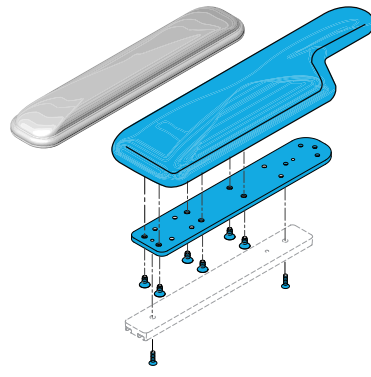

Gel Trough Armrest Pad

Quickie Power, Sedeo Pro ____ Gel Trough Assembly ____ Side,

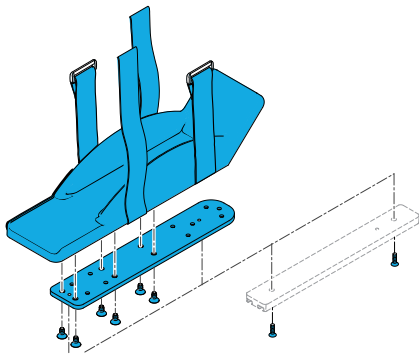


Size	Side	Width	Product Codes
Large	Left	4.5" x 14.5"	1600-163350
Large	Right	4.5" x 14.5"	1600-163351
Medium	Left	3.5" x 13"	1600-163348
Medium	Right	3.5" x 13"	1600-163349

Sedeo Pro Legrests

Swingaway Mounts


1600-163482

Quickie Power, Sedeo Pro/Lite Swingaway
Hanger Mount Right Side

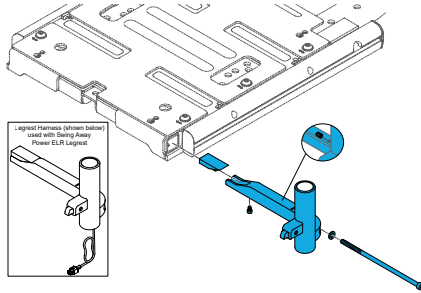


1600-163483

Quickie Power, Sedeo Pro/Lite Swingaway
Hanger Mount Left Side

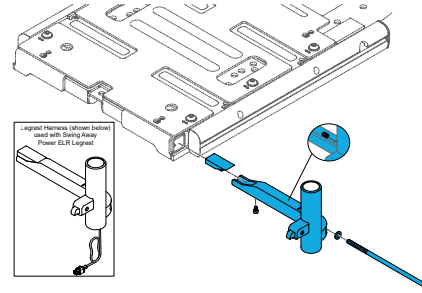

Swing Away 70 Degree Assembly

Quickie Power, Sedeo Pro Swingaway 70 Degree
hanger & Extension Mount Footplate Assembly for
_____ Side

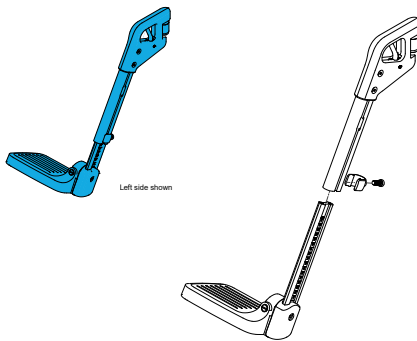


Size	Side	Product Codes
15"-16"	Left	1600-163370
15"-16"	Right	1600-163369
17"-18"	Left	1600-163372
17"-18"	Right	1600-163371
19"-20"	Left	1600-163374
19"-20"	Right	1600-163373
21"-22"	Left	1600-163376
21"-22"	Right	1600-163375

Swing Away 70 Degree Hangers


1600-22505215-001

Quickie Power, Sedeo Pro Swingaway 70 Degree
Hanger Only Left Side

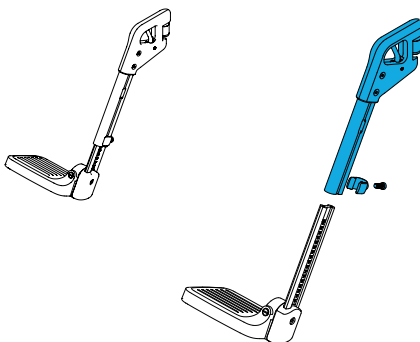


1600-22505215-002

Quickie Power, Sedeo Pro Swingaway 70 Degree
Hanger Only Right Side

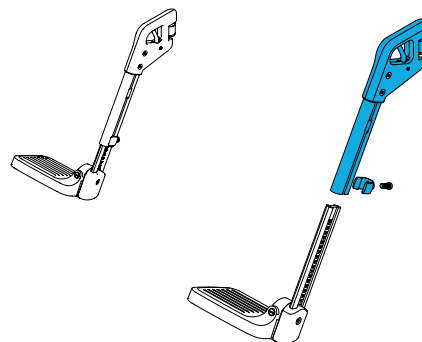

Swingaway Manual ELR Assembly

Quickie Power, Sedeo Pro Swingaway Manual ELR hanger & Extension Mount Footplate Assembly for _____ Side

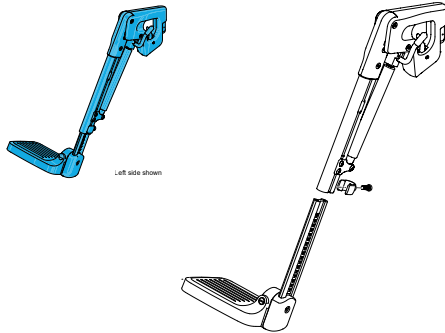


Size	Side	Product Codes
15"-16"	Left	1600-163378
15"-16"	Right	1600-163379
17"-18"	Left	1600-163380
17"-18"	Right	1600-163381
19"-20"	Left	1600-163382
19"-20"	Right	1600-163383
21"-22"	Left	1600-163384
21"-22"	Right	1600-163385

Swingaway Manual ELR


1600-22505214-001

Quickie Power, Sedeo Pro Swingaway Manual ELR Hanger Only Left Side

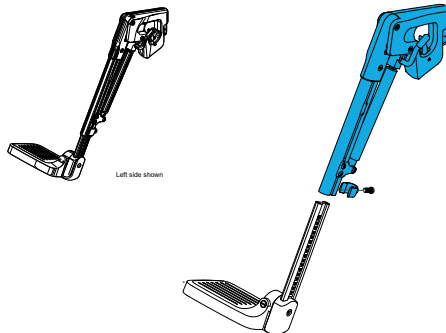


1600-22505214-002

Quickie Power, Sedeo Pro Swingaway Manual ELR Hanger Only Right Side

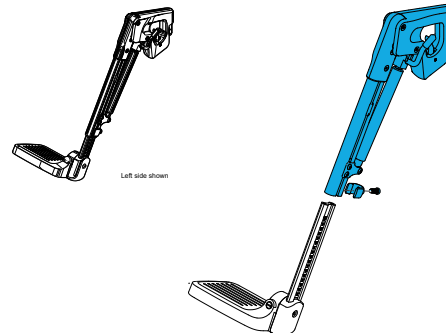

Swingaway Power ELR Assembly

Quickie Power, Sedeo Pro Swingaway Power ELR hanger & Extension Mount Footplate Assembly for _____ Side

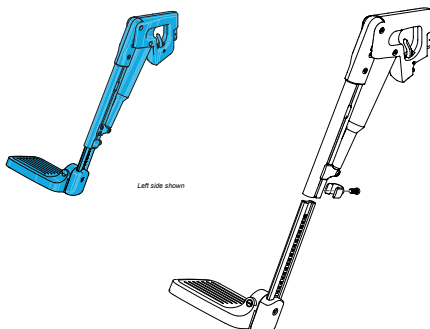


Size	Side	Product Codes
15"-16"	Left	1600-163386
15"-16"	Right	1600-163387
17"-18"	Left	1600-163388
17"-18"	Right	1600-163389
19"-20"	Left	1600-163390
19"-20"	Right	1600-163391
21"-22"	Left	1600-163392
21"-22"	Right	1600-163393

Swingaway Power ELR Parts


1600-22505216-001

Quickie Power, Sedeo Pro Swingaway Power ELR Hanger Only Left Side

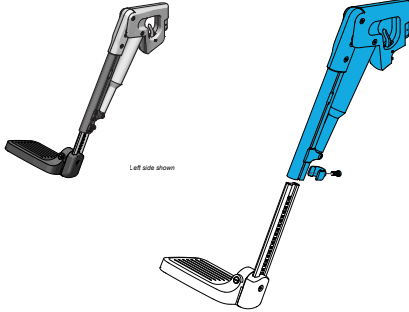


1600-22505216-002

Quickie Power, Sedeo Pro Swingaway Power ELR Hanger Only Right Side

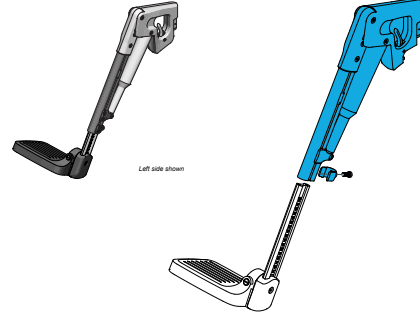


1600-22511004

Quickie Power, Sedeo Pro/Lite Swingaway Hanger Mount, Harness Cable

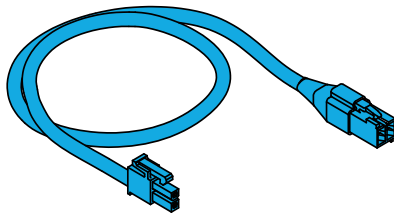


1600-22511005

Quickie Power, Sedeo Pro Swingaway Power ELR Right or Left Actuator-ISM Cable

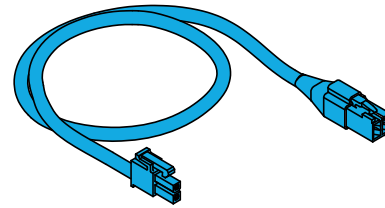

Swingaway Hanger Footplates


Quickie Power, Sedeo Pro Extension Mount Footplate Assembly _____" Universal Per Each

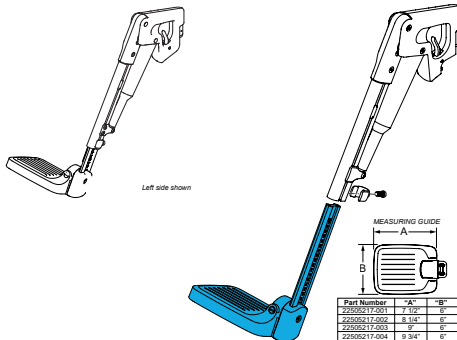

Width	Product Codes
15-16 "	1600-22505217-001
17-18 "	1600-22505217-002
19-20 "	1600-22505217-003
21-22 "	1600-22505217-004


1600-90005040

Quickie Power, Sedeo Pro Footplate Coupler Kit

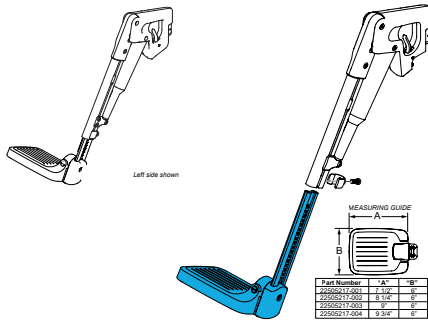

Knee & Calf Pads


Quickie Power, Sedeo Pro Upholstered Calf pad, Pair Assembly for _____" Wide Chairs, Includes Mounting Brackets

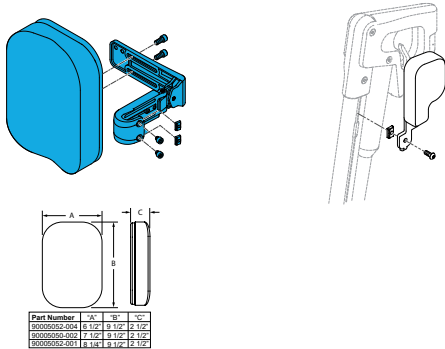

Width	Product Codes
15-16"	1600-163403
17-20"	1600-163404
21-22"	1600-163405


1600-9010430

Quickie Power, Sedeo Pro Integral Style Calf Pad Assm for 18-20" Wide Chairs

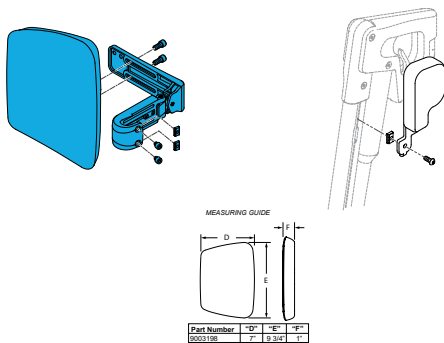


1600-9010160

Quickie Power, Sedeo Pro Calf Pad Mounting Bracket Assembly Each

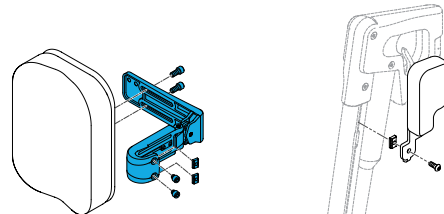


Quickie Power, Sedeo Pro Upholstered Calf Pad Pair for _____ " Wide Chairs

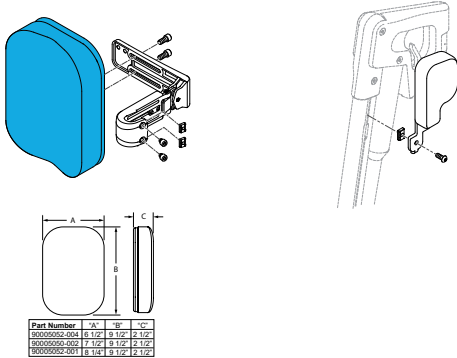

Width	Product Codes
15-16"	1600-90005052-004
17-20"	1600-90005050-002
21-22"	1600-90005052-001


1600-22509128-002

Quickie Power, Sedeo Pro Knee Pad/Cushion For Right Side Swingaway Hanger

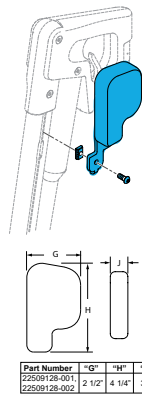


1600-22509128-001

Quickie Power, Sedeo Pro Knee Pad/Cushion For Left Side Swingaway Hanger

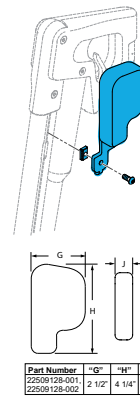

Power Centre Mount Mechanical Ext



Quickie Power, Sedeo Pro Power Center Mount Assembly _____ Footplate With _____ Calf Pads

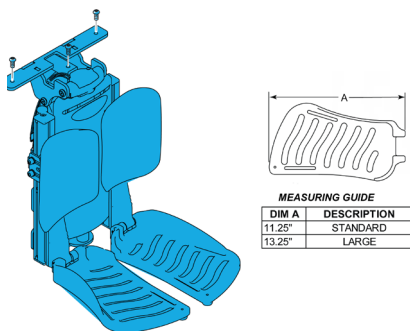

Footplates	Calf Pads	Product Codes
Standard	Standard	1600-163394
Large	Standard	1600-163395
Large	Large	1600-163396

Power Centre Mount ELR Power Ext

Quickie Power, Sedeo Pro Power Center Mount ELR with Power Ext Assembly _____ Footplate With _____ Calf Pads

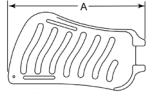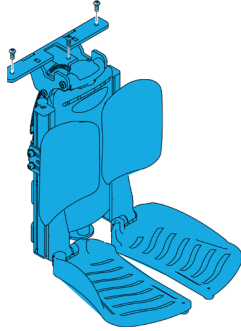

MEASURING GUIDE


DIM A	DESCRIPTION
11.25"	STANDARD
13.25"	LARGE

Footplates	Calf Pads	Product Codes
Standard	Standard	1600-163400
Large	Standard	1600-163401
Large	Large	1600-163402


Power Centre Mount Parts

1600-22505027

Quickie Power, Sedeo Pro Power Center Mount Calf Pad Assembly Per side, Order with 1600-131065

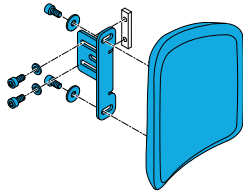


MEASURING GUIDE

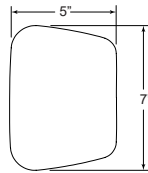


1600-131064

Quickie Power, Sedeo Pro M6 Channel Nut

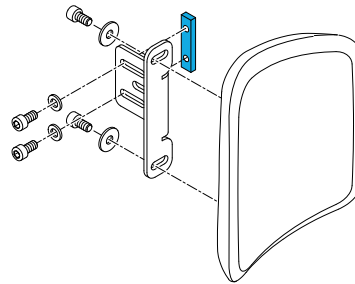

1600-136475

Quickie Power, Sedeo Pro Power Center Mount Shroud/Cover and Hardware

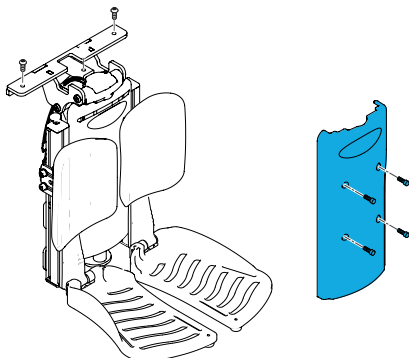


1600-136473

Quickie Power, Sedeo Pro Power Center Mount Small Parts Kit.

POWER CENTERMOUNT SMALL PARTS KIT							
ITEM #	ILLUSTRATION	DESCRIPTION	QTY	ITEM #	ILLUSTRATION	DESCRIPTION	QTY
Not Shown		TUBE FOOTREST EXTRUDE CAP	2 Right, 2 Left	10		SHCS M6 X 1 X 16 FULL, PL, SP	4
Not Shown		FHCS M5 X 0.8 X 10 PL ROHS	4	3		BEARING IGUS GFM-1012-05	4
Not Shown		BHCS, M5 X 0.8 X 8 FULL, PL, SP	4	Not Shown		BOSS ACTUATOR	4
8		M6 CHANNEL NUT FTPLT	4	Not Shown		FHCS, M5X0.8X25, FULL, PL, SP	4
Not Shown		SHCS, M5X0.8X20 FULL, PL	2	Not Shown		ACT-MBX1.25-X12 PL	1
Not Shown				Not Shown		SHCS, M5X0.8X16, FULL, NP, SP	2


1600-163193

Quickie Power, Sedeo Pro Power Center Mount, Standard
Footplate Kit Only

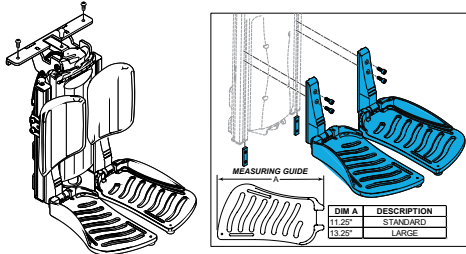


1600-163194

Quickie Power, Sedeo Pro Power Center Mount, Large
Footplate Kit Only

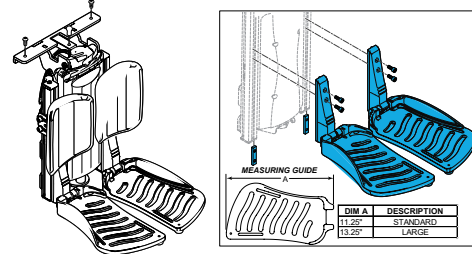


1600-130320

Quickie Power, Sedeo Pro Centre Mount Legrest Tube/Rail
fits Left or Right

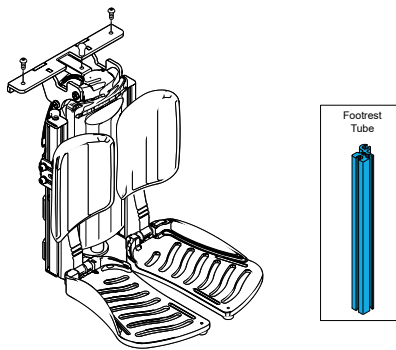


1600-249756

Quickie Power, Sedeo Pro Power Centre Mount Legrest
elevation Actuator Kit

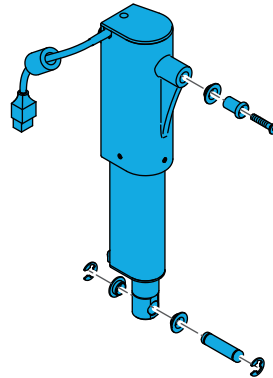


1600-163447

Quickie Power, Sedeo Pro Power Centre Mount Calf
Pad (6"x7") Assembly Right side

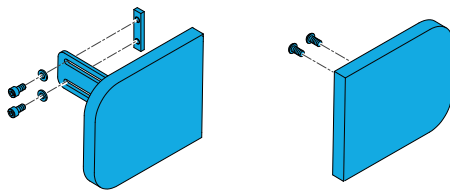


1600-163446

Quickie Power, Sedeo Pro Power Centre Mount Calf
Pad (6"x7") Assembly Left side

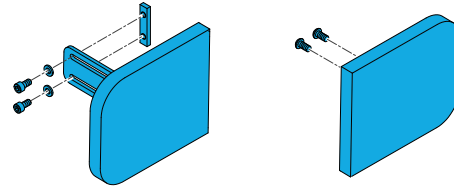

1600-130320


Quickie Power, Sedeo Pro Power Centre Mount Calf
Pad Mounting Kit Per Side

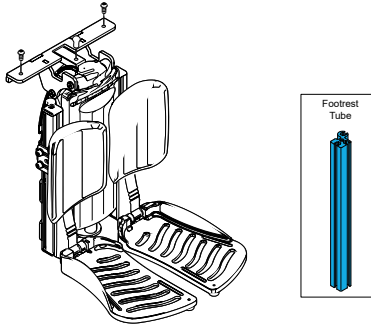

Power Centre Mount Wiring


1600-248515

Quickie Power, Sedeo Pro Power Center Mount Actua-
tor Harness/Cables Control Side

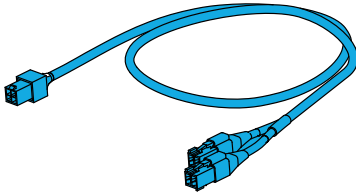


1600-248516

Quickie Power, Sedeo Pro Power Center Mount Actuator
Harness/Cables Actuator Side

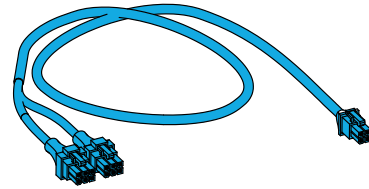


1600-248743

Quickie Power, Sedeo Pro Power Center Mount, Single
Actuator Channel 3 Actuator Cable 500mm

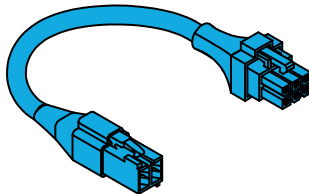


1600-22511004

Wiring for Power Swing-Away Legrest - IMS to Actua-
tor Harness

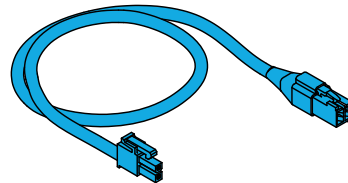

1600-22511005

Quickie Power, Sedeo Pro Power Swingaway Hangers
Actuator Harness/Cables Legrests to ISM

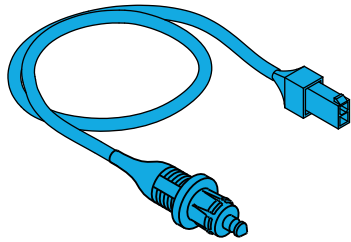


Sedeo Lite Legrests

Angle Adjustable Centre Mount


1600-164260

Quickie Power, Sedeo Lite Angle Adjustable Center
Mount Legrest

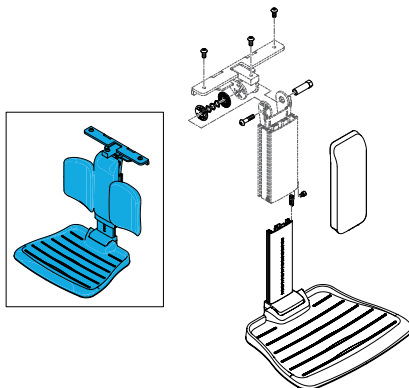


1600-22505115

Quickie Power, Sedeo Lite Angle Adjustable Center
Mount Legrest Calf Plate Assembly

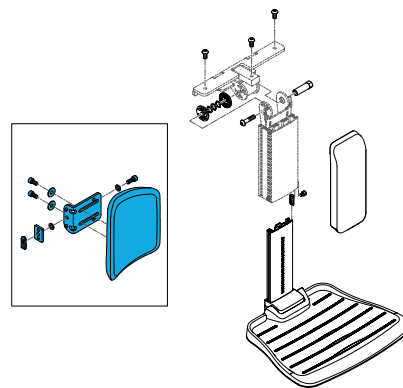

1600-253789


Quickie Power, Sedeo Lite Angle Adjustable Center
Mount Legrest Hardware Kit

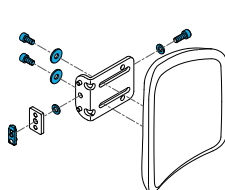

HARDWARE KIT	
	CLAMP NUT PROFILES 1
	SHCS ISO4762 M6x12 8.8 DPS NP 2
	WASHER ISO7093 6.4 A2 ZNS 2
	SHCS ISO4762 M6x16 12.9 DPS NP 1
	WASHER ISO7089 6.4 A2 ZNS 2


1600-22505105

Quickie Power, Sedeo Lite Angle Adjustable Center
Mount Footrest lower Assembly Including Footplate

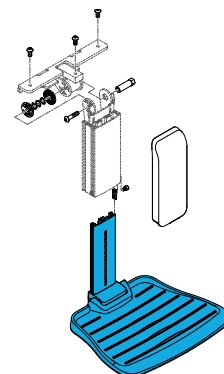


1600-22505116

Quickie Power, Sedeo Lite Angel Adjustable Center
Mount Footrest Adjustment Wheel Kit

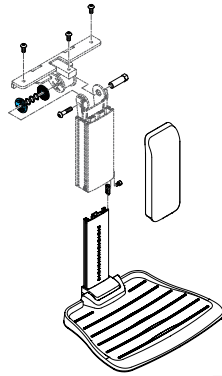

1600-253788


Quickie Power, Sedeo Lite Angle Adjustable Centre
Mount Footrest Hardware Kit

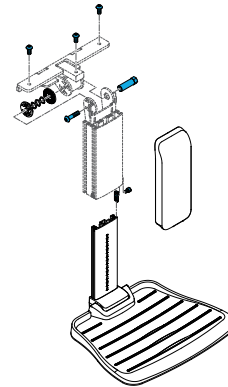

Fixed Centre Mount Assembly


1600-163573

Quickie Power, Sedeo Pro/Lite Fixed Centre
Mount Short

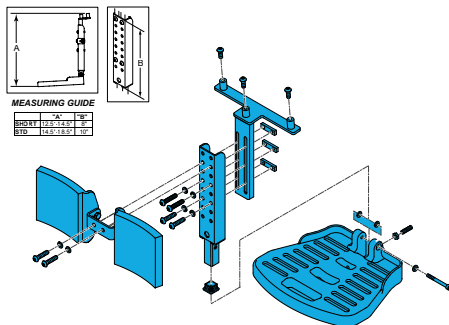

1600-163574


Quickie Power, Sedeo Pro/Lite Fixed Centre
Mount Long

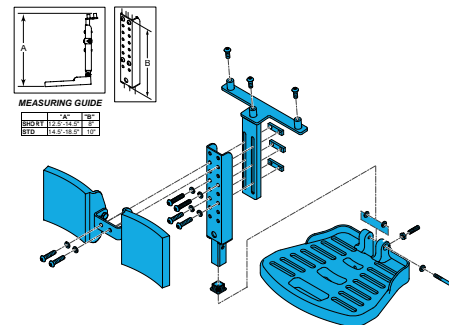

Fixed Centre Mount Parts


1600-103118

Quickie Power, Sedeo Pro/Lite Fixed Centre Mount,
Calf Pad and Hardware

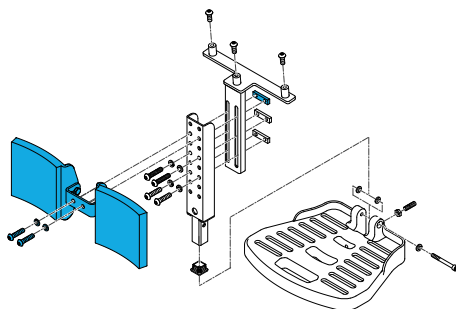


1600-101685

Quickie Power, Sedeo Pro/Lite Fixed Centre Mount
Footplate Assembly

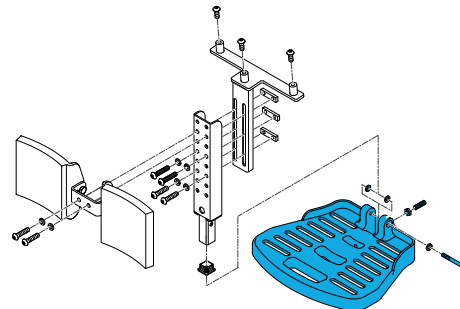

Fixed Centre Mount Dual Footplate

Quickie Power, Sedeo Pro Fixed Center Mount Assembly _____ Dual Footplates With _____ Calf Pads

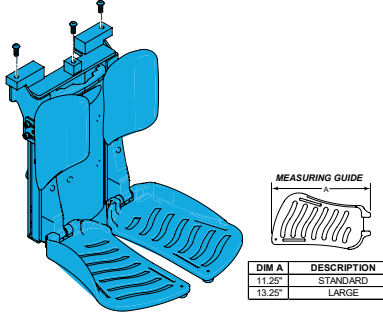


Footplates	Calf Pads	Product Codes
Standard	Standard	1600-163453
Large	Standard	1600-163454
Large	Large	1600-163455

Parts


1600-136475

Quickie Power, Sedeo Pro Fixed Center Mount Dual Footplate Shroud/CoverKit With Fasteners

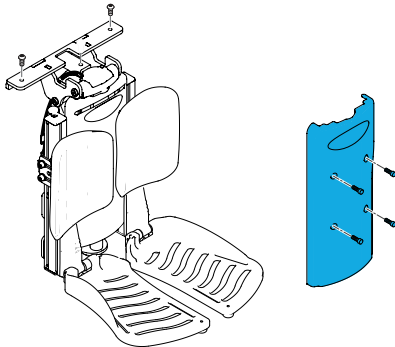


1600-136489

Quickie Power, Sedeo Pro Fixed Center Mount Dual Footplate Small Parts Kit

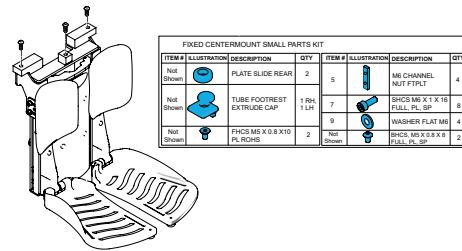


1600-163193

Quickie Power, Sedeo Pro Fixed Center Mount Dual Footplate, Replacement Standard Footplate Kit, Pair

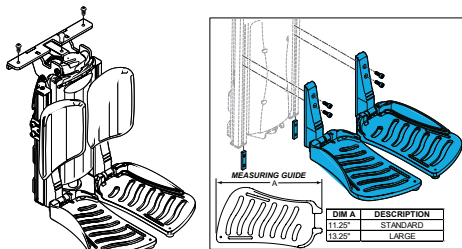


1600-163194

Quickie Power, Sedeo Pro Fixed Center Mount Dual Footplate, Replacement Large Footplate Kit, Pair

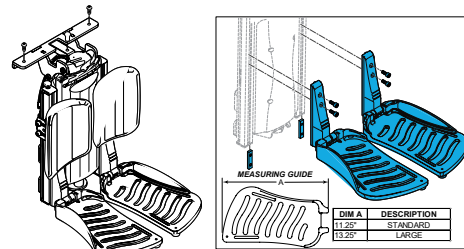

1600-130320


Quickie Power, Sedeo Pro Centre Mount Legrest
Tube/Rail fits Left or Right

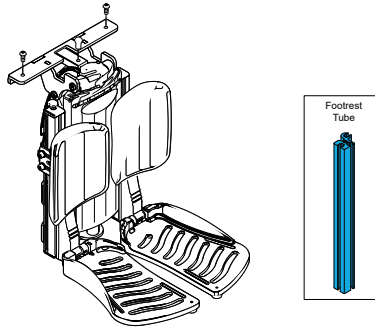

Knee Pads


1600-125582

Quickie Power, Sedeo Pro Rail Mounted Knee Support
Assembly Right Side, With 4" Pad

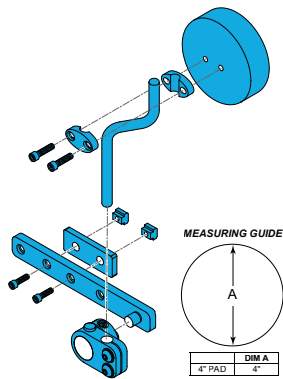


1600-125583

Quickie Power, Sedeo Pro Rail Mounted Knee Support
Assembly Left Side, With 4" Pad

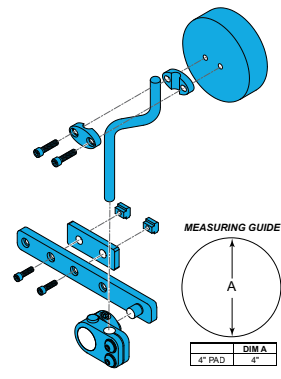


1600-KAD11XX

Quickie Power, Sedeo Pro Rail Mounted Knee Support
4" Pad only

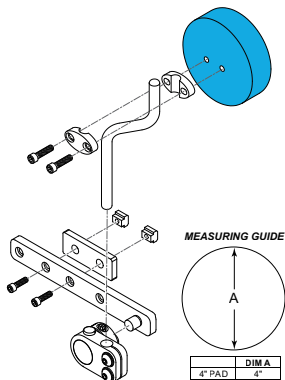


1600-22505200-002

Quickie Power, Sedeo Pro Swingaway Amputation
Support Right Side

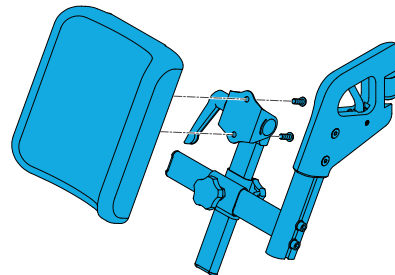

1600-22505200-001


Quickie Power, Sedeo Pro Swingaway Amputation
Support Left Side

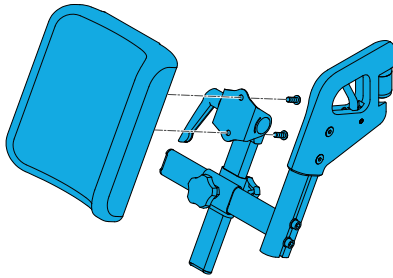

Headrests


1480-1010957

Quickie Power, Sedeo Standard Headrest

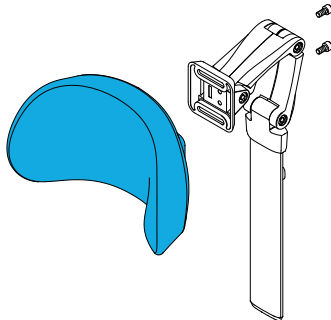


1480-248225

Quickie Power, Pro Back Headrest Moun

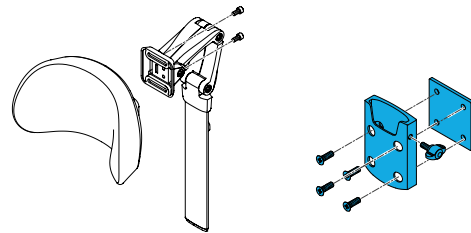


1480-21809210

Quickie Power, Sedeo Headrest Mount

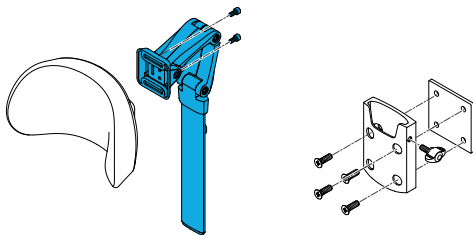


1480-245912

Quickie Power, Sedeo-Axys Headrest Assembly

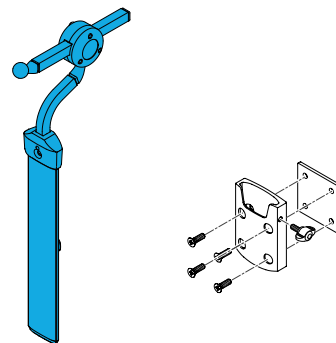


1480-245914

Quickie Power, Sedeo Linx Headrest Assembly

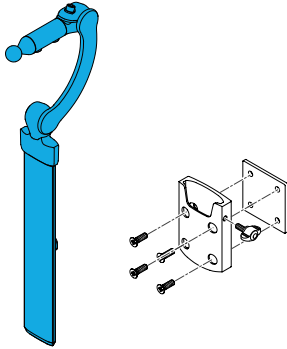


1480-245915

Quickie Power, Sedeo-Cobra Flip Headrest Assembly

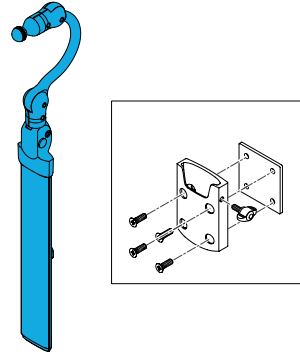


1480-248715

Quickie Power, Pro J3 Back Hedrest Thick Mount;
16" - 18" width

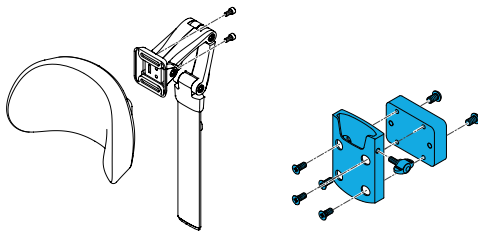

1480-248716

Quickie Power, Wide Pro J3 Back Headrest Thick Mount;
20" - 22" width

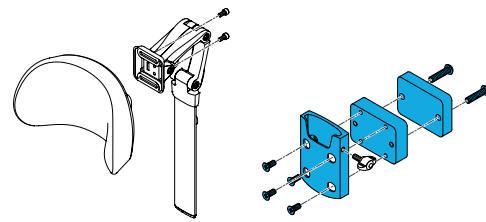


Joystick

CJSM2

1615-D51595


RNET, CJSM2 Joystick, Complete with Mk1 Button Keypad

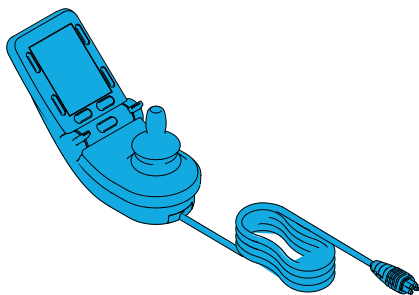

CJSM1


1615-D51607

RNET, CJSM Joystick, Complete With Jack Ports (compatible with Actuator Switches)

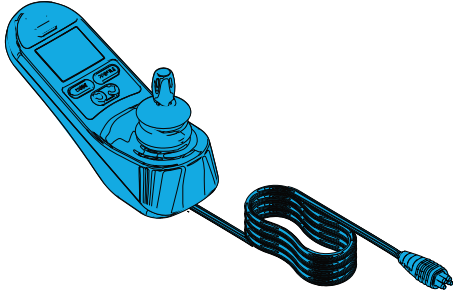

1615-D51153


RNET, CJSM Joystick, Complete With Lights and Jack Ports (compatible with Actuator Switches)

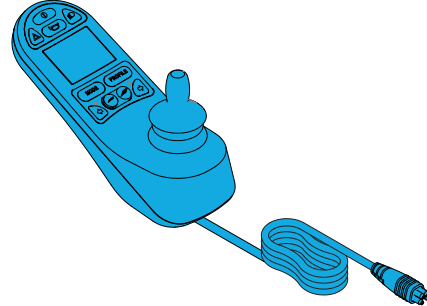

LED

1615-D51122


RNET, JSM Joystick, LED basic joystick complete with Jack Ports

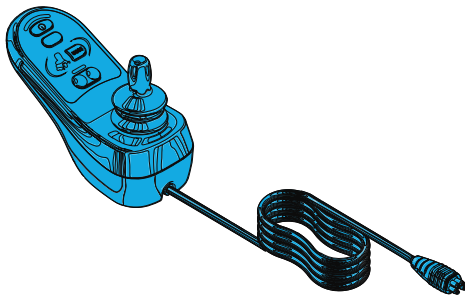

Mounts

1600-163451


Quckie Power, Sedeo Pro HC Retractable Joystick Mount Assembly Right Side

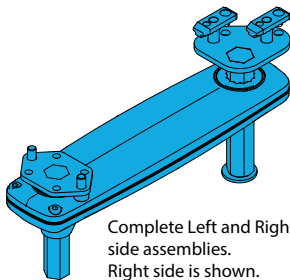

Complete Left and Right side assemblies. Right side is shown.

1600-163452


Quckie Power, Sedeo Pro HC Retractable Joystick Mount Assembly Left Side

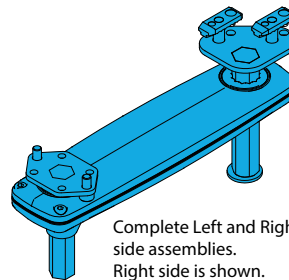

Complete Left and Right side assemblies. Right side is shown.


1600-120593

Quickie Power, Sedeo Pro & Lite Round Tube
Swingaway Joystick Mount Assembly Right Side

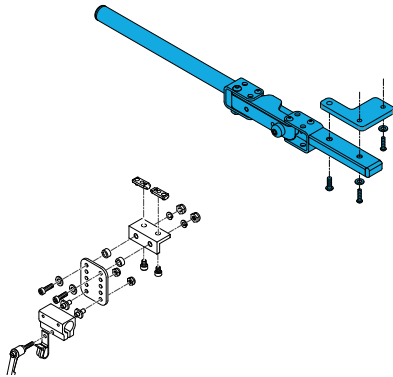


1600-120594

Quickie Power, Sedeo Pro & Lite Round Tube
Swingaway Joystick Mount Assembly Left Side

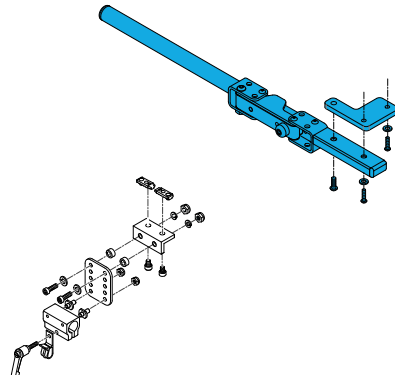


1600-164202

Quickie Power, Sedeo Pro Round Joystick Mount
Assembly

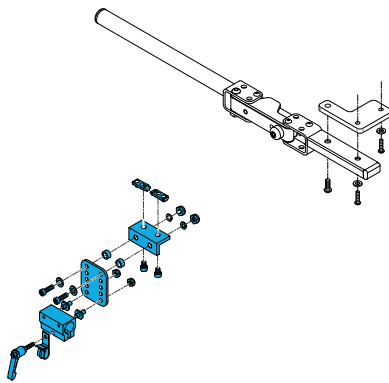


1600-164204

Quickie Power, Round Joystick Pro Mount Swingaway
Assembly RHS

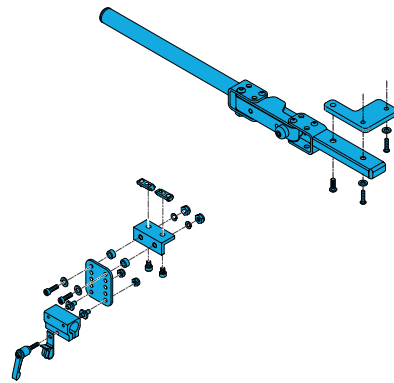


1600-164203

Quickie Power, Round Joystick Pro Mount Swingaway
Assembly LHS

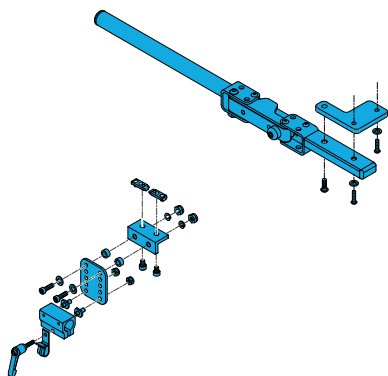


1600-163449

Quickie Power, Sedeo Pro & Lite HC Fixed Joystick Mount
Right Side

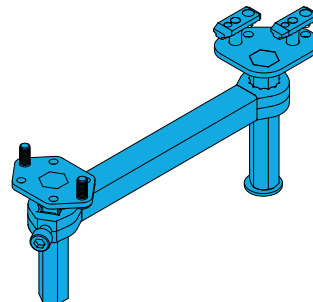

1600-163450


Quickie Power, Sedeo Pro & Lite HC Fixed Joystick
Mount Left Side

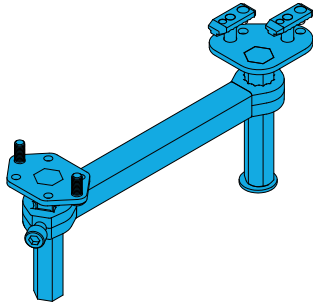

Parts


1615-SA81500L12

RNET CJSM2, Joystick Spare Part, 90 Deg Input RNET
Cable 120 cm

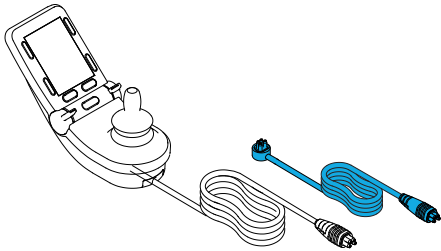

1615-SA81095

RNET Accessory, Joystick Knob, Softball

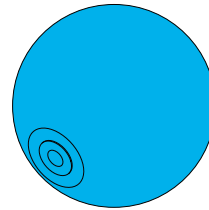


1615-P75755

RNET Accessory, Joystick Knob, Standard



1615-SA72410

RNET Accessory, Joystick Knob, T Bar

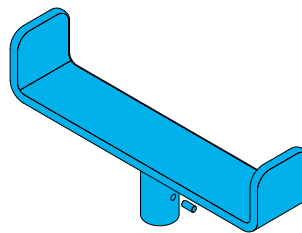

1615-SA72434

RNET Accessory, Joystick Knob, Old Style Sponge ball



1615-SA72435

RNET Accessory, Joystick Knob, Stick Knob SB1



1615-SA72439

RNET Accessory, Joystick Knob, Mushroom M49



1615-SA72441

RNET Accessory, Joystick Knob, Solid Ball B39



1600-101

Bodypoint Joystick Handle 3" Wide U Shape



1600-102

Bodypoint Joystick Handle 4" Wide U Shape



1600-110

Bodypoint Joystick Handle Grooved Mushroom



1600-107

Bodypoint Joystick Handle Rubber Dome




Joystick Repairs Now Available.

If you are needing any parts, Medifab are now credited to process Refurbs on any RNET technology. Please contact parts@medifab.com with any questions

Attendant Joystick

1615-D50882


RNET, Attendant Joystick, With Female Bus Connector

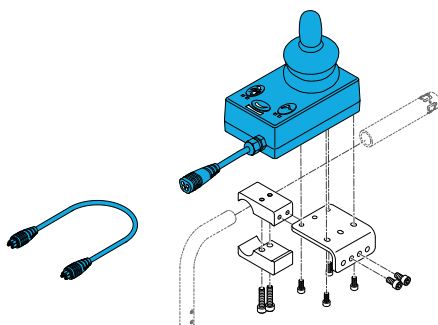

Attendant Mounting Systems


1600-131268

Quickie Power, Universal Universal Attendant Control 7/8" Mount

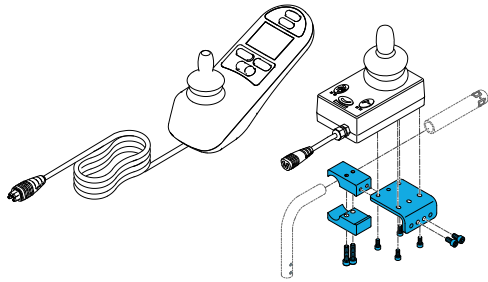


1600-22809050-001

Quickie Power, Sedeo Lite Attendant Control Mount Assembly Left Side

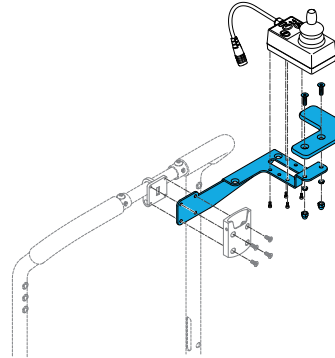


1600-22809050-002

Quickie Power, Sedeo Lite Attendant Control Mount Assembly Right Side

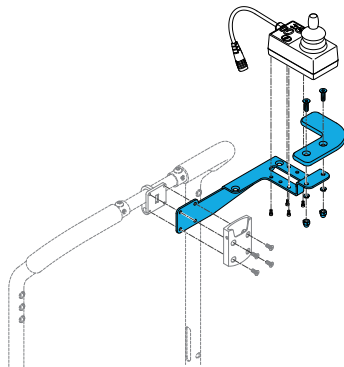

1600-108316-050


Quickie Power, R-Net Bus Cable/Harness 500mm

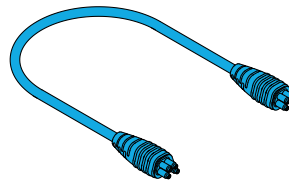

Dist Block to ISM or
Dist Block to Bluetooth mounted on Seat


1600-9315-LHS

Quickie Power Attendant Control Assembly LHS

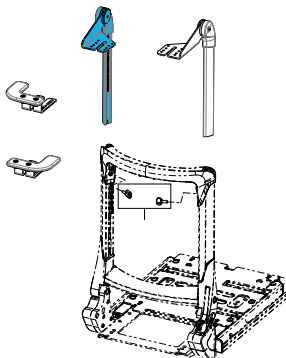


1600-9315-RHS

Quickie Power Attendant Control Assembly RHS

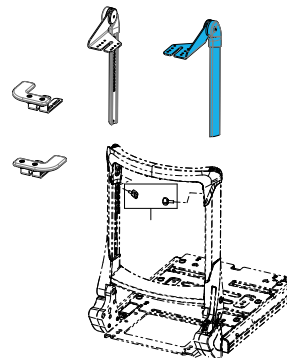


1600-22503415-002

Quickie Power, Palm Rest for Sedeo Pro Attendant
Control Mount LHS

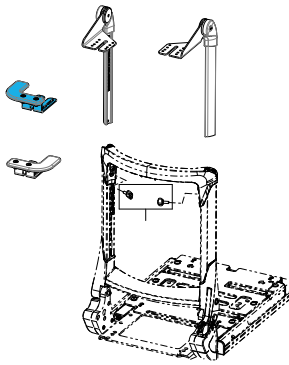

1600-22503415-001


Quickie Power, Palm Rest for Sedeo Pro Attendant
Control Mount RHS

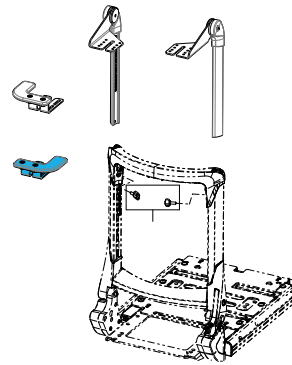

Accessory Electronics


1615-D50903

RNET, Power Module 120 Amp

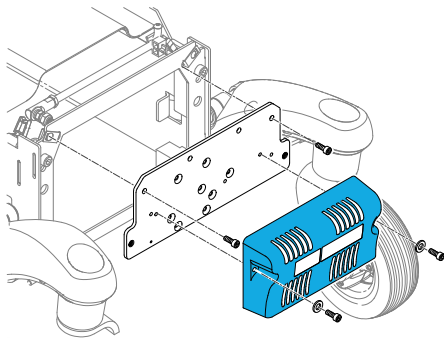


1600-241648

Quickie Power, RNET Light Module Only

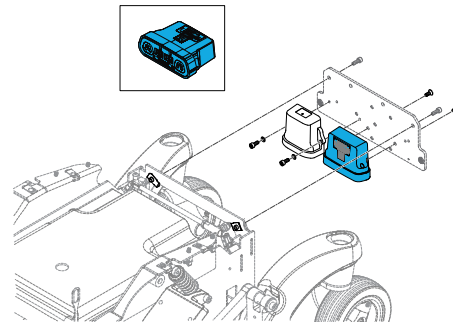


1600-015575

Quickie Power, Motor Encoder Module

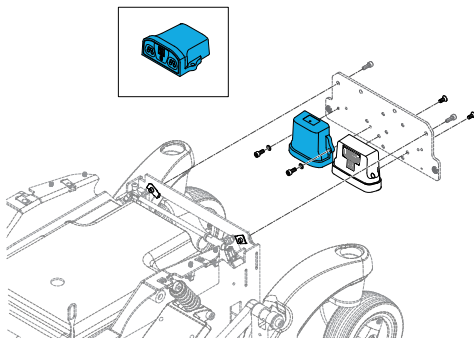


1615-D50602

RNET, ISM (Intelligent Seating Module) 6 Channel
with Lights.

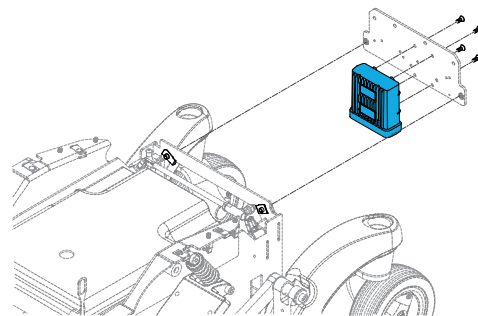


1600-D50996

Quickie Power RNET/VR2 HandHeld Programmer

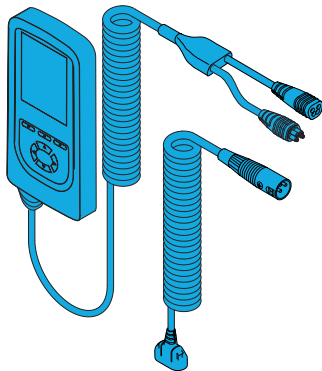


1615-D50609

RNET, Programmer Dongle, Dealer Version

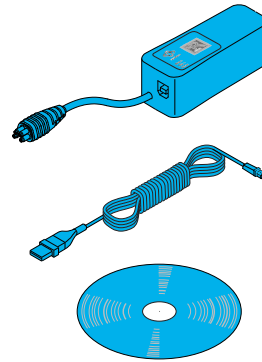


1615-D51508

RNET, ASM (Advanced Stability Module)

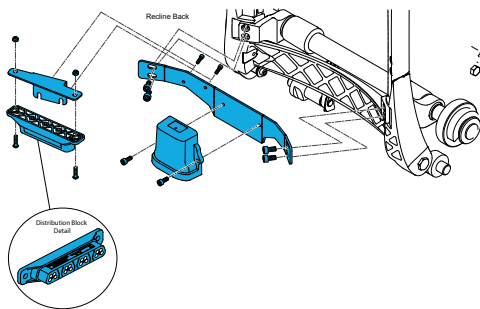


1600-164131

Quickie Power, Accelerometer Stability Module/Gyro Module and Distribution block assembly

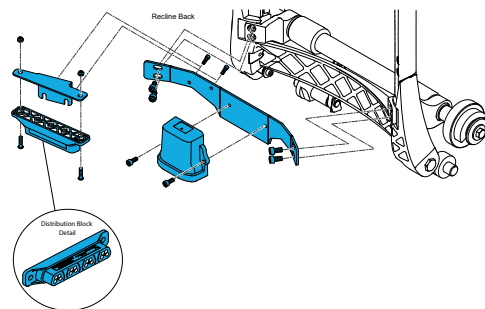

1600-108312

Quickie Power, RNET Distribution Block

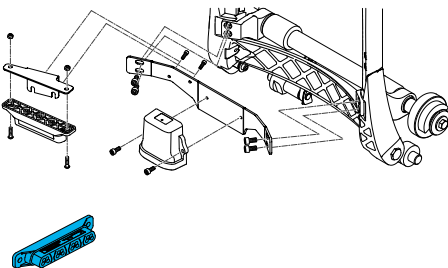


RNET Cables

Bus Cables/Harness

1615-SA77525L03


RNET, Bus Harness Cable (M-M), 0,3m long

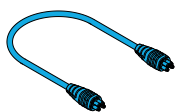

1C Controller Joystick Port to Bluetooth, or Dist Block or Bluetooth to Output Module without Power Recline, or Controller Joystick Port to ISM in Base

1615-SA77525L05


RNET, Bus Harness Cable (M-M), 0,5m long

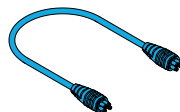

1A Dist Block to ISM or Dist Block to Bluetooth mounted on Seat

1615-SA77525L10


RNET, Bus Harness Cable (M-M), 1m long

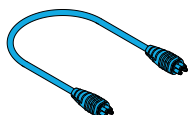

1D Distribution Block to Attendant Control, or Rnet Ext to Dist Block with Tilt/Lift combo

1615-SA77525L12


RNET, Bus Harness Cable (M-M), 1,2m long

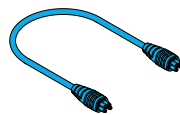

1B Non-expandable Rnet Bus Ext to ISM with Legrests only, or Rnet Bus Ext to Dist Block without Tilt/Lift combo, or Dist Block to Omni or Output Module with Power Recline

1615-SA77525L15


RNET, Bus Harness Cable (M-M), 1,5m long

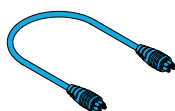

1B Non-expandable Rnet Bus Ext to ISM with Legrests only, or Rnet Bus Ext to Dist Block without Tilt/Lift combo, or Dist Block to Omni or Output Module with Power Recline

1615-SA77525L20


RNET, Bus Harness Cable (M-M), 2m long

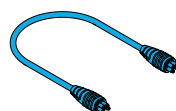

1B Non-expandable Rnet Bus Ext to ISM with Legrests only, or Rnet Bus Ext to Dist Block without Tilt/Lift combo, or Dist Block to Omni or Output Module with Power Recline

1615-SA78640L05

RNET, Bus Harness Extension Cable (M-F), 0.5m long

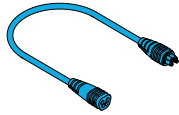


2B

Controller Joystick Port or ISM in Base or Bluetooth in Base to Rnet Ext without Tilt or with Tilt/Lift combo

1615-SA77525L10

RNET, Bus Harness Extension Cable (M-F), 1m long

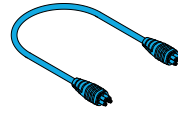


1D

Distribution Block to Attendant Control, or Rnet Ext to Dist Block with Tilt/Lift combo

1615-SA81331

Harness R-Net Bus Splitter

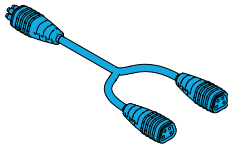


3

Used with CTRL+5 with Joystick, Omni, and Attendant

Battery Harness


1600-164136

Quickie Power, Battery Harness 100A Kit

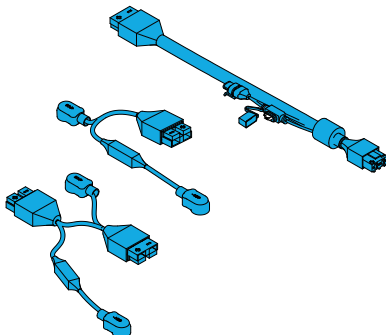

1600-249005


Quickie Power, 15A Mini Fuse

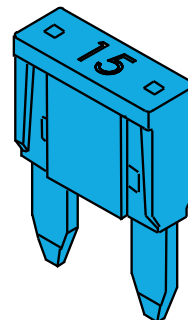

Wiring


1600-248506

Quickie Power, Harness Inhibit Shortening Plug I3

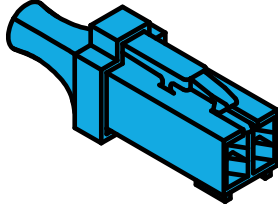


1600-21211005-100

Quickie Power, Sedeo Pro Actuator 150cm Extension Cables

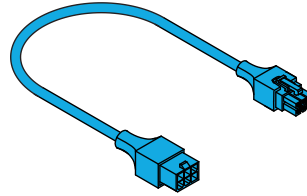


1600-21211005-150

Quickie Power, Sedeo Pro Actuator 150cm Extension Cables

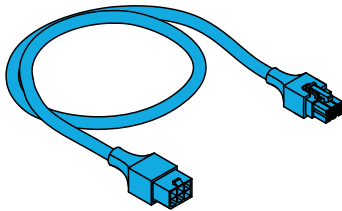


1600-248490

Quickie Power, Sedeo Pro Tilt/Lift Combo Lift Actuator E Break PM7 Assembly

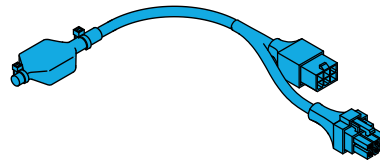


1600-248517

Quickie Power, Sedeo Pro Tilt/Lift Combo Harness Cables

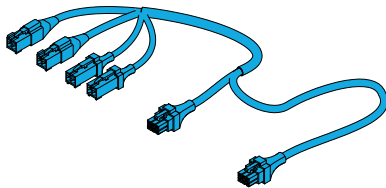


1600-248515

Quickie Power, Sedeo Pro Power Center Mount Actuator Harness/Cables Control Side

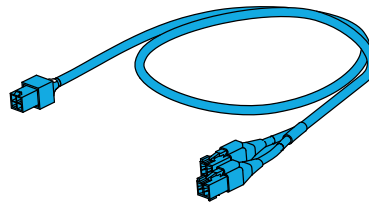


1600-248516

Quickie Power, Sedeo Pro Power Center Mount Actuator Harness/Cables Actuator Side

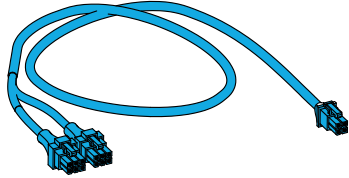


1600-248743

Quickie Power, Sedeo Pro Power Center Mount, Single Actuator Channel 3 Actuator Cable 500mm

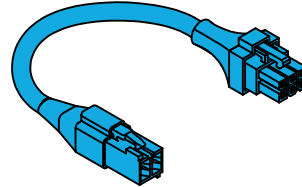


1600-22511004

Quickie Power, Wiring for Power Swing-Away Legrest - IMS to Actuator Harness

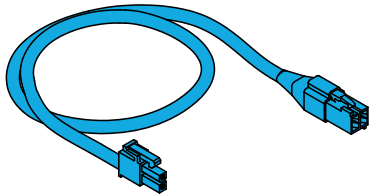


1600-22511005

Quickie Power, Sedeo Pro Power Swingaway Hangers Actuator Harness/Cables Legrests to ISM

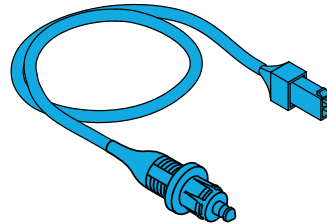


1600-249002

Quickie Power, Recline Actuator Cable (Check Specifics)

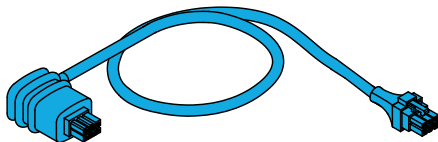


1600-22511003

Quickie Power, Recline Actuator Cable (Check Specifics)

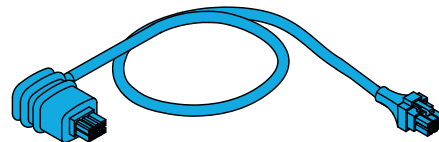


1600-249247

Quickie Power, Sedeo Pro Power Recline Actuator connector cable for chair with Lift

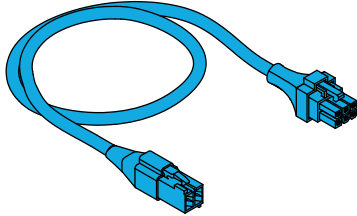


1600-249997

Quickie Power, Sedeo Pro Power Recline Home Cable, No Switch

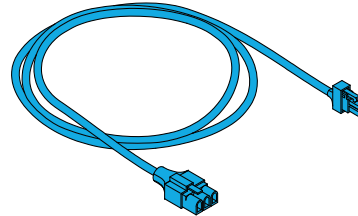

1600-250084


Quickie Power, Sedeo Pro Power Recline Home Switch, No Cable

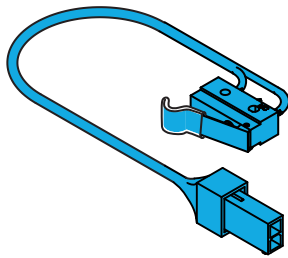

Lighting


1600-20009025-002

Quickie Power Lighting, LED Amber Indicator Signal Light Per Each

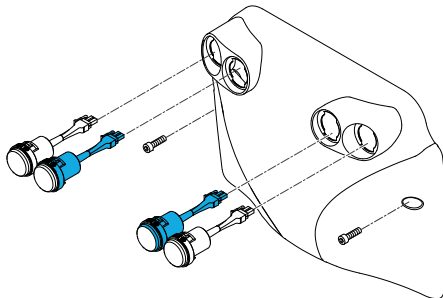


1600-20009025-001

Quickie Power Lighting, LED Red Tail Light Per Each

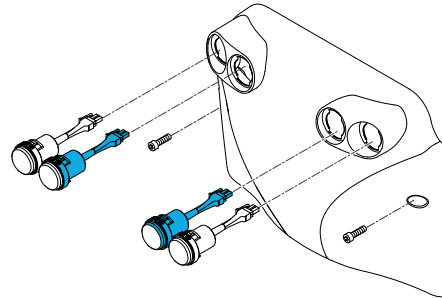


1600-244585

Quickie Power ISML Harness; Light Module
Harness/Cables; Order x 2 for complete set

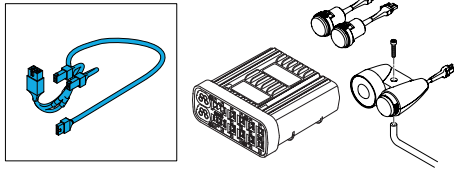


1600-163566

Quickie Power Lighting with ISML- Retrokit

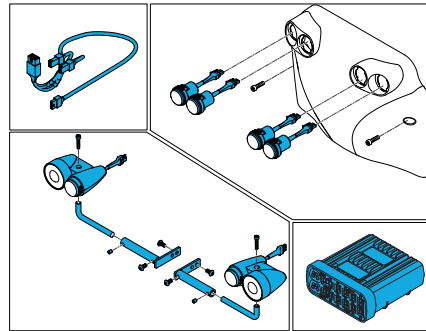


1600-243298

Quickie Power Front Left Light Assembly

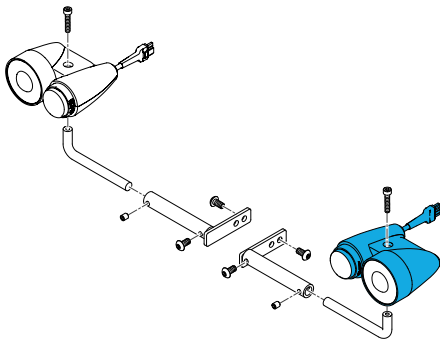


1600-243301

Quickie Power Front Right Light Assembly

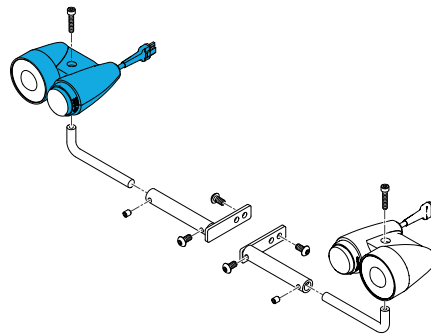

1600-160912

Quickie Power, Lighting Front assembly

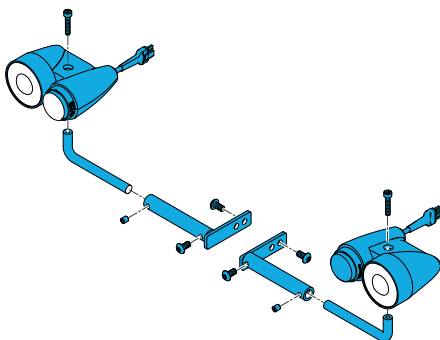

medifab

Distributed in NZ by Medifab Ltd

22 Detroit Drive,
Rolleston, 7675
New Zealand

0800 543 343

solutions@medifab.com

www.medifab.com



DISTRIBUTED BY: