


Secure-Fix Vehicle Harness


Secure-Fix Transport waist belts provide security for individuals who are confused or have challenging behaviour.

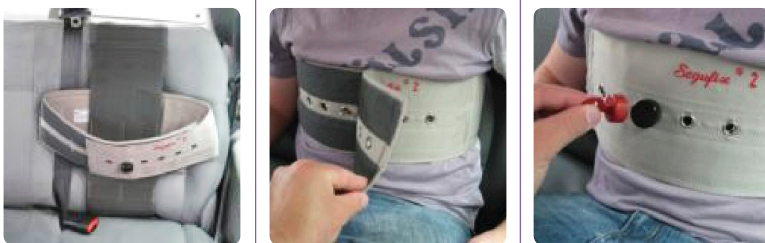

Place the backrest strap around the backrest and Velcro® in place.

Velcro® the waist strap around the user. Engage the anti-escape catch, and connect the vehicle safety belt.

To disengage, use the magnet release key.

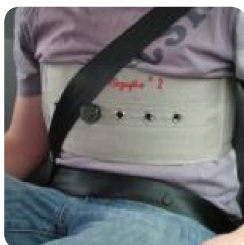

PATENTED MAGNET-RELEASE SYSTEM

Positive anti-escape catch system cannot be disengaged without the magnet key.



ACCESSORIES/UPGRADES

Pen/Key Release Buckle Upgrade
4620-6370-031

Anti-Escape Buckle Upgrade
4620-6370-026



Spare Magnetic Locking Cap
4700-0026-000

Spare Magnetic Locking Key
4700-0030-000



| Size | Waist Circumference | | | Backrest Strap Circumference | | | |
|------|---------------------|---------------|-----------------|------------------------------|---------------|---------------|---------------|
| | Small | Medium | Large | Small | Medium | Large | Extra Large |
| | 1 (Child) | 2 (Adult) | 3 (Large Adult) | Size 1 | Size 2 | Size 3 | Size 4 |
| Code | 4620-6361-017 | 4620-6362-017 | 4620-6363-017 | 4620-6181-296 | 4620-6182-296 | 4620-6183-296 | 4620-6184-296 |
| CM | 50 - 70 | 70 - 102 | 100 - 138 | 90 - 110 | 110 - 130 | 130 - 150 | 150 - 170 |

To order a Secure-Fix Vehicle Harness, both a Waist Strap and a Backrest Strap must be selected.