

# Starlight

SAFETY FEATURES

Our fully adjustable special needs seat with inbuilt tilt function

**INBUILT ADJUSTMENT** to the rear of the seat allows the backrest height to be increased for growth

**5 POINT COMFORT HARNESS:** with shoulder and abdominal padding, and centre pull adjuster

Large abdominal pad **PROTECTS CHILD** at harness buckle

**CRASH TESTED** to ECE R44-04

**SWIVEL BASE** Pivots out to the left (or right side if specially configured) to ease transfers

Uses **VEHICLE SEATBELT** to secure the child



# Starlight

Fully adjustable special needs seat, offering excellent posture support options and an inbuilt tilt function. Starlight is suitable for young children from 9kg, and has 10cm of backrest height adjustment to allow the child to grow. Maximum weight limit is 36kg.

- Ensure the **Starlight's back is in contact with the vehicle seat back** (no big gap behind headrest).
- There is no tether for this seat, but **ensure the clamp bow is slotted correctly between the seat and backrest of the car, OR that the ISOFIX/LATCH connectors are correctly in use.**
- Familiarise yourself with the weight limit on your child's seat, and ensure they have **not outgrown** this.
- Ensure the **car's seatbelt is used EVERY time**. Ensure your child's weight is appropriate for the selected belt path. **The seatbelt can be used behind the child until they are 18kg.** Please refer to the labels on the side of the seat for alternative belt path options and their corresponding weight capacities.
- Ensure **the shoulder straps are firmly secure, and are level with the child's shoulder height**, if these are below the child's shoulder height, the seat needs to be grown – please contact your Occupational Therapy team.
- Please note (for vehicle seat belt used in front of child) **the red belt guide at the shoulder is to be used to ensure the belt is sitting correctly**, is kept low across the child's hips and is kept off the child's face. The lower portion of the belt is to be threaded through the belt guide spaces on either side of the child's thighs.
- Periodically **check for loose or missing parts.**
- Keep child's fingers **clear of moving parts and swivel latches.**

Contact the car seat specialists for more information:  
0800 543 343  
[solutions@medifab.com](mailto:solutions@medifab.com)

Distributed by **medifab**<sup>®</sup>

The quick reference guide does not replace the requirement to read the user manual and safety information therein.



Ensure seat is in contact with the vehicle seat back



Ensure shoulder straps are firmly secure



Ensure child has not outgrown the weight capacity limit



Starlight has two methods of routing for the vehicle seat belt; **behind** the child for weights between 9-18kg and as a booster seat with the belt **over the front of the child** from 15-36kg. Please consult the user manual for detailed instructions on how to restrain your Starlight.