



## EQUAGEL COMPONENT



### Hand Washing Instructions (Preferred Method)

- Use warm, soapy water
- We recommend a liquid dish soap with a light perfume
- Air dry if possible. You may dab dry with a towel so long as the towel will not leave much lint behind. Do not tumble dry the cushion (with or without heat). A drying cabinet that allows the gel to rest uncompressed, unstretched, and unstressed is acceptable with a maximum heat of 43 degrees Celsius

### Machine Washing Instructions

- Machine washing is also acceptable provided the following guidelines are followed
- Do not use an agitating washer, only one that allows the cushion to remain still and uncompressed and not stretched or stressed
- Maximum water temperature should be 40 degrees Celsius
- Air dry if possible. You may dab dry with a towel so long as the towel

### Sanitation or Additional Smell Removal Instructions

- To remove odors, we recommend a liquid dish soap with a light perfume
- If you feel you must use a bleach to sanitize or remove odors, use a highly diluted bleach solution bath. Use of low-diluted or undiluted bleaches or sanitizing agents may damage the integrity of the gel over time
- Air dry if possible. You may dab dry with a towel so long as the towel will not leave much lint behind. Do not tumble dry the cushion (with or without)

---

## REMOVABLE BLACK COVER

- Machine wash with normal, non-bleaching detergent in cold to warm water
- Air dry on a rack or tumble-dry on a NO HEAT setting to minimize the possibility of shrinkage
- Stains may be pre-treated or hand-washed with non-bleaching cleaners