

# Carrot XL

—  
SAFETY FEATURES

Catering for safety  
and the importance  
of keeping the user  
in a supported  
posture for travel.



# Carrot XL

Carrot XL caters for safety and the importance of keeping the user in a supported posture for travel. This larger sized car seat is suitable for older children and adult users who will not fit into the Carrot 3000.

- Ensure Carrot XL back is in contact with the vehicle seat back (no big gap behind headrest).
- Strap at the rear of the Carrot XL's headrest is to be secured around vehicle's headrest, or backrest.
- If the car has individual separation in the rear seats, the lower horizontal strap at the rear of the Carrot XL is to be secured.
- Ensure the car's seatbelt is used EVERY time. This is to be securely fastened over the child and seat FIRST, before applying the harness. Please note this seat does not have capacity to have a belt threaded through the back as it is a booster seat. You are able to purchase and install a padded sleeve for the seatbelt if you feel the belt is going to irritate your child.
- Ensure the shoulder straps are firmly secure, and are level with the child's shoulder height, if these are below the child's shoulder height, the seat needs to be grown – please contact your Occupational Therapy team.
- If a cross strap is being used, please ensure this is sitting no higher than the top being level with the child's underarm.
- Ensure child has not outgrown the weight capacity which is 36-75kg. If child is approaching this weight limit, consider what would be next option.
- Please note the red belt guides at the side of the headsupport are to be used to ensure the belt path is sitting correctly, keeping the safety belt away from the child's face.
- Periodically check for loose or missing parts.


Contact the car seat specialists for more information:  
0800 543 343  
[solutions@medifab.com](mailto:solutions@medifab.com)

Distributed by **medifab**

The quick reference guide does not replace the requirement to read the user manual and safety information therein.



Ensure back is in contact with the vehicle seat back

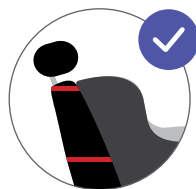


Ensure back straps are firmly fixed around the vehicle's headrest and seat



Ensure shoulder straps are firmly secure



Ensure child has not outgrown the weight capacity limit

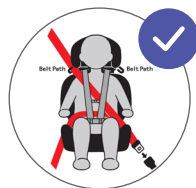

The car's seatbelt must be worn OVER top of the child and UNDER car seat harness