

LapStacker® Flex



Hold everything.

"Absolutely life changing. I use it for everything from gardening, shopping, cooking - anything in day to day life." - Rebecca Rodley

Retractable carry system for your wheelchair.



Zip.



Clip.



Go.

Lightweight (11oz / 300grams)

Enhanced mobility without burdening your shoulders or the performance of your wheelchair.

Magnetic buckles

Effortless and intuitive, the buckles snap together like magic.

Finger loops

Accessible finger loops make pulling the straps together a pleasure, even if you have limited hand function.

Easy install

Takes less than 5 minutes. No tools needed.

Adjustable

Suitable for wheelchairs 12 - 22" (30 - 56cm wide).

Retractable straps

A 48" (1.2m) strap system on each side offers fast secure and versatile carrying.

See it in action



Made with love by
adaptdify®

Quick fitting guide.

LapStacker Flex has an adaptable mounting system. Use this quick fitting guide to see the common installations and clearance required.

Manual rigid.




Horizontal Install

LapStacker Flex attaches horizontally around rigid frame wheelchairs.



Clearance


The mounting strap passes over the seat base. Check if you have space.

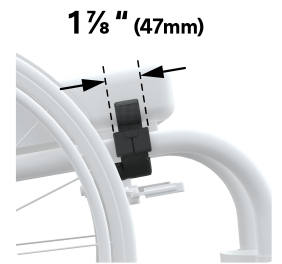

Dimensions

Existing accessories can often be moved to accommodate LapStacker Flex.

Manual folding (single frame).

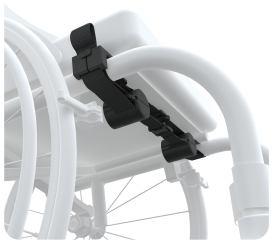


Horizontal Install

LapStacker Flex attaches horizontally around single frame folding wheelchairs.



Clearance

The mounting straps pass under the raising seat base. Check if you have space.

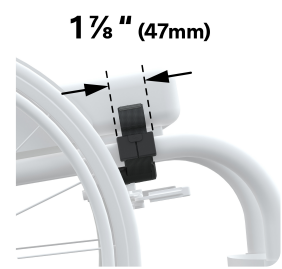

Dimensions

Existing accessories can often be moved to accommodate LapStacker Flex.

Manual folding (dual frame).



Vertical Install

LapStacker Flex attaches vertically between the top and bottom parallel tubes of folding wheelchairs.



Clearance

Check clearance between top and bottom tubes.



Dimensions

Existing accessories can often be moved to accommodate LapStacker Flex.

Other wheelchairs.

LapStacker Flex can be fitted to other wheelchairs with suitable clearance.

It has been tested with lightweight power wheelchairs such as Whill and Moving Star, off-road chairs like GRIT Freedom Chair and wheelchairs with complex frame designs like the Panthera-X.

This is not an exhaustive list, so please check the information above and the full installation guide for other fitting information.

Full installation guide.

medifab

0800 543 543 (NZ)
solutions@medifab.com
medifab.com