

Folding Mechanism

Bring the black lever out to fold the chair by also pushing the backrest forward.



Use the strap below the seat to keep the chair folded while storing or transporting it.



To unfold, undo the Velcro strap and push both the seat and back opening the chair.

The chair can stand while folded



Quickie 50R

User Guide



medifab

medifab.com

Controller



Red Leavers at the Back

Bring both leavers (right and left) down to disengage the motors and turn the chair into manual mode.

Bring both leavers up to engage the motors and turn the chair into power mode.

Please note the controller won't function and its lights will flash when the chair is in Manual Mode.

Armrest

Button to flip back armrest

