



This installation checklist is made available to facilitate the delivery of this special purpose car seat or restraint only, it does not replace the need to consult the user manual as required.





1 - Client and vehicle details

Date of Installation		Client age at time of installation	
Client Name			
Client weight at time of installation	IMPORTANT: check user weight does not exceed maximum user weight prior to commencing installation: 36kg (Kidsflex Size 2) 75kg (Kidsflex Size 3)		
Main vehicle that client uses for the majority of their travel	Make and model:		
	Year of manufacture:		
Other vehicles used for transport have been checked for compatibility with the Kidsflex		<input type="checkbox"/> Yes	<input type="checkbox"/> No

2 - Installation of Kidsflex into vehicle

It is recommended images are taken of each completed installation step. These images can be stored in both AT supplier and prescribing therapist's client record system for future reference.

Mandatory Installation Points – tick when each step is complete

- ISOFIX connection made through factory installed ISOFIX points when an audible click is heard. Test connection by gently pulling on both connection points. If no ISOFIX is present, abort delivery of Kidsflex
- Adjust angle of tilt to suit individual user needs (if required) 
- Adjust angle of backrest recline to suit individual user needs (if required) 
- Kidsflex has been pushed rewards in vehicle seat so its backrest lies flat against the vehicle seat backrest
- Tighten the four grub screws (two screws either side) to secure the ISOFIX adaptor to the Kidsflex seat base
- Adjust position of headrest to suit the head height of the user (if required) 
- Adjust position of lateral supports to suit needs of user (under direction from child's OT/PT) 
- Adjust Kidsflex inbuilt harness to suit the size of the user
- Adjust vehicle seat belt shoulder retainer to suit height of the user
- Vehicle seatbelt is directed correctly as per the user manual:
 - Upper section of the seat belt is fed through the red belt guide located at the side of the back support
 - Lower section of the seat belt passes through the red belt guide on the lower part of the seat base closest to the seat belt origin
 - Vehicle seat belt then passes over the top of the child and the secured Kidsflex inbuilt restraint. Both the upper and lower sections of the vehicle seat belt pass through the red seat belt guide located on the opposite side of the seat base and are buckled into the receiver

- Ensure the webbing of the restraint and vehicle seat belt is not twisted or trapped
- Remove any residual slack from the vehicle seat belt

3 - Parent/caregiver training on the Kidsflex

Training and education has been given to parent/caregiver on how to:

- Remove and reinstall the Kidsflex from vehicle
- Adjust the restraint
- Use and fit all accessories delivered with the Kidsflex
- Use the swivel base

IMPORTANT

There are differences in the road laws between each state and territory in Australia regarding requirements of transporting children with disabilities and medical conditions. Please refer to the MACA website www.macahub.org/resources/forms-and-templates to find out about requirements and processes that apply to the specific state or territory that the Kidsflex will be used in.

All below parties to complete and sign the below

- Completion of sections 1, 2 and 3 of this form
- Parent/caregiver has been supplied with Parent Safety Checklist relevant to product
- Prescribing therapist to be supplied a copy of completed form and upload to client records stored by the prescribing therapist

AT Product Consultant name and organisation:	Parent/caregiver name:	Prescribing therapist name and organisation:
AT rep signature:	Parent/caregiver signature:	Prescribing therapist signature:
Date:	Date:	Date:

AT rep to retain and store copy of completed form. Parent/caregiver can request copy of completed form.