

SleepShape W Leg Cushion

Care Instructions



IMPORTANT CONSUMER INFORMATION NOTICE:

This instruction leaflet contains important information regarding the care and maintenance of this product.

SUPPLIER:

Please pass this instruction leaflet on to the end user of the product.

USER:

Before using this product, read care instructions and save for future reference.

medifab

Care Instructions for SleepShape W Leg Cushion Cover
Care Instructions: SleepShape W Leg Cushion 20241008

Serial Number:

Cleaning Instructions

The W Leg cushion is supplied with an outer washable cotton over. The following instructions apply to this outer cover.

1. The outer cover should be regularly washed to keep it fresh for use and especially when it is soiled.
2. Only **NON** bleaching household washing powder can be used to wash the W Leg cushion cover.
3. The W Leg cushion cover should be cleaned in a standard household washing machine and should be spun on a **slow-spin-cycle**, 800 rpm or less is preferable.
4. The cover should not be washed any warmer than **40 degrees CELSIUS**, or 40°.
5. The cover can be air-dried or tumble dried on low heat.



The INNER cover of the W Leg cushion is a wipe-able PU material.

1. Only clean with mild detergent and other water-based household cleaners.
2. Do not use solvents or abrasive cleaners on the inner cover.
3. Do not immerse or soak in water as the foam filling will deteriorate.
4. Additional foam filling can be purchased if over time it becomes compressed.

Steps

1. Adhere to the manufacturer's instructions detailed above at all times.
2. Remove the outer cover from the inner waterproof pillow. Insert the pillow outer cover **ONLY** into the washing machine. Add non bleaching washing powder and set the washing machine as per instructions.
3. Remove from the washing machine, allow the product to air dry or tumble dry on a low heat. Then insert the waterproof inner into the outer cover. Fasten the zip and re-attach the leg straps (if required) and the W Leg cushion cover is ready to use.

Storage

- Do not store in wet or damp conditions as this will damage the cover.
- It should be stored in a room above freezing but below 40°C. For extended storage, it is recommended to store in a plastic, sealed container.

Maintenance Instructions

- Check that all seams and sewn areas are intact before cleaning.
- Ensure at all times that the integrity of the cushion is maintained.
- If pooling of moisture occurs ensure that an incontinence pad is placed on the base of the mattress to absorb excess moisture.
- Check before use that no objects are within the W Leg Cushion cover.

medifab

Distributed by Medifab Ltd

solutions@medifab.com

medifab.com

DISTRIBUTED BY: