



This installation checklist is made available to facilitate the delivery of this special purpose car seat or restraint only, it does not replace the need to consult the user manual as required.

1 - Client and vehicle details

Date of Installation		Client age at time of installation	
Client Name			
Client weight at time of installation		Hero XL max user height/weight - Check user weight has not exceed this prior to installation: 125cm/36kg	
Main vehicle that client uses for the majority of their travel	Make and model:		
	Year of manufacture:		
Other vehicles used for transport have been checked for compatibility with the Hero XL		<input type="checkbox"/> Yes	<input type="checkbox"/> No

2 - Installation and fitting of Hero XL

It is recommended images are taken of each completed installation step. These images can be stored in both AT supplier and prescribing therapist's client record system for future reference.

Mandatory Installation Points – tick when each step is complete

- Identifies pre installed lower tether anchorage points (only present in some vehicles).
- If vehicle does not have pre installed lower tether anchorage points** – Install lower tether anchorage strap around the seat rails of the either the front passenger or driver's seat. Choose the closest available hole when fitting the straps to ensure the anchorage point is as close to the floor of the vehicle as possible. Pull on the anchorage to tighten.
- Repeat above process to install the second lower tether anchorage strap around the second seat rail of the same vehicle seat.
- The heel of the Hero XL is positioned so that the yellow vertical line on the heel never crosses the front edge of the vehicle seat.
- The tilt angle of the Hero XL is adjusted to suit the user.
- The Hero XL is unzipped to open the belt clamp. Both the upper and lower sections of the vehicle seat belt are fed through each seat belt slot and the belt clamp, and then buckled into the receiver.
- All excess slack from vehicle seat belt is removed - press down on the Hero XL for a firmer installation. Close the belt clamp, ensure the vehicle seat belt is not twisted and close the zipper on Hero XL.
- If using the depth extension piece, the available leg room is adjusted to suit the user by adjusting the position of this piece and securing on each side of the Hero XL through tightening the red wingnuts.
- Straps at the front of the Hero XL with hooks on the end are connected and tensioned to lower tether anchorages.

- Firmly press down on the Hero XL in a side to side motion to correctly tension the straps.
- Foot prop is deployed and angled away approx. 10 degrees. Foot prop is extended downwards and makes solid contact with the vehicle foot well (the Hero XL may need to be lifted slightly to extend the foot prop to the appropriate length). Foot prop is secured through hearing an audible click.
- Vehicle seat belt is checked again to ensure seat belt has not become loose when deploying foot prop. Repeat seat belt tensioning step only if seat belt has become loose when deploying foot prop.
- Side protection impact pad installed to side of the Hero XL closest to the vehicle door.
- Headrest, inbuilt harness and lateral support position adjusted to suit the size and needs of the user.

3 - Parent/caregiver training on the Hero XL

Training and education has been given to parent/caregiver on how to:

- Remove and reinstall the Hero XL
- Adjust the restraint
- Use and fit all accessories delivered with Hero XL

IMPORTANT

There are differences in the road laws between each state and territory in Australia regarding requirements of transporting children with disabilities and medical conditions. Please refer to the MACA website macahub.org/resources/forms-and-templates to find out about requirements and processes that apply to the specific state or territory that the Hero XL will be used in.

All below parties to complete and sign the below

- Completion of sections 1, 2 and 3 of this form.
- Parent/caregiver has been supplied with Parent Safety Checklist relevant to product.
- Prescribing therapist to be supplied a copy of completed form and upload to client records stored by the prescribing therapist.

AT Product Consultant name and organisation:	Parent/caregiver name:	Prescribing therapist name and organisation:
AT rep signature:	Parent/caregiver signature:	Prescribing therapist signature:
Date:	Date:	Date:

AT rep to retain and store copy of completed form. Parent/caregiver can request copy of completed form.

