Ipai-LGT

SAFETY FEATURES

Fully adjustable special purpose car seat

Ensure the HEADREST is adjusted as your child grows taller

POSTURAL SUPPORTS

should be adjusted to ensure effective positioning tailored to your child's needs

Ensure the RECLINING BACKREST is in contact with the vehicle seat back The VEHICLE SEAT BELT must always be used over the child and the car seat



OUICK CHECK FITTING GUIDE

Ipai-LGT

Installation

- The Ipai-LGT should be installed in the back seat of a vehicle
- The mandatory points of installation are:
 - Use of the vehicle seat belt at all times
 - Ensure the Ipai-LGT back is flush with the vehicle seat back
- Use of the ISOFIX system is optional

Use

- If using the ISOFIX, ensure its secured before the start of each journey
- Place the child in the seat, ensuring the shoulder straps are firmly secure and are level with the child's shoulder height.
 If these are below the child's shoulder height, the seat needs to be grown
- Ensure the buckle is done up at the front of the child, you should hear an audible click
- The harness should the be tightened via the buckles at the bottom of each shoulder padding section, a snug and supportive fit should be ensured
- Ensure the car's seatbelt is used EVERY time. This is to be securely fastened OVER the child and seat
- Please note the red belt guide above the shoulder is to be used to ensure correct path of the belt and is kept off the child's face and neck. The lower portion of the belt is to be passed over the red belt guide spaces on either side of the seat base
- Familiarise yourself with the weight limit on your child's seat and ensure they have not outgrown this. For the lpai-LGT this is 36kg
- Periodically check for loose or missing parts



Ensure child has not outgrown the weight capacity limit



Ensure back is in contact with the vehicle seat back



Ensure shoulder straps are firmly secure



The harness should the be tightened via the buckles at the bottom of each shoulder padding section, a snug and supportive fit should be ensured.