

SYMMETRISLEEP WASHING INSTRUCTIONS

DESCRIPTION	WASHING INSTRUCTIONS
VELCRO SHEET Both full length fitted sheet with pull cord and old style with velcro fastenings	40°C OR LESS WASH ON REDUCED CYCLE. DO NOT IRON, TUMBLE DRY OR USE CHLORINE BLEACH.
BRACKET S	WIPE CLEAN WITH A SOFT CLOTH MOISTENED WITH WATER AND NEUTRAL DETERGENT. DO NOT USE BLEACH OR BIOLOGICAL CLEANERS.
BRACKET PADS	MACHINE WASH AT 40°C OR LESS WITH MILD DETERGENTS. DO NOT TUMBLE DRY.
SOFT KNEE BRACE	COOLOVER INSERTS - CAN BE WASHED UP TO 40°C WITH MILD DETERGENTS. DO NOT TUMBLE DRY. PU COVER - WIPE CLEAN WITH A SOFT CLOTH MOISTENED WITH WATER AND NEUTRAL DETERGENT. DO NOT USE BLEACH OR BIOLOGICAL CLEANERS.
SIDE LYING LEG SUPPORT	WIPE CLEAN WITH A SOFT CLOTH MOISTENED WITH WATER AND NEUTRAL DETERGENT. DO NOT USE BLEACH OR BIOLOGICAL CLEANERS
LOWZONE OVERMANTLE or Mattresses All with PU covers	WIPE CLEAN WITH A SOFT CLOTH MOISTENED WITH WATER AND NEUTRAL DETERGENT. DO NOT USE BLEACH OR BIOLOGICAL CLEANERS. MACHINE WASH OR DRY CLEAN UP TO 60°C (SOME SURFACE WRINKLING MAY OCCUR - THIS HAS NO ADVERSE EFFECT ON THE FABRIC'S PROPERTIES). HANG OR TUMBLE DRY (ON A COOL SETTING).
AIRMANTLE Coolover Or Bamboo and PU base	WASH WITH MILD DETERGENTS AT 40°C. DO NOT TUMBLE DRY.
AIR SHEET	WASH WITH MILD DETERGENTS AT 40°C. DO NOT TUMBLE DRY.
BAMBOO FITTED SHEET	60°C OR LESS WASH ON REDUCED CYCLE. CAN BE TUMBLE DRIED. DO NOT IRON OR DRY CLEAN.
COOLOVER FITTED OVER SHEET	40°C OR LESS WASH ON REDUCED CYCLE. CAN BE TUMBLE DRIED. DO NOT IRON OR DRY CLEAN.
Cushions- Ivory or blue	PU COVER - WIPE CLEAN WITH A SOFT CLOTH MOISTENED WITH WATER AND NEUTRAL DETERGENT. DO NOT USE BLEACH OR BIOLOGICAL CLEANERS. MACHINE WASH OR DRY CLEAN UP TO 60°C (SOME SURFACE WRINKLING MAY OCCUR - THIS HAS NO ADVERSE EFFECT ON THE FABRIC'S PROPERTIES) BAMBOO COVER - 40°C OR LESS WASH ON REDUCED CYCLE. DO NOT IRON, TUMBLE DRY OR USE CHLORINE BLEACH. COOLOVER COVER - 40°C OR LESS WASH ON REDUCED CYCLE. CAN BE TUMBLE DRIED. DO NOT IRON OR DRY CLEAN.
SAFETYSLEEP CUSHIONS Lowzone crumb filled cushions with PU inner cover, safetysleep main cover and bamboo outer cover	SAFETYSLEEP COVER - MACHINE WASH AT 40°C OR LESS WITH MILD DETERGENTS. DO NOT TUMBLE DRY. PU COVER - WIPE CLEAN WITH A SOFT CLOTH MOISTENED WITH WATER AND NEUTRAL DETERGENT. DO NOT USE BLEACH OR BIOLOGICAL CLEANERS. MACHINE WASH OR DRY CLEAN UP TO 60°C (SOME SURFACE WRINKLING MAY OCCUR - THIS HAS NO ADVERSE EFFECT ON THE FABRIC'S PROPERTIES) BAMBOO COVER - 40°C OR LESS WASH ON REDUCED CYCLE. DO NOT IRON, TUMBLE DRY OR USE CHLORINE BLEACH.

