

medifab



GLIDE CENTRO
ENDLESS POSSIBILITIES & OPTIONS



Glide Centro

Mid Wheel Drive

Australian designed and manufactured the Centro is designed to give outstanding stability, flexibility, indoor & outdoor performance and drive capabilities with trouble free service for many years.

Glide Centro Electric Wheelchair comes with Active Pozi-Trak System (APT)- 6 wheel fully independent suspension system. Manoeuvrable, narrow and compact for a perfect blend of indoor and outdoor performance. Long range 73 amp batteries for all day freedom. Available with electric lift and tilt. All Glide Centro Electric Wheelchairs come with a completely customisable seating platform. Also available in paediatric sizes.



Glide Centro Bariatric

Mid Wheel Drive

The Glide Centro Bariatric is a heavier duty version of the standard Centro. This model caters for the larger and heavier user.

With its safe working load of 250kg and seat width of up to 73cm the Glide Centro Bariatric wheelchair is suitable for heavier patients. All features and options of standard Glide Centro are available for bariatric version.

Manoeuvrable for a perfect blend of indoor and outdoor performance. All Glide Centro Bariatric Wheelchairs come with a completely customisable seating platform.



Glide Centro XT

Mid Wheel Drive

The all new Centro-XT is an Extreme Terrain power chair designed for users who enjoy off pavement driving.

The XT is based on a standard Centro and fitted with low pressure wide knobby Drive tyres and 9" front castors. Fitted with high torque 4 pole motors and 120 Amp power module. The XT gives unsurpassed outdoor performance and terrain following capabilities. It still maintains most of the features of the standard Centro and can be scripted to suit each individual client.





Centro- Lift-Anterior-Reach Mid Wheel Drive

Transfer - Reach - Get Closer

The all new Centro Lift-Anterior-Reach chair is design for users who want the versatility of a standard wheelchair while offering a vertical seat raise, the ability for a standing transfer with the anterior tilt function and also a reach forward option when in a raised seat position to perform task generally not achievable with standard lift seat chairs.



Centro- Vertical Lift Mid Wheel Drive

Centro Vertical lift chair comes standard with a Power Tilt Seating platform. The 30cm range of seat lift allows the user an infinitely variable seat height which greatly improves independence and quality of life.

Vertical lift system also enhances social interaction by bringing users to a level to engage better with their peers. The Vertical Lift system can be programmed to suit user needs and can allow the chair to be driven whilst at an elevated position.

All the options and accessories you would expect from a Centro are available with the Vertical Lift version. The Vertical Lift Centro can be customised to your individual requirements with the addition of options such as Power recline, Power elevating leg supports, lights and custom seating.



Glide Centro Electric Wheelchairs can adapt to any brand of seating.

Active Pozi-Trak drive system (APT)

APT is a unique suspension system patented by Glide and was developed to enhance the performance of its latest Mid Wheel Drive wheelchair. The APT system is a three functional operation which increases traction on the Drive Wheels, aids with Kerb Climbing and acts as an Anti Pitch device. The suspension system also eliminates nose dive when descending road side kerbs.

The drive wheels are set positive meaning when the wheelchair is on a level ground a preload of up to 20kg (depending on spring set up type) is on each drive wheel. This unique system greatly improves traction for driving and braking. It also performs extremely well in emergency down hill braking.

Whilst giving an unsurpassed ride & comfort, this system is a very low maintenance and has eliminated the need for complex moving parts that need constant servicing or adjustments. Also, with the combination of varying spring set ups, your wheelchair can be fined tuned to suit your driving preference.

Specifications

	Glide Centro Lift-Anterior-Reach Vertical Lift	Glide Bariatric	Glide XT
Max Recommended User Weight	175kg*	250kg*	175kg*
Weight	85kg	84kg	85kg
Operation Weight	132-140kg	134-142kg	134-142kg
Rear Castors	7" pneumatic (opt. 6" or 8")	7" pneumatic (optional 6" or 8")	8" pneumatic (opt. 9")
Front Castors	8" pneumatic (opt. 7")/ 8" solid fill	8" pneumatic (optional 7")/ 8" solid fill	9" pneumatic
Drive Wheels	3.00-8 (14")- optional 14" or 12" wide	4.10/3.50-6 (12")	145/70-6 - Low pressure Knobby
Suspension	APT	APT	APT
Overall Length	900mm (foot plate removed)	900mm (foot plate removed)	980mm (foot plate removed)
Overall Width	620mm (wheel base)	620mm (wheel base)	740mm (wheel base)
Seat Width	31, 34, 37, 40, 43, 46, 49, 52, 55cm (custom width available)	49, 52, 55, 58, 61, 64, 67, 70, 73cm (custom width available)	31, 34, 37, 40, 43, 46, 49, 52, 55cm (custom width available)
Seat Depth	31, 34, 37, 40, 43, 46, 49, 52, 55cm	40, 43, 46, 49, 52, 55, 58, 61cm	31, 34, 37, 40, 43, 46, 49, 52, 55cm
Seat Height	46cm (top of rail using 0-46 set up)	46cm (top of rail using 0-46 set up)	46cm (top of rail using 0-46 set up)
Low Seat Height Option	42cm	N/A	N/A
Back Height	1030mm Top of push handle (custom height available)	1030mm Top of push handle (custom height available)	1030mm Top of push handle (custom height available)
Seat Tilt	4-50std (optional 0-46 degree)	4-50std (optional 0-46 degree)	4-50std (optional 0-46 degree)
Arm Rest	Flip up (adjustable)	Flip up (adjustable)	Flip up (adjustable)
Leg Rest	Swing away detachable	Drop-in swing away detachable	Single Centre Mount
Frame Colour	Choice	Choice	Choice
Brakes	Electronic and manual park	Electronic and manual park	Electronic and manual park
Controls	R-net LCD std	R-net LCD std	R-net LCD std
Battery (260Lx170Wx220H)	70-90 amp/hr, Sealed	70-90 amp/hr, Sealed	70-100 amp/hr, Sealed
Range	40km max. with 70amp/hr	40km max. with 70amp/hr	40km max. with 70amp/hr
Crash Tested to AU Standards	YES	YES	YES

* AS 3695 Full compliance

Options

	Glide Centro Lift-Anterior-Reach Vertical Lift	Glide Bariatric	Glide XT
Power Tilt	0-46 or 5-50	0-46 or 5-50	0-46 or 5-50
Anterior Tilt	Available	Available	Available
Anterior Tilt with reach	Available	N/A	Available
Vertical Lift	Available	N/A	Available
Power Recline	Available	Available	Available
Centre Mount Power	Available	Available	Available
Elevating Leg Rest			
Swing Away Power	Available	Available	Available
Elevating Leg Rest			

Supplied by Medifab Ltd
www.medifab.co.nz
solutions@medifab.com
 0800 543 343