

# Grillo Gait Trainer

Grillo easily adjusts and is tool-free for user growth, transportation or multi-patient use. Measurement bars allow for quick, easy and accurate adjustment.

Sturdy walking aid designed for children through to adults with physical disabilities



**Constant centre of gravity** with patented height adjustment, allows for postural correction with the user in situ

**Precise & continuous adjustments.** Versatile with a range of interchangeable components.

**Huge growth range** means Grillo will accompany the user's growth for a long time within a single size

**Basic or complex configurations**

## Anterior or Posterior?



ANTERIOR

Switch frame configurations if user needs change by reversing the wheels



Direction walking

41-63 cm



POSTERIOR



## STANDARD PACKAGES

### ANTERIOR PACKAGE contains



- Pelvic and trunk support
- D Handle
- Pelvic harness
- Front direction locks
- Drag brakes
- Anti-reverse locks
- Foot activated brakes.



Size Mini

Size Small, Medium, Large

### POSTERIOR PACKAGE contains

- Pelvic support
- Pelvic harness
- Trunk support
- Adjustable handle bars
- Direction locks
- Drag brakes
- Anti-reverse locks
- Foot activated brakes



Size Mini

Size Small, Medium, Large

Size	Posterior Code	Anterior Code	Pelvic Support Height	Trunk Support Height	Support Circumference	User Height Range	Max Load	Footprint (cm)
Mini	5750-0140-000	5750-0011-000	41-63 cm	52-77 cm	53-79 cm	75-90 cm	35 kg	57 (W) x 73 (L)
Sml	5750-0151-12B	5750-0051-12B	49-84 cm	69-100 cm	55-80 cm	90-135 cm	40 kg	65 (W) x 75 (L)
Med	5750-0153-34B	5750-0053-12B	54-100 cm	76-125 cm	67-103 cm	110-160 cm	90 kg	76 (W) x 91 (L)
Lrg	5750-0155-05B	5750-0055-05B	63-117 cm	81-143 cm	78-115 cm	Up to 190 cm	110 kg	80 (W) x 111 (L)

## ACCESSORIES

### Leg Divider with Moving Leg Cuffs

Mini 5750-5400-000  
Sml 5750-5401-1-2  
Med 5750-5403-3-4  
Lrg 5750-5405-005



### Padded Saddle Seat for Anterior or Posterior

Min/Sml 5750-0211-1-2  
Med 5750-0213-3-4  
Lrg 5750-0215-005



### Arm Gutter Supports

Ant, Sml 5750-5501-1-2  
Ant, Med/Lrg 5750-5503-3-4  
Post, Sml 5750-5511-1-2  
Post, Med/Lrg 5750-5513-3-4  
*Arm Straps Available*



### Handlebars for Posterior

Sml 5750-5711-1-2  
Med 5750-5713-3-4  
Hand grips Sml 5750-5721-1-2  
Hand grips Med 5750-5723-3-4



### Height Adjust, Folding Seat for Posterior

Sml 5750-0201-1-2  
Med 5750-0203-3-4



### Lumbar Support for Posterior Frame

Sml 5750-2501-1-2  
Med 5750-2503-3-4



### Weight Bars

Sml 5750-8431-1-2  
Med 5750-8433-3-4



### Assistant Push Handle S/M/L 5750-8600-000



### Ergonomic Handles

Ant 5750-5730-000  
Post 5750-5735-000



### Head Support

S/M 5750-4600-000



### Rear wheels feature:

Anti reverse Device: Stops wheel from going backward.  
Adjustable Friction: To reduce speed of movement. 4 levels of adjustment.  
Hand brake compatibility  
Wheel stop brake: useful while positioning the child and during transportation.



### Front swivelling wheels feature:

Directional locks: prevent front wheels from steering, ensure linear stability and allow the user to focus their residual abilities only on linear walking rather than on lateral control.



### Quick adjustments


Adjustments are made with levers or handles without need for tools, and can be made with the user in the frame.

### Graduated bars

On all adjustable supports, to allow reset of the exact position after Grillo has been folded or used.

### Simple & quick folding

Grillo folds without requiring any components to be removed.

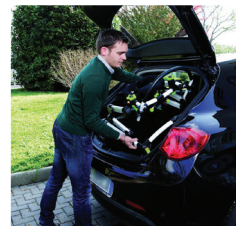

### Independent trunk /pelvis supports

Ergonomic and fully adjustable in width/circumference, height, tilt, and forwards/back.

### Ergonomic harness

Has a slight abduction effect without generating pressure. Adjustable, removable and machine washable.

### Easily stored and transported

Grillo folds in a single operation, reducing size for storage and transportation.