

# J3<sup>®</sup> BACK

ORDERING GUIDE





## JAY® J3® Backs

The most versatile JAY back for all SHAPES, SIZES, and LIFESTYLES

The J3 back product line offers a multitude of width, height, and contour depth backs to fit your client. The backs are adjustable with numerous hardware choices allowing for easy installation and quick removal.



J3 Carbon Fiber Option

### What makes the J3 unique?

- Five contour shapes provide individualized side-to-side stability.
- Four support heights and shapes allow you to select the ideal backrest for your client's torso height and clinical presentation.
- Carbon Fiber option weighs as little as 2.2 lbs. with hardware.
- Accessories include WHITMYER® headrests, lateral thoracic supports, and the CradleU Lateral.
- The J3 HV Back uses heating/ventilation technology to help keep your client's microclimate stable.

#### PEDIATRIC OPTION:

The JAY Zip® pediatric back features smaller sizes, lower weights, and contour shapes for active kids and teens.

## Universal Hardware

Four hardware options optimized for varying requirements, wheelchair types and easier installation:

- Patented auto-lock hardware securely locks into place once back pins are inserted into receiver – no extra step needed!
- Quick-release levers allow for fast back removal.
- Fits back cane widths  $\frac{3}{4}$ " to 1  $\frac{1}{8}$ " and able to mount to any QUICKIE Power wheelchair with adapter bracket.



2-pt. Quick Release  
TH Hardware

HARDWARE	SH	TH	RH	FS
CONTACT	2-pt.	2-pt.	2-pt.	4-pt.
ANGLE	+/- 20°	+/- 20°	+/- 10°	–
WIDTH	2"	2"	2"	2"
DEPTH	2.25"	2.25"	1.75"	–
WEIGHT CAPACITY	300 lbs.	500 lbs.	–	500 lbs.

## Functional & Custom Options

- Dual layer, multi-density foam to enhance durability and comfort.
- Layered 3DX™ Spacer Fabric and Lycra® Stretch cover promotes airflow and wicks moisture keeping your client cool and dry.
- Color options for the back panel available through JAY Your Way.



3DX Spacer Fabric

## TABLE OF CONTENTS

<b>Step 1</b>	Contour Depth	5
<b>Step 2</b>	Level of Support	6
<b>Step 3</b>	Back Support Height	7
<b>Step 4</b>	Backrest Width	8
<b>Step 5</b>	Mounting Hardware	9
<b>Recap</b>	Smart Part Number Model	10
<b>JAY Your Way</b>	Options	11
<b>Accessories</b>	Headrest Pads	12
	Headrest Mounting Hardware	12
	Laterals	12
	Chest Strap Harness	13
	Vanity Flap	13
	Spine-Align Kits	13
	Spine-Align Components	13
<b>Appendix</b>	Reference Drawings & Tables	14-15




TH - 2-pt.  
Quick-Release  
Mounting  
Hardware

J3 Posterior Deep Back

# Step 1

## Contour Depth

For a custom-fit J3 Backrest, follow this **5-Step Guide** to determine your **J3 Smart Part Number**.

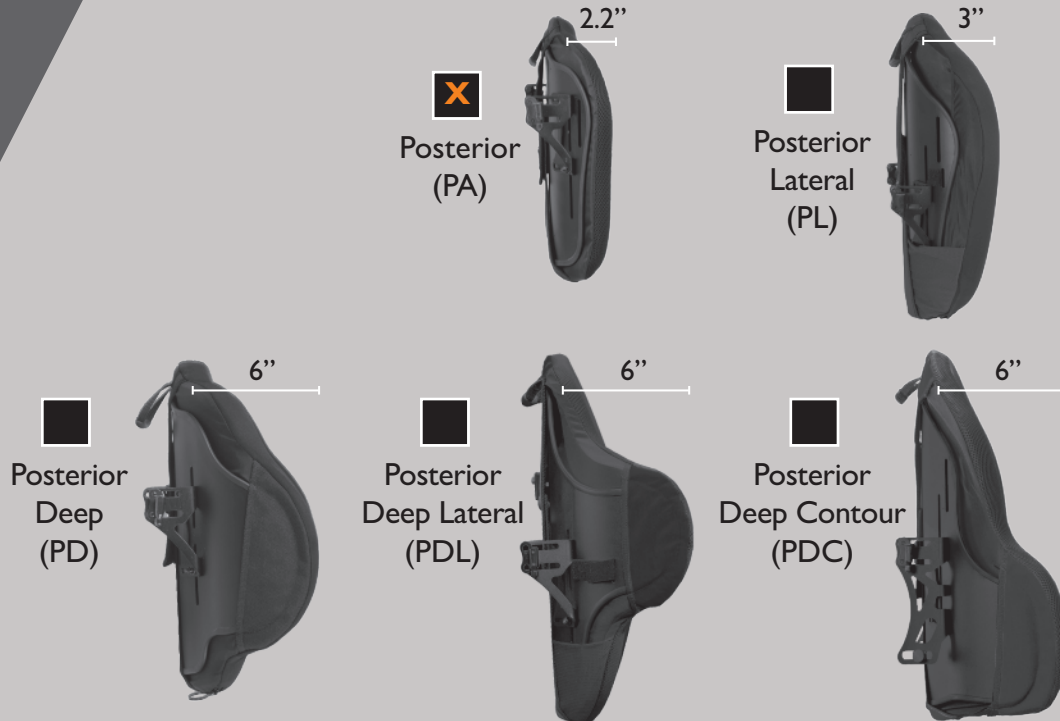

### Step 1 Determine Contour Depth.

How much lateral support is needed: PA, PL, PD, PDL & PDC

J3	PA	—	—	—	—
J3 Part Numbers begin with "J3"*	Step 1	Step 2	Step 3	Step 4	Step 5

**Note:** To order a J3 HV Back, begin with "J3HV".


**Note:** To order a J3 Carbon Fiber (CF) Back, begin with "J3CF".

\*The part number that is generated over the next several pages is for illustrative purposes. Actual part numbers may vary. Please reference the J3 Back, J3 HV Back, and J3 CF Back order forms on [www.SunriseMedical.com](http://www.SunriseMedical.com) for a full list of options and availability.

## Step 2

### Level of Support

Each individual, depending on their functional needs and level of trunk stability, requires a different level of support.

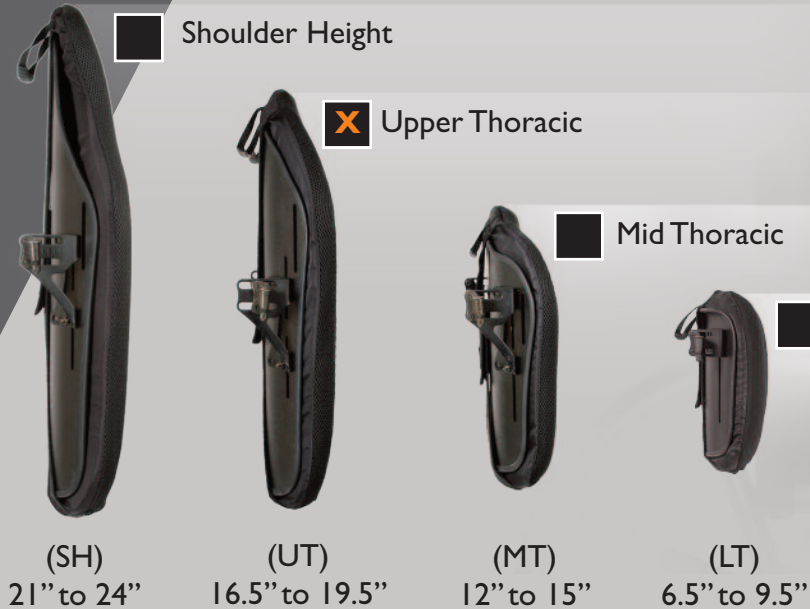

### Step 2 Determine Level of Support.

Choose from Lower Thoracic up to Shoulder Height.

<b>J3</b>	<b>PA</b>	<b>UT</b>	—	—	—
J3 Part Numbers begin with "J3"	Step 1	Step 2	Step 3	Step 4	Step 5

## Step 3

### Back Support Height

Within each level of support (*Step 2*), we offer short, medium, and tall height options to accommodate differences in the user's torso height.



Short (S)

LTS 6.5"  
MTS 12"  
UTS 16.5"  
SHS 21"



Medium (M)

LTM 8"  
MTM 13.5"  
UTM 18"  
SHM 22.5"



Tall (T)

LTT 9.5"  
MTT 15"  
UTT 19.5"  
SHT 24"

### Step 3 Determine backrest height.

In (*Step 2*), the level of support is chosen. Each of these support levels are offered in Short, Medium, and Tall. These two steps together determine backrest height.

<b>J3</b>	<b>PA</b>	<b>UT</b>	<b>M</b>	—	—
J3 Part Numbers begin with "J3"	Step 1	Step 2	Step 3	Step 4	Step 5

# Step 4

## Backrest Width

**Example:**  
Trunk widths measuring  
between 13" to 15",  
the PA backrest is 16"





Posterior (PA)



Posterior Lateral (PL)



Posterior Deep (PD)  
Posterior Deep Lateral (PDL)  
& Posterior Deep Contour (PDC)

WIDTH	12	14	16	18	20	22	24	26	12	14	16	18	20	12	14	16	18	20
 <p><b>Chair Back Cane Width</b> Based on outside back cane to outside back cane</p>	12 to 14	14 to 16	16 to 18	18 to 20	20 to 22	22 to 24	24 to 26	26 to 28	12 to 14	14 to 16	16 to 18	18 to 20	20 to 22	12 to 14	14 to 16	16 to 18	18 to 20	20 to 22
 <p><b>Trunk Width</b> Measure the widest part of the trunk supported by the back</p>	11	13	15	17	19	21	23	25	9.5	11.5	13.5	15.5	17.5	9	11	13	15	17

### Step 4 Determine the Backrest Width.

Measure the widest part of the trunk covered by the back or measure the chair back cane width. Reference the chart to select which width works best for the contour chosen in **Step 1** on page 5.

J3	PA	UT	M	16	—
J3 Part Numbers begin with "J3"	Step 1	Step 2	Step 3	Step 4	Step 5



## Step 5

### Mounting Hardware

**RH** hardware available on **LT** support levels. **TH** and **FS** hardware available on **MT**, **UT**, and **SH** support levels. **SH** hardware available on all support levels.



**SH**

2-point, Quick-Release hardware that offers 2.25" of depth adjustment and +/- 20° of back angle adjustment.



**TH**

2-point, Quick-Release hardware that is optimized for use with a tilt-in-space wheelchair, includes a lengthened bracket and attachment pin to accommodate up to 500 lbs, and offers 2.25" of depth adjustment and +/- 20° of back angle adjustment.



**RH**

2-point, Quick-Release hardware that is optimized for use with an ultra lightweight wheelchair, weighs 25% less than SH hardware, and offers 1.75" of depth adjustment and +/- 10° of back angle adjustment.



**FS**

Increased stability supports more force, 500 lb. weight capacity with hardware and over 22" wide shells.

**Step 5** Select mounting hardware.

**J3**

**PA**

**UT**

**M**

**I6**

**SH**

J3 Part Numbers begin with "J3"

Step 1

Step 2

Step 3

Step 4

Step 5

## Recap

### SMART PART NUMBER

Example Smart Part Number:  
**J3PAUTMI6SH**



**Step 1**    **J3**    *J3 Back*

**Step 1**    **PA**    *Posterior support, meets 2.2 inch posterior contour.*

**Step 2**    **UT**    *Designed for supporting the back through the upper thoracic spine.*

**Step 3**    **M**    *Medium height back in the upper thoracic support position.*

**Step 4**    **16**    *Designed to fit wheelchairs 16-18 inches in width.*

**Step 5**    **SH**    *SH attaching hardware.*

J3



## Options

**JAY Your Way** is a modification program designed to deliver the benefits of customized seating but with the ease of ordering standard JAY products.



From clinical improvements to colored covers, **JAY Your Way** modifies cushions and backs specifically to address users' unique individual needs and preferences. Modifications include JAY bases, fluid inserts, covers, backrests and JAY products. Most modifications are made at no additional charge. With **JAY Your Way**, products are easily modified to meet your clients' needs and individual styles. For more information go to: [SunriseMedical.com/JAY-Your-Way](https://SunriseMedical.com/JAY-Your-Way)



## Accessories

### Headrest Pads



PLUSH



C-PLUSH



Contour  
Cradle



Heads Up®



Adjust-A-PLUSH

### Headrest Mounting Hardware



AXYS  
Mount



LINX  
Mount



LINX<sup>2</sup>  
Mount



ONYX  
Mount



Cobra  
Mount



Cobra FB  
Mount



Pro  
Mount

### Laterals



Fixed Lateral



Angle Adjustable  
Swing-Away Laterals



CradleU® Lateral

## Chest Strap Harness



Contour Style



Classic Style



Center Opening  
with Buckle

## Vanity Flap



## Spine-Align Kits




Spine-Align Mini Kit I



Spine-Align Mini Kit II

## Spine-Align Components

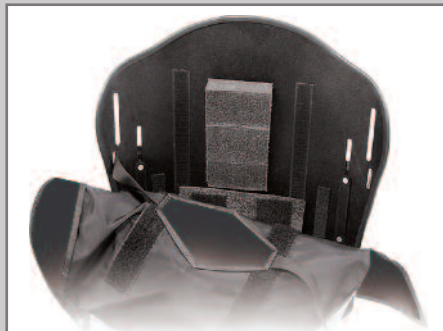

The components slip between the shell and the cover. It is not necessary to open the foam cover.



Small Lumbar

Large Lumbar

Lateral Shim

Posterior/  
Lateral Pelvic

Lateral Shim



Curved Shim

Flat Shim

Small Bean Bag


Large Bean Bag



# Appendix

## PA

**JAY J3 Posterior (PA) Backrest**  
**E2613 (<22" Wide)**  
**E2614 (>22" Wide)**

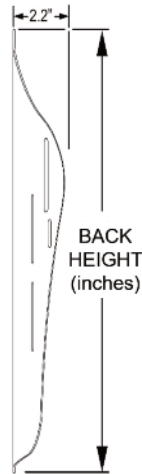


### WIDTH

12" 14" 16" 18" 20" 22" 24" 26"

		WIDTH							
		12"	14"	16"	18"	20"	22"	24"	26"
LT	S 6.5"								
	M 8"								
	T 9.5"								
MT	S 12"								
	M 13.5								
	T 15"								
UT	S 16.5"								
	M 18"								
	T 19.5"								
SH	S 21"								
	M 22.5"								
	T 24"								

## PL

**JAY J3 Posterior Lateral (PL) Backrest**  
**E2615**

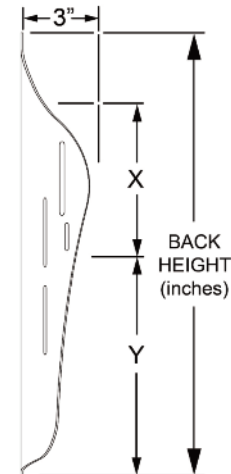

### WIDTH

12" 14" 16" 18" 20"

		WIDTH				
		12"	14"	16"	18"	20"
LT	S 6.5"					
	M 8"					
	T 9.5"					
MT	S 12"					
	M 13.5					
	T 15"					
UT	S 16.5"					
	M 18"					
	T 19.5"					
SH	S 21"					
	M 22.5"					
	T 24"					

# PD




# PDL

# PDC

**JAY J3 Posterior Deep  
(PD) Backrest  
E2620**

**JAY J3 Posterior Deep Lateral  
(PDL) Backrest  
E2620**

**JAY J3 Posterior Deep Contour  
(PDC) Backrest  
E2620**

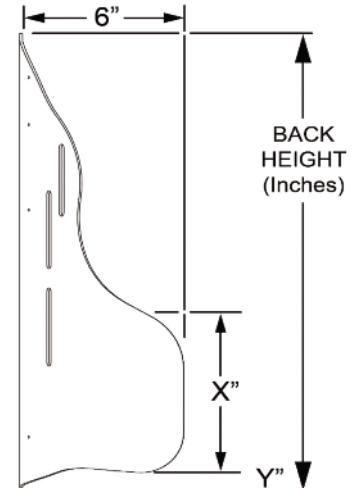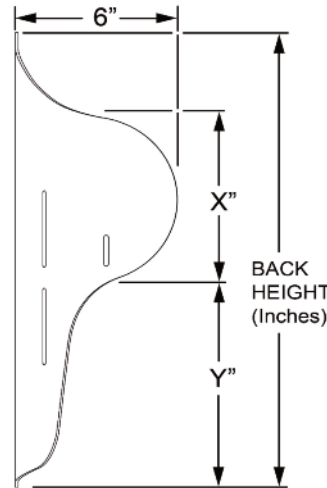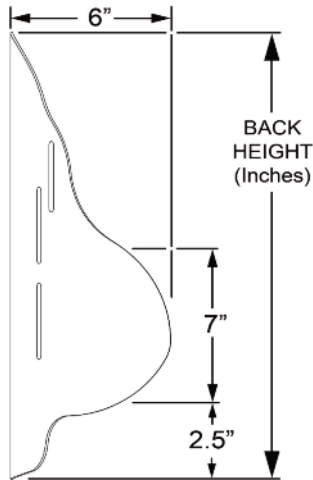

## WIDTH

## WIDTH

## WIDTH

12" 14" 16" 18" 20"

	12"	14"	16"	18"	20"
LT	S 6.5"				
	M 8"				
	T 9.5"				
MT	S 12"				
	M 13.5"				
	T 15"				
UT	S 16.5"				
	M 18"				
	T 19.5"				
SH	S 21"				
	M 22.5"				
	T 24"				

14" 16" 18" 20"

	14"	16"	18"	20"
LT	S 6.5"			
	M 8"			
	T 9.5"			
MT	S 12"			
	M 13.5"			
	T 15"			
UT	S 16.5"			
	M 18"			
	T 19.5"			
SH	S 21"			
	M 22.5"			
	T 24"			

14" 16" 18" 20"

	14"	16"	18"	20"
LT	S 6.5"			
	M 8"			
	T 9.5"			
MT	S 12"			
	M 13.5"			
	T 15"			
UT	S 16.5"			
	M 18"			
	T 19.5"			
SH	S 21"			
	M 22.5"			
	T 24"			



**J3 Back also comes in  
Carbon Fiber and  
Heating & Ventilation options.**



J3 Heating and Ventilation Option



J3 Carbon Fiber Option

**For more information go to:  
[SunriseMedical.com/J3Back](https://www.SunriseMedical.com/J3Back)**

Sunrise Medical® and JAY® are trademarks of Sunrise Medical.



Sunrise Medical (US) LLC, Fresno, CA 93727  
MK-100153 Rev. C ©04.2019

800.333.4000

[www.SunriseMedical.com](http://www.SunriseMedical.com)

