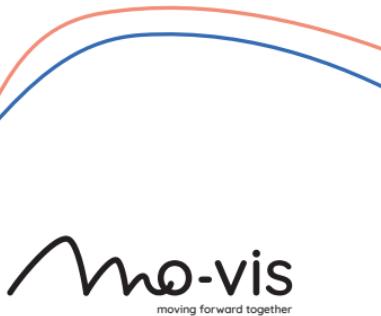
# User manual

Edition 5, July 2021

# Hand Warmer

Hand Warmer (P012-62)

# Contents

<b>About this manual .....</b>	<b>7</b>
User manual .....	7
mo-vis bv .....	7
<b>User manual (English) .....</b>	<b>8</b>
Warning labels .....	8
Support, scrapping & recycling .....	9
Limited liability .....	10
Intended use .....	10
Features .....	11
Operation .....	11
Parts and accessories .....	13
Qualifications .....	16
First time use .....	16
Cleaning .....	17
Warnings and limitations .....	17
Dimensions .....	18
<b>Brugermanual (Dansk) .....</b>	<b>20</b>
Advarselsmærkater .....	20
Support, bortskaffelse og genbrug .....	21
Ansvarsbegrænsning .....	22
Tilsigted brug .....	22

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# About this manual

## User manual

Thank you for choosing a mo-vis product!

This manual contains **useful and important information** about your device.  
**Please read it carefully before use and store safely for future reference.**

Our team (or your authorized dealer) will be happy to answer your questions.

## mo-vis bv

Biebuyckstraat 15D . 9850 Deinze . Belgium

<http://www.mo-vis.com> . [contact@mo-vis.com](mailto:contact@mo-vis.com) . +32 9 335 28 60

# User manual (English)

## Warning labels

*Please read this manual, the safety instructions and warning texts carefully, in order to reduce the risks associated to the device. Our products are safe under normal and reasonably foreseeable operating conditions.*

**NOTE:** This symbol indicates general notes and information.

**CAUTION:** This symbol indicates caution for a hazardous situation that, if not avoided, could result in minor or moderate injury.

**WARNING:** This symbol indicates a warning for a hazardous situation that, if not avoided, could result in death or serious injury.

# Support, scrapping & recycling

## *Technical support*

**TROUBLE:** In case of technical problems:

- Contact your dealer
- If your dealer is not available or unknown, please contact mo-vis: [contact@mo-vis.com](mailto:contact@mo-vis.com) or +32 9 335 28 60.

Always state the product code and device serial number when contacting us. This ensures you are provided with the correct information.

## *Spare parts and accessories*

Contact mo-vis or your dealer for more information about spare parts and accessories.

## *Scrapping & recycling*

**CAUTION:** For scrapping, adhere to your local waste legislation. Dispose of obsolete electronic parts responsibly in accordance with local recycling regulations.

## Limited liability

*mo-vis accepts **no liability** for personal injury or damage to property that may arise from the failure of the user or other persons to follow the recommendation, warnings and instructions in this manual.*

**CAUTION:** This product should only be installed by a qualified service engineer.

**NOTE:** All mo-vis manuals can be found at <http://www.mo-vis.com> where they can be consulted in PDF format.

**NOTE:** In case any serious incident occurs in relation to this device, this should be reported immediately to mo-vis and the competent authority of the Member State in which the user is established.

## Intended use

The Hand Warmer's main functionality is to keep the area around a wheelchair input device warm. This increases the comfort of the users and their ability to control the input device.

It is intended to be fitted on any power chair with 24V connection.

## Features

The Hand Warmer:

- is an air heater, powered by the battery power (24V) of a power chair.
- is fully adjustable to individual possibilities and needs with well-thought-out electronics (mini USB connection).
- recycles air instead of using air from the cold environment: it draws air in at the front, heats it and forces it out again at the front using an internal fan. This results in a huge advantage with regards to the power consumption and efficiency.
- a mechanical switch (e.g. mo-vis Twister) can be immediately plugged in (on/off, select power levels) although you also have an on-board push button.
- LED indicates the status of the device.

**NOTE:** A Perspex dome-shaped 2-hands hood is available to better maintain the air temperature around the hands.

## Operation

At the back of the device, a jack connection (1) and push button (2) are provided.

A button connected to the jack connection has the same functionality as the on-board push button.

The functionality of the button is to cycle through the levels (power settings) of the device. See *Installation Manual* for more information.

The green LED at the front indicates the status of the device:

MODE	LED INDICATOR
Sleep mode	Off
Stand-by	Short on, very long off
Active level 1	Short on, long off
Active level 2	Short on, medium off
Active level 3	Short on, short off
Fault	number of flashes, long off (see <i>Installation Manual</i> for more information on the errors and fault log)

## Parts and accessories

### *Hand Warmer set*

A Hand Warmer set consists of the following parts:

EXAMPLE	PRODUCT DESCRIPTION
 A black rectangular device with a ribbed base and a small circular logo on top, connected by a short black cable to a circular connector.	P012-61 Hand Warmer unit
 A coiled black cable with two connectors at the ends. The left end has a black cylindrical cap, and the right end features a gold-colored XLR-style connector.	P012-40 Hand Warmer extension lead 2000 mm

EXAMPLE	PRODUCT DESCRIPTION
	<p>M012-45 Hand Warmer mounting set:</p> <ul style="list-style-type: none"> <li>• M004-04 Q2M Clamp D6 half + bolt 14 mm</li> <li>• M004-74 Q2M Rod D6 50 mm (1 piece)</li> <li>• M012-40 Hand Warmer dual lock (22*9*3.5 mm): 2 pieces</li> <li>• M004-90 Q2M Rod D6-D8 60mm</li> </ul>

*D-P012-62-70-MX Hand Warmer User manual*

*D-P012-62-70-M1 Hand Warmer Installation manual*

#### ***Optional accessory: Hand Warmer Dome set (M012-31)***

To enclose the area around the hand and to better maintain the air temperature, a Perspex dome-shaped hood is available as an option.

This dome:

- reduces heat loss, resulting in a decreased power consumption, without compromising comfort.
- is delivered with 4 hook and loop fastener pads to attach the dome to any flat surface (e.g. a tray).
- has opening that can easily be adjusted, as Perspex is material that can be cut or shaved.

**CAUTION:** Use proper tools to adjust the dome openings. Warranty is void for any fault as result of adjusting the dome.

## Qualifications

**CAUTION:** Only a qualified service engineer may install the device.

**CAUTION:** An incorrect programming of the wheelchair or device electronics, may cause damage to the devices or injury to the user.

## First time use

**CAUTION:** Only a qualified service engineer may install the device.

**CAUTION:** An incorrect programming of the wheelchair or device electronics, may cause damage to the devices or injury to the user.

**CAUTION:** A qualified service engineer should assist during first time use of the device and make sure that all functions are clear to the user, including what to do in case of problems or doubt.

## Cleaning

**WARNING:** As dust and dirt could lead to reduced functionality, all parts of the device should be cleaned on a regular basis (monthly) or whenever needed.

- Gently remove dust and dirt with a damp cloth.
- Use only non-aggressive disinfectant cleaning agents.

**CAUTION:** Do not immerse in water or use excessive amounts of liquid.

## Warnings and limitations

**NOTE:** This product complies with the limit values for Electromagnetic Compatibility (EMC) with respect to power wheelchairs, as set out in

the harmonized standards for the EU in the Medical Device Regulation, No. 2017/745.

### **WARNING:**

At full power, the device will produce warm air (up to 80°C/158°F). It is advised to leave a minimum distance between the hand and the Hand Warmer of at least 10 cm (4 in).

If the distance is less, it is necessary to limit the maximum heater temperature (see *Installation manual*).

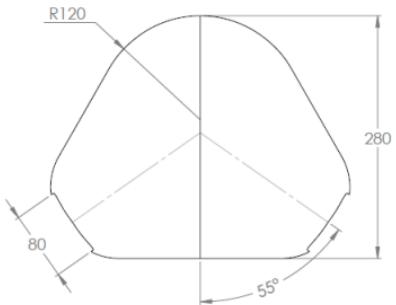
**WARNING:** Check daily if the Hand Warmer is still in its correct position and at least 10 cm away from the hands.

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# Support, bortskaffelse og genbrug

## Teknisk support

**FEJLFINDING:** In case of technical problems:

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EKSEMPEL	PRODUKTBESKRIVELSE
 A black rectangular device with a ribbed front panel and a small circular logo, connected by a short black cable to a circular DC power connector.	P012-61 Hand Warmer unit
 A black coiled extension lead with a gold-plated XLR male connector at one end and a black female connector at the other.	P012-40 Hand Warmer extension lead 2000 mm

EKSEMPEL	PRODUKTBESKRIVELSE
	<p>M012-45 Hand Warmer mounting set:</p> <ul style="list-style-type: none"> <li>• M004-04 Q2M Clamp D6 half + bolt 14 mm</li> <li>• M004-74 Q2M Rod D6 50 mm (1 stk.)</li> <li>• M012-40 Hand Warmer dual lock (22*9*3,5 mm): 2 stk.</li> <li>• M004-90 Q2M Rod D6-D8 60mm</li> </ul>

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## Rengøring

**ADVARSEL:** As dust and dirt could lead to reduced functionality, all parts of the device should be cleaned on a regular basis (monthly) or whenever needed.

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**PAS PÅ:** Do not immerse in water or use excessive amounts of liquid.

## Advarsler og begrænsninger

**BEMÆRK:** This product complies with the limit values for Electromagnetic Compatibility (EMC) with respect to power wheelchairs, as set out in the harmonized standards for the EU in the Medical Device Regulation, No. 2017/745.

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BEISPIEL	PRODUKTBESCHREIBUNG
 A black rectangular electronic device with a ventilation grille on the front and a power cable with a standard DC connector on the left side.	P012-61 Hand Warmer unit
 A black coiled extension lead with a gold-plated XLR male connector on one end and a standard DC connector on the other.	P012-40 Hand Warmer extension lead 2000 mm

BEISPIEL	PRODUKTBESCHREIBUNG
	<p>M012-45 Hand Warmer mounting set:</p> <ul style="list-style-type: none"> <li>• M004-04 Q2M Clamp D6 half + bolt 14 mm</li> <li>• M004-74 Q2M Rod D6 50 mm (1 Stück)</li> <li>• M012-40 Hand Warmer dual lock (22*9*3,5 mm): 2 Stück</li> <li>• M004-90 Q2M Rod D6-D8 60mm</li> </ul>

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D-P012-62-70-M1 Hand Warmer-Installationshandbuch

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## Reinigung

**WARNUNG:** As dust and dirt could lead to reduced functionality, all parts of the device should be cleaned on a regular basis (monthly) or whenever needed.

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**ACHTUNG:** Do not immerse in water or use excessive amounts of liquid.

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**ANMERKUNG:** This product complies with the limit values for Electromagnetic Compatibility (EMC) with respect to power

wheelchairs, as set out in the harmonized standards for the EU in the Medical Device Regulation, No. 2017/745.

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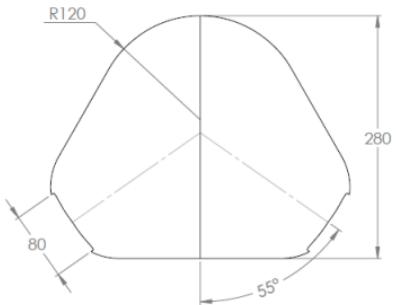
**PROBLEMA:** In case of technical problems:

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EXEMPLE	DESCRIPTION DU PRODUIT
 A black rectangular device with a ribbed base and a small circular logo on top, connected by a short black cable to a circular connector.	P012-61 Hand Warmer unit
 A coiled black cable with two connectors at the ends. The left end is a black cylindrical connector, and the right end is a gold-colored XLR-style connector.	P012-40 Hand Warmer extension lead 2000 mm

EXEMPLE	DESCRIPTION DU PRODUIT
	<p>M012-45 Hand Warmer mounting set :</p> <ul style="list-style-type: none"> <li>• M004-04 Q2M Clamp D6 half + bolt 14 mm</li> <li>• M004-74 Q2M Rod D6 50 mm (1 pièce)</li> <li>• M012-40 Hand Warmer dual lock (22*9*3,5 mm) : 2 pièces</li> <li>• M004-90 Q2M Rod D6-D8 60mm</li> </ul>

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## Nettoyage

**Avertissement:** As dust and dirt could lead to reduced functionality, all parts of the device should be cleaned on a regular basis (monthly) or whenever needed.

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**ATTENTION:** Do not immerse in water or use excessive amounts of liquid.

## Avertissements et limitations

**REMARQUE:** This product complies with the limit values for Electromagnetic Compatibility (EMC) with respect to power

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## Pulizia

**AVVERTENZA:** As dust and dirt could lead to reduced functionality, all parts of the device should be cleaned on a regular basis (monthly) or whenever needed.

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**AVVERTENZA:** Do not immerse in water or use excessive amounts of liquid.

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**NOTA:** This product complies with the limit values for Electromagnetic Compatibility (EMC) with respect to power wheelchairs, as set out in the harmonized standards for the EU in the Medical Device Regulation, No. 2017/745.

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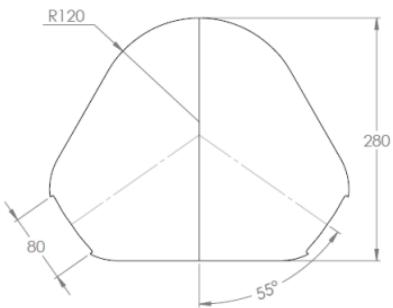
**PROBLEM:** In case of technical problems:

- Kontakt forhandleren
- Hvis forhandleren er ukjent eller ikke tilgjengelig, kan mo-vis kontaktes: [contact@mo-vis.com](mailto:contact@mo-vis.com) eller +32 9 335 28 60.

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VOORBEELD	PRODUCTBESCHRIJVING
 A black rectangular device with a ventilation grille on the front and a power cable with a 2-pin connector on the left side.	P012-61 Hand Warmer unit
 A black coiled extension lead with a 2-pin connector on one end and a gold-plated XLR connector on the other.	P012-40 Hand Warmer extension lead 2000 mm

VOORBEELD	PRODUCTBESCHRIJVING
	<p>M012-45 Hand Warmer mounting set:</p> <ul style="list-style-type: none"> <li>• M004-04 Q2M Clamp D6 half + bout 14 mm</li> <li>• M004-74 Q2M Rod D6 50 mm (1 stuks)</li> <li>• M012-40 Hand Warmer dual lock (22*9*3,5 mm): 2 stuks</li> <li>• M004-90 Q2M Rod D6-D8 60mm</li> </ul>

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## Reinigen

**ATTENTIE:** As dust and dirt could lead to reduced functionality, all parts of the device should be cleaned on a regular basis (monthly) or whenever needed.

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**LET OP!**: Do not immerse in water or use excessive amounts of liquid.

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**OPMERKING:** This product complies with the limit values for Electromagnetic Compatibility (EMC) with respect to power

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EXEMPEL	PRODUKTBESKRIVNING
 A black rectangular electronic device with a ventilation grille on the front and a power cable with a 2-pin connector on the left side.	P012-61 Hand Warmer unit
 A black coiled extension lead with a gold-plated XLR male connector on one end and a 2-pin connector on the other end.	P012-40 Hand Warmer extension lead 2000 mm

EXEMPEL	PRODUKTBESKRIVNING
	<p>M012-45 Hand Warmer mounting set:</p> <ul style="list-style-type: none"> <li>• M004-04 Q2M Clamp D6 half + bult 14 mm</li> <li>• M004-74 Q2M Rod D6 50 mm (1 del)</li> <li>• M012-40 Hand Warmer dual lock (22*9*3,5 mm): 2 delar</li> <li>• M004-90 Q2M Rod D6-D8 60mm</li> </ul>

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## Rengöring

**VARNING - RISK FÖR MASKINSKADA:** As dust and dirt could lead to reduced functionality, all parts of the device should be cleaned on a regular basis (monthly) or whenever needed.

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**ANM:** This product complies with the limit values for Electromagnetic Compatibility (EMC) with respect to power wheelchairs, as set out in the harmonized standards for the EU in the Medical Device Regulation, No. 2017/745.

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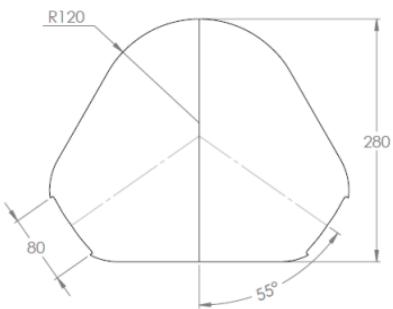
**Go to our website for more information on our products or share  
your experience with us via email.**