

Ipai-Reha

SAFETY FEATURES

Fully adjustable
special needs
car seat



Ipai-Reha

Fully adjustable special needs car seat, offering excellent posture support options and a backrest recline feature.

- Ensure the **Ipai-Reha's back is flush with the cars seat back or headrest** if this is non removable (no big gaps).
- There is no tether for this seat, but **ensure the clamp bow is slotted correctly between the seat and backrest of the car, OR that the ISOFIX/LATCH connectors are correctly in use.**
- Ensure child **has not outgrown the weight capacity which is 36kg**. If child is approaching this weight limit, consider what car seat would be next option.
- This seat has the capability to be very easily grown in back height and seat length. **If you are raising the headrest height, please ensure you are also bringing up the shoulder strap height to the appropriate level** using a 8mm spanner. If you are unsure please contact your Occupational Therapy team.
- Ensure the **car's seatbelt is used EVERY time**. This is to be securely **fastened OVER the child and seat**. Please note this seat does not have capacity to have a belt threaded through the back as it is a booster seat. You are able to purchase and install a padded sleeve for the seatbelt if you feel the belt is going to irritate your child.
- Please **note the red belt guide above the shoulder is to be used to ensure the belt is sitting correctly**, is kept low across the child's hips and is kept off the child's face. The lower portion of the belt is to be threaded through the belt guide spaces on either side of the child's thighs.
- Periodically **check for loose or missing parts**.
- Keep child's fingers **clear of moving parts and swivel latches**.



Ensure back is in contact with the vehicle seat back



Ensure shoulder straps are firmly secure



Ensure child has not outgrown the weight capacity limit



The car's seatbelt must be worn over top of the child and car seat harness

Contact the car seat specialists for more information:
0800 543 343
solutions@medifab.com

Distributed by **medifab®**