POWER RECLINE ADAPTER MOUNT COMPATIBILITY GUIDE


Use this guide to help select Spex products compatible to fit to the **Quantum TRU-Balance 3** System.

Step 1

Select a compatible back support

Back Support	Back Support Spex (Classic, Manta, Vigour, SuperShape)		XLella	XLella ²
Compatibility	✓	\checkmark	1	×

(i) Take Note of Compatibility Guide Assumptions

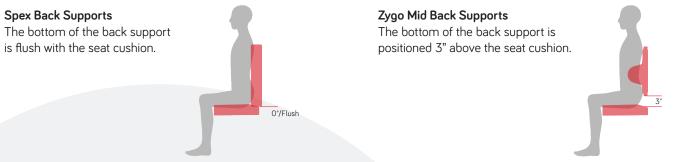

Step 2

Determine compatible back support size

Back Support	Hand with Stude Slim Meuntine Plank?	Compatible Size	
	Used with Stylo Slim Mounting Block?	Width	Height*
Spex	Included Stylo Slim Mounting Block	14 - 21"	16 - 25"
	Excluded Head Support Hardware		14 - 25"
Zygo Mid	Included Stylo Slim Mounting Block	14 - 20"	14 - 18"
	Excluded Head Support Hardware	14 - 20	13 - 18"

*When seat cushion height is the same as the Spex Classic Cushion.

Deel, Comment	System	Shell Width	Shell Height	Compatible cushion to back support gap	
Back Support				Including Stylo Mount	Excluding Stylo Mount
XLella	TRU-Balance 3	22 - 28"	16 - 20"	2.75 - 6.5"	0 - 7.25"

*When seat cushion height is the same as the XLella Cushion.

Take note of head support setup considerations

Position silver ball mount (shown in red) above the top edge of the Quantum TB3 mounting plate when the wheelchair is in an upright sitting position.

This prevents interference of Stylo head support hardware during power recline function.

