

# Fabric Care Instructions

**IMPORTANT CONSUMER INFORMATION NOTICE:**

This instruction leaflet contains important information regarding the care and maintenance of this product.

**SUPPLIER:**

Please pass this instruction leaflet on to the end user of the product.

**USER:**

Before using this product, read care instructions and save for future reference.

**medifab**

Fabric Care Instructions 20211104

# Information Table

The following table provides detailed procedures for caring for and washing of the following fabric products.

Fabric Name	Appearance	Properties	Fabric Composition	Washing Instructions	Available For
<b>Cotton Towelling</b>	Variety of colours available	Slight stretch, absorbent, soft and warming. Small soft loops on fabric.	90% Cotton, 10% PES	Hand or machine wash on gentle cycle - 60°C. Do not bleach, dry clean, wring or leave wet. Do not tumble dry. Do not iron.	Custom made items & CareWave (CareWave variant towelling can be slightly firmer)
<b>Natural Merino Wool</b>	Variety of colours available	Stretchy, soft, with natural cooling and warming properties	100% Merino	Hand or machine wash on gentle cycle - 40°C. Low spin. Low tumble dry - 40°C. Cool iron. Use approved wool detergent and wash with like colours. Do not bleach, dry clean, wring or leave wet.	Custom made items
<b>Coolover</b>	Cream knit	Stretchy and comforting, highly absorbent and impregnated with Coolover technology to absorb and store heat, and then release it when required.	Main fabric: 60% Modified Acrylic, 40% Viscose with stearate. Sides: Polyester mesh.	Machine wash at 75°C or less with mild detergents. Low tumble dry. Do not iron or dryclean.	Symmetrisleep sheets, positioning products and pillows, custom made items
<b>Outlast</b>	White woven cloth	Slight stretch, cool to touch, impregnated with Outlast technology to absorb and store heat, and then release it when required. Less overheating and night sweating, less chilling to help to sleep more soundly.	100% Polyester textile, with Silicone coating layer containing PCM (Phase Change Material) capsules	Hand or machine wash on gentle cycle - 40°C. Do not bleach, dry clean, wring or leave wet. Low heat tumble dry. Cool iron. Wet clean.	Custom made items, and available for covering CareWave, Poz'In'Form if necessary.

Disclaimer: The above washing instructions are provided by the manufacturers of the fabrics and are subject to change without notice.

**medifab**

22 Detroit Drive  
 Rolleston 7675  
 New Zealand

+64 3 307 9790

solutions@medifab.com

www.medifab.com